Access Handle

Access Handle

Access Handle

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Wall Mounted Grab Bar

Instructions for Installation	
Necessary tools:	1 power drill 1 screw driver
Additional material:	6 screws and rawl plugs of a shape and dimension suitable for the selected wall.
	Diameter of the thread: Max: 4,8 mm Diameter of the screw head: Max. 6,0 mm
Instructions:	
<ol style="list-style-type: none"> Make certain that there are no water pipes or electrical cables in the wall at the place, where you wish to mount the grab bar. This can be checked either on the drawings of the building or by using a special search device. Remove the 2 round plastic covers, which cover the screw holes by sliding these onto the middle of the grab bar. Hold the grab bar against the wall, where you want to mount it and draw pencil marks on the wall through the 6 holes in the wall mounting brackets, 3 on each side. Remove the grab bar, drill holes in the wall and insert the rawl plugs. Fasten the grab bar securely by inserting the 6 screws through the holes and by tightening these with a screw driver. Control that the grab bar is securely fastened and does not move weight is applied to it. Push the 2 round plastic covers back in place so that the screws are no longer visible. 	
Important: Should the grab bar break or distort during installation, the article should be replaced immediately.	
Cleaning:	
Wash the grab bar without removing it from the wall. Wash with non-corrosive detergents.	
Caution:	
Make sure that the wall selected is suitable to support the grab bar, that the grab bar fixed to the wall prior to use and do not exceed max weight indicated.	
Technical details:	
Material: ABS (wall mounting brackets and plastic covers), PVC (bar)	
Length:	RI-EH-12 305,0 mm / 12" RI-EH-16 406,5 mm / 16" RI-EH-18 457,0 mm / 18" RI-EH-24 610,0 mm / 24"
Diameter of bar:	36 mm
Max weight:	120 Kg