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
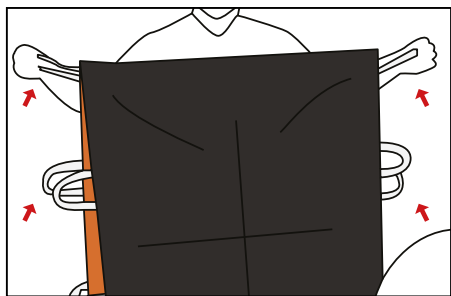
**UK** Hold the Sling Master using the two grey bands in the middle. Then take the two white bands and let the two grey ones fall in the middle

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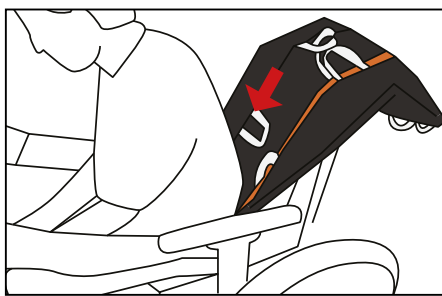Guide the Sling Master down the user's back. Always stand in front of the user for safety purposes

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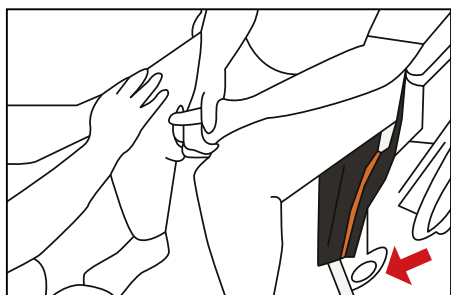
Tighten the Sling Master and let it slide under the user's bottom with a "sawing" motion

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
**UK** Tighten the Sling Master and let it slide under the user's buttocks with "sawing" movements

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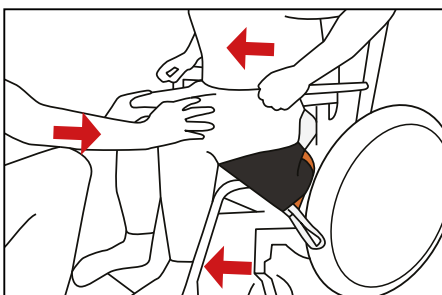
The user leans their upper body forward at the same time as the helper pushes and lets the user slide from side to side in the chair. The user must support their feet either with a foot support or with a Foot Master

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
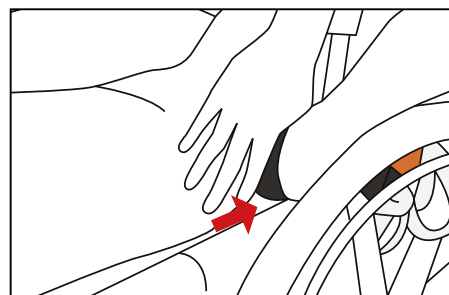
The Sling Master can be removed by grasping the handles and pulling it out from under the user

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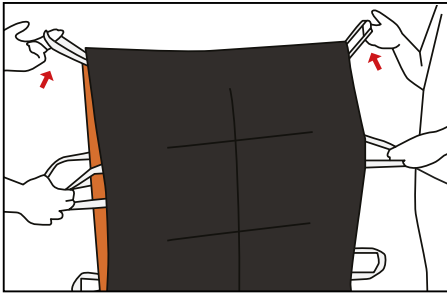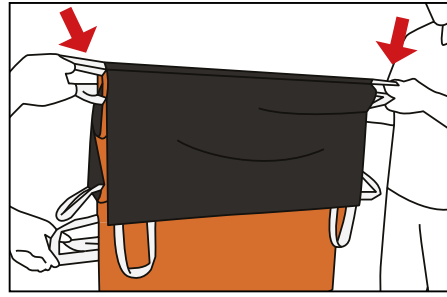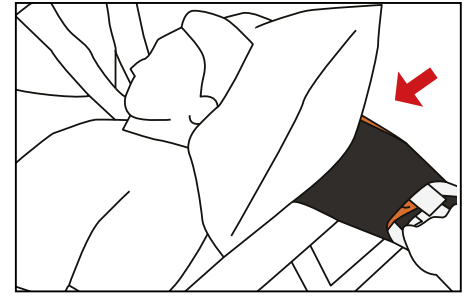
**UK** Raise the bed under the user's knee and head. Hold the Sling Master by grasping the two grey handles in the middle

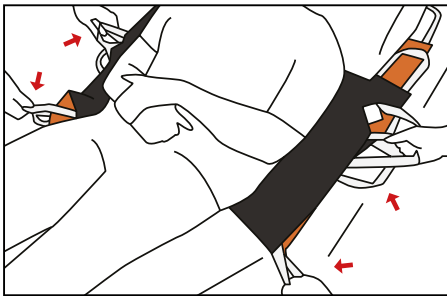
Hold the two white handles, and at the same time let the grey ones fall to the middle

Hold the Sling Master by the white handles. With the other hand, hold the pillow and push the Sling Master in under this

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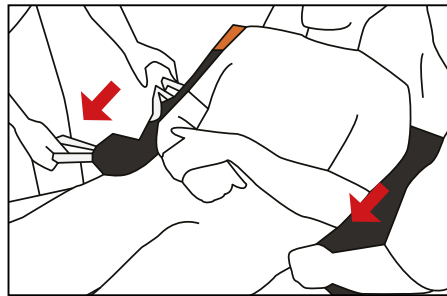
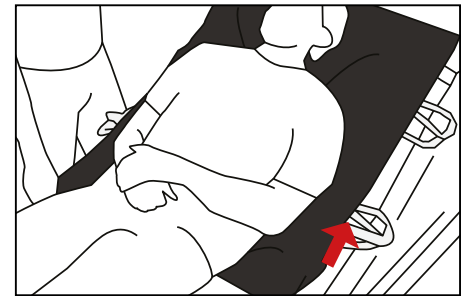
**UK** The helper pulls alternately in the white handles and makes "sawing" motions until the Sling Master has passed the user's shoulder blades

The helper grips the lowest grey handles on the Sling Master. The Sling Master folds out and slides down to the user's lower back

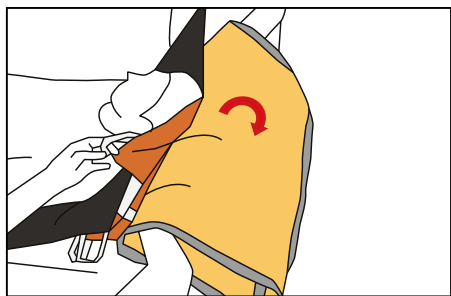
The Sling Master is now placed under the user from the lower back upwards. If the user is lying on a Silk Master, the Sling Master will be easy to place

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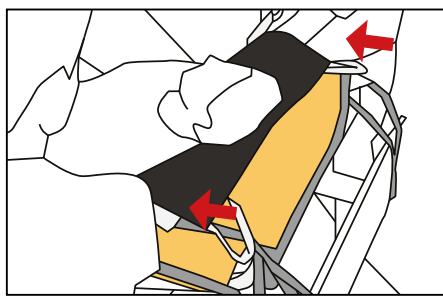
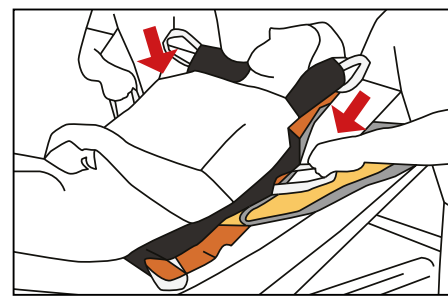
**UK** Fold the lifting sail together so that the back part of the sail hangs down with its outer side towards the head end and the foot straps are down to the side facing in towards the user's head

The helpers hold the hand surfaces in the lifting sail where it is folded

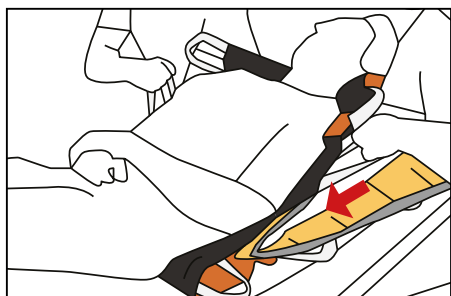
The helpers hold the two upper handles on the Sling Master and guide the sail a little in between the two sliding surfaces

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**UK** After this, the helpers hold the two corner handles on the Sling Master at the same time as guiding the sail down towards the lower back so that the sail glides into the Sling Master

The one helper pulls the leg straps on the lifting sail, whilst the other is holding the handles at the opposite end of the Sling Master. The lifting sail now slides into place

Hold the grey handles at the end of the Sling Master. Fold that part up, pull and slide that part down towards the user's lower back. The Sling Master is removed by pulling in the lowest sliding part and rolling it out under the user

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- Leveres i flere størrelser

- UK
- Easy to place under the lying and sitting user,
  - Reduces friction.

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**MC 001-1732** [ HMI: 49368 ]  
Master Turner Soft - **Large**

Størrelse / Dimensions / Größe:  
L 215 x B/W 150

**MC 001-2540** [ HMI: 49370 ]  
Master Turner Two In One Soft - **Large**

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
Størrelse / Dimensions / Größe:  
L 47 x B/W 4 cm

**MC 001-2500** [ HMI: 49873 ]  
Master Turner DUO Two In One Soft Bund / Bottom  
/ Unterteil - **Large**

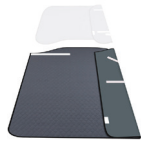Størrelse / Dimensions / Größe:  
L 160 x B/W 145 cm

**MC 001-2501** [ HMI: 49872 ]  
Master Turner DUO Two In One Soft Top / Top /  
Oberteil - **Large**

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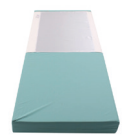
Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantly

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care**'s products with lowered safety rails

Never leave the user with master **care**'s products attached to the lift or with lowered safety rails

### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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These products comply with the standards for Class 1 products in the Medical Device Regulation (EU) 2017/745.

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