Invacare® Stileo

H171 / H172

en	Suction grab bar	
	User Manual	1
de	Saugerhandgriff	
	Gebrauchsanweisung	3
fr	Poignée de maintien à ventouses	
	Manuel d'utilisation	5
it	Barra di supporto a ventosa	
	Manuale d'uso	7
es	Asa de sujeción con ventosa	
	Manual del usuario	8
pt	Manípulo de sucção	
	Manual de utilização	10
nl	Handgreep met zuignappen	
	Gebruiksaanwijzing	12
sv	Suggrifhandtag	
	Bruksanvisning	14
da	Gribehåndtag med sugekop	
	Brugsanvisning	15
no	Sugekopphåndtak	
	Bruksanvisning	17
fi	Imukuppikahva	
	Käyttöohje	19

I General

en

I.1 Images

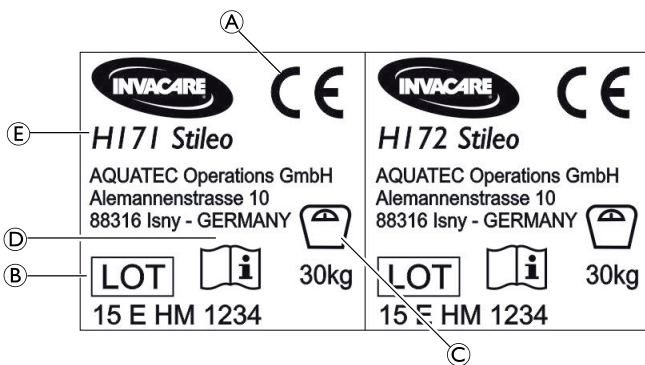
Fig. 1

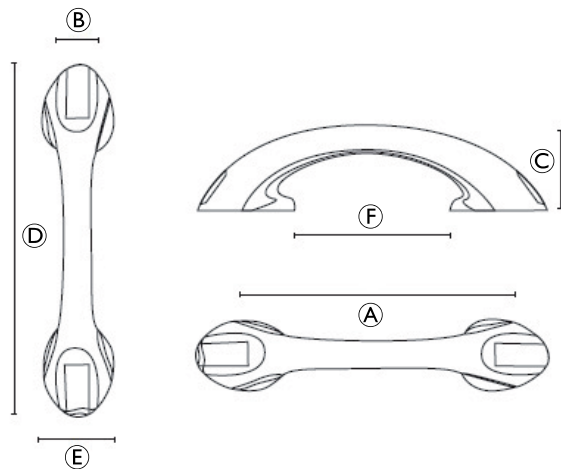
Fig. 2

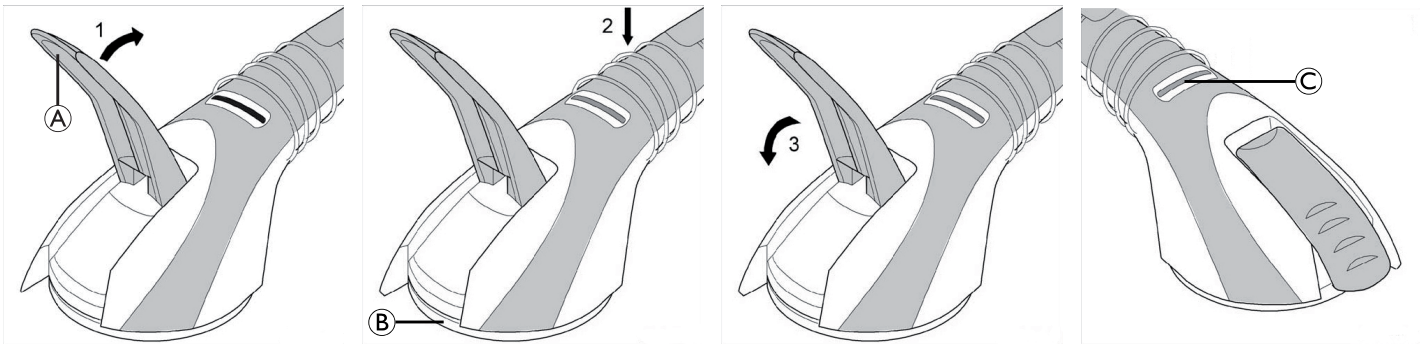
Fig. 3

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

Yes, you can.®

1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

1.3 Intended use

The suction grab bars were designed to support natural movement and not to bear the full body weight of the user. They are intended for short-term use, e.g. on journeys or as a temporary solution after a hospital stay. Any other use is prohibited.

1.4 Scope of delivery

- Suction grab bar
- User manual

1.5 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

1.6 Standards and regulations

Quality is of the utmost importance for our company; all processes are based on the ISO 9001 and ISO 13485 standards.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class I.

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare® in your country (for addresses, see back page of this manual).

1.7 Service life

The expected service life of this product is five years when used daily in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety

2.1 Safety information

WARNING! Risk of falling!

On unsuitable surfaces, the suction grab bar may not achieve the required suction force and may as a result suddenly be detached from the surface.

- Only use the suction grab bar on smooth, non-porous and firm surfaces, which are clean and dry, steamproof and imperviously sealed.
- Never attach a suction grab bar on a joint.

WARNING! Risk of falling!

- Do not use if defective.

WARNING! Risk of falling!

Through ageing, incorrect installation or as a result of a dry construction method, it is possible that the adhesive force of tiles and surface materials may be insufficient. For this reason, the prepared surface may become detached if load is applied to the correctly mounted suction grab bar.

- Check the stability of the prepared surface before using the suction grab bar for the first time.

WARNING! Risk of falling!

Different ambient conditions can reduce the suction force.

- Only use the suction grab bar at temperatures below 40 °C and up to a maximum altitude of 1000 m above sea level.
- Do not expose the suction grab bar to direct sunlight for prolonged periods.
- Never attach a suction grab bar to a window.

WARNING! Risk of falling!

Excessive or intermittent loads can lead to spontaneous loss of suction force.

- Before using the suction grab bar, check the suitability for the relevant user and his/her usage behavior.
- The maximum load that can be applied to the suction grab bar is 30 kg.

2.2 Labels and symbols on the product

Identification label (Fig. 1)

(A)	Conformity mark
(B)	Lot number
(C)	Maximum safe load
(D)	Read user manual
(E)	Product designation

Warning label

Warning label for safe usage. Refer to hazard statements in chapter 3.1 Installing the suction grab bar (Fig. 3), page 2.

3 Setup and usage

3.1 Installing the suction grab bar (Fig. 3)

The suction grab bar can be installed on sealed and smooth surfaces.

WARNING! Risk of falling!

Incorrect mounting may lead to the suction grab bar becoming suddenly detached from the surface. The check display on the suction grab bar allows the user to check if it is mounted correctly (it is not a vacuum display!).

- Ensure that the suction grab bar is undamaged, oil, dust and dirt-free before installing.

1. Release both folding levers (A).
2. Press and hold the suction grab bar equally with both suction pads (B) against a suitable surface.
3. Close both folding levers.

**WARNING!
Risk of falling!**

There are two displays © present on the suction grab bar indicating if the grab bar is installed correctly and stable.

- If one or both indicators display in red, the suction grab bar is not safely installed and must be repositioned.
- Make sure that the indicators on the suction grab bar both display in green before using the product.

Observe Warning label regarding the indicators:

	Red hand: Do not use the product when one or both indicators display in "red".
	Green check: Only use the product when both indicators display in "green".

3.2 Using the suction grab bar

As the following safety information does not encompass all possible circumstances, it is up to the user in the individual case to check before use whether the suction grab bar is suitable for the intended application, under the existing conditions.

**WARNING!
Risk of falling!**

If the suction grab bar is used for any purpose other than that intended, e.g. as a sports device for "chin-ups" or as a climbing aid, it can suddenly become detached.

- Only use the suction grab bar for its intended purpose.
- Do not use as sports equipment.
- Do not use as a climbing aid.

1. Grip the handle of the suction grab bar with one hand and carry out the desired body movement.

4 Maintenance

4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

**WARNING!**

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

4.2 Cleaning and disinfection

**IMPORTANT!**

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.

- Clean by hand.
- Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
- Max. cleaning temperature 30 °C.

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

4.3 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly

1526478-C

(© 4.2 Cleaning and disinfection, page 3), check the safety functions and reset if necessary.

4.4 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

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*All product components are corrosion resistant and Latex free.

Environmental parameters

Operating conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Atmospheric pressure	940 - 1060 hPa
Storage and shipping conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing
Atmospheric pressure	795 - 1060 hPa

Invacare® distributors

Belgium & Luxemburg:

Invacare nv
Autobaan 22
B-8210 Loppem
Tel: (32) (0)50 83 10 10
Fax: (32) (0)50 83 10 11
belgium@invacare.com
www.invacare.be

Danmark:

Invacare A/S
Sdr. Ringvej 37
DK-2605 Brøndby
Tel: (45) (0)36 90 00 00
Fax: (45) (0)36 90 00 01
denmark@invacare.com
www.invacare.dk

Deutschland:

Invacare GmbH,
Alemannenstraße 10
D-88316 Isny
Tel: (49) (0)7562 700 0
Fax: (49) (0)7562 700 66
kontakt@invacare.com
www.invacare.de

España:

Invacare SA
c/Areny s/n, Polígon Industrial de
Celrà
E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00
Fax: (34) (0)972 49 32 20
contactsp@invacare.com
www.invacare.es

France:

Invacare Poirier SAS
Route de St Roch
F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66
Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com
www.invacare.fr

Ireland:

Invacare Ireland Ltd,
Unit 5 Seatown Business Campus
Seatown Road, Swords, County Dublin
Tel: (353) 1 810 7084
Fax: (353) 1 810 7085
ireland@invacare.com
www.invacare.ie

Italia:

Invacare Mecc San s.r.l.,
Via dei Pini 62,
I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59,
Fax: (39) 0445 38 00 34,
italia@invacare.com
www.invacare.it

Nederland:

Invacare BV
Celsiusstraat 46
NL-6716 BZ Ede
Tel: (31) (0)318 695 757
Fax: (31) (0)318 695 758
nederland@invacare.com
www.invacare.nl

Norge:

Invacare AS
Grensesvingen 9, Postboks 6230, Etterstad
N-0603 Oslo
Tel: (47) (0)22 57 95 00
Fax: (47) (0)22 57 95 01
norway@invacare.com
www.invacare.no

Österreich:

Invacare Austria GmbH
Herzog Odilostrasse 101
A-5310 Mondsee
Tel: (43) 6232 5535 0
Fax: (43) 6232 5535 4
info@invacare-austria.com
www.invacare.at

Portugal:

Invacare Lda
Rua Estrada Velha, 949
P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47
Fax: (351) (0)225 1057 39
portugal@invacare.com
www.invacare.pt

Sverige:

Invacare AB
Fagerstagatan 9
S-163 53 Spånga
Tel: (46) (0)8 761 70 90
Fax: (46) (0)8 761 81 08
sweden@invacare.com
www.invacare.se

Suomi:

Algol-Trehab Oy
Karapellontie 6, 02610 Espoo
Puhelin 09 50991
Faksi 09 5099490
Sähköposti info.algoltrehab.fi@algol.fi
Internet www.algoltrehab.fi

Schweiz / Suisse / Svizzera:

Invacare AG
Benkenstrasse 260
CH-4108 Witterswil
Tel: (41) (0)61 487 70 80
Fax: (41) (0)61 487 70 81
switzerland@invacare.com
www.invacare.ch

United Kingdom:

Invacare Limited
Pencoed Technology Park, Pencoed
Bridgend CF35 5AQ
Tel: (44) (0) 1656 776 222
Fax: (44) (0) 1656 776 220
uk@invacare.com
www.invacare.co.uk

USA:

Clarke Health Care Products
7830 Steubenville Pike
Oakdale, PA 15071, USA
888-347-4537
www.clarkehealthcare.com

Canada:

Invacare Canada LP
570 Matheson Blvd E. Unit 8
Mississauga Ontario
L4Z 4G4, Canada
Phone: (905) 890 8300
Fax: (905) 501 4336

Aquatec Operations GmbH
Alemannenstrasse 10
D-88316 Isny

1526478-C 2015-08-25

Making Life's Experiences Possible™

Yes, you can.®