FOLLO STABILIS WALKER

 FOLLO FUTURA

We congratulate you on choosing a Follo Stabilis product. This instruction information makes it possible for you to take full advantage of the product from the first day in use. Please read these instructions carefully before taking the product into use.
Contents: Check list, unpacking/Installation, safety information, how to use, maintenance instructions, technical specifications and spare parts list

Flat packed Walkers are controlled according to our procedure in ISO 9001 system.

ORDER NO.:
COLOUR:
SERIAL NO.:

- ELECTRIC
 GAS 200N 250N

FUNCTION TEST:	<input type="checkbox"/> Arm rest/surface: No damages
<input type="checkbox"/> Bottom frame: Wheels, Screws, plastic parts	<input type="checkbox"/> Battery function
<input type="checkbox"/> Top frame: Surface, screws, plastic parts	<input type="checkbox"/> Labels: Serial number
<input type="checkbox"/> Glide tube: Welding, surface, Silicon grease	ENCLOSED:
<input type="checkbox"/> Gas: Adjusting handle	<input type="checkbox"/> Instructions
<input type="checkbox"/> Screws	

ASSEMBLED (date/sign.):	
-------------------------	--

END CONTROLLED (date/sign.):	
------------------------------	--

PACKING CONTROL: (date/sign.):	
--------------------------------	--

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9. Max bredd på bottenram.

Unpacking / Installation:

1. Make sure that the product has not been damaged (hurt) in transport.
2. Fit the walker according to assembly instructions.
3. Assemble all the accessories that have been ordered.
4. Test all the various functions and setting options.

Security Information:

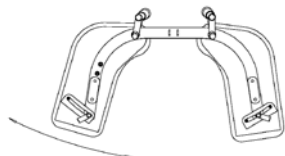
1. If there is any damage or defects in the walker should not be used until these are rectified.
2. The Walker is designed for indoor use.
3. When adjusting height and depression, make sure that nothing is pinched between the frame parts.
4. The Walker should never be charged with more than the approved static use weight.
5. Infusion stand has to be mounted parallel to the walker. (See illustration on page 16).
6. The Walker must not be modified or used with other accessories than the original. Producer responsibility for CE marking terminates any self-modification.
7. The electric walker should not be used with the charger attached to the wall outlet.

Instructions for use:

Height Control of walker with gas spring: The handle on the front of the top frame activates the gas spring. Squeeze the handle to the top frame to adjust the walker up or down. To lower the walker, squeeze the handle and load the armrest with body weight, then drop the handle at the desired level.

Height Control of walker with electric motor: The height of the armrests is adjusted with the controller to the desired height. The product is also delivered with maneuvering feature integrated into the handles.

Cushions adjustment: Loosen the handles. Adjust the cushions to the desired position, tighten the handles. (Make sure they are tightened). Please note: support handles follow the armrest setting and facilitates appropriate support and use.

Wheels locking: The rear wheels have locks. To unlock the brakes press down the "locking" with the foot. To release the brakes tilt "locking" up again with the toe.

Maintenance / inspection:

- Control at regular intervals (for example every third months):
1. That all screws are tight and all functions are working properly.
 2. That there is no damage or wear and tear on the walker and the walker is stable.
 3. That the braking effect on the brakes is adequate.
 4. That the telescopic tubes are lubricated regularly (with a thin layer of, for example, Vaseline or silicone).

The cushions are made of polyurethane foam, and can easily be wiped off with most detergents.

Attention! Alcohol-based products discolor the cushions. The motor, the control box and the controller should be wiped off with a slightly moist cloth.

Reconditioning:

- Reconditioning of the product must be performed by professionals.
- Before cleaning / disinfection, remove the cushions. Control Box is attached with Velcro must be easily removed from the base frame, a hand control follows with. Engine disassembles easily by removing two M10 bolts and nuts, see the groundbreaking design.
- After cleaning / disinfecting, wipe product carefully so that moisture dislodges. It is very important that all cavities are inspected for moisture.
- When the product is dry, Telescopic tube must be lubricated with silicone spray and top frame is moved up and down a few times so that the parts slide easily. After mounting of cushions, check the sideways adjust alignment and if the felt is in place.
- Replace worn parts and ordered by the supplier, see the exploded drawing and part list.
- All worn / damaged Information - and warning labels must be replaced. These ordered from the supplier.
- The product assembles in accordance with groundbreaking design.

Recycling:

The motor, the control box and the hand control are disposed of as electronic waste. The cushions are disposed of as combustible waste, while the stand etc. are sorted as metal.

Charge Routines:

When charging the Follo Stabilis Walker, the charger should always first be connected to the walker before plugging into the wall outlet. Disconnection is performed in the reverse order. The batteries are usually fully charged at delivery, but they should nevertheless be set to charge overnight before Follo Stabilis Walker is applied. The charging should always be done in room temperature. It should always be charged when the warning sound appears (a beep sound). The battery should never be completely emptied. The battery will not be destroyed by too much charge.

NOTE: The walker should NOT be used with the charging device (A) connected.

The charging takes place via the battery pack (B) or via the controller (C). The charging unit (A) is inserted into a hole on the left side products. The unit is charged via a common wall outlet.


Technical specifications

	Cm	Inch
Height min / max gas	88,6 / 129,7	35 / 51
Height min / max el.	92,8 / 128	36 ½ / 50
Width	81	31 ¾
Length	79,5	31 ¼
Width lower frame	70	27 ½
Minimum turn with lower frame	100	39 ¼


	Gass	Elektrisk
Weight	14,7 kg / 32,6 lbs	20,6 kg / 45,7 lbs
Maximum user weight	150 kg / 333,3 lbs	150 kg / 333,3 lbs

Height gas:

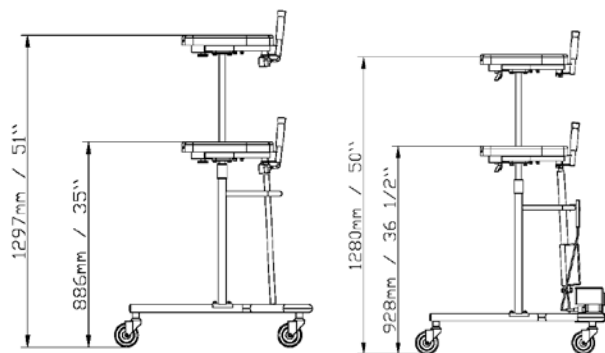Height El:

Width between cushions:

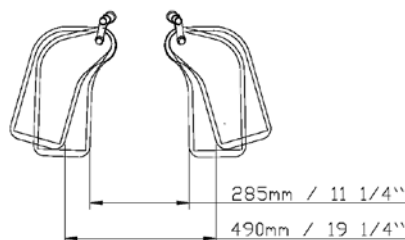Labelling

Serial no.:
MANUFACTURED BY
FOLLO FUTURA AS
P.B. 112-1430AS- NORWAY

**Maximal person
weight 150 kg.**

1. Manufacturer's name and address, serial number and max. load

 WWW.FOLLO-FUTURA.NO 

2. Web address.

3. CE label 4. Location of charging point 5. For indoor use and consult operator's manual

MAX LOAD 2,5KG

MAX LOAD 3,8KG

6. Max load on the basket. 7. Max load on the holder for oxygen bottles

3 4 2 - 2 0 - 1 - 1 1 I A N A R 0 -

8. Warning tilt hazard with use of infusion support, read instruction manual.

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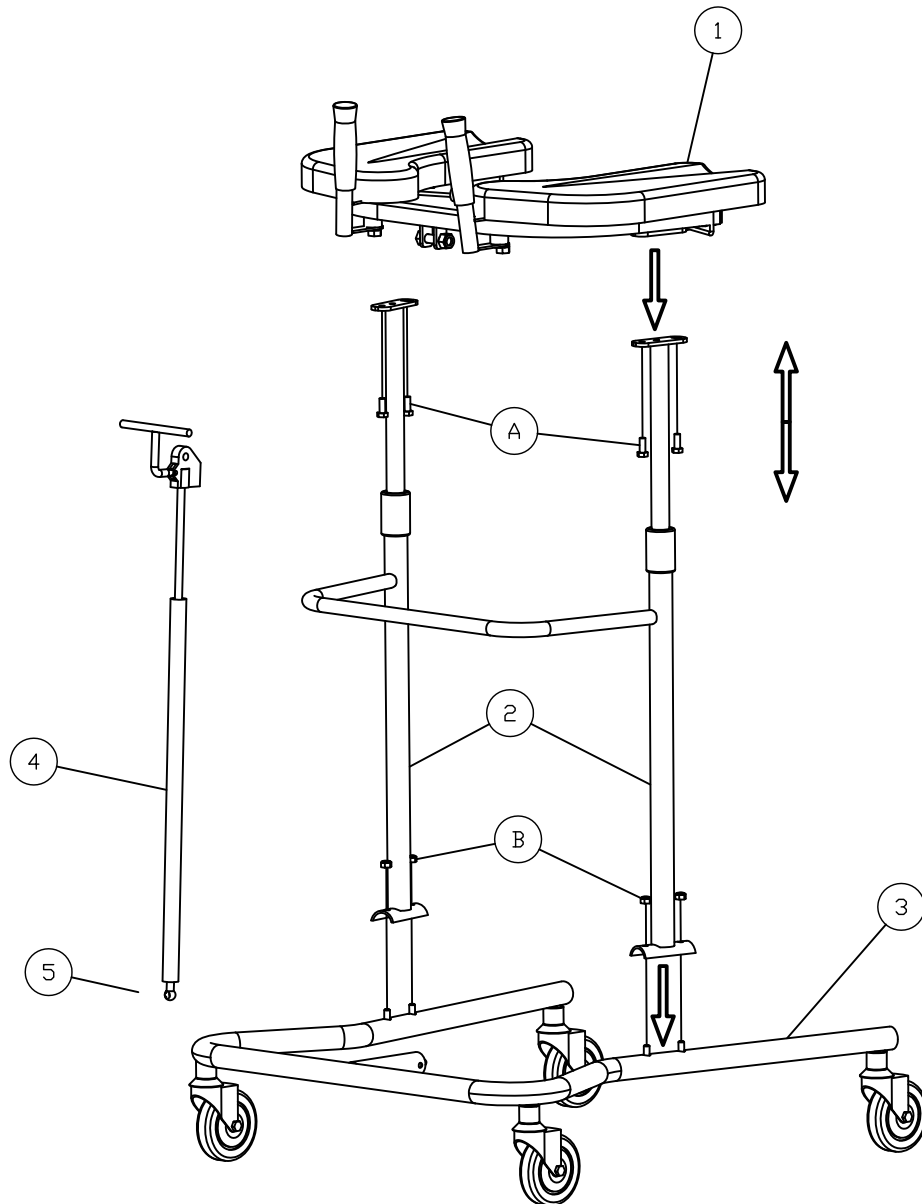
ASSEMBLY/DISASSEMBLY INSTRUCTIONS


ASSEMBLY:

1. Mount the top frame (1) to the rack (2). Screw the M8 screws (A) on, do not tighten.
2. Mount the top frame (1) and the rack (2) on the bottom frame (3). Screw the M8 nuts (B) tighten on.
3. Mount the gas / electric (4) lifting equipment on the Follo Stabilis Walker.
4. Lift the top frame (1) up and down to center the telescope tubes to the top frame. Tighten screws M8 (A) securely in.
5. Remember to lock the safety pin after you have mounted the gas spring on the lower frame.


DISASSEMBLY:

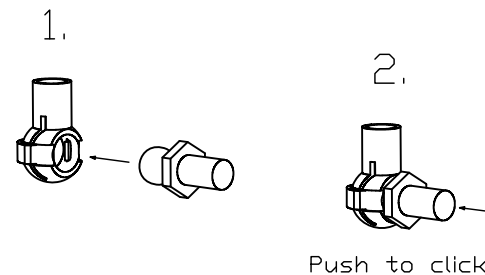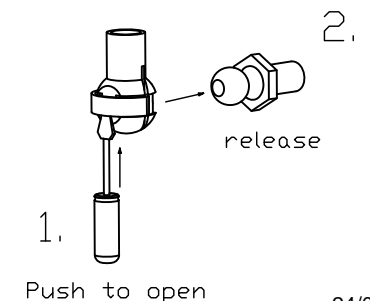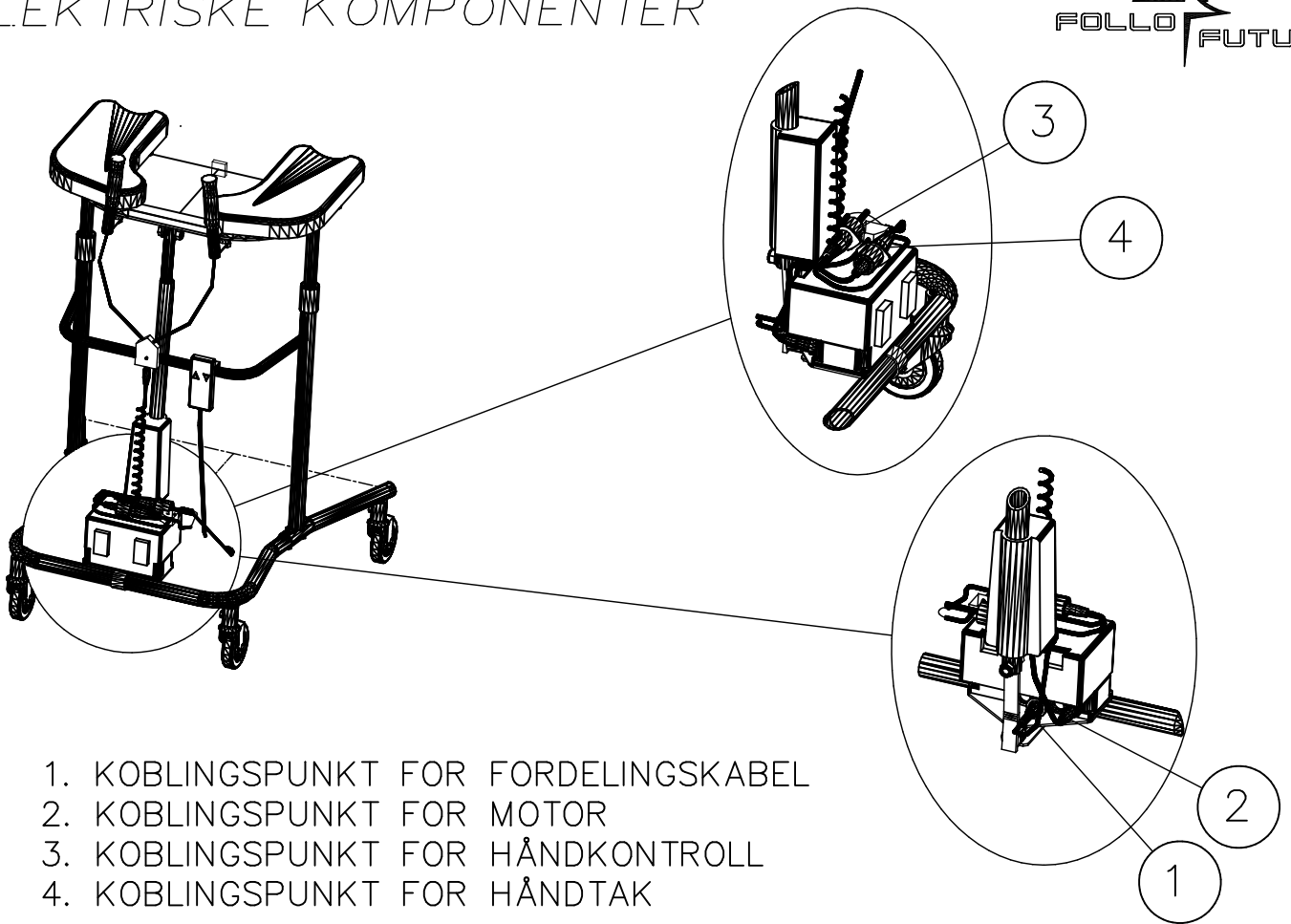
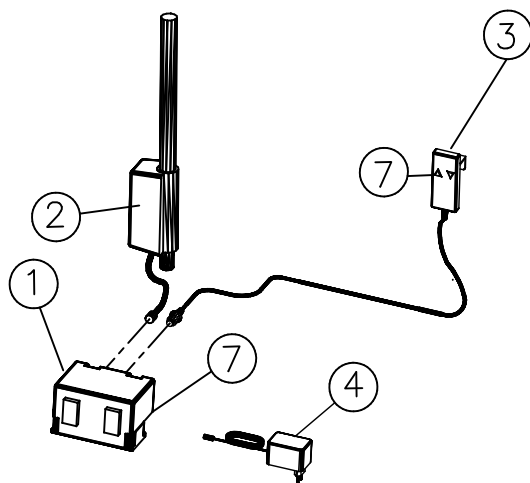
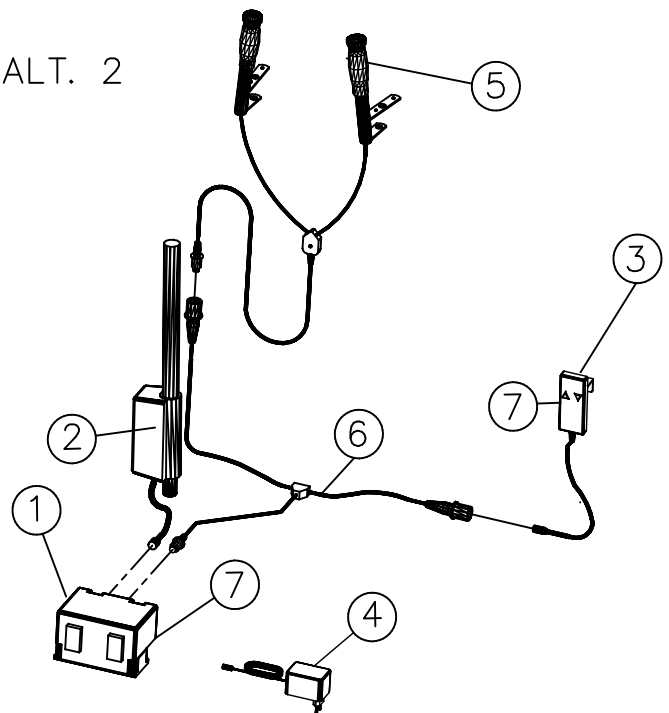
1. Disassemble the gas / electric lift mechanism.
2. Unscrew the top frame.
3. Unscrew the rack form the bottom frame.
4. Unscrew the wheels from the bottom frame.

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