Shower stool

Item No. 302070, 71, 72, 73, 78, 79 & 30207X

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione

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1. Introduction

Thank you for buying a high-quality Mobilex product. This user manual contains a description of the medical device and important guidance to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Indications

Shower stools are indicated as a seating device to be used under shower and in bathroom by elderly people, persons in wheelchairs and other people with limited mobility of the lumbar spine, knees or hips, and for all those who have difficulties performing everyday personal hygiene in standing position. Using the stool for purposes other than those indicated above, is prohibited.

Contraindications

Shower stool is not indicated for those patients who can't maintain proper body balance in a sitting and standing position. The stool must not be used if the user has problems with impaired perception. Contraindications cannot be related to one specific clinical picture and they depend on individual evaluation of potential user's health status. The choice of orthopedic aid appropriate for a given clinical picture must be always made with the help of a specialist physician or physiotherapist.

If you have any questions, please contact your dealer or Mobilex A/S.

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
--	--

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb on to the device
- The device must only be used on stable surface.
- When adjusting the device the user must take care not to squeeze the fingers or other body parts.

3. Description

Product description

The shower stool is designed for individuals who require seating support during showering or while bathing. It is made from aluminium tubing which is warm to the touch and corrosion resistant. No tools are required for assembly.

Parts supplied and mounting instructions

The shower stool is delivered in a carton with the following contents:

Parts supplied:

- ✓ 1 frame
- ✓ 4 legs
- ✓ 1 seat
- ✓ This user manual

Assembly of the stool:

The shower stool is delivered in a cardboard box in which the lower parts of the legs are included separately. To assemble the shower stool ready for use, insert the 4 footrests into the tubes provided and secure them at the desired height using the clips provided (see illustration below). Also check that the soft PUR seat is correctly seated on the frame, as it may have come loose from the stool during transport. It may be necessary to press the buttons on the underside of the PUR seat into the holes on the frame.

Note! Check that all 4 feet are set at the same height.

To check this, count the holes in the height adjustment where the clips are located.

Mounting of backrest (accessory, not included)

The backrest of the bath tub (accessory) is easily fitted by inserting the 4 locking clips provided into the corresponding tubes under the seat.

Set Up description

Adjusting the seat height (legs length)

The seat height of the shower stool should be adjusted to suit the user.

As a rule, the seat height is correctly adjusted when the user is sitting on the stool and both feet fully touch the ground.

1. Pull the locking clip (A) out of the stool's leg.
2. Adjust the telescopic leg tube to desired height.
3. Push the locking clip (A) in to the locking hole (B).
4. Repeat above steps for the other legs of the stool.

IMPORTANT! Don't adjust the seat of the stool when someone is sitting on it and always make sure all locking clips are mounted correctly.

Accessories

For the shower stool following accessories are available with article no. 302075 and 302076. Please contact your dealer.

Transport recommendations

The stool can be easily transported. Always assure that device during transportation is safely secured against unintended movements.

4. Technical Data:

Shower stool	Seat width 42 cm (302070+71)	Seat width 47 cm (302072+73)	Seat width 52 cm (302078+79)	Seat width 57 cm (30207X)
<i>Dimension (WxDxH)</i>	51,5 x 53,5 x 41,7-58,5 cm	56,5 x 53,5 x 41,7-58,5 cm	62,5 x 53,5 x 41,7-58,5 cm	68,5 x 53,5 x 41,7-58,5 cm
<i>Seat dim. (WxD)</i>	40,7 x 33,5 cm	45,2 x 33,5 cm	50 x 33,5 cm	50 x 33,5 cm
<i>Inner width</i>	42 cm	47 cm	52 cm	57 cm
<i>Outer width</i>	51,5 cm	56,5 cm	62,5 cm	68,5 cm
<i>Seat height</i>	41,7 - 58,5 cm	41,7 - 58,5 cm	41,7 - 58,5 cm	41,7 - 58,5 cm
<i>Armrest height</i>	60,4 - 77,1 cm.	60,4 - 77,1 cm.	60,4 - 77,1 cm.	60,4 - 77,1 cm
<i>Weight</i>	2,7 kg	2,9 kg	3,3 kg	3,86 kg
<i>Max. load</i>	150 kg	150 kg	200 kg	250 kg

Product label

The product label is located on main frame

Image of the product label shower stool 302070

5. Care & Maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives). Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional.

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary

Recommended service life: Normally 10 years depending on use.

Storage

We recommend storage in a dry place.

Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

6. Most common problems and solutions

Symptoms	possible cause	what to do?
Shower stool is not stable	Legs are adjusted to different heights	Adjust all legs to the same height






7. Disposal of the product

The device cannot be deposited with normal garbage. It must be delivered to the public station for re-use.

8. Guarantee terms

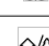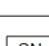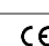Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly.

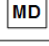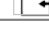The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Shower stool - item no. 302070-73+78-79+7X

ACCESSORIES LIST

Item no.	Accessories item no.
302070+72+78	302075
302071+73+79	302076
3020X7	302075

Harmonized norms used during conformity estimation:

PN-EN 12182:2012, PN-EN 581-2:2016-02, PN-EN 1022:2019, PN-EN ISO 14971:2012, PN-EN 1041:2009

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In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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Serial number:

SHOWER STOOL

1 The optional backrest, like the seat, is made of soft PUR and can easily be adjusted in depth.

2 Comes standard with a soft PUR seat that provides extra pressure relief and comfort.

3 During the development of the armrests, the focus was placed on ergonomics to make it easier for user to stand up from the stool.

4 The legs can be easily adjusted in height with clips and without tools. The pin of the clips is made of stainless steel.

5 Despite its high load-bearing capacity, the shower stool has a low dead weight and can be stacked easily.

6 The welded aluminium construction provides a lightweight and stable frame for safe use.