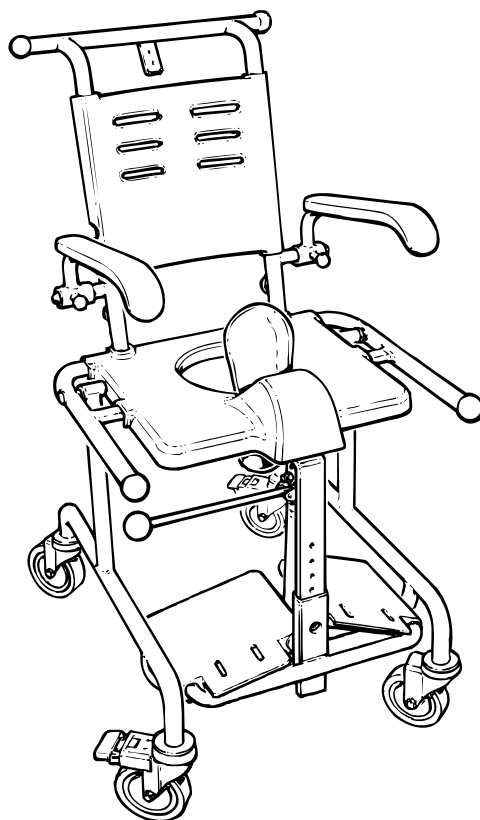
# Etac Tripp

Manual

etac®

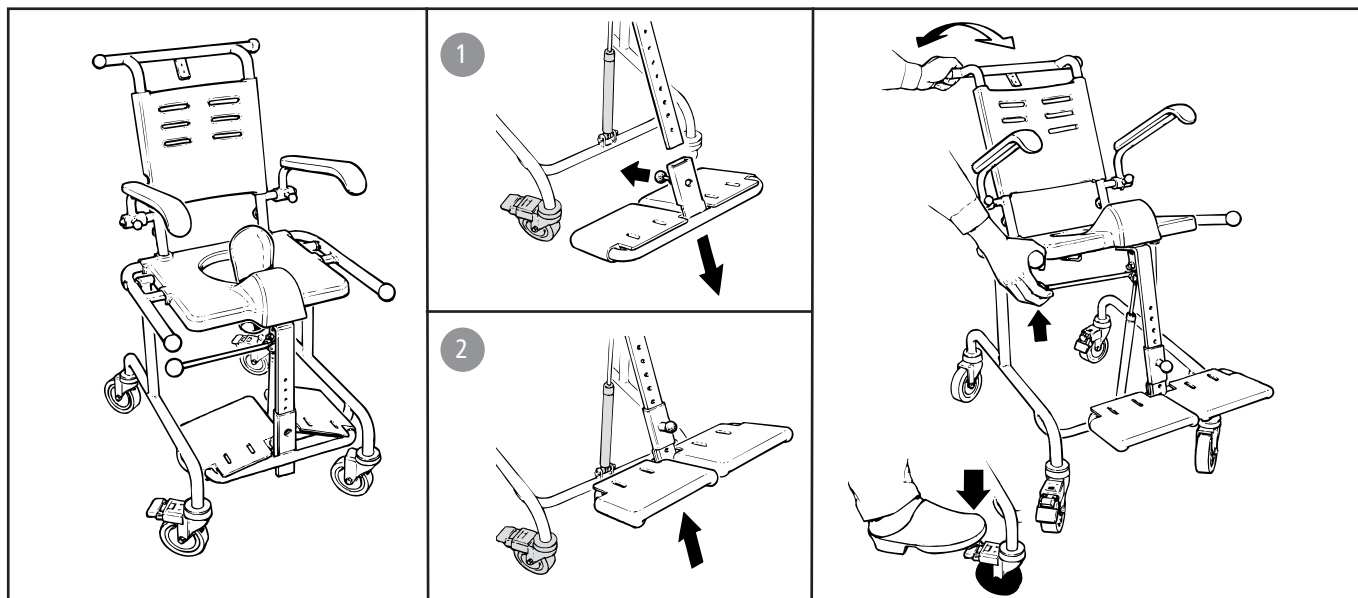
78056E 16-02-05

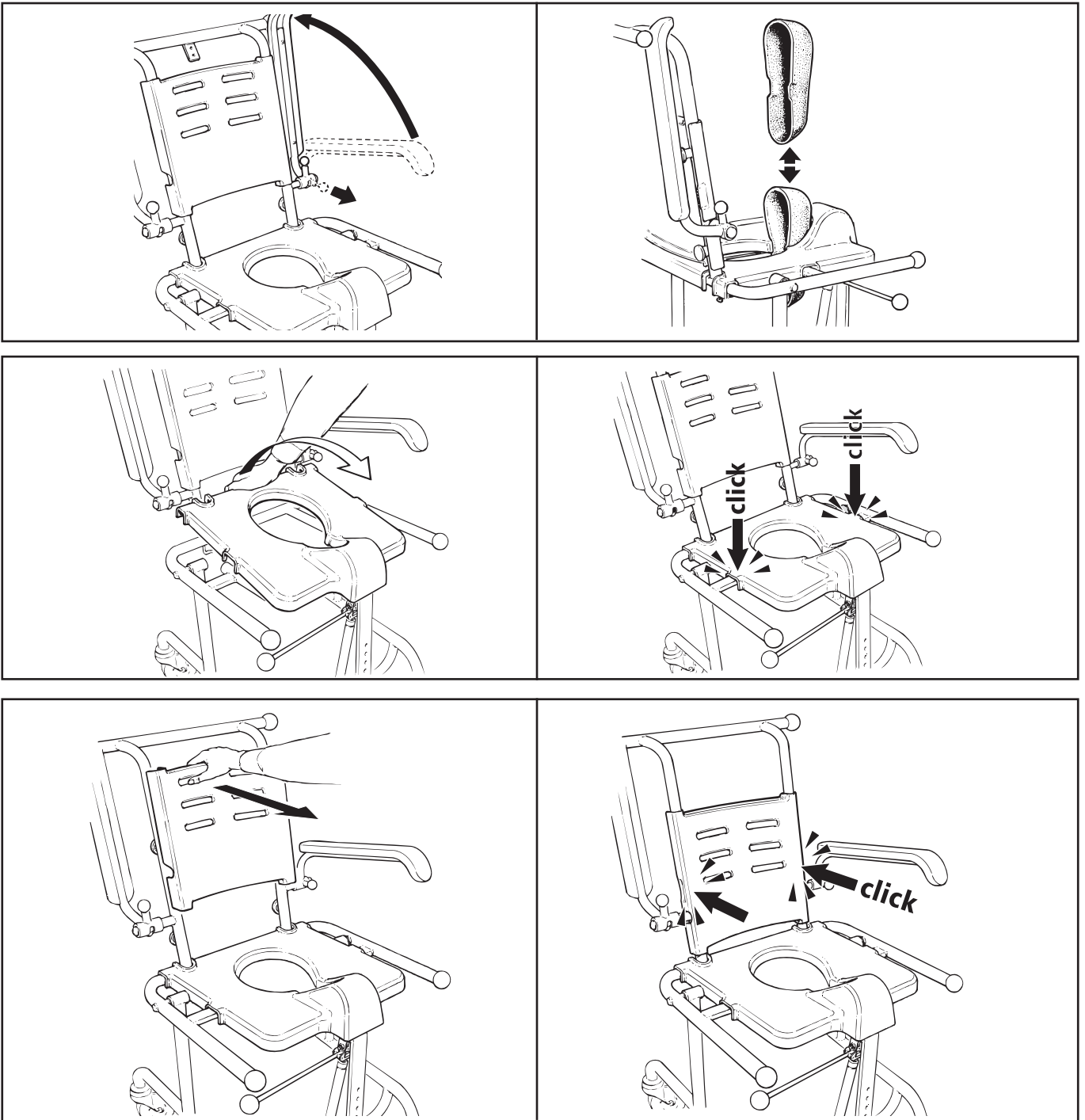
sv no da en de fi fr nl it es pt


A


**B**

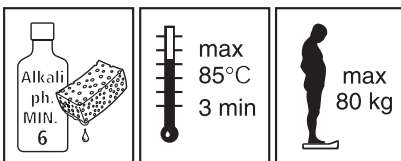**C**

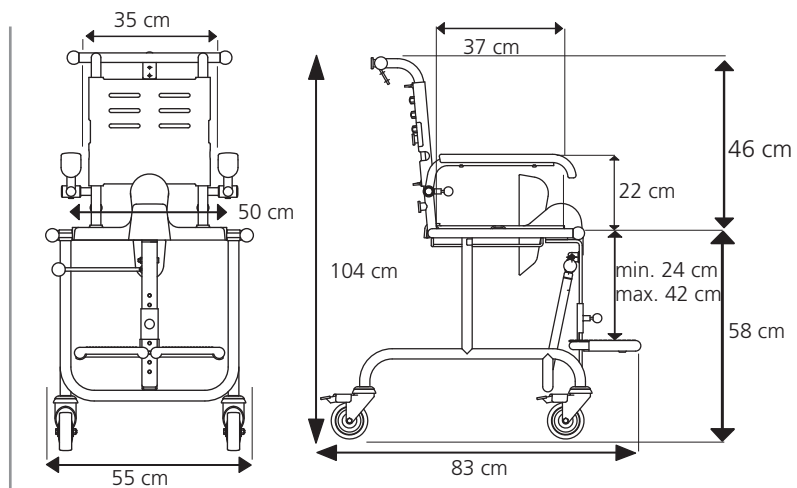17 kg

sv

Svenska

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English

**Intended use**

The Tripp is a mobile shower and toilet chair designed for use in facilitating intimate hygiene for persons with restricted mobility.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

**Installation, use..... Figur A**  
**Maintenance and safety..... Figur B**

1. Clean/disinfect the chair with normal non-abrasive cleaning agents with a pH value of between 5-9, or with 70% disinfectant solution. Rinse and dry.
2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for reconditioning.
3. Maximum user weight 80 kg.
4. Defective or incorrectly installed products must not be used. In case of damage, contact your dealer.
5. Heated chair can cause burns.

The materials used are corrosion-resistant.

The materials used are resistant to all general disinfectants.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

**Guarantee, service life**

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 7 years. For complete information regarding the service life of the product, see www.etac.com.

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**Installation, utilisation** ..... *Figure A*

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Etac A/S
R82, Inc.
Etac Supply Center AB