# system varioPLUS

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**User Manual**

**Mode d'emploi**

**Istruzioni d'uso**

**Manual de instrucciones**

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**Brugsanvisning**

**Bruksanvisning**

**Käyttöopas**

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
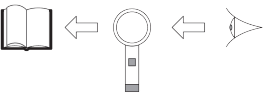
**ESCHENBACH**

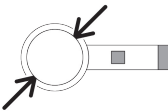				
<b>Ø 35 mm</b>	50,0 Dpt	12,5 x	<b>15 mm</b>	<b>40 mm</b>
<b>Ø 35 mm</b>	38,0 Dpt	10,0 x	<b>20 mm</b>	<b>40 mm</b>
<b>Ø 35 mm</b>	28,0 Dpt	7,0 x	<b>30 mm</b>	<b>60 mm</b>
<b>Ø 50 mm</b>	23,0 Dpt	6,0 x	<b>31 mm</b>	<b>100 mm</b>
<b>Ø 58 mm</b>	20,0 Dpt	5,0 x	<b>36 mm</b>	<b>140 mm</b>
<b>Ø 70 mm</b>	16,0 Dpt	4,0 x	<b>45 mm</b>	<b>150 mm</b>
<b>Ø 80 mm</b>	12,0 Dpt	3,0 x	<b>49 mm</b>	<b>250 mm</b>
<b>100 x 50 mm</b>	11,4 Dpt	3,9 x	<b>46 mm</b>	<b>200 mm</b>
<b>100 x 50 mm</b>	7,6 Dpt	3,0 x	<b>59 mm</b>	<b>250 mm</b>
<b>100 x 75 mm</b>	7,0 Dpt	2,8 x	<b>61 mm</b>	<b>250 mm</b>

Deutsch .....	4
English.....	6
Français.....	9
Italiano.....	12
Español.....	14
Nederlands.....	17
Dansk .....	20
Svenska.....	22
Norsk.....	24
Suomi.....	27
Polski.....	29
Česky .....	32
日本語.....	34




**159910:**

**1545:**  
2,5V 250 mA

**1599144:**

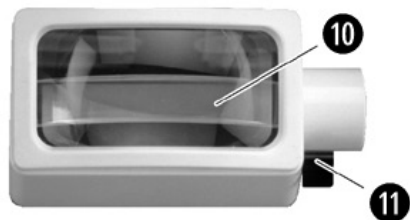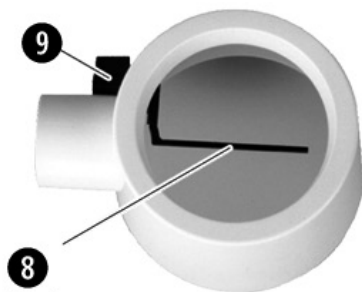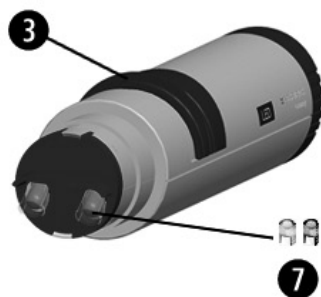## Deutsch

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## English

You have purchased a quality product from the Eschenbach company that was "Made in Germany" using state-of-the-art manufacturing processes. We congratulate you on your decision.

For the illuminated magnifier system vario plus, various magnifications and different lighting options are available. The illuminated magnifier acquired by you has been specifically assembled with this system according to your visual requirements. It may therefore differ from the illustration.

Should there be changes in your visual requirement, the lighting and magnification can be variably adjusted by changing the magnifier handle and magnifier pot.

The lens of your illuminated magnifier is made of high quality, unbreakable PXM<sup>®</sup> material, with a cera-tec<sup>®</sup> hard coating protecting it against scratches - scratch resistance is a crucial criterion for quality and durability.

## Safety Instructions

- ▶ **Danger of glare and injury! Never look directly into the sun or other bright light sources using optical devices! 1599144: Never look directly into the light source.**
- ▶ **Danger of fire! Lenses in optical devices can cause significant damage by focusing light to generate heat if handled or stored improperly. Be careful never to leave optical lenses lying in direct sunlight!**
- ▶ **Protect your magnifying light from impacts, moisture and excess heat. Never lay your magnifying light on a radiator or in direct sunlight.**
- ▶ **Inform other persons, particularly children, of these dangers.**

## Notes on usage

- ▶ **When the lens is not standing horizontal, pull the foot ❶ from the handle until it engages.**
- ▶ **If the foot bothers you, press the snap-in pin ❷ and slide the foot into the handle.**
- ▶ **Switch the illuminated magnifier on by sliding the switch ❸ in the direction of the lens.**
- ▶ **Now place the illuminated magnifier on the text to be read.**
- ▶ **Guide the illuminated magnifier with the handle ❹ over the text to be read.**
- ▶ **After use, switch the illuminated magnifier off by sliding the switch ❸ to the rear.**

1599144: If you do not like the colour of the light, your optician can adjust the light colour with the supplied coloured LED-Caps.

## Lamp bulb replacement

Models with bulbs only (159910):

- ▶ **Switch off the illuminated magnifier by pushing the switch [3] back towards the end.**
- ▶ **Pull the handle (159910) out of the lens barrel.**
- ▶ **Change the defective bulb for a new one.**
- ▶ **Only use bulbs specified (2,5 V, 250 mA).**
- ▶ **Insert the handle straight into the lens barrel. Ensure that the support [1] is pointing downwards.**

## Changing batteries

 All of the batteries (2 alkali-manganese batteries, C-size) should always be replaced. Pay attention to the correct polarity (positive pole in the direction of the light).

- ▶ **Switch the illuminated magnifier off by sliding the switch ③ to the rear.**
- ▶ **Turn the grey end cap ⑥ anticlockwise and then remove the end cap.**
- ▶ **Replace the old batteries ⑤ with new ones.**
- ▶ **Screw the grey end cap back into the handle.**
- ▶ **Insert the handle straight into the magnifier pot.**  
Ensure that the slot in the rear end cap points down.

## Magnifier pots with valuable extra function

### Pivoting reading line ⑧ as an orientation guide

The following magnifier pots have this function:

∅ 80, 12 Diopter, 3.0 x	∅ 70, 16 Diopter, 4.0 x	∅ 58, 20 Diopter, 5.0 x
-------------------------	-------------------------	-------------------------

- ▶ **Turn the lever ⑨ to swing in the orientation guide.**

### Pivotable additional magnification ⑩ of 4 Diopter.

To read small print or to highlight difficult passages of text.

The following magnifier pots have this function:

100x50, 7.6 Diopter, 3.0 x	100x75, 7.0 Diopter, 2.8 x
----------------------------	----------------------------

- ▶ **Turn the lever ⑩ to swing in the additional magnification.**

## Care Instructions

 When the illuminated magnifier is not in use, it should be covered.

Clean the housing with a moist cloth. Clean the lens with a soft, lint-free cloth (e.g. eyeglass cleaning cloth). For heavier soiling (e.g. fingerprints), moisten the cloth slightly.

**Do NOT use any soap solutions that contain softening agents, alcohol-based solvents or abrasive cleansers! The lenses could be damaged.**

## Warranty

Within the framework of the applicable statutory provisions, we provide a warranty for the functioning of the product described in this manual with regard to faults arising in connection with manufacturing errors or material errors. In event of damage due to improper handling, including damage from a fall or impact, no warranty claim exists. Warranty claims are only accepted upon presentation of the sale receipt!

## Disposal

When the device reaches the end of its useful life, do not dispose of it with normal household waste. This product is subject to European directive 2002/96/EC.

Dispose of the product or parts of the product through a certified disposal agency or your municipal disposal agency. Observe the currently applicable regulations. In case of doubt, contact your disposal agency. Dispose of all packaging materials in an environmentally friendly manner. Batteries and cells must not be disposed of in household waste! Provide for proper disposal.

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Eschenbach Optik GmbH