- Ⓓ **Montageanleitung**
- ⒼⒷ **Installation instructions**
- Ⓕ **Montage**
- Ⓝ **Montage van**
- Ⓘ **Istruzioni di montaggio**
- Ⓔ **Montaje instrucciones**
- Ⓟ **Instrukcja montażu**

Artikel-Nr.  
item number  
No de référéce  
Type Nr.  
Nr. Articolo  
N° de artículo  
Numer produktu

4


Warnhinweise  
Warning  
Mises en garde  
Let op  
Avvertenze importanti  
Advertencias  
Ostrzeżenia

5 - 11

Zweckbestimmung  
proper use  
affectation  
toepassingen  
conformità di norma  
Usò previsto  
Przeznaczenie

12 - 13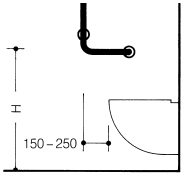

Montagehinweis  
Mounting information  
Conseil de montage  
Montage aanwijzing  
Indicazioni per il montaggio  
Indicaciones de montaje  
Wskazówka dotycząca montażu

14 - 15

Wartung  
Maintenance  
Entretien  
Onderhoud  
Manutenzione  
Mantenimiento  
Konserwacja

16 - 19

Pflegehinweise  
Care tips  
Conseils d'entretien  
Onderhoudsinstructies  
Consigli per la pulizia  
Indicaciones para el mantenimiento  
Pielęgnacja

20 - 23

CE 801.22.100 801.22.110 CE 802.22.100 802.22.110 801.22.1S 802.22.1S

801.22.300 802.22.300

CE

801.22.200 802.22.200

801.22.120 802.22.120

CE

CE

## Warnhinweise!

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## **WARNING!**

### **Use of HEWI L-shaped support**

The HEWI support is designed to withstand a maximum static working load of 150 kg. Do not use this support for weights in excess of this. Any non-compliance with loading limits can cause permanent damage to the support or the joint. In such cases, the support must be examined by a specialised company prior to the support being used again. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

Only use the HEWI support in the manner described in the instructions. Any improper use e.g. as a gymnastic apparatus or climbing aid can cause an accident with persons potentially suffering serious injury.

Assuming normal use, the support is of a flexible nature thanks to the materials used in its manufacture. This flexibility reduces the risk of injury if anyone bumps into the support. The fact that the L-shaped support bends to a certain extent under load is normal. Should however the L-shaped support become permanently deformed, you should immediately contact the company appointed to carry out maintenance work or HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300.

### **Installation of HEWI L-shaped support**

The smooth and safe functioning of the HEWI L-shaped support largely depends on its correct fixing to the given wall, which is why this work should only be entrusted to a licensed contractor (sanitary fitter). Serious impairment to the support's proper functioning can occur if its installation is not carried out professionally, potentially resulting in accidents with persons suffering serious injury. This is why we advise you not to install the L-shaped support yourself.

When the licensed contractor mounts the HEWI support, he should ensure that the entire area of the wall to which the wall mounting plate is to be fitted is flat (no projecting joints or tiles sticking out), that the structure of the wall is suitable for the installation of the L-shaped support and has no weak points. Furthermore, the choice of the right fixing material kit for the job is important; HEWI offers various types to suit the nature of the wall concerned. If in doubt, contact HEWI's after-sales service team (sanitary) on tel. +49 5691 82-300 for further assistance.

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(GB)

#### **Proper use in accordance with Directive 93/42/EWG**

HEWI L-shaped support rails are designed to be used in bathrooms and toilets.

They are particularly useful for patients in hospitals or persons with a physical disability who can hold on to them when they get into a bathtub or sit down on a toilet or on some other seat.

They act as a support particularly for patients in hospitals or persons with a physical disability who can hold on to them when they get out of a bathtub or get up from the toilet or from some other seat. The toilet roll holder is an optional extra that can be fitted to the support rail to accommodate a toilet roll and thus make it more easily accessible particularly for hospital patients or persons with a physical disability.

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### **Wartung**

The 'L-shaped' support rail in question is a medical product. It was therefore developed and produced in compliance with specific criteria (basic requirements as per 93/42 EWG).

Maintenance of the 'L-shaped' support rail or of the system as a whole should be performed on a 6-monthly basis and should encompass the following key points:

1. Inspection of fixing

The 'L-shaped' support rail should be mounted firmly at all fixing points and should fit tightly to the surface to which it is fixed. If necessary, the rose caps should be removed and the screws used to fix the rail in place tightened up. Any defects identified during inspection should only be remedied by persons qualified to do so.

2. Inspection of surface finish of 'L-shaped' support rail

3. The surface finish of the rail should have no fractures or other damage that could cause personal injury.

There should be no gaps at the joints between the straight elements and the bend elements.

### **Remaining risk**

The 'L-shaped' support rail was designed in compliance with specific criteria (see above) for use as a medical product. Incorrect usage or unprofessional installation of the support rail system can cause serious personal injury.

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### **Care tips**

HEWI products are easy-care. Generally speaking, an occasional wipe with a damp cloth is enough. If, however, you feel you have to use detergents, you should take note of the following information. Only use detergents with a pH-rating will do between 6 and 8. Detergents containing active substances such as phosphates, soap and tensides may be used. Detergents containing acids, alkalis, bleach or scouring agents may not be used. Nor should utensils with a scouring effect such as brushes etc. be used. Usage of any detergent requires full compliance with the manufacturer's instructions (concentration, soaking time etc.).

If this is not done, the surface finishes of the HEWI products may suffer as a result and sustain damage. HEWI accepts no liability whatsoever for damage occurring from incorrect handling

**For further information please contact  
HEWI Technical Support, tel. +49 5691 82-300.**

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HEWI Heinrich Wilke GmbH