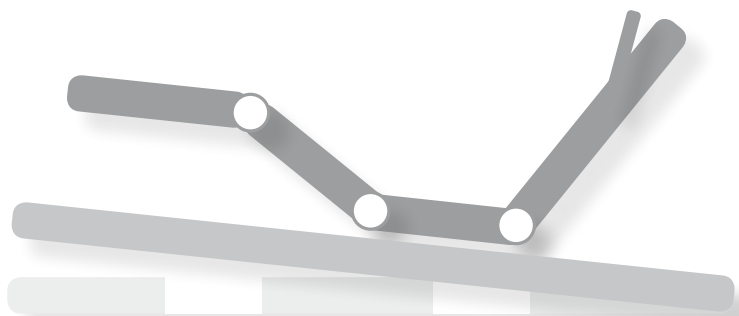
# Motorrahmen

Sitting frames / Sommier motorisé

Lattoflex 292 - 295

Lattoflex 392 - 395

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Installation and operating instructions

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**english**

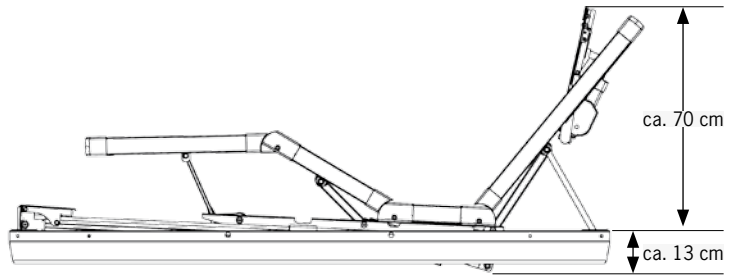
Important document. Please keep in a safe place.

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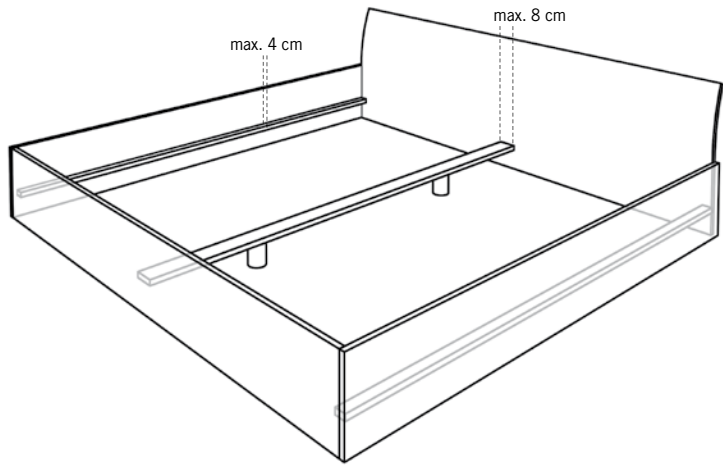
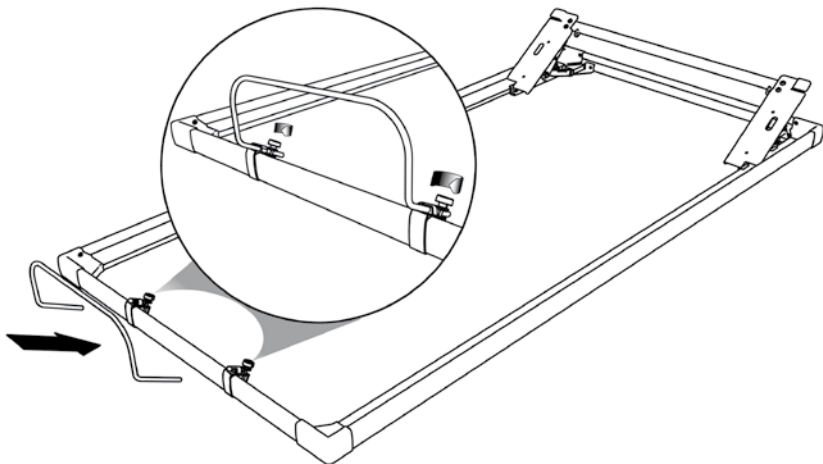
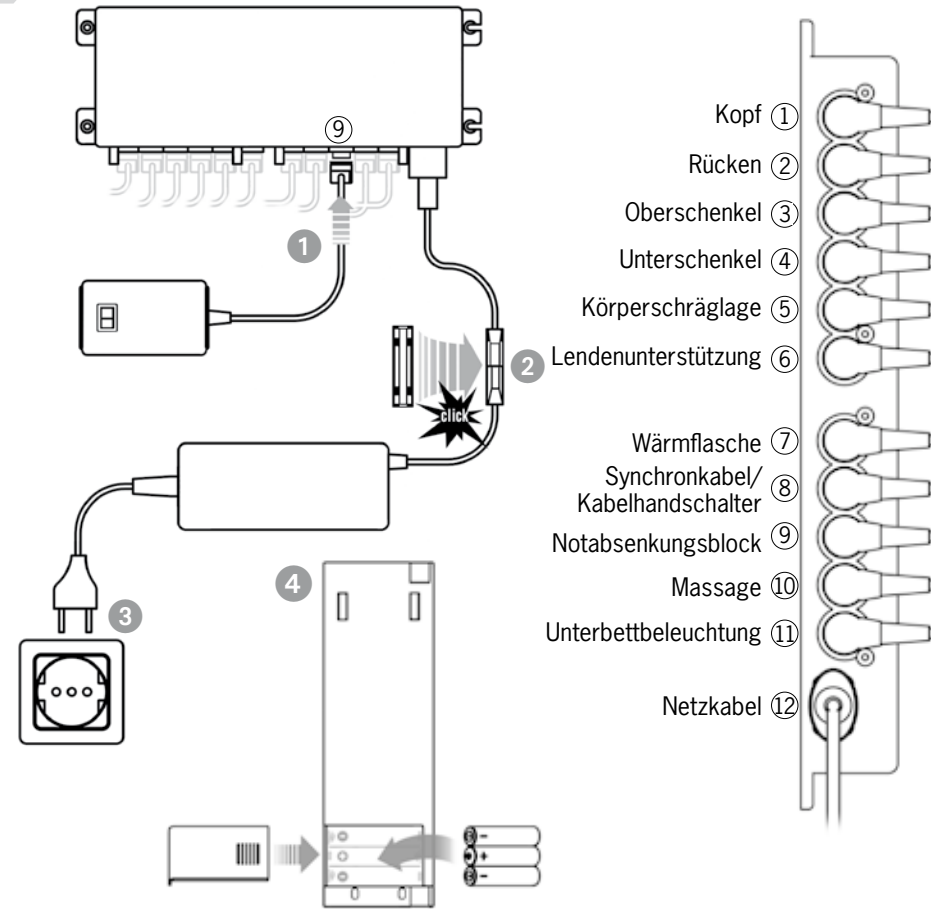
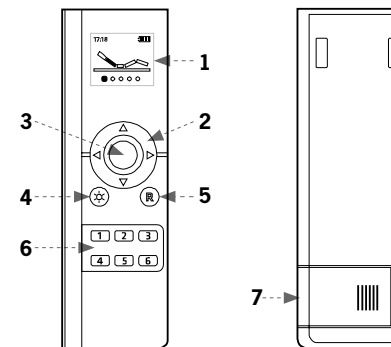
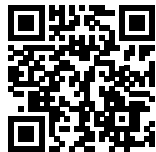
Have you got a smart  
phone? Please use the free  
Lattoflex app to control  
your motor frame - even  
throughout the house as  
long as you are in the  
Wi-Fi network. Load the  
Lattoflex app for iPhone or  
iPad from itunes, or from  
GooglePlay for android  
devices. Simply follow the  
QR code:

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You will need the Smart  
Remote Wi-Fi adapter to  
connect with the motor  
frame (available as  
accessory).

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## Contents

Installation and connection .....	10
The Lattoflex remote control .....	11
Adjusting and saving the settings.....	12
Emergency lowering .....	12
Safety instructions.....	12
Care instructions .....	12
Settings menu.....	13
Troubleshooting .....	14 - 15
Technical data .....	23

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## Installation instructions

### Frame type and serial number

The name plate is located at the head end on the left longitudinal beam below the raised head.

### Supplied accessories

**Installation accessories:** mattress holder (foot end), 4 non-slip foils

**Connection accessories:** radio remote control with batteries (3 x AAA), felt pocket, emergency lowering block, mains adapter, lead cable, plug fuse

### 1 Placing in a bed

Please heed the necessary space for the adjustable frame parts, e.g. in rooms with inclined roofs, superstructures over the bed or padded head sections. An installation depth of approx. 13 cm (floor clearance) is needed in the substructure for the adjustment mechanism and drive system.

Please always heed the spacing of the back part with its motorised adjustment, to the head part of the bed. There should be spacing of at least 1.5 cm here when the bed is laid flat. To rule out any unintentional slipping, please screw the frame to the bed at the pre-drilled points in the metal substructure.

### 2 Side support sections must not exceed 4 cm in width (middle frame 8 cm).

For beds with triangular supports (leg length max. 15 cm), please use the „Extension elements“ installation kit (available as accessory). For a stable bed structure, we always recommend a supporting point in the middle of the bed for triangular supports.

### Non-slip foil

Affix in the bed structure to the supporting point of the frame. The foils prevent any noise that could be caused by the frame slipping in the bed structure.

### 3 Mattress holder

Attach the supplied holder at the foot end. The plastic clamps are premounted.

## Connection and commissioning

### 4 Please proceed as follows to commission your motor frame:

- 1 Connect emergency lowering block to jack 9 of the control. Connect up all other accessory parts as appropriate.
- 2 Connect the mains adapter to the control and clip the plug fuse in position.
- 3 Connect the motor frame to the mains.
- 4 Unpack the remote control and insert the batteries

### Remote control teach-in:

On delivery, once the batteries have been positioned the remote control is automatically in the menu to proceed with teach-in for a motor frame.

1. Trigger the search mode by pressing the middle button in the control cross.
2. Within 30 seconds, briefly press the button on the emergency lowering block.
3. The remote control shows „Teach-In Ok“ to confirm that the teach-in was successful; the control in the motor frame beeps.
4. Briefly press button R on the remote control. The display tells you to reset the unit:

5. To conclude the teach-in procedure, now press button R until the picture of the motor frame appears on the display.

## The Lattoflex remote control

### Bi-directional radio connection

Your motor frame has been delivered with an intelligent radio remote control. There is a bi-directional i.e. two-sided radio connection between the remote control and the control unit in the motor frame. This offers you two crucial advantages:

- The display only shows those motors and accessories that are actually installed in your frame. Components connected up later on are automatically added to the remote control menu after a corresponding teach-in session.
- At all times, the display shows the exact position of all adjustable parts and the function status of all accessories.

### 5 Overview

- 1 - Display
- 2 - Control cross
- 3 - Menu change button
- 4 - Torch
- 5 - Reset
- 6 - Comfort buttons for favourite positions
- 7 - Battery compartment

### Changing the batteries

Make sure that you have spare batteries ready as soon as there is only one red bar in the battery display. You will need 3 batteries or 3 rechargeable batteries, size AAA. Simply push open the cover of the battery compartment on the back of the unit. Make sure you insert the batteries with the positive and negative ends pointing in the correct direction.

### Motor frame control quick start

The remote control switches on automatically when moved or when a button is pressed. If no adjustment is made for 10 seconds or if the remote control is not moved, it switches to the sleep mode.

The 6 comfort buttons are preset on delivery. This lets you quickly enjoy the full range of functions. The presets can be changed at any time and replaced by your own favourite positions (see page 6, save settings). For motor adjustment, press the corresponding number button until the frame has reached its end position.

Button 1 = reading and sitting position

Button 2 = raised legs

Button 3 = relax position

Button 4 = light on/off (stage 3 / 10 min), optional

Button 5 = socket on/off, optional

Button 6 = electrical hot-water bottle on/off (stage 3 / 15 min), optional

### 6 Menu overview

The Lattoflex remote control combines control of the motor adjustments and all installed Lattoflex components in one single control unit. The various levels shown in Fig. 6 appear on the remote control, depending on the installed drives and function components.

### Change between the levels:

by repeatedly pressing the menu change button.

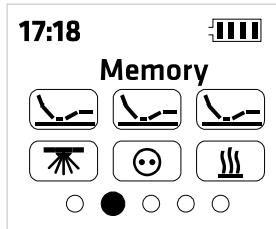
**Return to the motor adjustment level:** by briefly pressing the R button (Reset).

The remote control switches on when moved or when a button is pressed. The Lattoflex logo always appears first, followed by the motor adjustment.

Underbed lighting, socket receiver and hot-water bottle are available as accessories. Please contact your Lattoflex dealer.

### Adjusting settings and saving them if necessary


The menu change button runs in a cycle through the various functions and the settings menu. On reaching the required function, simply select the element to be adjusted with the „right“ or „left“ button. Then adjust the setting with the „up“ and „down“ buttons. You can go to the „memory“ view in every menu level simply by pressing the „right“ or „left“ button.

Here the settings that have just been adjusted can be saved as your favourite positions. To do so, press and hold one of the six comfort buttons

for two seconds in the memory mode. The selected button flashes on the display to confirm that the setting has been saved.

### Example: frame adjustment

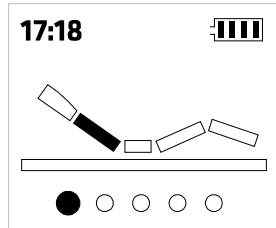Use the „right“ and „left“ arrow buttons to select the frame part to be moved. The selected element is shown in yellow. Use the „up“ and „down“

arrow buttons to adjust the frame part. Hold the buttons until the motors have reached the required position. Press and hold the R button for all motors to move to the flat position at the same time.

### Emergency lowering

In the event of a power failure, the motors can be adjusted to the flat position with the button on the emergency lowering block. In

this case, the motors move very slowly and one after the other. Please note that the emergency lowering function almost completely discharges the batteries so that they must be replaced afterwards (2 x 9 V E block).

### Safety instructions

Do not climb on the frame when it is being adjusted and when it is raised. The frame is designed so that it normally poses no risk of getting clamped or crushed during operation. Even so, please make sure that there is nobody, in particular no child, under the frame or in a position to reach into the mechanism when raising or lowering the frame. Lattoflex 292-295, 392-395 and all equivalent models are inlay frames for bed structures. They are only intended for use according to the conditions stated in these instructions and only for domestic use.

Make sure that the cables are never clamped, placed over sharp edges or crushed between moving parts of the frame. Only use original parts to replace damaged connection leads. Shortening the support beams or dismantling attachment parts poses a risk of damage. Do not store any items under the frame. Bedding in particular can be damaged by the motors.

### Cleaning and care

Your Lattoflex spring base does not need any special care. Simply wipe it down from time to time with a damp cloth. Please do not use any solvents or benzene, or any caustic or scouring cleaning agents! Always take the mains plug out of the socket before cleaning. Use silicone or preferably Flonium to treat any noises coming from the running surfaces of the sliding function or plastic bearings. Never use commercially available oil or grease.

## The settings menu in the remote control

7 All functions and parameters of the remote control can be adjusted and adapted as required. When reading these instructions, please refer to the illustrations on the rear fold-out page.

### Fig. 1 and 2:

The brightness of the display and keyboard lighting can be adjusted in 6 levels.

### Fig. 3:

Adjust the time and date. To go to the settings mode, please press the menu change button here, and again after adjusting the settings. Use the 4 arrow buttons to adjust all settings. Then press the menu change button again.

### Fig. 4:

Child lock. To enable and disable, press and hold the menu change button for at least 10 seconds. The lock symbol shows the status in the display.

### Fig. 5:

Teach-in/out: after pressing the menu change button, use the „right“ or „left“ arrow buttons to select the following functions: Search (teach new device), Remove (remove a device after teach-in) or Exit (to quit the teach-in/out area). Select the required function with the menu change button and follow the instructions given in the display.

Teach-in at the motor frame: see page 4.

Teach-in at the socket receiver: select Search with the menu change button, adjust the coding switch on the socket receiver to POS B (teach-in), select the required channel at the receiver and press the menu change button to confirm. The next free channel is chosen as default (max. 6). Set the coding switch on the socket receiver back to POS A (operation). This concludes the commissioning procedure. Please also refer to the separate instructions.

### Fig. 6:

To reset the remote control to the factory settings, please proceed as follows: press the menu change button and confirm the next query with Yes. The teach-in procedure now has to be repeated for all devices, see page 4, remote control teach-in.

### Without illustration: test

This point does not contain any relevant functions.

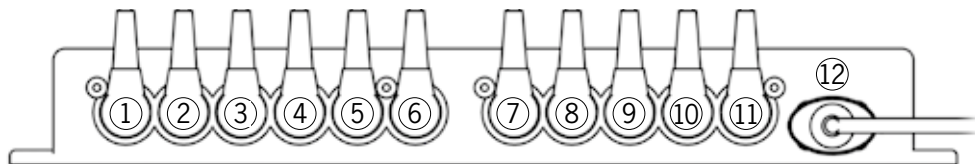
### Fig. 7:

Info: this shows the software status and serial number. You should have this information at the ready when making any technical queries.

## Troubleshooting

### Jack assignment

- |                          |   |
|--------------------------|---|
| ① Head                   | ⑦ Hot-water bottle                        |
| ② Back                   | ⑧ Synchronous cable/<br>cable hand switch |
| ③ Thigh                  | ⑨ Emergency lowering block                |
| ④ Calf                   | ⑩ Massage                                 |
| ⑤ Inclined body position | ⑪ Underbed lighting                       |
| ⑥ Lumbar support         | ⑫ Mains lead                              |

### Frame does not react

Cause	Remedy
Mains plug disconnected/ mains lead interrupted.	Connect mains plug / check plug connections on the mains adapter.
Socket without power.	Check house fuse.
Remote control mixed up. Remote control no longer in teach-in status.	Repeat teach-in process for remote control.
Control overheated because of continuous operation. Beep sounds when activated.	Leave to cool down for approx. 20 min.

### Remote control does not react, no display or keyboard lighting

Cause	Remedy
Batteries flat	Change batteries, 3 x AAA.
Batteries incorrectly inserted.	Remove batteries, insert correctly. Check + and -. See marking in housing.
Remote control disabled.	Remove a battery and insert it again.
Remote control defective.	Replace remote control.

### One of the motors is not working

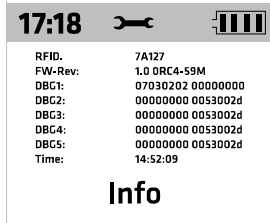
Cause	Remedy
The motor cannot be selected on the remote control	Repeat teach-in process for remote control.
Plug not inserted correctly in jack / discon- nected.	Insert plug <b>as far as it will go</b> .
One of the motors has overheated because of continuous operation.	Leave to cool down for approx. 20 min.
One of the jacks on the control unit is defective.	Connect motor to another jack. See jack assignment on the left. Use the remote control to select the corresponding frame part and test the motor.
Motor defective.	Replace the motor.

### Emergency lowering does not react

Cause	Remedy
Emergency lowering block plug not inserted correctly in the jack	Insert plug in jack 9 <b>as far as it will go</b> .
Batteries in emergency lowering block are flat.	Replace batteries, 2 x 9V block (6LR61)

### No lowering of back and/or thigh motor

Cause	Remedy
Frame collides with resistance and is swit- ched off by the fuse.	Raise motors and remove resistance

If you cannot solve the problem, please contact your Lattoflex dealer. If the remote control is still working, it can help to send the system data to Lattoflex. These can be found on the „Info“ display in the „Settings“ menu. If possible, send a photo of the screen content. This information will be helpful in making a quick analysis of the problem.

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## Technical Data

The motors work with safe low voltage of max. 29V DC.	
Mains connection	100V – 230V, 50Hz to 60Hz (single phase alternating current), max. 2.0 A
Switch-on period	2 min ON at maximum continuous load, 18 min OFF
Protection class	IP20
Emergency lowering	2 x 9V E-block batteries
Standby	approx. 0.8 W
Possible connections	6 motors, electrical hot water bottle, massage system, LED underbed lighting, cable hand switch, synchronous cable, emergency lowering block

## Radio remote control

Power supply	3 x 1.5V AAA batteries or 1.2V rechargeable batteries
Radio frequency	2.4 GHz, bidirectional connection as per EN 30440-1 V1.6.1 und EN300440-2 V1.4.1, transmitting power <1mW
LED torch	0.1 W
Operable receiver	motor frame, max. 6-socket receiver
The products are fitted with a radio transmitter and may only be operated within the scope of EN 300440. Regional regulations must be heeded where applicable.	

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Thomas GmbH+Co. Sitz- und Liegemöbel KG · Postfach 1464 · D-27424 Bremervörde