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**GB Mobilex A/S****Shower Board 302030**  
User's manual

The shower board is a medical device which intended use is to give stable & easy access and seating support when disabled persons use a bathtub. It is capable of being adjusted to suit the majority of bathtub widths.

**Security issues:** Only use the product for the purpose it is intended for. Don't climb on the product. Use the product only in good technical condition. Do not overload the product. Use the product only on flat, smooth bath walls. Always check stability of product before use. Make sure the product is adjusted to the correct width before use. **WARNING! Be careful not to trap your fingers in one of the board's holes or slots.**

**Assembly and use:** The handle must be fitted into the 2 locating holes at one end of the board. The screws are fixed to the angle from the underside of the board. Locate & secure the individual posts into the required positions at 4 long slots on the board to fit your bathtub width. The posts are fixed by the screws from underside of the board. Please ensure there is no sideways movement of the seat & that the post surface is in contact with the bath walls from inside. Both ends of the seat must be fully supported by the bath sides.

**Technical data:**

Seat dim: 32 x 70,5 cm  
Max user's weight: 190 kg  
Weight: 2,7 kg

**Cleaning & Maintenance:** Clean the device regularly & keep it clean. Use soft cloth, warm water & neutral household washing agents. Do not use abrasives & bleach based cleaners.

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CE Mobilex A/S, Nørskovvej 1
**D Mobilex A/S****Badewannenbrett 302030**
**I Mobilex A/S****Asse da vasca 302030**