HOGGI[®]

BINGO Evolution Brugermanual

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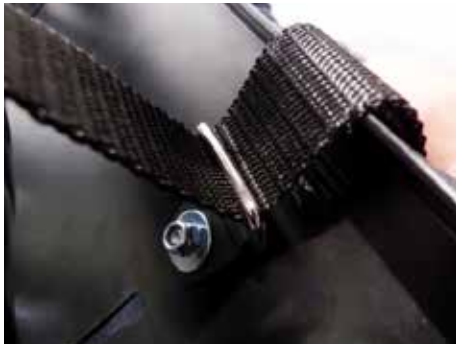
Position the buckle on the bottom side of the seat panel and insert the screw from the top side of the seat panel into the rear bore hole. Insert the washer and biloc nut. Proceed in the same way on the other side.

- Fix the seat upholstery.

- Guide the straps as shown.

- Insert the buckle and tighten it. Proceed in the same way on the other side.

- The illustration shows the attached fixing clamps of the groin strap.

- Guide the strap from the outside through the long holes of the side plate.

- Guide the end of the strap through the loop and tighten it.

- Thread the plugs onto the straps as shown and cross them.

Caution!

Excess strap ends on accessories may be trapped in the tilt mechanism and can avoid the engagement of the locking pins. In such a case the seat could fall off the mobility base. Shorten excess strap ends on accessories so that they can not be trapped.

To put on the groin strap proceed as follows:

- Place your child into the pushchair on the groin strap.
- The thigh straps are passed between the legs through and over the thighs, at that the part of the thigh belt with the buckle attached should lie on the pelvic bone.
- Insert the male part of the quick-release buckles located on the tensioning straps into the buckles on the thigh belts.
- Tighten the thigh belt by pulling the free end of the strap that comes out of the buckle.
- To disengage the quickrelease buckle, press the key on the buckle.

6.15 Fastgørelsesjakke

To attach the fixation jacket straps, rivet (from inside to outside) the six cam-lock buckles to the outside of the back plate.

Thread the shoulder straps of the fixation jacket through the shoulder slots to the rear. The centre "axillary straps" must be threaded at the appropriate height between the back frame tube and back plate. Thread the lower straps through the slots in the lower part of the back base (see also 6.9 Lap belt) to the rear.

Thread all straps through the open cam-lock buckles and clamp in position by closing the hinged cam.

Caution!

Excess strap ends on accessories may be trapped in the tilt mechanism and can avoid the engagement of the locking pins. In such a case the seat could fall off the mobility base. Shorten excess strap ends on accessories so that they can not be trapped.

Opening the fixation jacket:

- Open the quick-release buckles on the shoulder part of the fixation jacket.
- Pull the zipper down and open the fixation jacket.
- Lift your child out of the seat.

Closing the fixation jacket:

- Place your child into the seat.
- Close the zipper.
- Tack together the fasteners on the shoulders.

- Readjust the buckles if necessary.

6.16 Bryst- og skuldersele

To attach the chest-shoulder harness, rivet (from inside to outside) the four cam-lock buckles to the outside of the back plate. Guide the shoulder straps for the chest-shoulder harness backwards through the designated slots. Thread the lower straps through the slots in the lower part of the back base (see also 6.9 Lap belt) to the rear. Thread all straps through the open cam-lock buckles and clamp in position by closing the hinged cam.

Caution!

Excess strap ends on accessories may be trapped in the tilt mechanism and can avoid the engagement of the locking pins. In such a case the seat could fall off the mobility base. Shorten excess strap ends on accessories so that they can not be trapped.

To put the chest-shoulder harness on:

- Before seating the child in the pushchair, release the four cam-lock buckles on the back plate.
- Pull the chest and shoulder pad slightly to the front, open the hinged cams at the lower end of the pad and pull the straps out.
- Place the straps on the sides of the seat.
- Place your child into the seat.
- Pull the chest and shoulder pad over the head onto the chest.
- Next, thread the straps into the lower cam-lock buckles of the pad all the way to the stop rivets and close the buckles.
- Then tighten the lower straps from behind.

- Make sure that the chest and shoulder pad is positioned symmetrically on the pelvis and close the buckles on the lower straps.
- Set the upper body of your child upright against the back.
- Finally, tighten the upper straps behind the seat and close the buckles on the upper straps

The fit and position of the harness is correct when the straps are not touching the child and when the beaded edge of the harness is not in contact with the strap guide slot.

The harness provides the best upper-body support when the strap guide slot is approx. 2-3 cm above the child's shoulder.

6.17 Abduktionsblok

The **BINGO Evolution** can be supplied with a fixed or a swing away abduction block.

You can swing away the swing away abduction block for an easy get in.

To swing away:

- Position in front of the **BINGO Evolution** and press with one hand the blocking clamp as shown and swing away the abduction block with the other hand downwards.

The illustration shows the swing away abduction block.

To remove:

- Push the button and move the abduction block to the front.

To put on:

- Insert the abduction block and move it up to fix it.

6.18 Gribebøjle med polstring

Guide the longer part of the grab rail into one side of the retainer for accessories.

After this guide the shorter end of the grab rail by pressing the snap button into the other retainer for accessories.

To remove the grab rail, stand sideways looking in the direction of the seat unit. Grasp the top of the retaining bracket from top to left and right. Take off the grab rail by pressing the snap buttons and pull the rail upwards.

The cover is easy to remove by opening the zipper.

6.19 Terapibord

Insert the tray with the vertical round tubes into the retainer for accessories on the seat unit.

Press the snap buttons and ensure that they point to the side.

To detach the therapy tray see 6.18.

The tray top is adjustable in depth and angle by operating the clamping lever (A). After loosening the screw connections (B) the height adjustable models can be adjusted in the height.

The tray can only be used if the seat is mounted facing forward in driving direction.

Trays must not be loaded with objects heavier than 5 kg (11 lb).

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The retainers for accessories are used to attach the grab rail and the therapy tray.

The retainers for accessories can be mounted in different depth positions to make the distance between child and grab rail or therapy tray variable.

A modification to another depth position by the dealer is possible.

- This illustration shows the retainers for accessories for **BINGO Evolution Mini**.

6.21 Vintervarmer

Zippers allow the upper part of the winter warmer to be opened or removed.

Place the winter warmer in the stroller and tie with the straps.

Upper straps to the frame of the back insert.

Lower straps to the lower area of the back base.

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After removing the upper side of the winter warmer the sheep skin insert gets inserted into the winter warmer.

6.23 Regnslag

With the zip-fastener open pull the rain cape over the child's head, then unfold the upper end over the backrest and the lower end around the footrest.

6.24 Kaleche inkl. adapter og regnslag

First mount the canopy adapters on the back slot. Lose the screws on the back panel and move the back insert out of the back base.

- Insert the adapters as shown.

- Position the adapters into the shown height.

- Tighten the canopy adapters firmly. Proceed in the same way on the other side.

- Insert the male pins of the canopy into the canopy adapters on both sides.

Adjustable ratchet joints are used to set the angle of the canopy.

- Spread the canopy and wrap over the backrest of the seat.

Canopy with rain cover:

Take the rain cover out of the bag attached to the wrap, unfold the rain cover, pull it over the unfolded canopy and put it over the footrest assembly.

The angle adjustment of the backrest is limited because of the transparent rain cover. The angle adjustment is adapted to a seat and knee angle of 90°. Use the space in tilt function to adjust a stationary position. (see 5.17).

6.25 Ledsagerbremse

The **BINGO Evolution** mobility base can be fitted with an attendant brake. The brake mechanism can be operated with a single lever and a cable switch or with two independent brake lever.

Adjustments:

The attendant brake set is pre adjusted.

Nevertheless, if adjustments become necessary just after installation or after a period of use there are following options:

At the exit of each brake housing a cranked set screw leads the brake cable away from the tyre of the rear wheel. After loosening the counter nut the pre tension of the individual (left or right) brake can be adjusted.

Tighten the counter nut firmly again and ensure that the crank of the set screw shows towards the rear frame, away from the tire.

6.26 Tromlebremse

BINGO Evolution can be fitted with drum brakes too. The drum brakes are activated from the push-bar.

Each drum brake is operated by a brake lever. To apply the drum brake squeeze the brake lever.

With the finger tips press the rocking catch, until it locks with a click and release the brake handle.

When the brake lever is squeezed again, the rocking catch is unlocked and the drum brake can be released.

After loosening the counter nut the initial tension can be adjusted by screwing the guide of the bowden cable.

Whenever you change any settings, make sure that you firmly tighten any screws that have been loosened.

6.27 Buggy-Board / Kiddy-board

To transport a second child **BINGO Evolution** can be supplied with a detachable buggy or kiddy board.

Buggy- and kiddy board are applicable for children at the age of 2 years and resilient up to 20 kg.

6.28 Solskygge

BINGO Evolution can be supplied with a sunshade.

For this purpose, the adapter is screwed to the handlebar as shown. The sunshade can be removed from the adapter via a plug-in system. The adapter remains on the frame when folding the rehab buggy.

6.29 Headphone set

The stereo headphone set consists of two very flat speakers with a diameter of 36 mm and a spiral audio cable with a 3.5 mm stereo jack plug.

The stereo headphone can be retrofitted in headrest supports with spacer fabric.

6.29 Pusletaske

- Black Nylon diaper bag

An extra big changing mat and a thermal bag for a drinking bottle or emergency medication are included. Several storage compartments offer space for diapers, wet wipes and your smartphone.

Attaching heavy bags or similar to the push handle can have a negative influence on the stability.

7 Transport

7.1 II bilens bagagerum

Basically you can store your **BINGO Evolution** in the trunk of your car as well with or without seat unit on as well as with or without wheels on.

To achieve the most compact folding package fold the backrest and remove the rear wheels.

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7.2.1 Common information

- Check that your rehab pushchair is suitable for a crash test.
- Check that the vehicle is equipped and compatible for transporting your rehab pushchair.
- There should also be enough space for safe transport.
- Ensure an upright sitting position during transport.

Please note that improper use of a rehab pushchair in a wheeled ambulance can be dangerous beyond typical use. Failure to follow this advice will result in serious injury or death. We recommend that, wherever and whenever possible users transfer to the seats installed in the motor vehicle and use the corresponding vehicle restraint systems, because this is the only way to ensure optimum protection of the passengers in case of an accident.

Your **BINGO Evolution** rehab pushchair was successfully tested in accordance with ANSI/RESNA WC-4 (except size 0/Mini) and ISO 7176-19 (size 1, 2, 2XL) (Crash Test). It is, however, possible to use your **BINGO Evolution** rehab pushchair as a seat in a motor vehicle, if our "Tie down Kit" as well as appropriate restraint systems are used. Your **BINGO Evolution** is admitted for use as a seat in a motor vehicle yet.

CAUTION! Important note for BINGO Evolution Mini:

With the **BINGO Evolution Mini** it is absolutely necessary to additionally position and fix the child on the seat using a chest-shoulder harness or a fixation jacket. This is the only way to ensure that especially small children (with a body weight of 12-25 kg) are safely held back by / with the 3-point restraint system in the event of a frontal impact.

7.2.2 Securing the rehab pushchair during bus transport

The rehab pushchair may only be used forward in the direction of travel.

The rehab pushchair was dynamically tested in the direction of travel while the dummy was attached with lap belt and shoulder strap.

- 1) If the user finds itself in the rehab pushchair, the rehab pushchair has to be positioned forward and has to be fixed with the fastening and safety belts according to the instruction for use of the restraint system manufacturer (fastening belts referred to WTORS need to fulfil the requirements of ISO 10542 or SAE J2249).
- 2) The wheelchair has not been tested for transportation in another position. The transportation in a lateral directed position for instance is not allowed at all.
- 3) Depending on the size of the pushchair the maneuverability can be impaired, so that the turning of the pushchair is not or only partially possible, to position the stroller forward in the direction of travel.
- 4) The pushchair has to be fixed with a restraint system referred to ISO 10542 or SAE J2249 with fixed belts in the front and adjustable belts in the back. Usually this concerns snap hooks/ s-shaped hooks as well as plug closures. The restraint systems usually consist of 4 single belts, which have to be attached to the 4 edges of the pushchair. The attachment points of the pushchair restraint systems are marked with the international hook symbol.
- 5) The rehab pushchair may also be supplemented for transport with other positioning and fixation systems. However, these are not a substitute for passenger and rehab pushchair restraint systems and may limit user comfort.

Without consulting the manufacturer, no changes may be made to the attachment points of the pushchair or to components of the chassis and frame or they may not be exchanged. Failure to comply with these requirements, the stroller must not be transported in vehicles.

In order to reduce the risk of injury to vehicle occupants, tools that are not specifically designed for crash safety, should be removed and stored separately in the vehicle safely such. eg crutches, loose pillows, therapy tables etc.

7.2.3 Buckling the rehab pushchair passenger

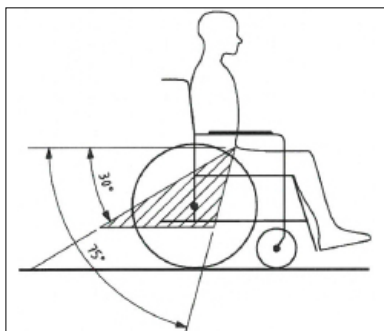
The user has to be buckled up with the lap belt as well as the shoulder belt.

Shoulder and lap belts should be used to minimize and / or avoid potential injury from impact on vehicle components.

- 1) The person executing the attachment should be trained in the handling of the system.
- 2) Before transportation the following adjustments of the rehab pushchair have to be carried out:
Seat: 0° - 5°
Backrest: 90° - 100°
Footrest: 90°
- 3) The angle between lap belt and horizontal has to be 30° - 75°. An angle close to 75° is preferable.

- 4) The shoulder belt has to run across to chest and shoulder. The belt may not butt at the neck and may not be attached loose from the shoulder.
- 5) The belt strap may not be twisted.
- 6) The user's head should be additionally secured by a separate and permanently mounted headrest in the bus.
- 7) Please lock the drum brake firmly
- 8) Please lock the manual brake firmly
- 9) Remember, however: In the event of a traffic accident, the risk of injury can only be minimized and not ruled out, even if the passenger and rehab pushchair restraint systems are used correctly.

Belt restraints should make full contact with the shoulder, chest, and pelvis and pelvic belts should be positioned low on the pelvis near the thigh-abdominal junction.

Lap belt and shoulder belt need to butt plane on the body.

As far as possible all additives should be detached and stored securely:

- crutches, loose cushions and therapy trays

You should not rely on the occupant restraint systems unless they are marked in accordance with the requirements of ISO 7176-19: 2008.

Care should be taken that the occupant restraint is positioned so that in the event of an accident, the release button is not triggered by pushchair components and results in unintentional opening of the seat belts.

Before reusing the pushchair after a collision or an accident with a bus, the pushchair may no longer be used for transport in a bus until it has been checked for possible damage by authorized personnel and then released again.

Attachment points, front according to ISO 7176-19

Attachment points, rear according to ISO 7176-19 for lap belt

Attachment points, rear according to ISO 7176-19

8 Maintenance / Service and Repairs

8.1 Maintenance / Service and Repairs

Your **BINGO Evolution** is CE approved. The manufacturer herewith guarantees that this medical product as a whole conforms to the basic safety and performance requirements in accordance with Annex I to Regulation (EU) 2017/745 of the EU Parliament and of the Council. The wheelchair should be checked for correct function before every use. Nuts with self-assurance should only be used once. After repeated loosening nuts must be replaced. The items listed in the following table must be checked by the user at the indicated intervals.

Check	daily	weekly	monthly
Function test of the brake/wheel lock	X		
Check of screw connections		X	
Air pressure (indicated on the side wall of the tyre)		X	
Visual inspection of wearing parts such as wheels and bearings			X
Contamination on bearings			X
Quick release (rear wheel)	X		
Check firmness of the footrest		X	
Function test of the tilt mechanism			X
Function test of the adjustment back angle			X

Should any defects become obvious, please contact your authorized HOGGI dealer to eliminate them. We also recommend that you have your **BINGO** serviced by your authorized dealer every twelve months.

Instructions for cleaning and maintenance

- Clean all frame components and plastic parts using mild detergents only. (e.g. Sagrotan)
- Padding parts can be washed at 40 °C. If washed in a washing machine, put them in a linen bag or a pillow case.
- In most cases, wiping with a damp cloth is sufficient
- Do not use your BINGO Evolution pushchair in salt water.
- Keep sand or other particles from damaging the wheel bearings
- If your BINGO Evolution gets wet, towel-dry it as soon as possible.
- Hair and dirt particles generally accumulate between the caster wheel and fork. This can restrict the caster wheels from rotating smoothly. Remove the caster and thoroughly clean the fork and caster using a mild detergent
- The rear wheels feature a quick-release system. To keep this system operational, ensure that no dirt adheres to the quick-release axle or axle housing. The quick-release axle should also be lightly lubricated regularly with resin-free sewing machine oil.
- Screw connections should be checked frequently, in particular when beginning to use the wheelchair and after any adjustment. If a screw connection becomes loose repeatedly, consult your dealer.

8.2 Operating life BINGO Evolution

The expected operating life of the **BINGO Evolution** rehab pushchair is 5 years, depending on intensity of use and maintenance. We recommend an annual inspection by an authorized **HOGGI** dealer. In case of an interference or a damage of the wheelchair it has to be checked immediately by a dealer or a medical supply store.

9 Storage

BINGO Evolution can be folded in one piece.

Care instructions!
Clean **BINGO Evolution** before storage.
Please observe the care instructions on page 51.

10 Recycling and disposal

BINGO Evolution is made from recyclable materials.

The product packaging as well as all metal, aluminum and plastic parts can be recycled.

Disposal must be carried out in accordance with the respective national, statutory regulations.

Please ask the city / municipal administration for local ones waste disposal company.

11 Specifications

	Mini	Size 1
Seat width	130 - 250 mm	180 - 300 mm
Seat depth	120 - 270 mm	160 - 340 mm
Tilt in space seat unit In driving direction Opposite driving direction	45° 45°	45° 45°
Backrest height	370 - 510 mm	450 - 630 mm
Recline backrest In driving direction Opposite driving direction	80° - 180° 80° - 180°	90° - 160° 90° - 160°
Lower leg length	160 - 340 mm	160 - 370 mm
Wheel diameter rear	290 x 60 mm 300 x 60 mm	290 x 60 mm 300 x 60 mm
Wheel diameter front (rigid / swivel)	190 x 50 mm 200 x 48 mm	190 x 50 mm 200 x 48 mm
Load capacity * Seat unit Mobility base	25 kg 60 kg	50 kg 60 kg
Weight** Seat unit Mobility base	6,4 kg 8,6 kg	6,4 kg 8,6 kg
Overall length Maximum Minimal	1015 mm 910 mm	1015 mm 910 mm
Overall width Maximum Minimal	630 mm 620 mm	630 mm 620 mm
Height	1110 mm	1110 mm
Weight of the heaviest part (seat unit)	8,6 kg	8,6 kg

CAUTION!

*Accessories and add-ons reduce the remaining load capacity for the passenger proportionally.

CAUTION!

**The weights refer to a BINGO Evolution in basic configuration.

11 Specifications

	Size 2	Size 2 XL
Seat width	230 - 350 mm	230 - 350 mm
Seat depth	220 - 400 mm	300 - 450 mm
Tilt in space seat unit In driving direction Opposite driving direction	45° 45°	45° 45°
Backrest height	540 - 750 mm	620 - 830 mm
Recline backrest In driving direction Opposite driving direction	90° - 160° 90° - 160°	90° - 135° 90° - 135°
Lower leg length	160 - 400 mm	160 - 400 mm
Wheel diameter rear	290 x 60 mm 300 x 60 mm	290 x 60 mm 300 x 60 mm
Wheel diameter front (rigid / swivel)	190 x 50 mm 200 x 48 mm	190 x 50 mm 200 x 48 mm
Load capacity * Seat unit Mobility base	50 kg 60 kg	50 kg 60 kg
Weight** Seat unit Mobility base	7,4 kg 8,7 kg	7,4 kg 8,7 kg
Overall length Maximum Minimal	1015 mm 910 mm	1015 mm 910 mm
Overall width Maximum Minimal	680 mm 670 mm	680 mm 670 mm
Height	1110 mm	1110 mm
Weight of the heaviest part (seat unit)	8,7 kg	8,7 kg

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HOGGI GmbH