
# DORMILET

Flexi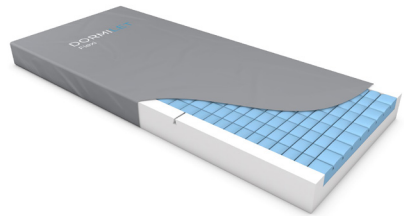

USER-  
MANUAL

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### INDHOLD

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## 1. INTRODUCTION

DORMILET Flexi is a foam mattress with unique cutouts in both the frame and core. This combined with a special slide zone offers good pressure relief and great comfort for the bedridden.

DORMILET Flexi consists of two different types of foam. The bottom of the mattress and the side structure are made of firm foam. This foam frame has specially developed cuts, which means that the mattress is aligned with the bed when for instance the back or leg support is elevated. Inside the foam frame there is softer foam which also has a unique cutouts. These cutouts provides the foam with unusually great adaptability to the curvatures of the body, which offers great pressure distribution and comfort.

DORMILET Flexi has a very special slide zone. The slide zone is under the entire upper part of the body and is designed to reduce shear and friction between the bedridden person and the mattress when the back support is elevated - e.g. when a sitting position in the bed is made.

The structure means that the friction and shear normally existing between the bedridden person and the mattress is now relocated to occur inside the mattress between the two different types of foam. This not only reduces shear and friction under the back but also the pressure under the pelvis is reduced considerably.

DORMILET Flexi is a class 1 medical device and is CE marked according to MDD 93/42/EEC.

### 1.1 Intended use

DORMILET Flexi is designed for use with a hospital- or carebed with a side rail in an appropriate size and as part of an overall care program for the prevention or treatment of pressure ulcers.

### 1.2 Target group

The target group is the active/moderately active bedridden person who has

up to medium risk of developing pressure sores or who has already developed sores up to and including category 2.

DORMILET Flexi allows a maximum user weight of 160 kg.

## 2. BEFORE USE

### 2.1. Unpacking

DORMILET Flexi is delivered flat and wrapped in transparent plastic. This way of delivery means that the mattress can be used immediately after unpacking.

First check that the package is intact. Remove the plastic. Use of a knife should be avoided due to the risk of damaging the mattress cover.

### 2.2. Installing

The mattress must be placed directly on the mattress support platform. The section of the mattress with printed feet must be placed at the foot end. The surface with print is facing up.

Always check that the mattress has a size suited to the bed and side rails. Also, make sure that mattress retainer is adjusted to the mattress width.

To ensure the greatest possible impact of the mattress slide feature, the sheet must be placed loosely around the mattress head end.

## 3.0 USAGE

DORMILET Flexi has two handles on both sides. The handles must ONLY be used for transporting the mattress - not to transferring the bedridden.

## 4. MAINTENANCE

### 4.1. Service and repair

DORMILET Flexi requires no service or repair except cleaning according to cleaning instructions mentioned in section 4.2.

### 4.2. Cleaning procedures

It is important to follow the instructions below before reusing the product for a new client or patient.

Check the cover for damages when cleaning. Is the cover damaged it must be repaired or replaced.

1. Let the cover stay on the mattress. Make sure that the cover is zipped properly.
2. Wash Mattress cover using hot water (60°C) containing detergent. Dry with a cloth.
3. Feces and blood stains must be removed as soon as possible with cold water.
4. For heavy contamination use a Hypochlorite solution 1,000 parts per million available chlorine.
5. If required, the Mattress Cover may be removed and machine-washed at a temperature of 90°C, for not less than 10 minutes.

Wash at 90°C

Tumble dry on low

Do not dry clean

Do not bleach

Do not iron

Please contact Vendlet ApS if you have any questions regarding cleaning.

## 5. TECHNICAL DATA

LOT no	See inside of mattress cover	
Dimensions*	800 x 2000 x 140 mm	
	850 x 2000 x 140 mm	
	880 x 2000 x 140 mm	
Nominal density Range	Frame: 28 kg/m <sup>3</sup>	Core: 50 kg/m <sup>3</sup>
Nominal Hardness Range	Frame: 170 Newton	Core: 145 Newton
Firetesting	BS EN 597-1 (cigarette test) BS EN 597-2 (match test) BS 6807 (crib 5)	
Weight	9 kg	
Life expectancy	5 years	

\*Possibility of manufacturing special mattresses in other widths. Extension pieces of 10, 15, or 20 cm in width can be purchased separately

## 6. STORAGE

Store the mattress flat in a dry place with normal indoor temperature.

Store the mattress so it can not be subjected to bumps, impact or other mechanical stresses. Avoid exposing the mattress to direct sunlight.

## 7. SYMBOLS

Max.user weight  
160 kg

No  
smoking

Medical Devices Directive  
93/42/EEC

Do not use sharp  
instruments

Warning

## 8. GENERAL INFORMATION

- The mattress may be re-used.
- Do not place the mattress on or against a heating source.
- Do not use heating blankets or hot water bottles with the mattress.
- DORMILET Flexi meets the latest requirements for fire safety. Vendlet ApS however recommend strongly that you do not smoke by the bed or while lying in bed.
- Do not use needle, knife or other sharp instruments directly on the mattress, as this will potentially damage the mattress.
- Intended use is the care sector such as nursing homes, hospices and hospitals.

## 9. WARRANTY

There is a two-year warranty on DORMILET Flexi.

The warranty applies to manufacturing defects. It does not apply to normal wear or damage to the product, which is caused by negligent or improper handling.

## 10. DISPOSAL

DORMILET Flexi must be disposed as ordinary combustible waste.

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All rights reserved.