
DRY-GO UV™

USER INSTRUCTIONS

The Widex DRY-GO UV drying station dries your Widex hearing aids and earmoulds, and offers optimal care.


1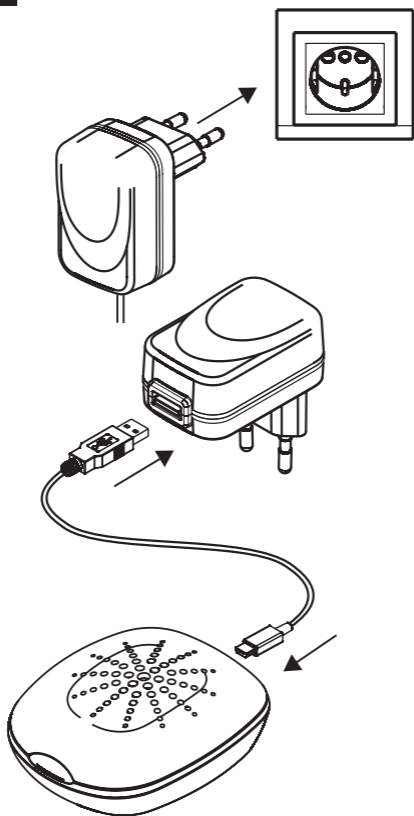

2

2

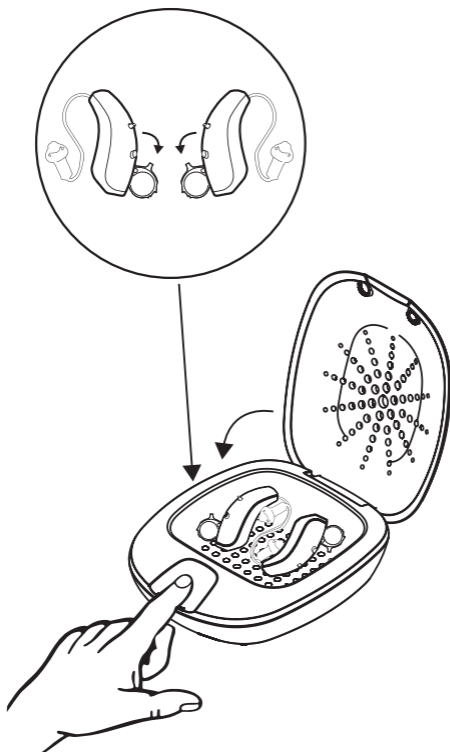3

Operation

1. Insert the USB cable into the 5V power supply and the Micro-USB connector into the socket on the rear side of your drying station. Illustration 1.
2. Plug the power supply into an AC power socket. Illustration 1.
3. Open the lid of your Widex DRY-GO UV.
4. Remove the batteries from the hearing aids. Place the pre-cleaned earmoulds and hearing aids in the drying station with the battery drawer open. Illustration 2.
5. Briefly touch the sensor button and then close the lid. Illustration 2.
 - The drying programme starts, and will stop automatically after three hours.
 - An LED light indicates that the unit is on.
 - A blinking LED light indicates that the UV-C light is on for the first eight minutes of the drying period.

This stops automatically after eight minutes.

- Towards the end of the three-hour drying period, the UV-C light will automatically turn on again for a further eight-minute period.
 - After three hours, the drying station automatically shuts down and the LED light switches off.
6. Open the lid and remove the dried and hygienically-cleaned hearing aids and earmoulds.

Maintenance and operation

- We recommend wet wipes for cleaning your Widex DRY-GO UV.
- You can stop the drying process by pressing the sensor button for more than three seconds.
- The Widex DRY-GO UV is very energy-efficient: you only need to unplug the power supply in the case of prolonged absence.
- The drying station has an automatic safety function that turns off the UV-C

light if the lid is opened during the UV-C programme.

Warnings

- Avoid contact with water.
- Do not use the Widex DRY-GO UV near basins, etc.
- Do not disassemble the drying station.
- Use only the 5V power adapter supplied.
- Children must be supervised to ensure that they do not use the drying station as a toy.
- The drying station is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the drying station, unless they are supervised for their own safety by a responsible person, or receive instructions on how to use the drying station.
- Do not look into the UV-C light, as this can harm your eyes and your skin.

US

DRY-GO UV™

USER INSTRUCTIONS

The Widex DRY-GO UV drying station dries your Widex hearing aids and earmolds, and offers optimal care.

Operating instructions

1. Insert the USB cable into the 5V power supply and the Micro-USB connector into the socket on the rear side of your drying station. Illustration 1.
2. Plug the power supply into an AC power socket. See illustration 1.
3. Open the lid of your Widex DRY-GO UV.
4. Remove the batteries from the hearing aids. Clean your earmolds and place them along with your hearing aids in the drying station with the battery drawer open. See illustration 2.
5. To begin the drying process, touch the sensor button and close the lid. See Illustration 2.
 - The LED light will indicate that the unit is functioning and will blink during the first eight minutes that the UV-C light is on.
 - Towards the end of the drying cycle, the UV-C light will automatically turn on for another eight minutes

- After three hours, the drying station automatically shuts down and the LED light will turn off.
6. Open the lid and remove the dried and hygienically-cleaned hearing aids and earmolds.

Maintenance

- We recommend wet wipes for cleaning your Widex DRY-GO UV.
- You can stop the drying process by pressing and holding the sensor button for approximately three seconds.
- The Widex DRY-GO UV is very energy-efficient but should be unplugged when it is not in use for a prolonged period of time.
- The drying station has an automatic safety function that turns off the UV-C light if the lid is opened during the UV-C program.

Warnings

- Do not use the Widex DRY-GO UV near water.
- Do not disassemble the drying station and only use the 5V power adapter included with this product.
- Children must be supervised to ensure that they do not use the drying station as a toy.
- The DRY-GO UV should only be operated by those who have sufficient knowledge of appropriate use or under supervision
- Do not look into or touch the UV-C light when the device is on as this may cause injury to your eyes and skin.

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flow-med GmbH · Silcherstraße 4
flow-med GmbH · Silberstraße 4