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**EN**

**⚠ Safety instruction**

- Check the step, adjustment knobs and rubber feet regularly for damage. In the event of defects, the step must no longer be used
- Use the step only on flat, non-slip surfaces
- Always take support of a helping person /nurse while using the step
- Always step into the middle of the tread, never on the edge, Danger of tipping and falling!
- Before entering the step, dry your feet completely
- Do not use oil- or fatty shower gels / cleaning agents, risk of slipping!

*In case of serious incidents in connection with this product please contact the manufacturer or the competent authority.*

**Warranty:**

*According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly.*

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**EN**

#### **Intended use:**

The bathtub step is a medical device and can facilitate getting in or out of the bathtub or the overcoming of the edge of the tub. The step is height-adjustable in 3 steps from 21-26 cm.

#### **Indication:**

Difficulties in overcoming the edge of the bathtub due to different illnesses (general weakness, severely restrictive walking difficulties, etc.)

**Contraindication:** strong dizziness, disorientation, no balance control, lack of stability, limited mental abilities, so that use is not or only possible at risk. The use of the step requires the unconditional presence and assistance by a trained carer.

#### **Versions:**

There are different models of steps, the selection should be made together with the specialist personnel. Testing, assembly and instruction in use must only be done by authorized, trained specialists. The model is designed for the domestic use, may only come into contact with intact, uninjured skin and not be used across patients. Another or more far-out use as the mentioned above is not intended. For the resulting damages, there is no liability on the part of Sundo GmbH, the risk is borne exclusively by the user.

#### **Assembly / Before first use**

First, read the instruction for use in full and keep it until the end of the product life. The step is delivered pre-assembled. Check whether the step is complete and undamaged. In the event of damage, the aid must be not used, please contact your dealer. Let yourself be advised about the ideal height adjustment. It is done with the help of the four legs. Press the silver-coloured push button and pull/slide the leg onto the appropriate height. Then leave the push button in the desired borehole click. Make sure that the button exits the hole in a perceptible and audible way. Do the same with the other three legs. Attention, all four legs must be set to the same height, otherwise there is a risk of falling!

#### **Use:**

Before each use check that the height adjustment buttons are correctly locked in place, the step is undamaged and placed secure. The step must only be used on flat and non-slip ground. The bathtub should be equipped with a non-slip mat. For every change in or out of the tub seek the help of a nurse, if necessary also an at the edge of the tub mounted entry aid. First, carefully climb with one foot, then climb with the second foot on the tread plate, and hold on to the caregiver. Then carefully climb with a leg over the edge of the tub onto the bottom of the tub. If you have a safe stand, proceed with the second leg in the same way. Proceed in reverse order to exit.

#### **Cleaning / Disinfection**

The step can be cleaned with a damp cloth or sponge. Use an oil-, grease- or solvent-free cleaner. Make sure that no water enters the pipes. All parts can be treated with commercially available, chlorine-free disinfectants. Attention, please observe the application regulations of the manufacturers!

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#### Sundo Homecare GmbH,
Homecare Enterprise Co., LTD. No.488, Lunmei Rd., Changhua City, Changhua County 500, Taiwan (R.O.C.)