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
UK

Place the cover on the mattress. See illustration  
This also applies for BASIC ST Cover

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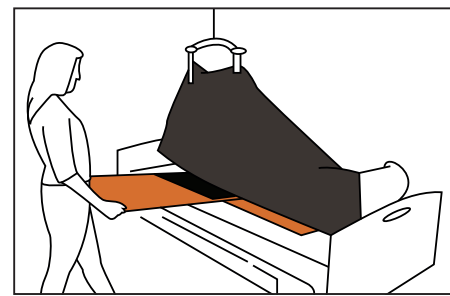UK

Attach the lift to the turn sheet on the side where the sliding flap opening is located. Raise the lift until the shoulder and bottom are no longer touching the surface beneath

Insert the sliding flap under the user at the points of least friction (ankle, hollow of the knee, lower back or neck). Slide the straps in so they are pointing forward

Mount the lift on the opposite side, and raise the side up. Pull out the sliding flap

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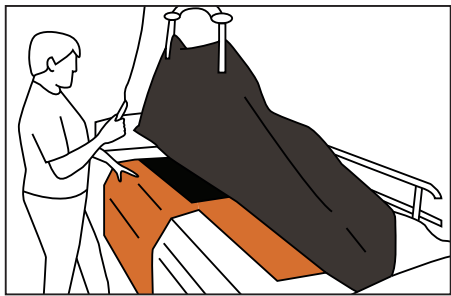
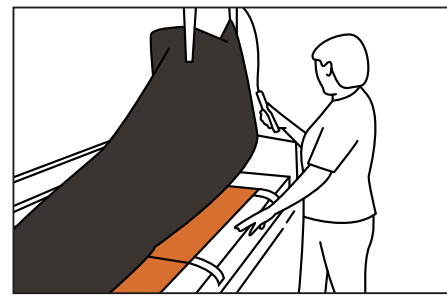
**UK** Attach the lift to the turn sheet on the opposite side of the sliding flaps opening. Raise the lift until the shoulder and bottom are no longer touching the surface beneath

Double up the sliding flap and slide it under the user using both hands. Smooth out the sliding flap so it is spread evenly across the cover's non-slip surface. Finish by maneuvering the straps under the user


Then, attach the lift on the opposite side. Raise the lift until the user's shoulder and bottom are not touching the surface beneath. The sliding flap straps should now be visible and can be fastened

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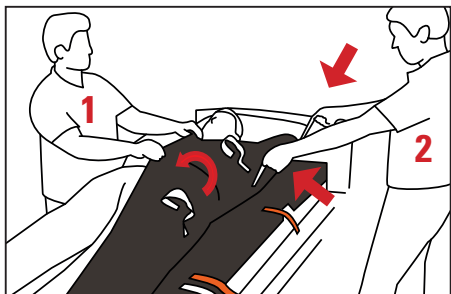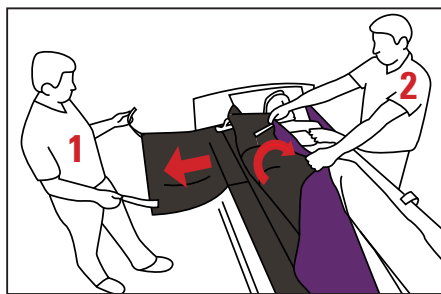
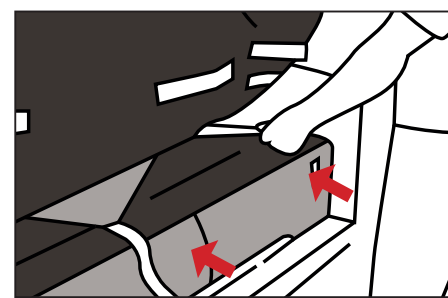
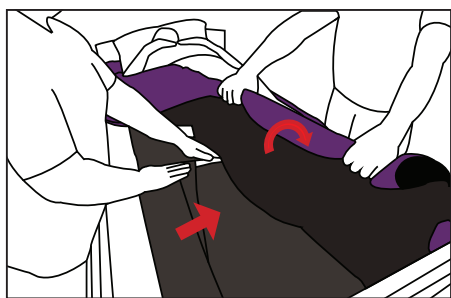
**UK** Person 1 helps position the user onto their side. See illustration. The sliding flap straps should now be visible under the user

Person 2 loosens the sliding flaps two straps and folds them. Fold the sliding flap under the user with straps at the front

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Stropper låses fast på BASIC Cover

**UK** Person 1 holds the turn sheet and helps position the user on their side. Double up the sliding flap and fold it under the user with straps at the front

Person 2 holds the turn sheet up on the opposite side. Person 1 takes hold of the straps and pulls the sliding flap into position under the use

Fasten the straps onto the BASIC Cover

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

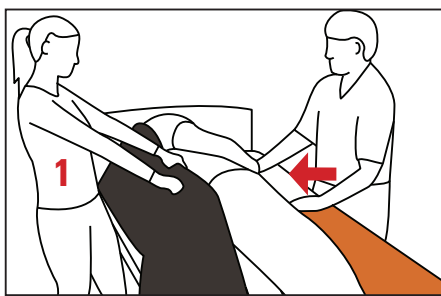
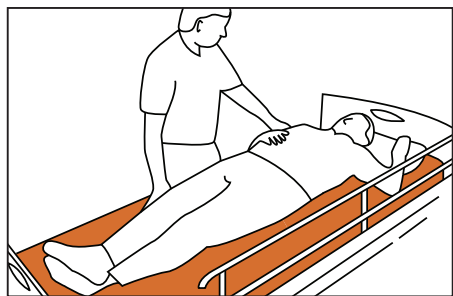
**UK** Cross the user's legs. Lead the user's hand to touch their opposite shoulder. Slide the pillow and shoulder toward the side that you need to turn the user

Person 1 pulls the turn sheet and helps position the user on their side

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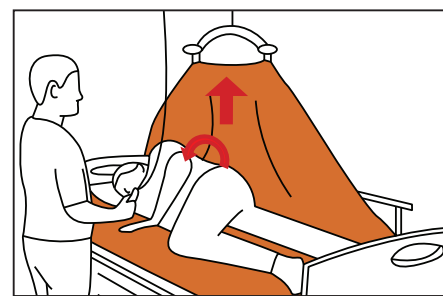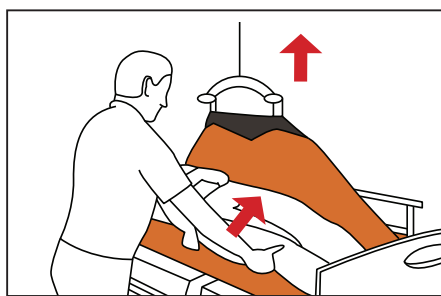**UK** Cross the user's legs and pull their pillow and shoulder toward the side that you need to turn the user. Attach the turn sheet's straps to one side of the lift

Lift the user slightly above the surface beneath. Person 1 then positions the user toward the bed guard. Use the Master Comfort Stomach Wedge for additional relief and support during turn



Raise the lift. Turn the user over onto a lateral position

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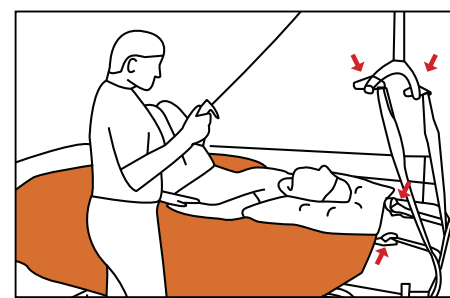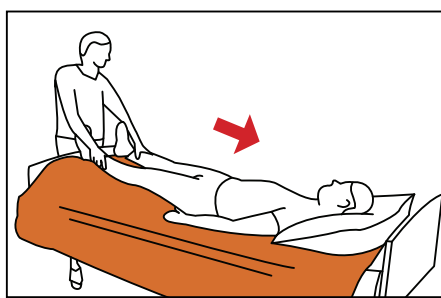UK

Elevate the foot of the bed

Person 1 stands at the foot of the bed and makes a weight transfer of the user's ankle/calf. The user then slides toward the foot of the bed

Alternatively, dismount the bed's headboard. Attach the Master Turner® pull straps to the turn sheet's straps at the head of the bed, pulled under the bed-frame and hooked onto the lift. Raise the lift to pull the user into an upright position in bed

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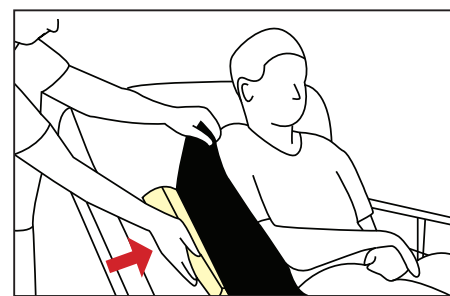
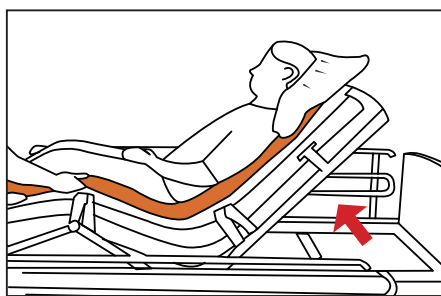
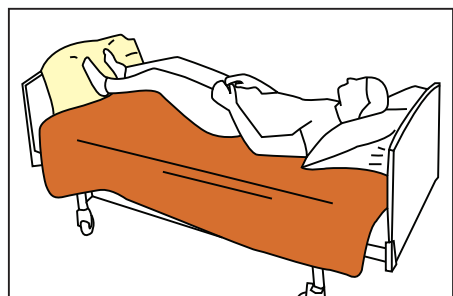
UK

Activate the adjustable bed under the user's knees. Place the Master Comfort Washing Wedge at the foot of the bed

Elevate the head of the bed

Place the Master Comfort Support Wedges under the turning sheet on either side of the user

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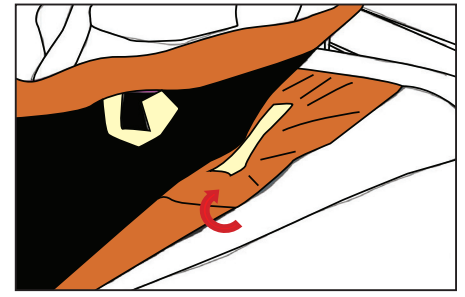
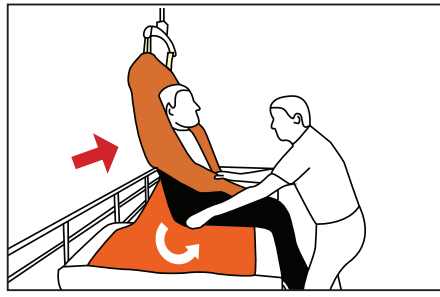
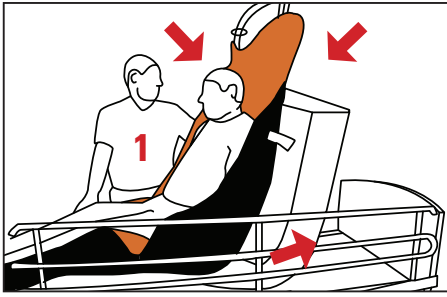
**UK** When this method of transfer is used, person 1 must ensure that the sliding flaps opening is facing the helper. Elevate the adjustable bed under the user's knees. Elevate the head of the bed. Attach the turning sheet's straps to the main section of the lift and raise until it fits snugly behind the user's back

Position the user's legs over the edge of the bed

The user can then slide into the desired seating position. In order to avoid sliding forward, the BASIC Cover sliding flap can be pulled back to make the BASIC Cover anti-slip visible

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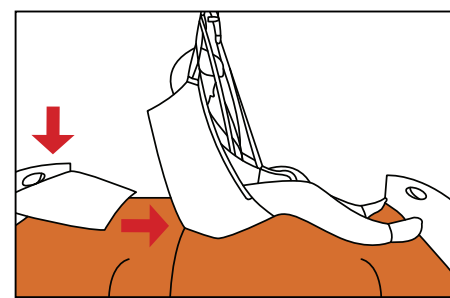
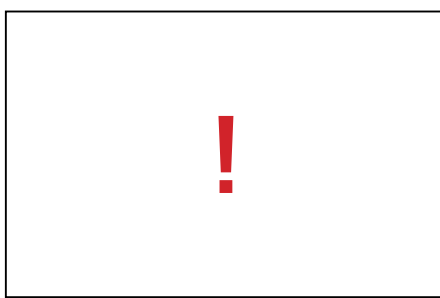
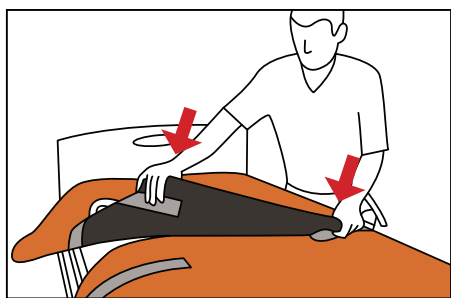
**UK** Position the bottom piece with the sliding part against the cover. The two soft straps must be facing upward toward the middle of the bed. Place the top piece with the sliding part against the cover

Remove the washing straps and save for later use. Attach the straps of the top piece to the bottom piece. Be careful not to work over the middle of the bed and that there are no unintended folds in the system. Pull up the top and bottom pieces until they are touching the headboard



Position the user so their lower back is positioned over the system in the middle of the turning sheet. See illustration

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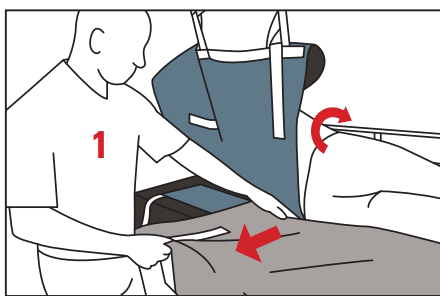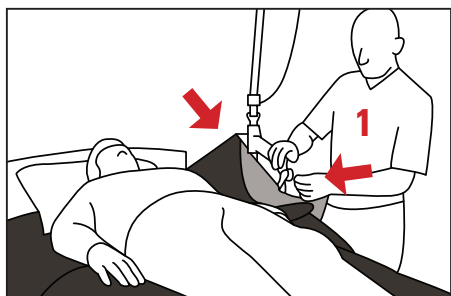**UK** Fasten the straps on the one side of the top piece to the lift. Follow the instructions provided in 2.2 on page 5

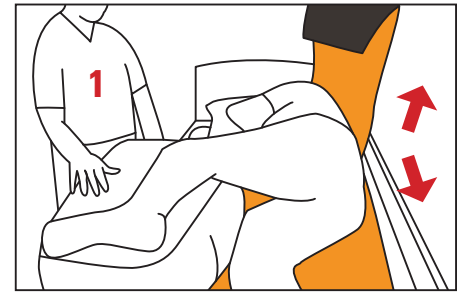
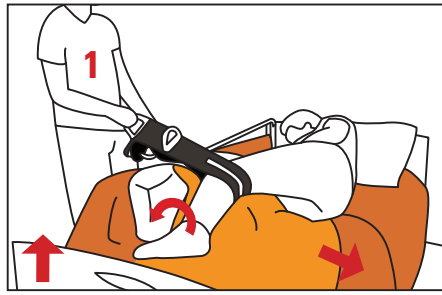
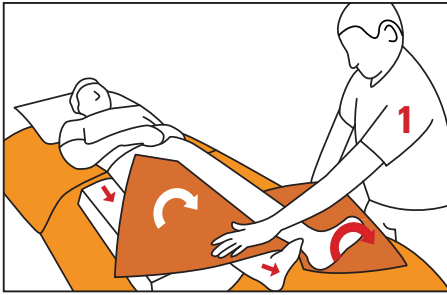
If the user is heavy or uncooperative, the top piece can be fastened to a lift to help keep the user in a lateral position

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**UK** Before the user is moved into a lateral position, position the Master Comfort Washing Wedge from the user's hip and down on the side you wish to turn the user. Fold a Nylon Master slide piece across the washing wedge and in between the user's legs

Turn the user as outlined in 2.2 on page 5. Extend the pull arm by positioning the Master Grip under the hollow of the knee. Use the bed guard to keep the washing wedge in place and pull the Master Grip until the leg has slid up on the Washing Wedge. Remove the Nylon Master slide piece

Remove the slide piece once the user is in a stable position. Detach the bottom piece from the top piece. Then, keep the user in a lateral position using the lift and top piece

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# Fordele / Advantages / Vorteile

## BASIC Cover / BASIC Cover / BASIC Bezug

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UK

- Easy to place on the mattress and remains stable
- Less heat accumulation
- Easy to remove and open sliding flap
- Non-slip on both sides offers better safety in connection with the use of turning sheets

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UK

- Helpers can now keep the user in that position without excess strain or discomfort
- Used for personal care in bed
- Used for dressing users in bed
- Allows users with very limited mobility to be mobilised
- Reduces the risk of pressure ulcers/shear
- Easy to transfer users further up in bed
- Easy to position users into an upright position (with or without lift)
- Easy to move a user's legs into bed

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UK

- Helpers can now keep the user in that position without excess strain or discomfort
- Used for personal care
- Used for dressing users in bed
- Allows users with very limited mobility to be mobilised
- Reduces the risk of pressure ulcers/shear
- Easy to transfer users further up in bed
- Easy to position users into an upright position (with or without lift)
- Incontinence membrane prevents liquid from penetrating into the mattress

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UK

- In addition to the possibilities offered by the BASIC Turn Sheet, the divisible version can provide lateral support in situations that would otherwise require two helpers
- Put slings on users in a lying position
- Practical if the user responds inappropriately to being turned over too often
- Shields staff during hygiene work around the user's lower body

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Washing temperature: **40 - 85°C**

Tumble dry: **max 60°C**

Airdrying: extends durability significantl

Do not use softener.

Use detergent without bleach and chlorine-containing compounds.

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### WARNING

There is a risk of the user slipping over the edge of the bed

Never leave the user lying on master **care's** products with lowered safety rails

Never leave the user with master **care's** products attached to the lift or with lowered safety rails

### IMPORTANT

- These products must not be used if they show critical wear or are otherwise damaged
- Always use the products in accordance with this manual
- Always check the stitching on these products - both before and after use
- If in doubt about the proper use of the products, please contact master **care**/your dealer immediately

- master **care** assumes no liability for accidents resulting in injuries if these products are not used in accordance with this manual
- Trading with master **care** is conducted in accordance with the company's at all time applicable Trade and Business Conditions
- **Velcro straps must be closed by washing and drying**
- **During wash and drying, the washing straps MUST be applied to the velcro on the toppiece**

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These products comply with the standards for Class 1 products in the Medical Device Regulation (EU) 2017/745.

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