TOMTAR

BEDIENUNGSANLEITUNG ^{DE}

OWNERS MANUAL ^{EN}

FAKTO 

Version 1.1.0.

ONLINE Version

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Contents

- 1. Signs and Symbols _____ 13
- 2. Common Warnings _____ 13
- 3. Contents of Delivery _____ 14
- 4. Indication of Use _____ 14
- 5. Cleaning & Recirculation _____ 14
- 6. Product Description FAKTO (P/N 329NPU) _____ 15
- 7. Operating the FAKTO Rollator _____ 16
 - 7.1 Folding and Locking..... 16
 - 7.2 Braking and Park Brake 17
 - 7.3 Push Tubes Height Adjustment 17
 - 7.4 Push Grips Adjustment 18
 - 7.5 Brake Cable Adjustment 18
 - 7.6 Cable Clips..... 19
 - 7.7 Tray 19
 - 7.8 Basket 19
- 8. Inspections & Maintenance _____ 20
- 9. Product Labeling _____ 20
- 10. Optional Equipment for FAKTO _____ 20
- 11. Specifications _____ 21
- 12. Warranty _____ 22

Dear Consumer.

Your new FAKTO rollator was designed on today's technical standards and with the greatest attention and care to detail.

Please carefully read the owner's manual in order to make yourself familiar with the FAKTO rollator, and in order to properly operate the rollator. This manual will provide all necessary information for a long-lasting use.

Your Team DIETZ

1. Signs and Symbols

Attention! Urgent Notice

Please read and follow instructions!

Operation related Warning

Please read and follow instructions!

2. Common Warnings

-  Read and follow the instructions given in this manual.
-  Never exceed the load limits of the rollator and its accessories!
-  Use the rollator only on solid and even leveled surface.
-  Never operate the rollator when it is folded!
-  On folding and unfolding the rollator, always lock the park brake!
-  Always make sure that the locking lever is closed when operating the rollator.
-  Never adjust the push tubes beyond the maximum height limit.
-  Never exceed the described angle of the push grips.
-  On folding and unfolding action dress or body parts may become jammed.
-  Never utilize the rollator for e.g. transporting heavy objects or as a stand up aid. Do only utilize the rollator for its designated purpose as a walking aid.

3. Contents of Delivery

- Rollator
- Basket
- Acrylic-Tray
- Cane Holder
- Owners Manual

On reception of your new rollator please check the delivery for completeness and damage. If damage is evident, please contact your seller immediately.

The owners manual is considered part of the product and shall remain with the product at all times including change of owner.

4. Indication of Use

The FAKTO rollator designated purpose is such as a walking aid for handicaped people with the provision of a seat for temporary seating.

5. Cleaning & Recirculation

Cleaning

Use mild soapsuds with no scouring additives for cleaning.


Use only alcohol biased commonly available disinfectants for disinfection.

Never use steam degreasers nor cleaners. Never use scouring or caustic cleaning agents.

Recirculation

Please have the Rollator technically properly inspected and if required repaired, thoroughly cleaned and disinfected prior to reuse.

6. Product Description FAKTO (P/N 329NPU)

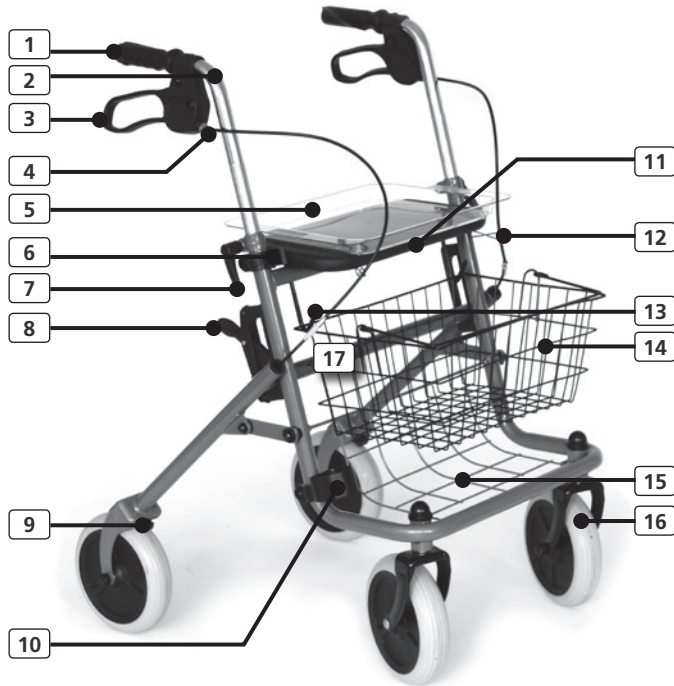Pic. 1

1 Push Grips	10 Cane Holder Lower Part
2 Push Tubes	11 Seat
3 Brake Grips	12 Brake Cable Clips
4 Brake Adjustment Screw	13 Basket Hooks
5 Tray	14 Basket
6 Cane Holder Top Part	15 Rack
7 Wingnut	16 Steer Wheels
8 Locking Lever	17 Labeling
9 Brake pads	

7. Operating the FAKTO Rollator

Make yourself familiar with your new walking aid prior to the first walking experience.

Before starting the operation your rollator needs to be adjusted for your individual demands. Part of this adjustment is the height and angle adjustment of the push tubes including the push grips and the connection of the brake cable clips to the seat.

Unfold the rollator by spreading the front and rear frame in opposite directions until it blocks. Secure the locking lever to the down position (Pic. 2) . Immediately lock both park brakes. The rollator must now be standing secure and all wheels must touch the ground surface evenly (Pic. 1).

7.1 Folding and Locking

1. After each time the rollator is unfolded and ready for operation, the locking action must be properly locked by operation of the locking lever. On locked, the lever must be engaged in the down position!
2. In order to fold the rollator the locking lever must be pushed to the upper position. Only then can it be folded.

-  Never operate the rollator when the proper operation of the locking lever is not possible.
-  Always pay full attention when locking/unlocking the rollator. Nearby parts could lead to impingement of fingers or hands.

Pic. 2

7.2 Braking and Park Brake

The operation of the brakes and the park brakes is performed by the brake grips.

1. For braking during walking and manoeuvring the brake levers must simultaneously be pulled up.
2. In order to apply the park brake the brake lever must be pushed down until it locks. Use the end part of the brake lever for an easy operation.

To disengage the park brake pull the brake lever up into the neutral position.

Pic. 3


7.3 Push Tubes Height Adjustment

The push tubes can be adjusted to meet your required height for a safe and comfortable operation.

1. The lowest push tube height is achieved by moving the tubes into the frame until it blocks.
2. The highest position requires attention. Never adjust the push tubes any higher than the main top knurling! Always make sure that the lower small knurling is solidly inside the side frame.
3. After final height adjustment securely tighten the wing grip for a solid binding and to avoid any movement of the push tubes.

The operation of the rollator with push tubes adjusted higher than advised displays a significant risk due to instability!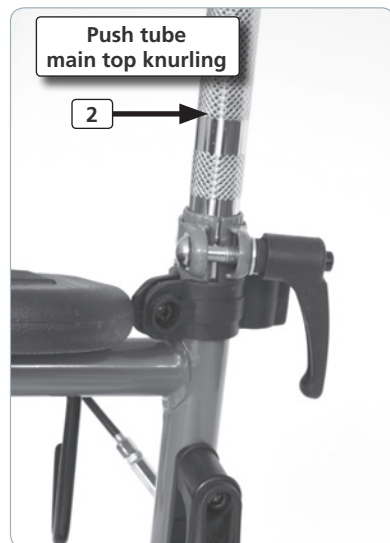


Pic. 4

7.4 Push Grips Adjustment

The push grips must be adjusted so that the push grips are aligned in straight forwards orientation with the rollator. See pic. 5.

 Never adjust the push grips exceeding the described range. Upon exceeding this range the stability and safety are compromised.

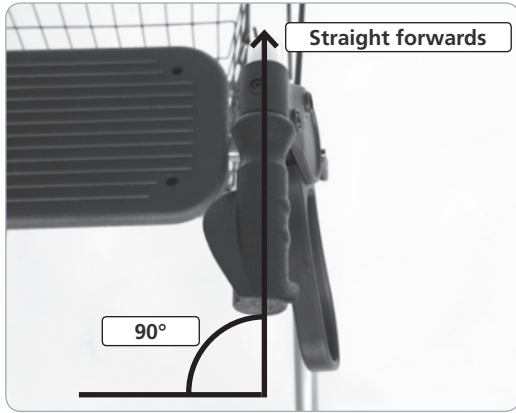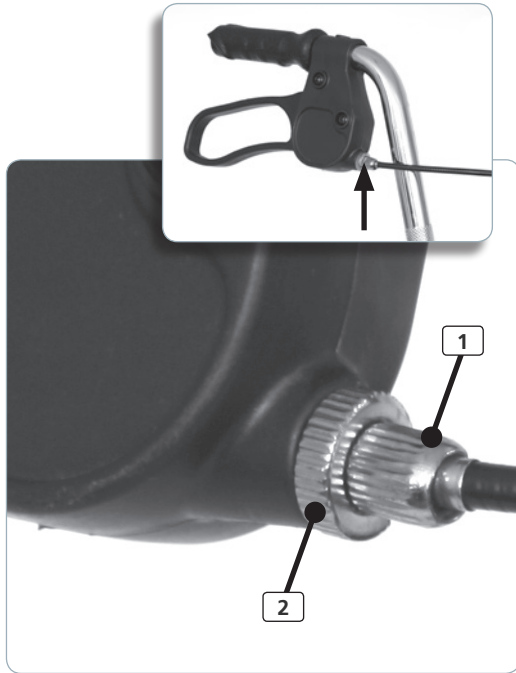
Pic. 5

7.5 Brake Cable Adjustment


The adjuster screw is found on the brake grip front.

1. To increase brake cable tension turn the screw counter-clockwise.
2. The screw is secured by a counter nut. After each adjustment the screw needs to be countered with the nut to avoid loosening and by that losing brake power.

 Never unscrew the screw so that it is not secured by the threads inside the brake grip housing. A loose cable screw will result in the lack of braking ability. Refer to chapter "Inspections".

Pic. 6

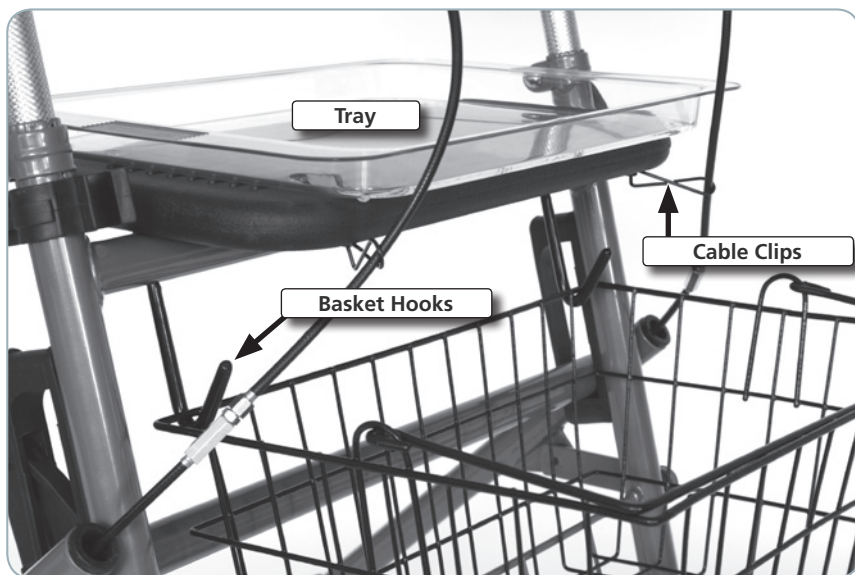Pic. 7

7.6 Cable Clips

The FAKTO is equipped with cable clips on the brake cables.

For a proper cable routing please attach both clips to the hooks that are located on the frontal left/right bottom side of the seat, by clicking the clips into the hooks..

7.7 Tray

Place the supplied tray on the seat.

- ⚠ The tray must be placed on the seat so that the guides interlock with the receivers on the seats surface. This, in order to avoid movement of the tray.
- ⚠ Never place hot objects on the tray.

7.8 Basket

Assemble the basket to the rollator by hanging it into the basket hooks. Before loading the basket check for proper seating of the basket.

8. Inspections & Maintenance

Wheels/Tires

If a wheel does not spin freely you may replace the bearing or the entire wheel. Check the tire surface for excessive wear. If the ground touching surface of the tire is flat you will need to replace the wheel. Flatend surface on the tire has a negative effect on braking power and may make brake cable adjustment impossible.

Steer Forks (Front Wheels)

The steer forks (castors) must rotate freely with minimum play. If this is not the case you could replace the castor bearing or the entire steer fork.

Brakes/Brake Grips

Check the braking performance prior to each operation of the rollator. If the braking performance is below expectations you may adjust the brake cables. If by adjustment no sufficient brake power can be achieved, than you need to replace the brake pads and readjust the brake cables. On heavily flat worn tires you need to first replace the wheels, and at the same time the brake pads.

Checkliste and intervals on common maintenance:

Checkpoint	daily	monthly	yearly	Adjustment
Brake Performance	●			User
Bearing play	●			User
Cleaning ¹		●		User
Seat and Frame Stability			●	Technician
Bearing Greasing			●	Technician

9. Product Labeling

Type Label: — |

TOMTAR FAKTO Standardrollator

S/N: 329000000

P/O: 1300333

ArtNr.: 329NPU

Max. Belastbarkeit (kg) Max. Load (lbs)

Rollator: 130 Rollator: 260

Ablage: 5 Rack: 10

Korb: 5 Basket: 10

Tablett: 2 Tray: 4

03/2013

DIETZ GmbH Reha-Produkte
D-76307 Karlsbad-Ittersbach GER | +49 (0) 7248 9186-0

10. Optional Equipment for FAKTO

- Mini-LED Clip Lights, red and whitev
- LED Beam, white
- Umbrella for rain or sun
- Rollator-Foldable Garage

1) Use only ordinary detergents and sanitizers. Never make use of a high pressure washer.

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11. Specifications

1 Type	Standard Rollator
2 Frame Material	Steel Tubing, welded
3 Varnish	Powder Coated
4 Color	RAL 7037, Sapphire-Gray
5 Tires Material	PU, non marking
6 Wheel Size	200*50mm
7 Weight (w/accesaires)	18.2lbs (19.6lbs)
8 Max. User Weight	130kg
9 Length*Width*Height	68*57*80.5cm
10 Folded, L*W*H	25*57*91cm
11 Pushbars Height	min. 80.5cm; max. 99cm
12 Seat Height	61.5cm
13 Brakes	Rear wheels left and right
14 Brake Grips	Pair left/right w/ park brake function
15 Max. Load Tray	2kg
16 Max. Load Basket	5kg
17 Max. Load Rack	5kg

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12. Warranty

All warranty is in regard to imperfections that are related to errors in production.

The warranty periode is 24 month from the shipping note on or latest post delivery.

The warranty is invalidated by:

Improper use, excessive wear and tear and intentional damage. As well as damages that derive from improper maintenance and improper cleaning substances or materials.

Claims can only be followed if a full comprehensive description and photo evidence is provided. On request we shall require the damaged product in house for evaluation.

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Remarks

Errors and technical changes reserved.

We continuously strive for product improvements. We reserve the right for a change in specifications without prior notice.

Applicable Law

Contractual relationships with our customers are to be executed on basis of German law only.

Place of fulfillment is Karlsruhe/Germany.

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TOMTAR is a registered brand by DIETZ GmbH Reha Produkte

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TOMTAR ist eine eingetragene Marke der DIETZ GmbH Reha Produkte