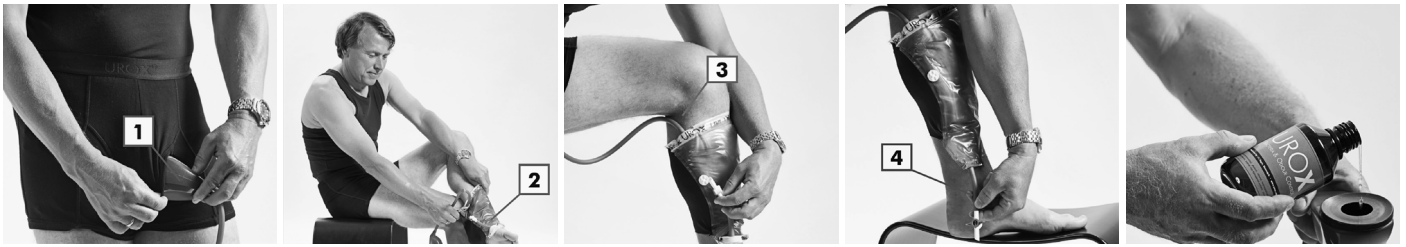
UROX

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GB // INSTRUCTION FOR USERS

Using the system for the first time

Wear the Urox fixation pants, the Cup (1) and Gaiter (2) as illustrated, and pull the tube out through the fly. Ensure that the penis fits securely through the membrane of the cup. The gaiter is worn on the left leg, with the tap sitting on the inside of the leg.
(1) Stand with your legs apart. (2) Cut the tube to the desired length allowing a few extra centimeters of slack to keep the tube from being pulled off the nozzle if you take a wide stride. (3) Push the end of the tube firmly down onto the inlet nozzle at the top of the Gaiter (4) (A drop of water may be used to assist). Check that the tap at the bottom of the Gaiter is closed (in line with the tube).

EMPTYING

Put your left foot on the side of the toilet, unhook the tap and point it down into the toilet to empty the urine out of the bag. Make sure there is no air left in the urine bag after emptying. Close the tap (in line with the tube). Fold back the tap into the hook.

TAKE THE SYSTEM OFF

Always empty the system before removing it. After you have emptied it, take the cup out through the opening of the fixation pants and pull the Gaiter out. It is recommended to keep the system assembled between usages.

CLEANING AND REUSE

All parts should be cleaned with UROX rinsing liquid each time the system has been used. We recommend that you make a habit of cleaning immediately after removal from the body. Allowing some tap-water to run into the cup and out through the tap at the bottom of the gaiter. Close the tap and add approx. 10 ml of rinsing liquid into the cup. Shake a bit, to make sure that the inner surface of the cup is wetted with the liquid, then let the liquid drain down through the tube and into the urine bag. Shake the urine bag allowing full cleaning of the inside of the bag with the rinsing liquid and empty the bag into the toilet. If needed, you can wipe the outside of the system using a damp cloth and rinsing liquid.

Cup and Gaiter can be used a month or more if properly cleaned and if carefully handled, but to minimize the risk of leakage we recommend that gaiters are replaced after a week's use. Avoid contact with solvents, like ethanol etc.

TIPS AND ADVICE

We recommend you practice using the system first in the comfort of your own home. Using the UROX while you are in the company of other people can be intimidating until you have become familiar with it. If you are afraid people will notice the sound from urine trickling into the bag, you may want to make a bit of noise by shuffling a newspaper, for instance.

Try to empty the bag whenever possible to avoid carrying the extra weight of the urine filled bag on one leg.

To avoid accidental leakage always ensure there are no bends or kinks in the tubing. Those who have a strong urine pressure flow are advised to stand when using the system, to maximize the flow capacity in the cup.

What happens if the urine bag is filled with air??

Air is pumped if the Cup is compressed as you move. Most the air will be released through the membrane in the cup. But if the membrane is tight the air cannot escape and will instead be pumped into the bag. This will close the non-return-valve at the inlet of the urine bag and lock the air in it. The advice is: Wear loose fitting trousers. Tight jeans are not recommended. Try a bigger size of membrane. Some users punch a 3-4 mm hole in the top of the cup. It will not cause leakage and the air is pumped out of the hole instead of into the bag.

The tubing has a "hard kink" and does not take normal shape immediately

If the tube was bent while storing, it will "kink" which will affect the flow. The solution is simple: Leave the tubing lying straight in a warm place (not too hot) for few hours until the tubing regains its normal shape, but it will always tend to kink at that spot if bent again. To avoid kinking, while wearing the System, the tube must go as straight as possible from the Cup to the Gaiter. Please cut the length accordingly.

The cup slides off penis

The reason is probably that the fixation pants are too big. Please remember that it is the fixation pants which keep the cup in place, not the membrane.

Men who have a combination of a large stomach and a very small penis may find it difficult to keep the cup in place, even with the correct fitting Urox® fixation pants.

Hygiene

People who are suffering from urinary symptoms and use incontinence devices are more exposed to fungus or bacterial growth than others.

We strongly recommend thorough cleaning of the system with Urox® rinsing liquid to prevent bacterial growth on the system and to maintain a completely clean and odor free system. Intimate body parts should be washed after the removal of the Urox system.

Using urox in bed

Urox is Intended for use when sitting and standing up, but many users have chosen to use It In bed anyway. Some will wear cup and fixation pants to bed, placing the gaiter on the floor next to them, but the easiest way is to place the system on a chair or a table next to the bed, in a place where It is easy to reach when the need arises.

Put the gaiter on the floor, grab the cup, and fit It on your penis. To avoid leakage, It is Important that the penis is completely Inside the cup.

When you remove the cup again, after you have used the system, lift It, to ensure that any remaining urine In It is drained down into the gaiter, and shake It a bit before putting It back.

It takes some practice to use UROX In this way, so we recommend you try It out a few times In daylight, before attempting It In the dark.

MATERIALS

The Urox® system does not contain latex. The membrane is made from medical grade silicone, the cup from plastic and the tubing from medical grade Vinyl.

Urologic ApS · Urologic ApS · Vedbæk Stationsvej 6,2 · 2950 Vedbæk · Danmark urox@urox.com

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