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GB Mobilex A/S**Round Shower Stool 302064**
User's manual

The round shower stool is a medical device which intended use is to give stable & easy seating support when disabled persons use the shower. It has an aluminum frame with height-adjustable legs & a round seat made of white plastic.

Safety Tips

Only use the device for the purpose it is intended for. Don't climb on the device. Make sure that the 4 feet are set to the same height. Use device only in good technical condition. Do not overload the device. Use device only on flat smooth surface. Always check stability of device before use.

Assembly and use

The device is delivered assembled, with the four legs set to the lowest position. Adjust the seat height of the chair to the size of the user by pressing the pin-buttons on the stool legs & adjusting the legs to the desired height. Make sure that all four legs are set to the same height & that the buttons on all 4 legs are safely locked.

Maintenance

Normal dirt on metal & plastic parts can be removed with standard cleaning agents & soft cloth. Do not use solvents or abrasives!

Technical data: Dim. Ø 33 cm,
Seat height-levels: 40-50 cm
Max user's weight: 130 kg
Product weight: 1,66 kg

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D Mobilex A/S**Runder Duschhocker 302064**
CE Mobilex A/S, Nørskovvej 1
I Mobilex A/S**Sgabello tondo da doccia**