Etac Komfort

Manual


etac®

75329C 21-05-19


sv da en de nl


A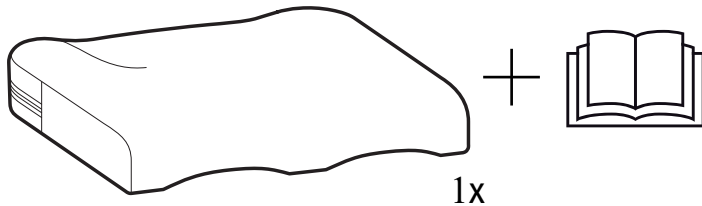

B

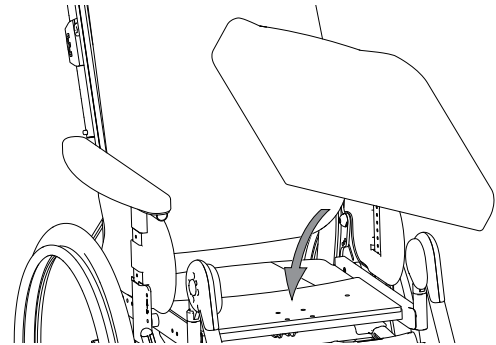C

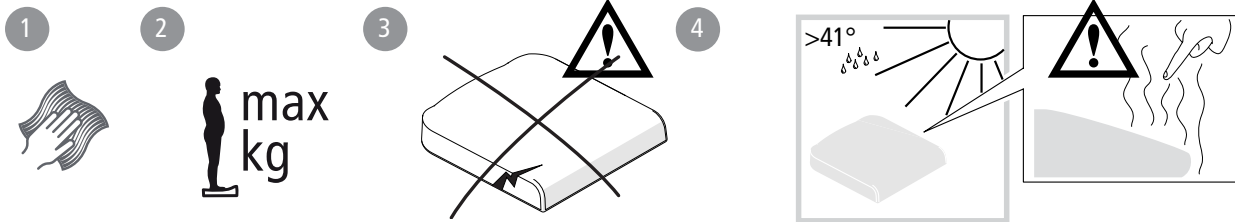D

<p>80 mm</p>	<p>460 mm 500 mm 560 mm</p>	<p>395 mm 445 mm 495 mm 545 mm</p>	
			643/930/1217 g
			752/992/1232 g
			811/1054/1297 g
838/1116/1394 g			

sv

Svenska

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en English

Intended Use:

The Prio seat cushion is a pressure redistributing cushion, designed for good seating comfort. The cushion is intended for users with good sensibility, who can vary their seating position and who have a low risk of developing pressure sores. The cushion should always be disinfected between users.

CE marking, tests:

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

Content of the delivery **Figure A**

Assembly **Figure B**

Maintenance and safety **Figure C**

1. Clean the cover as per the instructions on the product. Also refer to the Reconditioning instructions for Etac manual wheelchairs at www.etac.com.

The product requires no other maintenance.

2. Max. user weight 135 kg.

3. A defective product must not be used.

4. Heated products can cause burns.

Constituent materials are corrosion-resistant.

Constituent materials are resistant to common disinfectants.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Guarantee, service life

1-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 2 years. For complete information regarding the service life of the product, see www.etac.com.

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Etac A/S
R82, Inc.
Etac Supply Center AB