- GB Toilet seat raiser**
- D WC-Sitz Erhöhung**
- DK Toilet forhøjer**
- I Alza-water**
- S Toalettförhöjning**

Item no. **301102, 301104, 301106** (without lid)
Art. Nr. **301101, 301103, 301105** (with lid)
Varenr.
Codice

User manual

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

GB Toilet seat raiser

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Description & Intended Use

The Mobilex Toilet Seat Raiser is medical device which intended use is to give stable and easy to reach seating support while disabled persons use the toilet. Any other use than intended is prohibited

Important Safety Information

Use device only in good technical condition. Do not overload the device. Use the device only for purpose that it is intended for. Don't step on the device with your foot.

Set Up & Usage Description

Loosen plastic wing nuts on both sides of the device. Lift the existing toilet lid and place the device on toilet so it lies flat and stable. The inner bottom collar should be located inside the toilet hole. Tie both wing nuts so the device is locked in position. Ensure that the device is locked and stable before each use.

Technical Data

Dimensions; 5/10/15x37x40,5 cm
Inner hole diameter: 21x26,5 cm
Weight: 0,9/1/1,1 kg
Max load: 190 kg

Cleaning & Maintenance

Clean the device regularly & keep it clean. Use soft cloth, warm water and neutral household washing agents. Do not use abrasives or bleach based cleaners.

The device is basically maintenance free. Regularly check it's rigidity & stability on the toilet. Don't use damaged or broken device.

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Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

Declaration of Conformity (CE)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Danmark, declares that the

Mobilex toilet seat raiser

Item nr. 301101, 301102, 301103, 301104, 301105 and 301106

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment**

Lars Nygaard
Managing Director
Mobilex A/S

Skanderborg, 1/2-2017

.....
signature

	Raised Toilet seat
MOBILEX A/S Grønlandsvej 5 DK - 8660 Skanderborg Tel: +45 87 93 22 20 www.mobilex.dk	 ↓ Max. 190 kg
 Produced <input type="text"/>	Serial no. <input type="text"/>

Example of the product label