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Use:

The Shower Bench »HOD-1« wood look is a medical device and designed as a sitting and standing aid in the shower for people who have a limited stability / function of the lower extremities. The Shower Bench can facilitate or enable body care in the shower, but must only be used by people who can independently control their balance while sitting. The Shower Bench should only be used with the support of a caregiver.

Indication:

Limited standing ability, limited stability, general weakness

Contraindication:

Dizziness, impaired balance function, lack of trunk control, states of confusion, insufficient physical or mental abilities to use the device

Item description:

The Shower Bench is used for a comfortable and safer performance of body care while sitting in the shower. It is made of aluminum and plastic in wood look and can be adjusted into five different seat heights. The seat is equipped with a non-slip surface. The selection of the most suitable aid for the patient and the spatial conditions must be made together with the specialist staff. Testing, assembly and instruction in use must only be carried out by authorized, trained specialist personnel. The item should only be used with the help of an accompanying person/caregiver. The model is designed for domestic use, must only come into contact with intact, uninjured skin and must not be used across patients. Any use other than that mentioned other than that mentioned above is not intended. Sundo GmbH or the manufacturer is not liable for any resulting damages, the risk is borne exclusively by the user. Any serious incidents occurring in relation to this product must be reported without delay to the manufacturer or to the competent authority.

Before use / maintenance:

First, read this manual in its entirety and keep it until the end of the product's life. Please check whether the product is undamaged before first use and each use. Also check the safe position of the Shower Bench before each use, regularly check the screw connections and stability of the seat. In case of signs of damage or wear, the Shower Bench must not be used (anymore). Further maintenance is not required in compliance with the safety instructions.

Scope of delivery: 1 plastic seat, 2 aluminum bases with 4 height-adjustable legs, 5 Phillips screws with 5 washers

Required tool: 1 Phillips screwdriver

Assembly:

Rotate the seat with the bottom facing upwards. Take the base marked with (2) and fasten it with 2 screws (1 washer and 1 screw each) in the outer holes of the seat. Now the base marked with (1) is placed over it and also attached to the seat with 2 screws in the outer holes, the third screw in the middle is screwed through both bases. Tighten all screws tightly again. Now the height adjustment of the legs can be done.

Height adjustment:

The seat height of the Shower Bench is adjusted with the legs. To do this, press the brass-colored button on the base and slide the leg onto the base until the desired height is reached. The button must snap into place visibly, senselessly and audibly. Proceed with the other legs in the same way and make sure that all legs are adjusted to the same height.

Cleaning/disinfection:

Any impurities that are recognizable should be used with a damp cloth and Suitable, mild household cleaner should be removed immediately to avoid drying. Use soft cloths or sponges for cleaning and no sharp or abrasive detergents/objects, otherwise the surface could be damaged. If necessary, the Shower Bench can be treated with a commercially available, chlorine-free device disinfectant in the wipe disinfection process

Important: please pay attention to the application instructions of the manufacturer.

**Safety instructions:**

- ▶ *The body weight of the user may not exceed 136 kg!*
- ▶ *The Shower Bench may only be used by persons who can control their balance while sitting!*
- ▶ *If parts of the Shower Bench are bent or deformed, it isn't allowed to use it.*
- ▶ *All 4 feet must be set to the same height.*
- ▶ *Before each use, the safe stand must be checked!*
- ▶ *When adjusting the height, it must be ensured that the pushbuttons emerge visibly and noticeably from the bore!*
- ▶ *Be careful when using shower mats, risk of slipping and falling!*
- ▶ *Use the Shower Bench only for the intended purpose, do not stand on the aid!*
- ▶ *Repairs must only be carried out by authorized retailers using original spare parts.*
- ▶ *Soap and shampoo products increase the risk of slipping!*
- ▶ *The Shower Bench is not suitable for longer sitting!*
- ▶ *Do not use the Shower Bench in the sauna!*
- ▶ *For optimum safety, it is advisable to regularly check the rubber caps and replace them if necessary!*

Warranty: According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly. We reserve the right of modifications serving the technical improvement.

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Sundo Homecare GmbH,
Homecare Enterprise Co., LTD. No.488, Lunmei Rd., Changhua City, Changhua County 500, Taiwan (R.O.C.)