Kære kunde

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Dear customer

We are pleased that you have decided to use one of our high-quality products. This user manual contains a description and instructions for proper use of the product. It is important that you read this manual thoroughly before use.

Purpose and instruction in use of Ergonomic Baby Pillow.

BabyCloud's ergonomic baby pillow is developed by healthcare professionals, with extended experience with medical aids and hospital equipment. It is designed to prevent and treat Flat Head Syndrome on babies. The pillow should always be placed on an even surface or mattress. Place the baby's head on the center of the depression, this will reduce pressure to the back of the head by distributing the weight evenly. The pillow don't limit the child's ability to move, which is a good thing since head movement reinforce the neck muscles and prevent unhealth sleeping positions.

Recommendations to prevent SIDS.

- The safest place a baby can sleep is in their own bed.
- Babies should always sleep on their back.
- Avoid overheating. Recommended room temperature is between 18 and 20°C.
- Babies should only use a small and thin quilt that they can kick of themselves.
- Don't expose a baby to smoking.
- Never leave sleeping babies unsupervised.

Cleaning and washing instructions.

- Machine-wash the pillow at delicate cycle at 60°C.
- Handwashed in 95°C hot water for disinfection.
- Do not tumble dry the pillow.
- Do not bleach the pillow.
- Do not clean dry the pillow.
- Do not iron the pillow.

Warranty.

BabyCloud offers 2 years warranty against material and production defects on all products sold. BabyCloud cannot be held accountable for neglect and abnormal wear and tear or unauthorized alteration of the product by the user or a third party. The warranty does not cover normal wear and tear and does not include expenses in connection with a complaint. The warranty won't cover any the case of non-compliance or misunderstanding of this user manual. In case of doubt feel free to contact BabyCloud.