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### NORMATIVA

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## INSTRUCTIONS FOR USE

### NORMATIVE

This product is a Class I medical device and complies with Regulation (EU) 2017/745 of the European Parliament and of the Council of April 5, 2017 on Medical Devices.

This product should not be disposed of in the waste container. There are specific collection points where you can deposit them. For the disposal of the container and the product, strictly comply with the legal regulations of your community.

### INTENDED USE

Technical aid to prevent pressure ulcers.

This anatomic viscoelastic cushion is indicated for those who must remain in the same position for a long period of time, as prevention and treatment of bedsores, postpartum fistulas, haemorrhoids, etc. It has been specially designed for wheelchairs, due to its configuration it distributes the weight evenly over the entire surface, allowing the user to sit for long periods of time.

It is recommended for:

- Prevention of bedsores in high-risk patients and as a preventive method of treatment.
- Hip bursitis and Chronic idiopathic anal pain (CAD).
- Help for changes in posture, for people, incontinent or not, who remain sedentary for long periods of time.
- Restrictions that require control and postural stability from the pelvic region.
- Postpartum and post-operative pain relief and pelvic stability.

### TECHNICAL FEATURES

- Removable cover
- Measurements: 42 x 42 x 8 cm
- Interior Material: 80D visco injected polyurethane
- Exterior material: Removable and washable PU cover
- Net Weight: 1170 g
- Max Weight cap: 125 Kg

### LIFESPAN

The useful life of this product is influenced by their conditions of use and storage and by the regularity of their maintenance. By imperatively respecting the data provided in this instruction manual, it is possible to keep the item in perfect condition for 24 months.

**REF 3005**

## ANATOMIC MEMORY FOAM PRESSURE RELIEF CUSHION

**MD CE**

### SYMBOLS

|   |                |   |                      |
|---|----------------|---|----------------------|
| <b>REF</b>  | Product Ref    |  | Manufacturing Date   |
| <b>MD</b>   | Medical Device | <b>LOT</b>  | Lot Number           |
| <b>CE</b>   | EC Certificate |  | Instructions for use |
|  | Manufacturer   |  | Precaution           |
|  | Distributor    |  | Importer             |

#### **CARE INSTRUCTIONS**

- The cover can be washed periodically always by hand at a maximum temperature of 30° and with neutral soap. Made of PU, it does not allow ironing.
- The filling must be aired periodically.
- If the filling needs to be washed, it is recommended to do it by hand at a maximum temperature of 30°C.
- Do not use abrasive products, solvents or alcohols.
- It is not machine washable, does not allow tumble drying and does not allow ironing.
- Do not hang it up by its ends.

#### **PRECAUTIONS**

- Because it is a flammable product, do not expose it directly to the sun or a heat source.

#### **CONTRAINDICACIONES / CONTRAINDICATIONS**

- There are no contraindications for the use of this item.

#### **NOTICE**

Any serious incident related to the product must be reported by Sucesores de Manuel García, S.L: and to the competent authority of the Member State in which the user and / or patient are established.

#### **WARRANTY**

We guarantee our customers the products against any manufacturing defect for two years from the date of the invoice. For more information, see our general conditions of sale.

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