Mobilex AQUA aluminium shower stool

Item No. 302070, 71, 72, 73, 78, 79 & 30207X

Montage- und Gebrauchsanleitung
Montage- og brugervejledning
User Manual
Manuale d'uso e manutenzione
Installations- och bruksanvisning

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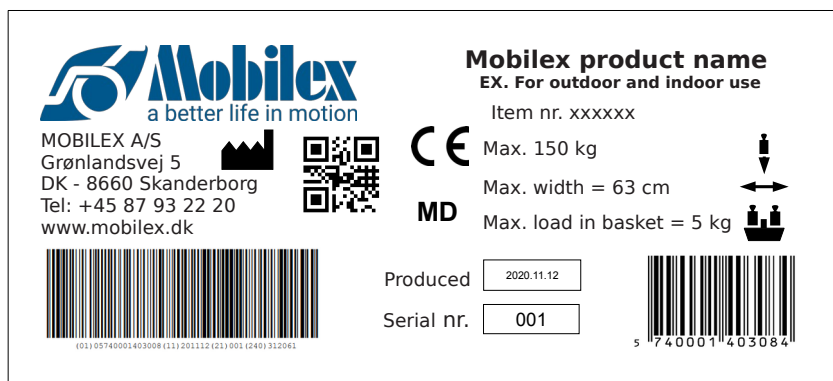
Mobilex product name
EX. For outdoor and indoor use

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1. Introduction

Thank you for buying a high-quality Mobilex product. This user manual contains a description of the medical device and important guidance to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

The shower stool is medical device which is intended to be used only as a seating option for showers and baths. It's purpose is to help to physically handicapped persons during their hygienic activities. Any other use is prohibited. Don't use this stool as a climbing aid.

If you have any questions, please contact your dealer or Mobilex A/S.

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
--	--

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- Don't climb on to the device
- The device must only be used on stable surface.
- When adjusting the device the user must take care not to squeeze the fingers or other body parts.

3. Description

Product description

The Mobilex shower stool is designed for individuals who require seating support during showering or while bathing. It is made from aluminium tubing which is warm to the touch and corrosion resistant. No tools are required for assembly.

Mounting of backrest

The backrest of the bath tub (accessory) is easily fitted by inserting the 4 locking clips provided into the corresponding tubes under the seat.

Set Up description

Adjusting the seat height (legs length)

The seat height of the shower stool should be adjusted to suit the user.

As a rule, the seat height is correctly adjusted when the user is sitting on the chair and both feet fully touch the ground.

1. Pull the locking clip (A) out of the chair's leg.
2. Adjust the telescopic leg tube to desired height.
3. Push the locking clip (A) in to the locking hole (B).
4. Repeat above steps for the other legs of the stool.

IMPORTANT! Don't adjust the seat of the stool when someone is sitting on it and always make sure all locking clips are mounted correctly.

Transport recommendations

The stool can be easily transported. Always assure that device during transportation is safely secured against unintended movements.

4. Technical Data:

Mobilex Aqua shower stool	Seat width 42 cm (302070+71)	Seat width 47 cm (302072+73)	Seat width 52 cm (302078+79)	Seat width 57 cm (30207X)
Dimension (WxDxH)	51,5 x 53,5 x 41,7-58,5 cm	56,5 x 53,5 x 41,7-58,5 cm	62,5 x 53,5 x 41,7-58,5 cm	68,5 x 53,5 x 41,7-58,5 cm
Seat dim. (WxD)	40,7 x 33,5 cm	45,2 x 33,5 cm	50 x 33,5 cm	50 x 33,5 cm
Inner width	42 cm	47 cm	52 cm	57 cm
Outer width	51,5 cm	56,5 cm	62,5 cm	68,5 cm
Seat height	41,7 - 58,5 cm	41,7 - 58,5 cm	41,7 - 58,5 cm	41,7 - 58,5 cm
Armrest height	60,4 - 77,1 cm.	60,4 - 77,1 cm.	60,4 - 77,1 cm.	60,4 - 77,1 cm
Weight	2,7 kg	2,9 kg	3,3 kg	3,86 kg
Max. load	150 kg	150 kg	200 kg	250 kg

Product label

The product label is located on main frame

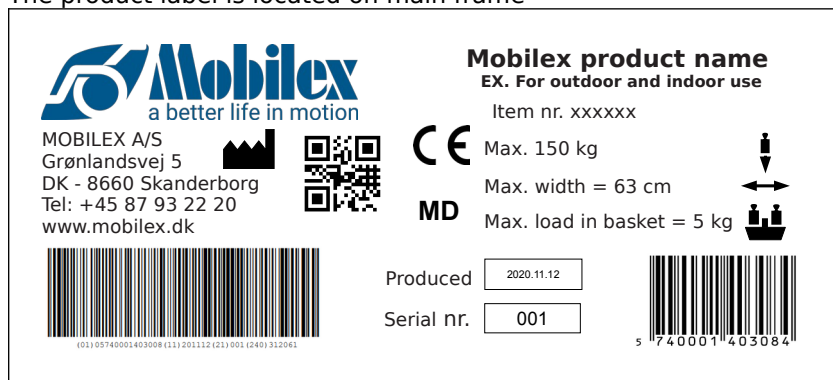
Image of the product label (not original)

5. Care & Maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives).

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional.

The device is maintenance free.

6. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Stool is not stable	Legs are adjusted to different heights	Adjust all legs to the same haeight

7. Disposal of the product

The device cannot be deposited with normal garbage. It must be delivered to the public station for re-use.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly.

The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Item no. 302070, 71, 72, 73, 78, 79 & 30207X

(Artikel Nr., Varenr., Item no., Codice, varenr.)

is designed and produced in accordance with the technical documentation and the basic requirements of the:

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AQUA

ALUMINIUM SHOWER STOOL

1 The optional backrest, like the seat, is made of soft PUR and can easily be adjusted in depth.

2 Comes standard with a soft PUR seat that provides extra pressure relief and comfort.

3 During the development of the armrests, the focus was placed on ergonomics to make it easier for user to stand up from the stool.

4 The legs can be easily adjusted in height with clips and without tools. The pin of the clips is made of stainless steel.

5 Despite its high load-bearing capacity, the shower stool has a low dead weight and can be stacked easily.

6 The welded aluminium construction provides a lightweight and stable frame for safe use.