JAY® J3 Back

Important Consumer Information

NOTE: This manual contains important instructions that must be passed on to the user of this product. Please do not remove this manual before delivery to the end user.

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read this entire manual and save for future reference.

WARRANTY REGISTRATION: To validate the warranty on this product, please complete and return the postcard attached to the enclosed instruction and warranty booklet.

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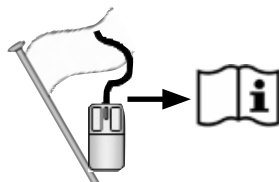
Sunrise Medical recommends that a clinician such as a doctor or therapist experienced in seating and positioning be consulted to determine if a J3 Back is appropriate.

⚠ WARNING!

Backs should only be installed by:

- an authorized Sunrise Medical supplier.
- a qualified and competent healthcare professional who has undergone appropriate certified training by either sunrise medical or their approved supplier.

THE JAY J3 BACK INTENDED USE

The Jay J3 Back is intended to provide a back support for a wide variety of wheelchair users. It is designed to provide simple, comfortable and effective support to help maximize function and increase sitting tolerance. In addition, it is lightweight, durable and easy to use.

The back can be installed onto a wide range of wheelchairs.

Maximum user weight:

300 lbs. (136 kg) – compact (and old standard hardware)

500 lbs. (227 kg) – heavy duty compact hardware

The intended lifetime of the back rest is 4 years.

⚠ WARNING

Installing a back support on a wheelchair may affect the center of gravity of the wheelchair and may cause the wheelchair to tip backwards potentially resulting in injury.

Always assess for the potential need for anti-tippers or amputee axle adapter brackets to be added to the wheelchair to help increase stability.

⚠ WARNING

Do not use J3 Back to push or lift wheelchair. Improper use may cause the back to unexpectedly detach from the wheelchair.

⚠ WARNING

Do not use the J3 back handle to lift wheelchair. It is intended only for back removal.

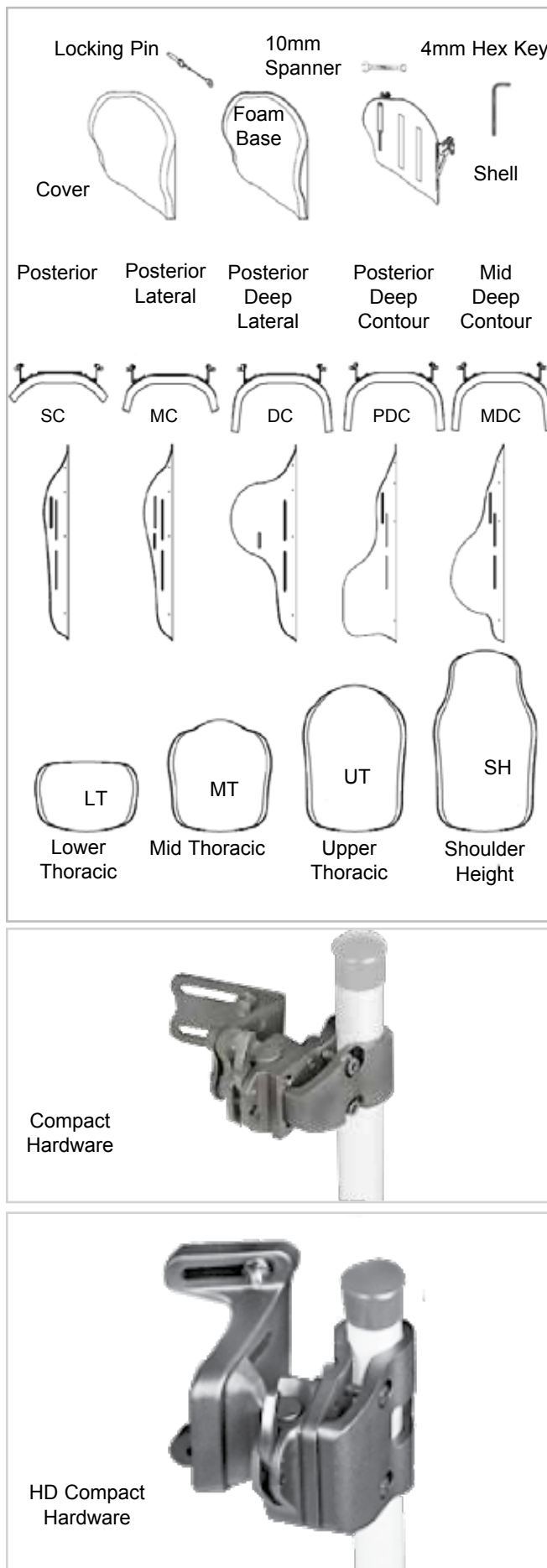
⚠ WARNING

Carelessness causes fire.

⚠ WARNING 

Please read this manual before usage.

J3 BACK

MAINTENANCE

Sunrise recommends that all fasteners be checked for wear, such as loose bolts or broken components every 6 months. Loose fasteners should be retightened according to the installation instructions. All fasteners should be tightened to torque specifications as stated in the hardware installation section. Contact your authorized Sunrise Medical supplier immediately to replace any broken components. Do not continue to use the system after identifying loose or broken components.

EASY CLEANING

NOTE: When disassembling the back for cleaning, note the position of the Velcro® Spine Align Components for correct reassembly.

To clean back cover

Remove cover by detaching Velcro. Detach cover from back shell. Locate zipper on bottom of the cover and unzip. Remove foam and turn cover inside out. Zip cover closed to avoid snagging. Machine wash in warm water (60°C). Drip or tumble dry on low heat.

NOTE:

- Do not dry clean covers or use industrial washers and dryers to clean cover.
- Do not steam autoclave.
- Do not bleach.

Reinsert the foam into the cover. Please note, the base has two layers of foam. When reinserting the foam, ensure that the thinner "light grey" foam is facing the shell. Once the foam is secured inside the cover, zip cover closed.

To clean back shell

Shell may be wiped clean with soap and water. No other maintenance should be required.

To clean positioning components

Positioning components can be wiped clean with soap and water. Do not immerse positioning components underwater. If you have any additional questions regarding cleaning procedures, please contact your authorized Sunrise Medical dealer.

Laundry Care Symbols	
	Machine wash in warm water (60° C)
	Only non-chlorine bleach, when needed
	Drip dry
	Do not iron
	Do not dry clean

DETERMINING COMPATIBILITY

Wheelchair Types

The J3 Back is designed to be compatible with most wheelchairs with the following exceptions:
 Wheelchairs that are angularly adjustable, recline or tilt and result in a back angle greater than 60 degrees from the vertical should not be used.

⚠ WARNING

Use with these types of wheelchairs may cause the back to accidentally disconnect from the wheelchair resulting in a fall and the potential for serious injury.

Wheelchair Dimensions

The J3 Back is designed to replace the wheelchair's sling upholstery. Prior to installation determine if the wheelchair used has compatible back canes, by measuring the width between your back canes as seen in Figure 1. Then using that measurement, check Table 1 to determine if the back will fit your chair.

Next check that the back canes are a compatible diameter. The J3 back hardware can be mounted on wheelchair back canes ranging from 3/4" (1.9 cm) to 1 1/8" (2.85 cm). Measure your back cane diameter (Fig. 2) and refer to Table 2 to ensure your back canes are compatible. If you cannot determine that the back canes are compatible please contact your authorised Sunrise Medical provider or Sunrise Medical customer service.

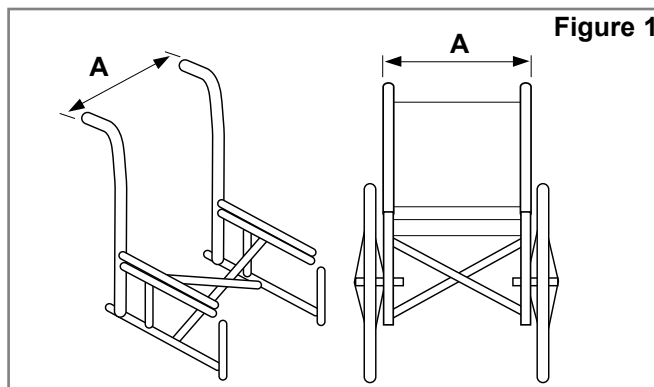
Figure 1

Table 1

J3 Back Width* (all contours)	Width of back canes from outside to outside of the tubes, (Fig.1-A)			
	Minimum Width		Maximum Width	
	inch	cm	inch	cm
12" 31cm	11.7	29.7	14.3	36.3
14" 36cm	13.7	34.9	16.3	41.3
16" 41cm	15.7	40.0	18.3	46.4
18" 46cm	17.7	45.1	20.3	51.6
20" 51cm	19.7	50.1	22.3	56.5

* Back Width is identified in your back's part number.
 Example: J3SCMTS36 is a 36 cm back.

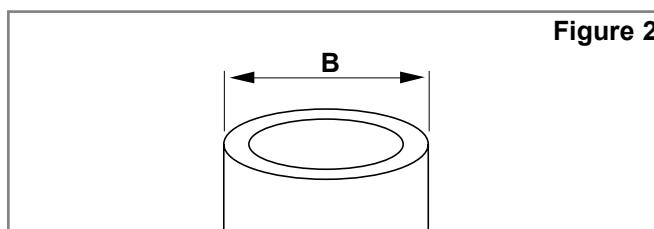
Figure 2

Table 2

J3 Back Width (all contours)	Cane diameter (Fig. 2-B)			
	Minimum width		Maximum width	
	inch	cm	inch	cm
14-20" (36-51cm)	3/4	1.9	1-1/8	2.85

INSTALLING THE MOUNTING HARDWARE

Please read the following instructions before beginning installation. To install the J3 Back, it is best to begin without a user in the wheelchair. Once the back has been installed and minor adjustments are needed the user can be seated back in the wheelchair.

Tools required (included with J3 Back)*

- 4mm hex key
- 10mm box end wrench

NOTE: Wherever torque settings are specified, it is strongly recommended that a torque meter (not included with Jay3 Back) is used to verify correct torque specification is achieved.

Hardware Installation Instructions (Figs. 3, 4, 5)

Valid for all hardware types (compact, heavy duty compact and old standard).

1. Remove the existing wheelchair back according to the wheelchair manufacturer's instructions.
2. Visually determine the desired location of the receivers on the wheelchair's back canes. The mounting hardware should be at approximately equal heights on each back post (Fig. 3) and parallel to the seat frame (Fig. 4). Higher locations on the back canes are generally better.
3. Attach the hardware receivers. (Fig. 5)
 - a. Loosen the clamp screws (A) with the enclosed 4mm hex key. **NOTE**– For large tube diameters, clamp screws may have to be removed completely.
 - b. Attach and align receiver to the wheelchair back canes (B).
 - c. Hand tighten clamp screws (A). **NOTE**– Receivers may require additional lateral adjustment to ensure proper alignment.

Attaching and Aligning the J3 Back Shell (Fig. 6)

- a. Loosen bracket nuts (C) and mounting pin nuts (F) using 10mm box-end wrench until hardware can move easily in all directions.
- b. Attach the J3 Back to the hardware receivers by inserting left and right mounting pins (D) in the left and right receivers (E).
- c. Locate back to the approximate location, relative to both chair and user.
- d. Hand tighten nuts on the inside of the mounting pin (F). **NOTE**– Back depth and angle may require additional adjustment to ensure proper fit to the user.
- e. Hand tighten bracket nuts (C). **NOTE**– Back height may require additional adjustment to ensure proper fit to the user.
- f. Test back release by pressing the release levers forward and removing the back off the chair. (See Removal and Replacement section for further details.) Proper alignment of the mounting hardware and back is achieved when smooth attachment and release can be easily done.
- g. If proper alignment has not been achieved, adjust the components until properly aligned.
- h. Once properly aligned, tighten the hardware receiver clamp screws. The recommended torque specification is 75 - 85 in-lbs (8.5 - 9.3 Nm).

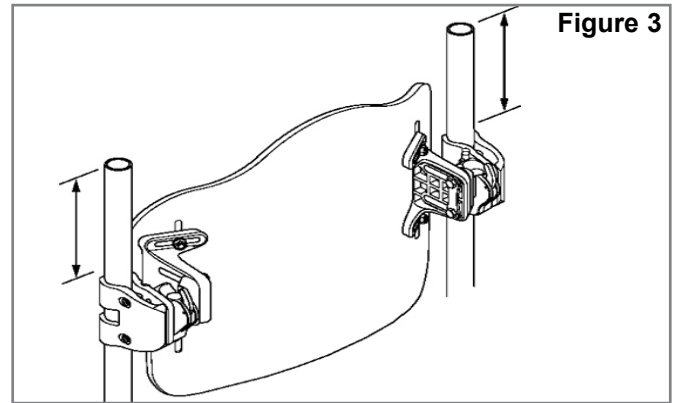
Figure 3

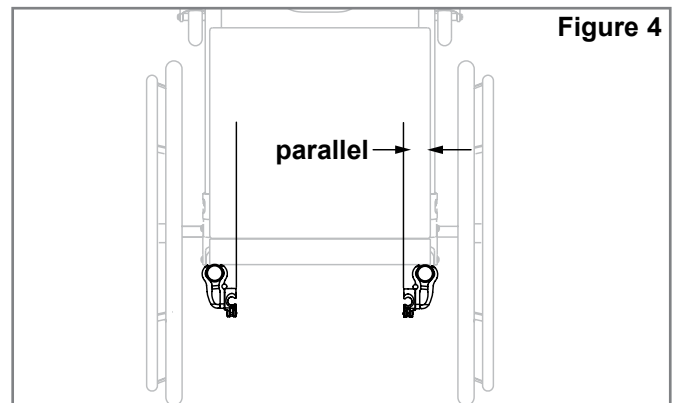
Figure 4

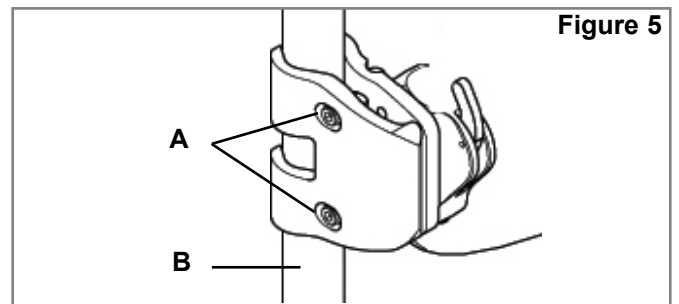
Figure 5

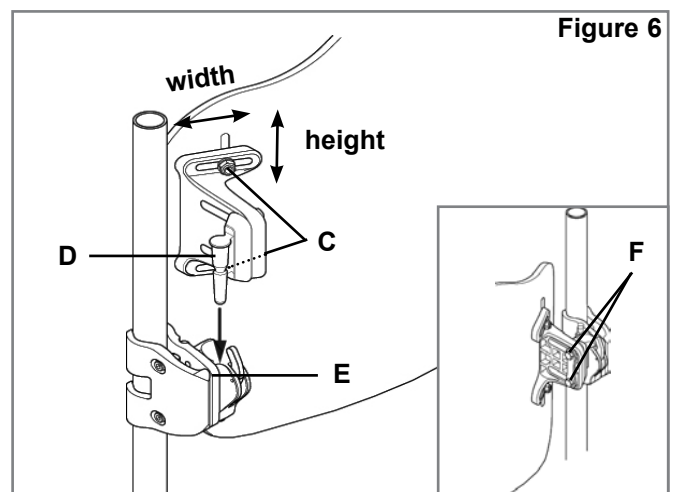
Figure 6

ADJUSTING J3 HEIGHT, DEPTH AND ANGLE

The first phase of the J3 Back hardware installation is complete. It is now the appropriate time to have the user seated back in the chair for personalizing the J3 Back. Do not have the user apply full pressure on the back as it is not fully tightened to the full torque requirements.

1. Height Adjustment (Fig. 10)

- With the user in the chair, identify the desired height of the back. Once identified, have the user remove pressure from the J3 Back by leaning or sliding partially forward if possible to allow the adjustment to take place.
- Loosen the bracket nuts (A) and adjust the height of the back shell to required position.
- Securely tighten the bracket nuts (A). The recommended torque specification is 75-85 in-lbs. (8.5 - 9.3 Nm).

NOTE: After height adjustment is complete, check that the width was not inadvertently changed.

2. Depth/Angle Adjustment (Figs. 11, 12 & 13)

Please note there are two types of hardware that can be ordered— standard hardware (Fig. 11) or the longer, adjustable hardware (Fig. 12). Both sets of hardware are adjusted the same, but the standard hardware allows for 2 inches (5 cm) of depth adjustment (Fig. 11) while the extended hardware (adjustable hardware) allows for a total of 3.4 inches (8,65 cm) of depth adjustability (Fig. 12).

- To adjust J3 Back depth and angle loosen left and right nuts (Fig. 13-B) on bracket slots.
- Adjust depth and/or angle and securely tighten. The recommended torque specification is 100-110 in-lbs. (11.3 - 12.4 Nm). Ensure that none of the depth adjustment nuts (Fig 13-B) will slip or rotate.
- If additional depth is needed, remove back from receivers. Remove screws (Figs. 11-D, 12-D) from receiver using the enclosed 4mm hex key, and reposition to desired depth. Note: Prior to replacing the screws the application of a suitable medium strength threadlocker e.g. loctite 243 is strongly recommended to the receiver and screw threads, if the back is to be repositioned. Tighten screws to 25-35 in-lbs. (2.8 - 4 Nm).

⚠ WARNING

Ensure that all hardware has been properly tightened once all adjustments have been completed and the J3 Back is ready for use.

⚠ WARNING

To ensure optimal security of the back fitment, the locking pin must be inserted before the backrest can be used, (Fig. 14 & 15).


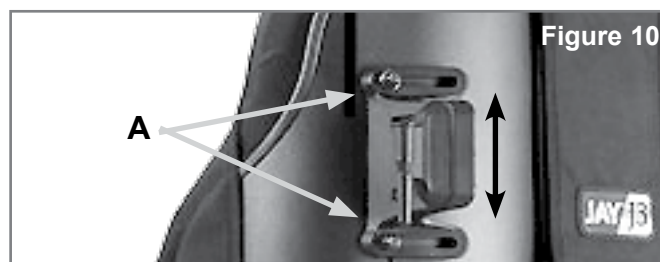
Figure 10

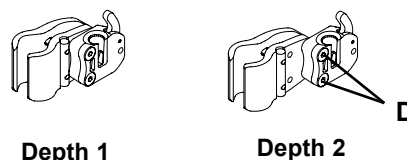
Compact Hardware– 2 depth settings Figure 11

Depth 1

Depth 2

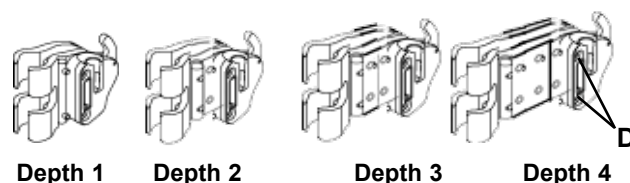HD Compact Hardware Figure 12

Depth 1

Depth 2

Depth 3

Depth 4

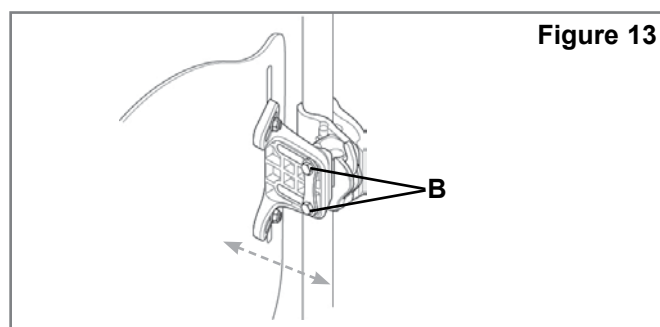
Figure 13

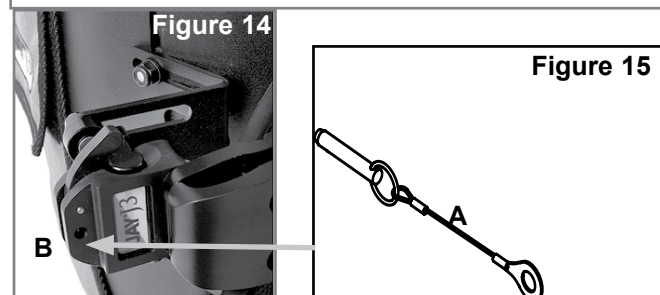
Figure 14


Figure 15

VANITY FLAP INSTALLATION

1. Lift upper flap of J3 Back Cover (Fig. 17)
2. Match hook to loop and attach Vanity Flap to the back of the J3 Back Cover (Fig. 17). Reposition upper flap of cover to sandwich the Vanity Flap between the flaps the upper and lower flaps (Fig. 18).
3. Remove Seat Cushion from wheelchair and sandwich "tail" of Vanity Flap between wheelchair support surface and Seat Cushion (Fig 20).
4. Adjust Vanity Flap as necessary by moving the tail in or out from under the Seat Cushion. If necessary the upper part of the vanity flap can be moved up or down on the Seat Back also.
5. Insert Hook & Loop Wrap ties through the appropriate grommets on the Vanity Flap, then wrap around the wheelchair frame and insert the leading end through the slot end of the Wrap (Fig. 21). Snug up and close outside gaps in Vanity Flap (Fig. 22).

Optional Storage Bag

NOTE: There is an optional storage bag available to attach onto the vanity flap, (Fig.16).

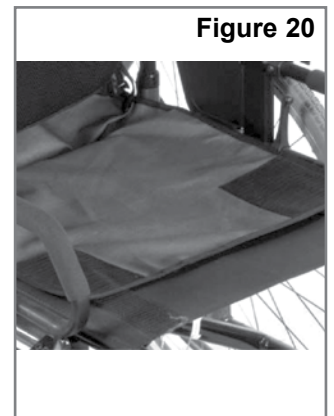**Figure 17****Figure 18****Figure 19****Figure 20****Figure 21****Figure 22****Figure 16****Figure 23****Finished J3 Vanity Flap Installation**

SPINE ALIGN POSITIONING COMPONENTS

Spine Align Positioning Components are designed to provide intimately shaped support within an “off-the-shelf” back. (Depending on your market, Spine Align components may be included with your back.) To install Spine Align please read the following directions.

1. Spine Align Positioning Component Installation

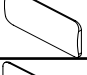
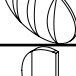
- a. Have the user remove pressure from the J3 Back by leaning or sliding partially forward if possible.
- b. Release Velcro strip on the top cover flap (Fig. 24) and lift the cover forward to reveal the inside of the J3 Back.
- c. Place the positioning component(s) on the Velcro inside of the shell (Fig. 25) and reattach the top cover flap. Pull down making certain that the top seam is aligned with the top of the shell. If seam does not line up, the bottom of the cover may need to be loosened (Fig. 26).
- d. Have the user test the current Spine Align setup for comfort and support and repeat steps a-c until adjustment is complete.
- e. Multiple configurations and combinations of Spine Align components can be used. If additional Spine Align shapes or sizes are required, please contact your dealer or Sunrise Medical.



NOTE: If you require further information on the J3 Spine Align system, please review our animation at www.sunrisemedical.com.

Installation of the J3 back is complete.

J3 ACCESSORIES

If you have ordered J3 accessories such as laterals, headrests or harnesses please consult the installation instructions related to those items.

Positioning Components	
Large Lumbar	
Small Lumbar	
Wedge	
Curved Shim	
Flat Shim	
Posterior/Lateral Pelvic	
Lateral Shim	
Large Bean Bag	
Small Bean Bag	

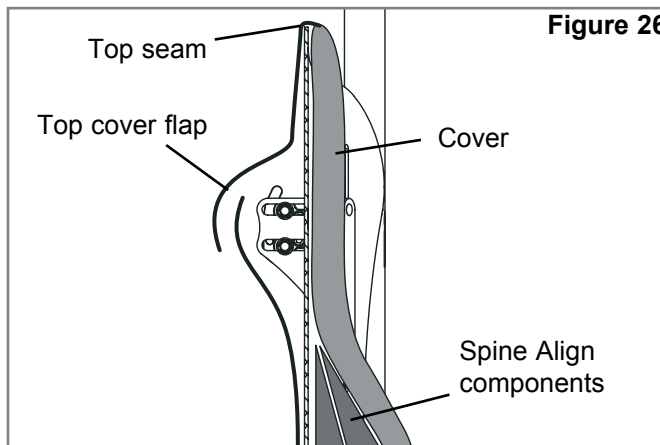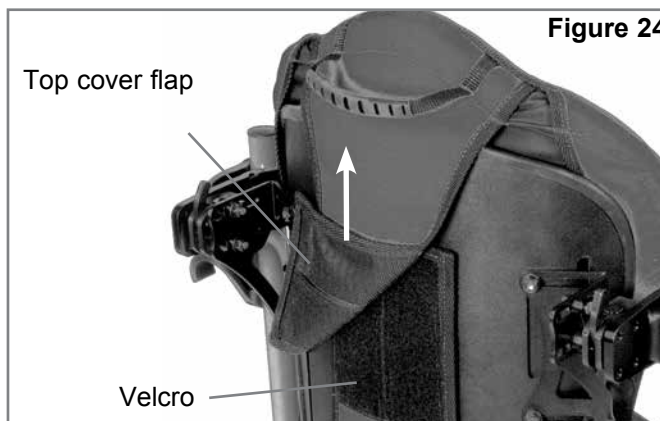BACK TRANSIT SAFETY

The J3 back has been dynamically tested for use in a motor vehicle. Please follow all installation, use and maintenance instructions within this manual as well as the transit instructions listed below.

WARNING

FAILURE TO HEED THESE WARNINGS COULD CAUSE SEVERE INJURY TO THE WHEELCHAIR RIDER OR OTHERS.

INSTRUCTIONS

1. If possible and feasible, the rider should transfer to the Original Equipment Manufacturer vehicle seat and use the vehicle restraint.
2. The distance between the top of the user's shoulder and the top of the back should not exceed 6.5" (16.5 cm).
3. The wheelchair must be labeled as appropriate for use as a seat in a motor vehicle, dynamically tested to the performance requirements of ISO Standard 7176-19 and installed, used and maintained according to the manufacturer's instructions.
4. If the original wheelchair seat was replaced, the new seat must be approved for wheelchair transit, installed and used as indicated in the manufacturer's instructions.
5. The Wheelchair Tiedown and Occupant Restraint System (WTORS) should be compatible with the specific wheelchair, used as indicated in the manufacturer's instructions and should comply with the performance requirements of ISO Standard 10542.
6. The wheelchair must be forward facing during transport.
7. In order to reduce the potential of injury to vehicle occupants wheelchair-mounted accessories such as trays and respiratory equipment should be removed and secured separately.
8. This product is intended for use by adults ranging in weight from 79 lbs (35 kg) to 300 lbs (136 kg).
9. Postural support devices such as pelvic positioning belts, anterior trunk supports or postural supports such as lateral trunk supports should not be relied on for occupant restraint in a moving vehicle unless they are labeled as conforming to ISO Standard 16840-4.
10. Alterations or substitutions should not be made to the structure, parts or components, without consulting the manufacturer.

REMOVAL AND REPLACEMENT

Removal

Following installation and adjustment, the back can be easily removed from the wheelchair if necessary. To remove, push each release lever forward to unlock (Fig. 27). Once both levers are in the forward position, simply lift the back straight up (Fig. 28) using the handle located on the top of the back.

Proper reattachment after removal

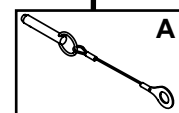
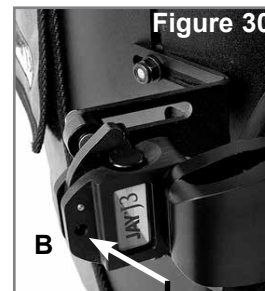
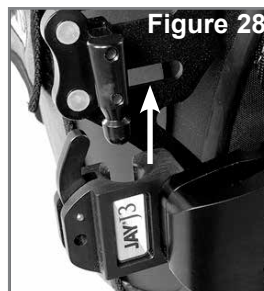
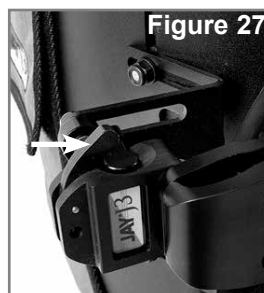
To reattach the back to the hardware receivers, simply line up the pins to the receivers and push into place (Fig. 29). If properly inserted the pins will “click” to a locked position.

⚠ WARNING

If you are uncertain if you have heard the locking “click”, pull up on the back using the handle to ensure that it has been securely locked.

Locking Pins (Fig. 30)

The J3 Back comes with two locking pins which, when secured, prevent the back from being released from the chair. To lock, insert the pins (A) into the mounting receivers (B). Simply remove the pins if the quick-release capability is desired.

CONTRAINDICATIONS:** WARNING**

Do not use or prescribe this backrest for anyone exceeding the stated Maximum User Weight.

RECYCLING / RE-USE:** WARNING**

- Risk of Injury
- Always have an authorized Sunrise Medical dealer inspect the product for damage or wear prior to transferring it to another user. If any type of damage is found, **DO NOT USE THIS PRODUCT.**
- Failure to heed these warnings may lead to severe injury.

This Product is suitable for re-use*:

- Have an authorized Sunrise Medical dealer inspect the shell, mounting hardware, foam and cover for damage or wear. If any damage is found, do not re-use.
- Clean and disinfect the shell, mounting hardware foam and cover in accordance with local government regulations. Refer to your local Infection Control Office for advice.
- After cleaning and disinfection, have a authorised Sunrise Medical dealer re-inspect the shell, hardware, foam and cover to insure that the cleaning/disinfection process did not cause any damage.

 WARNINGS:

- Failure to follow the re-use directions can lead to Severe Injury of the User.
- When being re-used, this product should be adapted and fit to the new user by a authorized Sunrise Medical dealer or clinician.

24 MONTH LIMITED WARRANTY**24 MONTH LIMITED WARRANTY**

Each Jay J3 Back is carefully inspected and tested to provide peak performance. Every Jay J3 Back is guaranteed to be free from defects in materials and workmanship for a period of 24 months from the date of purchase, provided normal use. Should a defect in materials or workmanship occur within 24 months from the original date of purchase, Sunrise Medical will, at its option, repair or replace it without charge. This warranty does not apply to punctures, tears or burns, nor to the back's removable cover.

Claims and repairs should be processed through the nearest authorized supplier. Except for express warranties made herein, all other warranties, including implied warranties of merchantability and warranties of fitness for a particular purpose are excluded. There are not warranties which extend beyond the description on the face hereof. Remedies for breach of express warranties herein are limited to repair or replacement of the goods. In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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Sunrise Medical
North American Headquarters
2842 Business Park Avenue
Fresno, CA, 93727, USA
(800) 333-4000
(800) 300-7502
www.SunriseMedical.com

Sunrise Medical
Thorns Road
Brierley Hill
West Midlands
DY5 2LD
England
Phone: 0845 605 66 88
Fax: 0845 605 66 89
www.SunriseMedical.co.uk

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Sunrise Medical Pty. Ltd.
6 Healey Circuit, Huntingwood, NSW 2148,
Australia
Phone: 9678 6600,
Orders Fax: 9678 6655,
Admin Fax: 9831 2244.
www.SunriseMedical.au

Sunrise Medical Poland Sp. z o.o.
ul. Elekronowa 6,
94-103 Łódź
Polska
Telefon: + 48 42 209 36 67
Fax: + 48 42 209 35 23
E-mail: pl@sunrisemedical.pl
Sunrise-Medical.pl

Sunrise Medical Canada Inc. -
Concord Head Office
237 Romina Drive, Unit 3
Concord , Ontario L4K 4V3
Canada
Telephone:(+1) 905 660 6704 (Administration)
Telephone 2: (+1) 888 703 9021 (Toll Free)
Fax:(+1) 905 660 2460 (Main)

©2011 Sunrise Medical (US) LLC
OM_Jay 3 Back_EU_ML_Rev. 4.0_2015

106109 Rev. G_Jay 3_Back_om.