 BRUGERMANUAL

 USER MANUAL

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## Contents

1. Introduction.....	3
1.1 Oline version.....	4
2. Labeling.....	5
3. Technical information .....	6
4. DK.....	7
5. UK.....	9
6. SV .....	11
7. NL.....	13

## 1. Introduction

Congratulations on your new PositionAid.

This is an original user manual for your PositionAid, positioning cushion.

In the following, we will use the term positioning cushions for PositionAid, unless otherwise specifically stated.

**⚠** It is important that you read the manual before using PositionAid.

PositionAid is a comfortable assistive device that can help healthcare personnel streamline workflows related to providing bedside care. PositionAid helps to secure and stabilize the patient during care. It comes in three different forms.

## 1.1 Oline version

The user manual and other information about PositionAid can be viewed at [www.danrehab.dk](http://www.danrehab.dk)

Scan the QR-code  
and find the manual.

## 2. Labeling

	Manufacturer of the relevant aid. With company name and address.
	Production date
	HMI-number
	The product meets all the requirements for CE marking.
	Medical Devices.
	The user manual must be thoroughly read before use.

### Warning

	Do not use the cushions as a substitute for a pillow.
--	---

### 3. Technical information

**PositionAid** is a cushions series that solves most positioning challenges in bed. It comes in three different forms and sizes. You can buy them individually or as a set with a bag to have them storage in. The complete set:

- 2 Wedge
- 1 Low
- 1 High

In the schedule you can see the different types of PostionAid.

Product name	HMI-no.:	Item no.:	Size	Foto
PositionAid complete set with bag.		7015		
PositionAid Pad Wedge ✓ 30° angel. ✓ Several possible positions. ✓ Can be combined.	133608	7010	50x30x15 cm	
PositionAid Pad Low ✓ Fold it to double height. ✓ Several possible positions. ✓ Can be combined.	133609	7011	50x30x4 cm	
PositionAid Pad High ✓ Really good for leg relief. ✓ Do not move easily. Several possible positions. ✓ Can be combined.	133610	7012	50x30x15 cm	

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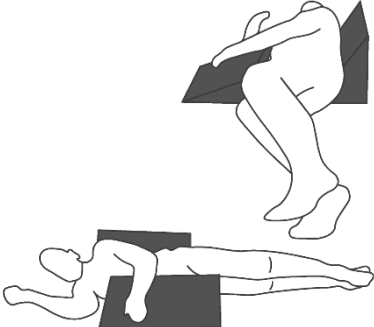
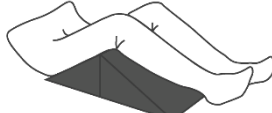
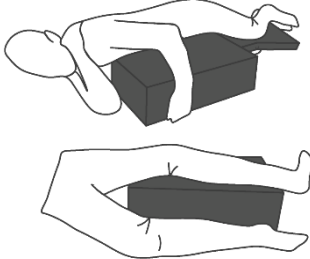
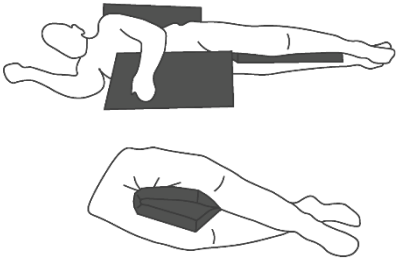
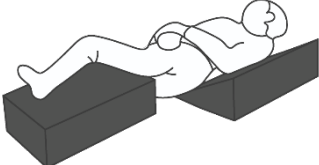
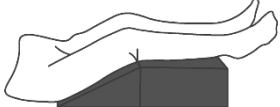
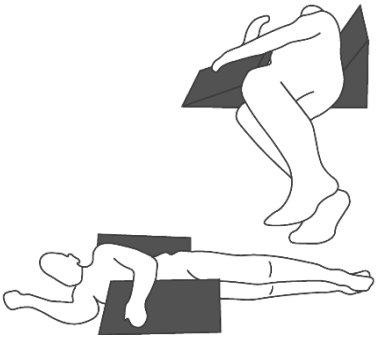
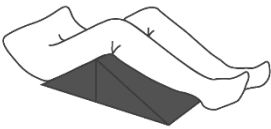
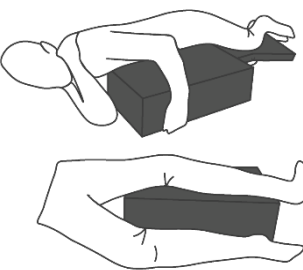
## 5. UK

PositionAid positioning cushions are developed with the purpose of improving the user's comfort. The cushions ensure that the citizens lay stable during, for example, care, and the staff therefore need to use less effort to stabilize the citizens. It spares the shoulders, arms, and back of the care staff. PositionAid can be used both in public settings and in private homes.

PositionAid is available in three versions - **Wedge, Low, High** (see section 4). PositionAid is all made with the same non-slipcover and firm foam filling - providing stability and comfort to the user.

### Application

The three different PositionAid can be used alone or together with each other during care for the user. Below are some examples of how the three different PositionAid: **Wedge, Low, High** can be used as an aid in providing care to the user.

<p>Wedge with its 30-degree angle can, among other things, be placed behind the citizen's back to provide support and ensure that the citizen does not fall backward during lower hygiene in bed. Can also be placed in front of the citizen, so they do not fall onto their stomach. This way, the care staff have both hands free to work.</p>	
<p>If a knee bend is desired for the resident, one can, among other things, combine two Wedges together.</p>	
<p>Pad high can, among other things, be placed in front of the citizen so they do not fall onto their stomach. Pad high can help relieve the citizen's legs as well as heels.</p>	

<p>Pad low can, among other things, be used between the legs for lower hygiene – the pad can be folded in half to achieve double height. Everything according to the wishes and needs of the individual resident.</p>	
<p>A combination of Wedge and Pad high is optimal in a 30° position. Wedge supports the citizen's back, and Pad High supports the citizen's legs.</p>	
<p>A combination of Wedge and Pad High can together help relieve pressure on the citizen's heels.</p>	

### Cleaning

The non-slipcover is breathable and waterproof – making it easy to clean with a wrung cloth and soapy water.

	<p>Tolerate hand washing.</p>
	<p>Does not tolerate bleaching.</p>
	<p>Do not dry clean.</p>
	<p>Does not tolerate tumbling drying.</p>
	<p>Does not tolerate ironing.</p>

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