## Mobilex rollator "IMPALA"

Art. 312060, 312061 & 312062

**Montage- und Gebrauchsanleitung**  
**Montage- og brugervejledning**  
**User Manual**  
**Manuale d'uso e manutenzione**  
**Installations- och bruksanvisning**

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. These user instructions contain a detailed product description and important guidelines for correct use of the product. You must read the instructions carefully before using the product. Please always observe all guidelines, in particular the security instructions, in order to use the product safely for a long time.

### Intended use of the device

The Mobilex rollator "Impala" is medical device which has been designed to increase the mobility of persons who have difficulties standing or walking. The rollator enables those persons to regain the ability to perform daily chores. The rollator is suitable for persons who as a result of various diseases need stable walking support. The rollator should only be used as a support for standing, walking and sitting. Any other use is prohibited.

### Indications

The device is indicated for use as walking, standing and seating support by persons who suffer different diseases of human motion system. The variety of such diseases is very wide but the symptoms are similar: people who because of their health status are unable to walk, stand or sit independently are indicated to use rollators. The final decision to choose the correct type of device depends on condition of individual patient and should always be taken by professional physician or physical therapist

**NB** The rollator should not be used by persons with balance problems whose muscles are severely weakened, thus unable to stand or walk on their own. The rollator should be recommended by a physiotherapist or a doctor.

If you have any questions, please contact your dealer or Mobilex A/S.

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Phone: +45 87 93 22 20  
Telefax: +45 87 93 17 77  
Email: info@mobilex.dk

## 2. Security instructions

- Use only the product if it is technically flawless
- In case of any product damage or faults please contact your dealer immediately
- Observe the data on the product label
- Use the product only for the described purpose
- Please notice that there is a risk of injury at all joints on the rollator
- Always check before use that the wheel brakes function perfectly
- Use the rollator only when it has been fully unfolded and the locking device has been activated
- The rollator should only be used on a flat stable surface, indoors or outdoors
- The seat may only be used when the brakes have been firmly locked
- Both brakes should always be activated or locked simultaneously
- Max. load of the rollator is 150 kg in total
- Max. load of the shopping bag is 5 kg
- It is not allowed to carry an extra person on the seat
- The product must not be changed without the manufacturer's written permission
- The rollator must always be braked when not in use

## 3. Description

### Product description:

The Mobilex rollator "Impala" has a powder coated steel frame with 4 wheels. The light-weight frame makes it easy to fold it together and offers easy storage and transport. The rollator runs easily and is sufficiently stable. The height of the underarm supports can be adjusted. The rollator is delivered with a mounted seat, crutch holder, back support and shopping bag.

### Parts supplied and mounting instructions

The Mobilex rollator "Impala" is delivered in a carton with the following content:

- ✓ 1 rollator including a mounted seat, push handles with brake cables, crutch holder, back support and shopping bag
- ✓ This user manual

### Control upon delivery

After unpacking the rollator please check if anything is missing-if all parts shown in the drawing have been delivered. If any parts are missing, please contact your dealer immediately.

### Unfolding the rollator

- Put the main frame with wheels on a flat surface. All wheels should touch the floor.
- Press the frame reinforcement rod downwards until the frame is fully opened. Push the seat pad down so it is in horizontal position.
- Always ensure the frame is stable and fixed when the rollator is opened.
- Insert the lower ends of push handles into holes at the top of the handle tubes as shown on the picture above.
- Adjust both push handles to the same height. Ensure the fixing holes in the push handle and the handle tube are parallel.
- Attach the fixing bolt through the parallel holes on both sides of the frame.
- Make sure that the fixing knobs (2 pieces) are securely locked so that the rollator is stable and parts can not move when forced.
- Attach the back rest. Insert the back rest tube ends into back rest attachments located on main frame as shown in the picture. You should hear a click sound to ensure that the back rest is safely locked.
- Always test to see that the rollator and attachments are properly and safely locked BEFORE using.
- Failure to follow these warnings can result in personal injury.
- To fold the rollator lift the seat pad first. Then pull the frame reinforcement rod upwards. The frame should fold.

### Function description

As the frame can be clapped together the rollator can fold, and as a result it requires very little space. Push handles can be freely adjusted to fit the size of your body. The seat allows you to rest.

- Adjust the height of the push handles so that when the user's arm is down along the body, the handles are at wrist height. Adjust the height by releasing the fixing knob located on the handle tubes and pull the bolt out of its place. Adjust the supports to the desired height, then have the bolt go through the holes and tie the knob securely. Repeat the for the other handle, so that both are at the same height.
- Care should be taken to ensure that ALL adjustments are secured and that castors and moving parts are in good working order before using.
- To stop the rollator simply pull the brake handles up towards hand grips and hold brake handles in the UP position to remain stationary. TO MOVE simply release the brake handles.
- To lock the brakes in engaged position for longer period of parking, push the brake handles down until you can hear the click sound. To unlock the brakes pull the brake handles UP until you can feel them loose.
- While using the rollator in a stationary position, the hand brakes MUST always be locked. You should always engage the brakes when you don't plan to move the device. Never leave the device without the brakes engaged when there is someone sitting in it! Never leave device on slopes without brakes engaged!

### Accessories

For the Mobilex "Impala" rollator there is a shopping basket available with item no. 312073 that can be added into the existing bag. Please contact your dealer for further information.

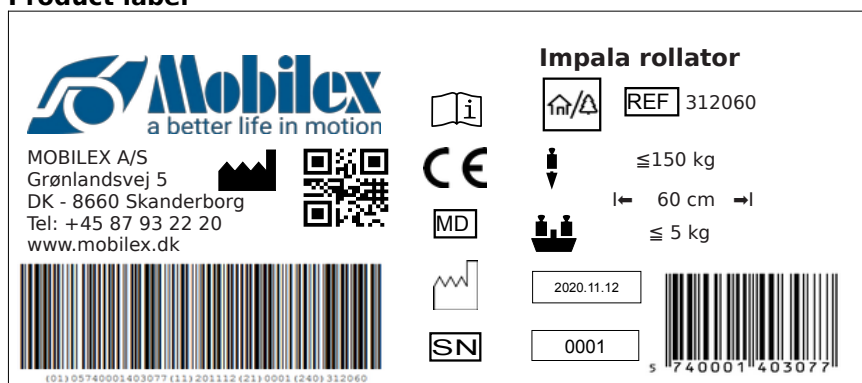
### Transport recommendations

It is recommended to transport the rollator in the folded position. Its construction allows easy unfolding and folding. You should check that the rollator is in a stable position during transport to avoid any deformation.

### 4. Technical data

Mobilex rollator Impala	Standard Item no. 312060	High Item no. 312061	Wide Item no. 312062
Height of the push handles (adjustable)	79 - 93 cm	90 - 105 cm	79 - 93 cm
Seat height	53 cm	63 cm	53 cm
Total width opened / folded	60 / 27 cm	63 / 35 cm	66 / 27 cm
Inner width between push handles	48 cm	49 cm	53 cm
Length (depth)	70 cm	75 cm	70 cm
Weight	7,4 kg	7,9 kg	7,5 kg
Max. load (rollator / bag)	150 kg / 5 kg	150 kg / 5 kg	150 kg / 5 kg
Brakes	2 lockable with parking function		
Wheel sizes	Front and back 200 x 35 mm		

### Product label

Product label for Impala standard 312060

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

### 5. Maintenance and care

Normal dirt can be removed with standard cleaning agents and a sponge or soft rag. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (do not use alcohol-containing products and no solvents or abrasives).

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary
- ✓ General retightening of the product

**Recommended service life:** Normally 10 years depending on use.

### Cleaning the wheels

It is recommended to clean the wheels in hot water with a mild cleaning agent. Lubricate the bearings with oil periodically or when the wheels start squeaking.

### Warning

If the product is exposed to very calcareous water, a binding and accumulation of lime on wheel bearings and other moving parts may occur. In this case, the product can be cleaned with a limescale remover and then lubricated with lubricating oil.

### Warning

When used in highly chlorinated environments extra strong corrosion may occur and the warranty will be void.

### Warning

Please note that if the frame is bent or broken, the product must not be recycled but must be replaced

### Storage

We recommend storage in a dry place.

### Transport recommendations

The device should be transported in a stabilized and secured way with no risk of sudden or dangerous movements during transport.

### Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

## 6. Troubleshooting

<i>Problems</i>	<i>Possible Causes</i>	<i>Solutions</i>
<i>The rollator is difficult to guide or push</i>	Dirt between wheels and forks	Remove dirt or hair between wheels and forks
<i>The brakes do not work or work only in one side</i>	The brake shoes are worn, or the brakes are not adjusted correctly	Adjust the brakes. Replace the brake shoes if needed (→ service centre)
<i>The seat does not fit in when the rollator is unfolded</i>	The locking device is in a wrong position or defective	Check that the locking device is in a correct position. Replace it if needed (→ service centre)

## 7. Disposal of the product

The product may not be disposed of with household waste but must be brought to the local recycling site.

## 8. Guarantee

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

## PRODUCT LIST

**Impala Rollator** Item numbers: **312060, 312061 & 312062**

## ACCESSORIES LIST

Item no.	Accessories item no.
312060, 312061 & 312062	312073

Harmonized norms used during conformity estimation:

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**In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /**

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