



CityCaddy operating instructions

PLEASE READ CAREFULLY BEFORE USE
AND KEEP IT FOR FUTURE REFERENCE





Dear user,

Thank you for choosing CityCaddy. Your companion in everyday life as a shopper, trolley and walking aid. Even when traveling, it promotes posture and helps to create a good quality of life.

To ensure that you enjoy walking with the CityCaddy, please read the operating instructions completely before using it.

In order to become familiar with the CityCaddy, it is recommended that you practice using it in a familiar and risk-free environment.

CityCaddy – Technical Description

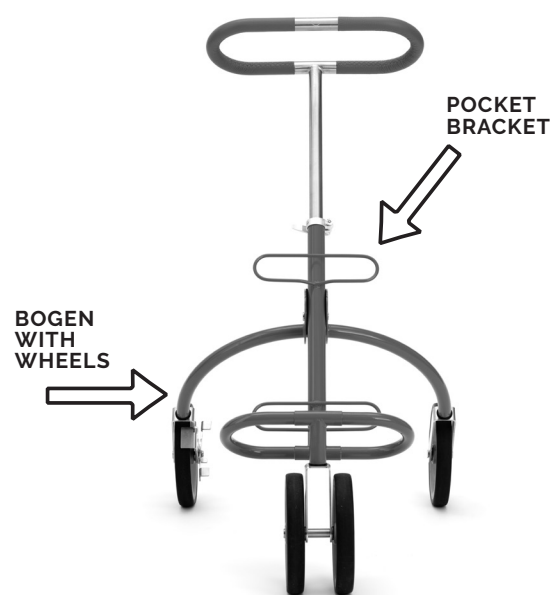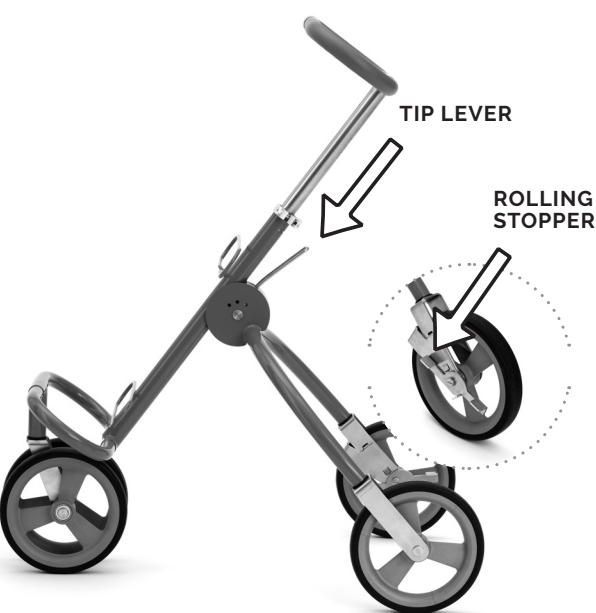

Commissioning



1. Take the CityCaddy on the center pole out of the box..



2. Hold the CityCaddy with both hands in front of your body, ...




3. so that you can reach the tipping lever with one hand.

2. SET UP AND PLACE IN STOP POSITION




1. Press the tipping lever briefly once to let move the bow with back wheels away from the center bar into the walking position.

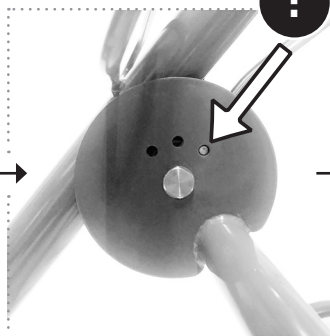


2. See how to recognize the correct position for sliding forwards.



3. Your CityCaddy is now ready to use.

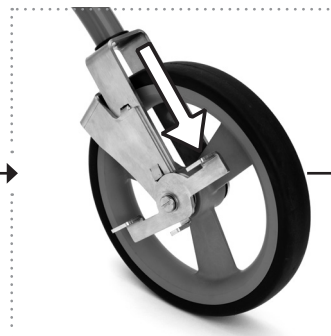


4. To prevent the CityCaddy from rolling away, press the roll stopper with your foot...

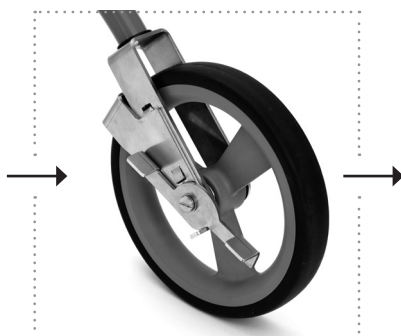


5. to the braking position.


3. ADJUST HANDLE HEIGHT

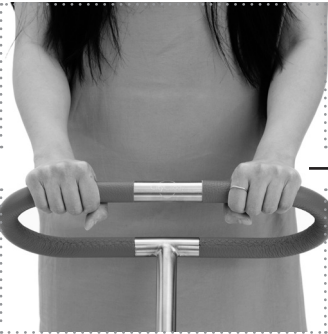


1. To adjust the height of the handlebar to your comfort ...

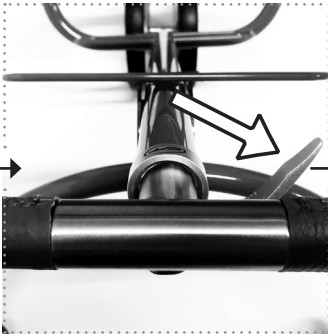


2. please loosen the quick release ...

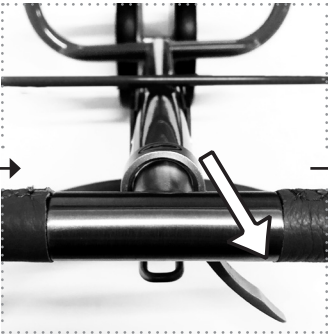


3. by moving it horizontally to the right.

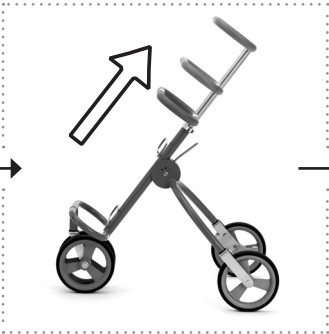

4. Pull the handlebar up until it reaches approximately one hand wide below your belly button.

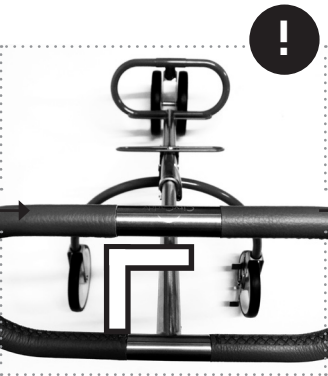

5. Align the handlebar exactly at a right angle to the wheels.

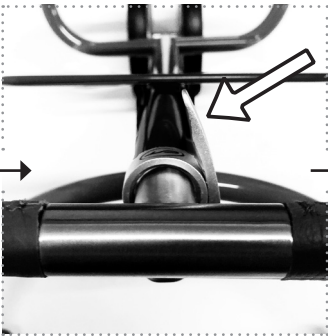

6. Lock the quick release by pressing it firmly down.




7. CHECK IF THE QUICK RELEASE IS TIGHT ENOUGH BY LEANING FIRMLY ON THE HANDLEBAR.

The heights of the handlebar is ideal when one hand wide below your belly button

Continuation of commissioning


4. ATTACH THE BAG

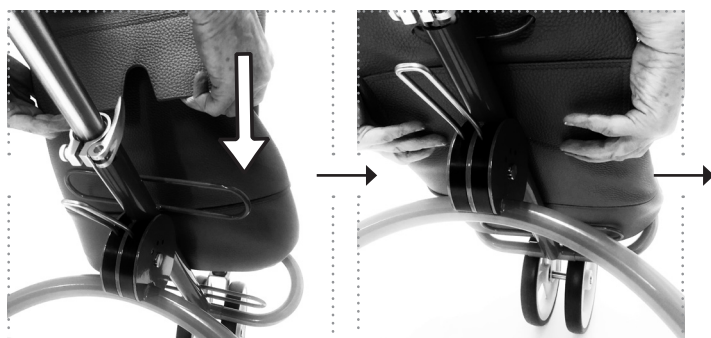

1. Now attach your CityCaddy bag by using the back flap ...

2. and slip it over the bag handlebar at the CityCaddy.

5. RELEASE THE ROLL-AWAY STOPPER

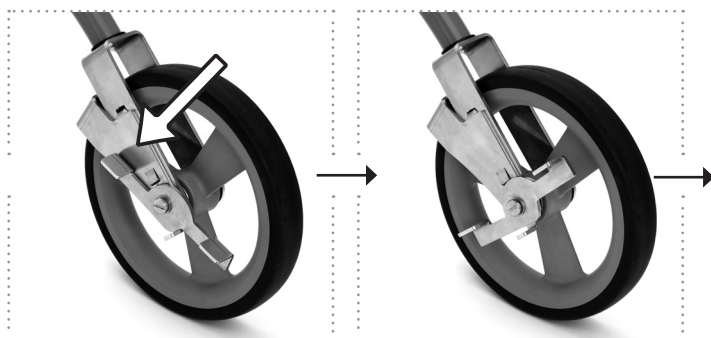

1. Before starting the journey: Press the roll stopper with your foot ...

2. to the driving position.

6. PUSH OFF



1. Now your CityCaddy is ready for the first ride.


Voilà, done!
Enjoy the
movement.

In use


On www.citycaddy.de a short video guide is provided to show the pushing and pulling.



1. PUSHING POSITION

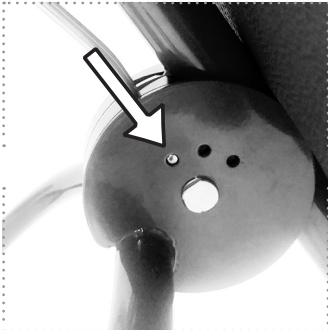


1. How to recognize the correct position for sliding forwards the CityCaddy.

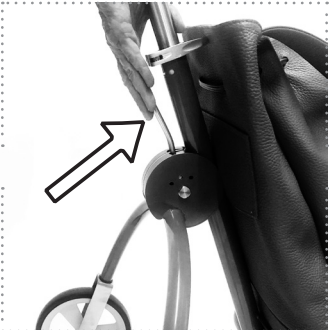

2. This allows you to reach the sliding position at any time: Press the tipping lever briefly to let moving the wheel arch away from the enter bar into the driving position.



2. PULL

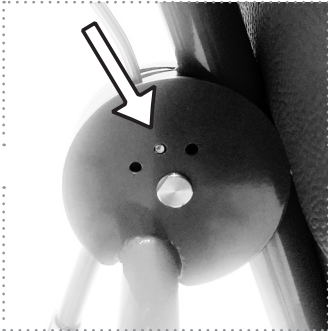

1. How to recognize the correct pulling position of the CityCaddy.



2. How to change from sliding to pulling position: Press the tipping lever briefly to let moving the wheel arch towards the enter bar until the CityCaddy is standing stable again (a "click" sound is heard).



3. STANDING (PULLING POSITION PLUS PRESSED ROLL STOPPER)



1. How to recognize the stop position of the roll stopper.




2. How to recognize the driving position (firmly pressed).




4. TRANSPORT OR STORAGE




1. To prevent the CityCaddy from rolling away, press the roll stopper with your foot.

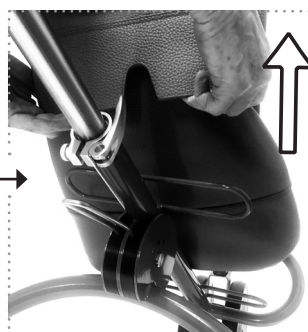


2. Now remove your bag and put it aside.

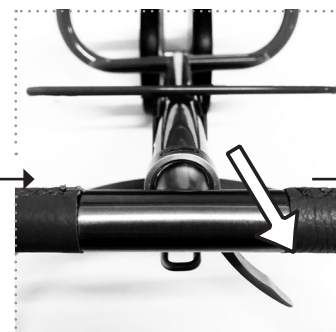


3. Loosen the quick release.

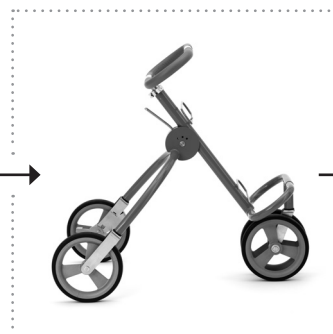


4. Push the handlebar down.

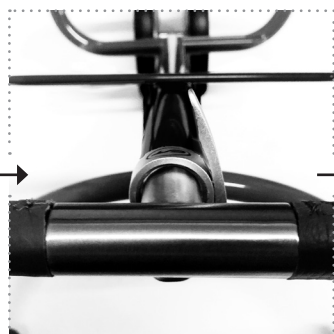

5. Straighten the handle again and close the quick release firmly.

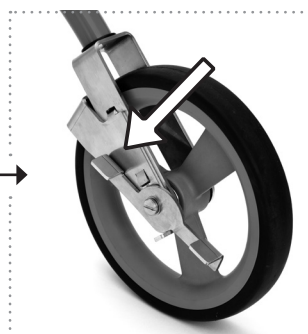


6. Release the roll stopper.

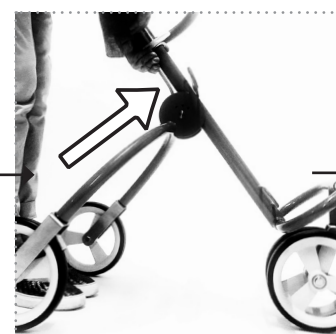


7. Press the tipping lever briefly to move the wheel arch completely towards the enter bar (a "click" sound is heard).



8. Now front and rear wheels are aligned.

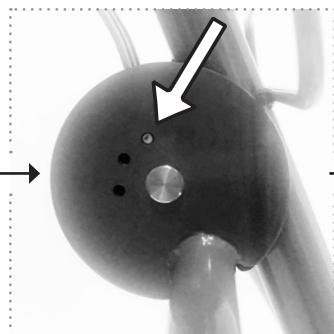


9. How to recognize the correct position.

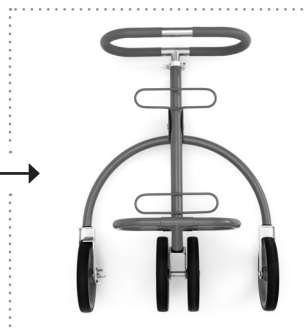

10. Now the CityCaddy is ready for transportation (e.g. in a trunk or closet).

Safety instructions

CURB

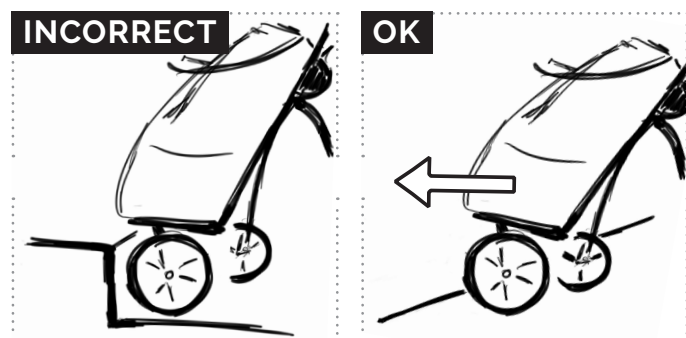

When crossing from the sidewalk to the street, always use a lowered access, not a high curb!

HEIGHT DIFFERENCES

Height differences should always be approached at right angles. In case of strong altitude drop use the pulling position (see page 6).

STAIRS



To climb stairs with the CityCaddy, adjust the pulling position (see page 6) and approach the steps centrally when ascending or descending.

Continuation of Safety Instructions

NOTE WHEN USING



ROLL STOPPER

Use the roll stopper when taking a break, e.g. leaning on the handle, riding in a bus.



QUICK RELEASE

The quick release must always be firmly closed when using the CityCaddy.



SECURE FOOTING

To stand safely, all wheels of the CityCaddy must touch the ground.



SMALLEST POSITION


Caution, in the folded position the CityCaddy does not stand on its own!

ALSO IMPORTANT:

- ! Fill the bag properly by placing heavy items at the bottom.
- ! Please note that heavy loads increase the risk of tipping.
- ! The CityCaddy is a lifestyle product, **not a medical device**.
- ! The use of the CityCaddy is not recommended in case of severe perception or balance disorders.

ATTENTION: On sloping terrain, the rolling speed can be higher than your own walking speed and this is increased even more when the CityCaddy is loaded.
Danger of falling!

Care instructions

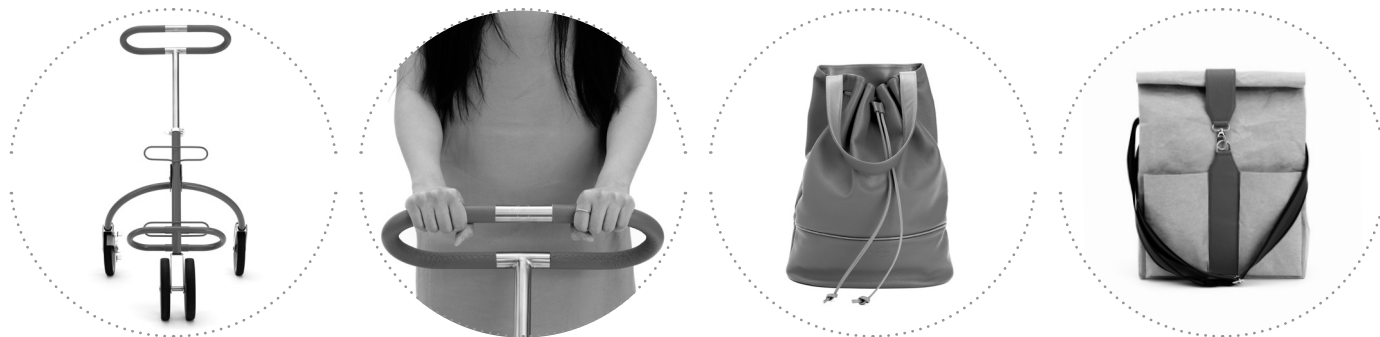

CITYCADDY FRAME AND HANDLEBAR

- If the CityCaddy frame or the wheels are dirty, please clean with water, soft detergent with a cloth. Then dry thoroughly with a soft cloth.
- If you get caught in a rain shower with the CityCaddy, allow the leather on the handlebar to dry slowly after using a soft cloth if necessary.

CITYCADDY BAGS

LEATHER BAGS, E.G. CC.PARIS

- Dust the CityCaddy bags with a soft, dry cloth.
- If you get caught in a rain shower with the CityCaddy, allow the leather on the handlebar to dry slowly after using a soft cloth if necessary.

BAG MADE OF WASHABLE PAPER WITH LEATHER TRIM, EG CC.MILANO

- Dust the bag with a soft, dry cloth.
- Stains can be removed with water, mild detergent and a cloth in cleaned with gentle movements.

Technical description / materials / dimensions



CITYCADDY FRAME

- CityCaddy material: stainless steel / painted steel
- CityCaddy handle: stainless steel with leather covering, hand-stitched
- Wheels: diameter 19 cm with ball bearings
- Weight: approx. 6.5 kg
- Dimensions in use: depth 60 cm / width 54 cm / height 100 cm
- Dimensions when folded: depth 30 cm / width 54 cm / height 80 cm
- Support load: max. 100 kg

CITYCADDY BAGS

CC.PARIS. BIG BUCKET BAG, COWHIDE LEATHER

- Chrome-tanned, aniline-dyed leather from European sources
- Lining material of the leather bag: Dinamica® MIKO microfiber fabric
- Bag dimensions: approx. height 46 cm / width 35 cm / depth 17 cm
- Load capacity: approx. 15 kg

CC.MILANO. ROLLIN' BAG, WASHABLE PAPER / COWHIDE / WEBBING

- Paper-plastic mixture consists of more than 60% cellulose.
The remaining ingredients are color pigments and latex.
The cellulose is obtained from trees from sustainably managed forests in Northern Europe.
- It is 100% vegan.
- The material does not contain pentachlorophenol, PVC or BPA and is neither harmful to the environment nor to health.
- Bag dimensions: approx. height 46 cm / width 35 cm / depth 17 cm
- Load capacity: approx. 15 kg

The bags are made of natural materials. Therefore, slight color variations may occur.

Guarantee

The warranty corresponds to the legally required 24 months from the date of purchase.



Warranty services include defects in the product that can be proven to be due to material or manufacturing defects. Damage caused by natural wear and tear, intent, negligent or improper operation or use is excluded from the warranty.

The guarantee is also void if unsuitable cleaning agents are used.

The CityCaddy is sustainable, stable, durable, repairable, recyclable.

Wear parts such as handlebars and wheels can be replaced.

Disposal

The product is made mostly of steel, supplemented by small plastic components. Only the wheels are made of mixed materials.

If you are unsure how to handle the recycable material, please contact your local authorities as different municipalities/recycling companies use different recycling methods.

CityCaddy Contact

CityCaddy UG has limited liability
Stresemannstraße 90
22769 Hamburg
Germany

Phone: +49 (0)176-32485338
E-Mail: info@citycaddy.de
www.citycaddy.de

Commercial Register: HRB 158213
Register court: Hamburg District Court



WWW.CITYCADDY.DE