

DESIGNED AND DEVELOPED IN NORWAY

Letsip® Pack 2 S-series

Letsip® S-Series:

Letsip® S1 width 20mm/.78"

Letsip® S2 width 25mm/.98"

Letsip® S3 width 45mm/1.77"

Letsip® S4 width 55mm/2.16"

Letsip® S5 width 65mm/2.55"



Use it, don't lose it!



Exercising the muscles in and around the mouth

www.letsip.com

www.letsip.com/2



Scan me

Exercising the muscles in and around the mouth

www.letsip.com

Letsip® S-Series:


Letsip® S1 width 20mm/.78"

Letsip® S2 width 25mm/.98"

Letsip® S3 width 45mm/1.77"

Letsip® S4 width 55mm/2.16"

Letsip® S5 width 65mm/2.55"

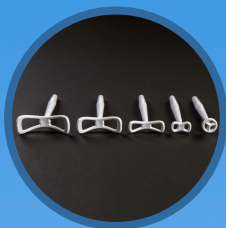

Letsip® S-series:

The different sizes of the Letsip S-series provide different exercises for the cheeks and lips. The biggest mouth piece stimulates the large parts of the cheek, whilst the smallest mouth piece, exercises the lips whilst stabilising the cheeks. Exercising mouth closure, stimulating cheeks and all musculature attached to lip musculature.

Cleaning and disinfecting:

We recommend that you lightly wash all products before use, excluding the Letsip® handle piece. All the pieces, excluding the handle, are dishwasher safe, up to 158°F / 70°C. The Letsip® handle piece is water resistant and can be disinfected with wipes or a light spray. Disinfecting your Letsip products is best done with LifeClean. We recommend using LifeClean, as it does not cause any visible damage to Letsip products. LifeClean administers the highest level of disinfection, coming into full effect within two minutes of proper application.

NOTE!

Cleaning Letsip products with spirits or washing the plastics in high temperatures, can change the shape and strength of the product over time. Disinfecting with alcohol may also over time, cause the discolouration of the handle. Follow the cleaning guide at www.letsip.com

WARNINGS & PRECAUTIONS

Keep out of the reach of unsupervised children. Caution is recommended when using Letsip products around the mucous membranes in the mouth, and on exposed and/ or irritated skin. Vibration on the throat is not recommended. Always check the product before use and discard the product at the first signs of damage or weakness.



DESIGNED AND DEVELOPED IN NORWAY

2 year guarantee



Letsip AS

Langelandsvegen 51, N-6010 Aalesund Norway
Phone +47 480 09 500 post@letsip.com www.letsip.com

PATENTED All products are protected by the international patent application



Material ABS Dow USA. Ball and string Surgical steel. Made in P.R.C.