

# FloorLine "gulv senge"



## FloorLine sengen er udført i 3 modeller



### FloorLine LTC

anvendes typisk i hjemmeplejen, institution og plejehjem.

Køre fra 9 til 70cm uden madras.



### FloorLine -i plus

anvendes typisk på institution og plejehjem.

Køre fra 10 til 80cm uden madras.



### FloorLine-i Plus

anvendes typisk på hospitaler.

Køre fra 10 til 80cm uden madras.

FloorLine sengene er alle udstyret som en moderne plejeseng, bare med den ekstra fordel, at de kan køre helt ned til gulvet!

Det betyder at sengefald bliver til et "sengerul" hvilket stort set fjerner risikoen for skader.


FloorLine reducerer behovet for opsyn. Det betyder mindre arbejdsbyrde for plejere.

Øget sikkerhed for bruger, plejepersonale og arbejdsgiver gennem reduktion af risiko.


FloorLine sengen er det rette valg til brugere, som:

- Risikerer at falde ud af sengen.
- Har ukontrollerede bevægelser.
- Har problemer med at løfte benene.

## Risiko for skader ved fald fra seng

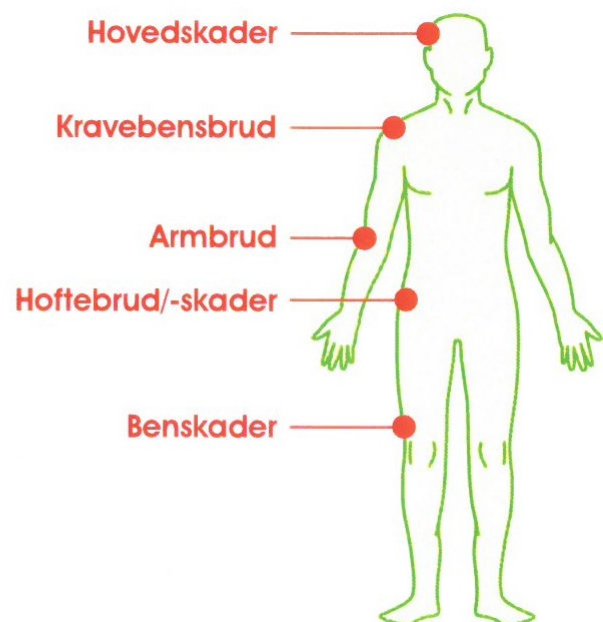

# Teknisk information FloorLine LTC

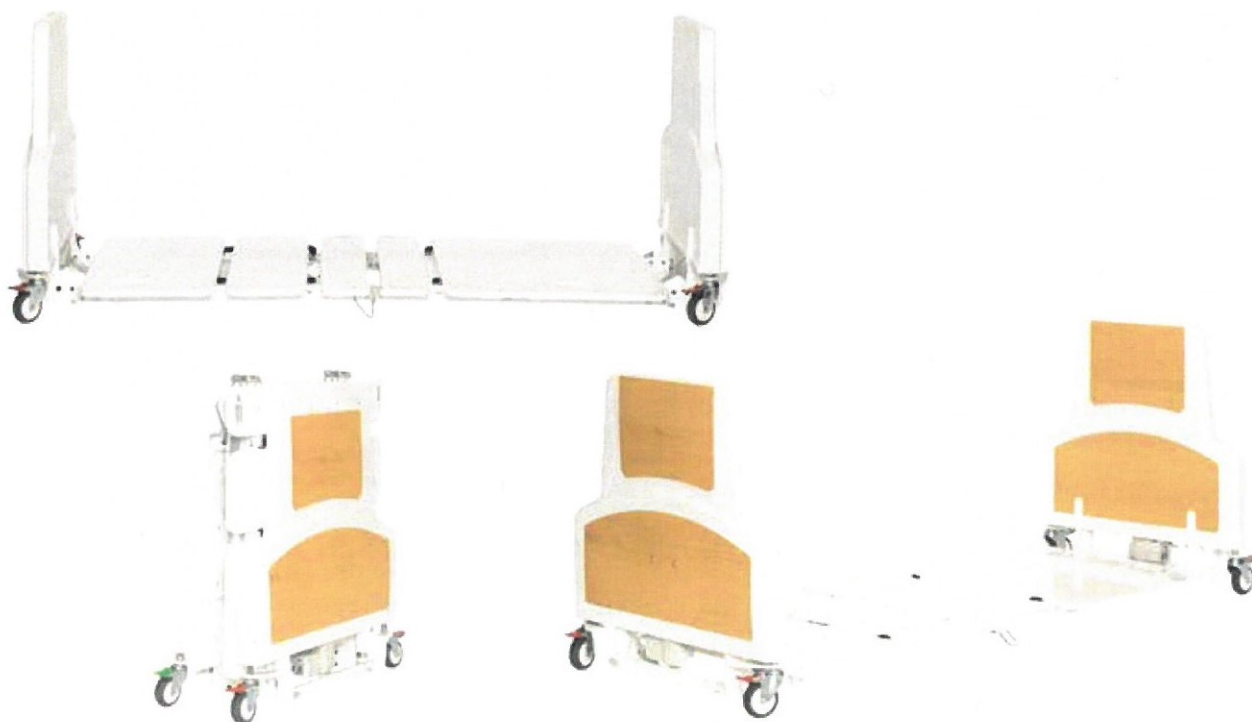


Part No.	Platform Position min- max	Sleeping Dimensions L x W	Bed Dimensions L x W	Functions	Compliance Standards	Maximum User Weight/ Safe Working Load	Product Weight
Europe: 90600 UK: 90602 USA: 90604 USA class 1: 90606 Australia: 90608 Europe with wooden end boards: 90601 UK with wooden end boards: 90603 USA with wooden end boards: 90605 USA with wooden end boards: 90607	9.5 cm - 70 cm [3.7" - 27.5"]	2000mm x 900mm [78.7" x 35.5"]  With Extension Kit 2175mm x 900mm [85.5" x 35.5"]	2150mm x 905mm [84.5" x 35.5"]  With Extension Kit 2325mm x 905mm	Auto Contour Button for Cardiac Chair Position, Handset w Lockout (Batteribackup as accessory, part no. 259051)	IEC 60601-2- 52:2009 EN1970: 2002 AS/NZS 3200.2:38: 2007	Maximum user weight: 150kg [330lbs]  Safe working load: 200kg (440lbs)	106 kg (234 lbs)



FloorLine LTC kan nemt foldes sammen.



## FloorLine-i

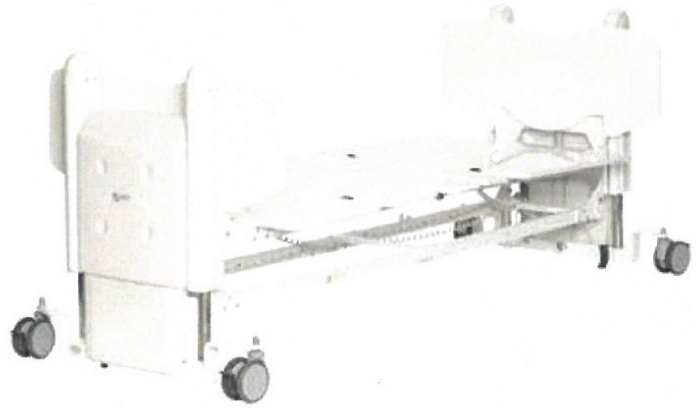

Part No. (including hand control and end boards)	Platform Position min- max	Sleeping Dimensions L x W	Bed Dimensions L x W	Functions	Compliance Standards	Maximum User Weight/ Safe Working Load	Product Weight
Europe: 90300 Australia : 90301 USA: 90302 UK: 90303	10cm - 80cm (4" - 31.5")	2000mm x 900mm (78.7" x 35.5")	2325mm x 925mm (91.5" x 36.5")	Twin castors Battery Back Up, Auto Contour Button for Cardiac Chair Position, Hand- set w/ Lockout	IEC 60601-2-52:2009 EN1970:2002 AS/NZS 3200.2.38:2007	Maximum user weight: 185kg (407lbs)  Safe working load: 250kg (550lbs)	138 kg (303 lbs)

## FloorLine-i Plus



Part No. (including hand control and end boards)	Platform Position min- max	Sleeping Dimensions L x W	Bed Dimensions L x W	Functions	Compliance Standards	Maximum User Weight/ Safe Working Load	Product Weight
Europe: 90500 Australia 1: 90501 Australia 2: 90502 USA: 90503 UK: 90504	10 - 80 cm (3.9" - 31.4")	2000mm x 900 mm (78.7" x 35.4")	2325mm x 925mm (91.5" x 36.4")	Twin castors Battery Back Up, Auto Contour Button for Cardiac Chair Position, Handset w Lockout CPR function	IEC 60601-2- 52:2009 EN1970:2002 AS/NZS 3200.2.38:2007	Maximum user weight: 185 kg (408 lbs)  Safe working load: 250 kg (551 lbs)	140 kg (309 lbs)




# Accessories

## Hanger for air mattress pump

Attachment to hold air mattress pump. To be mounted on the footend of the bed.

Part no.: PUMPBKT

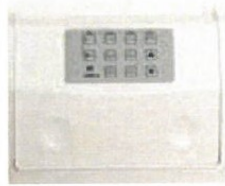

## Care Assist Rail

Designed to help patients safely in and out of bed.

H: 380 mm

B: 250 mm

Part no.: 90702



## Push handle

The push handle can be mounted on the foot end of the bed to improve the operation of the bed.

Part no.: 90708 (Included in the product for FloorLine-i Plus, accessory for FloorLine-i)



## IV pole (adjustable in height)

Adjustable IV pole that can be mounted on both ends of the bed.

Part no.: IVP



## Extension kit

Extension kit that extends the bed, with 175 mm to 2175 mm to accommodate more people.

Part no.: 90709 (FloorLine LTC)


Part no.: 90701 (FloorLine-i and FloorLine-i Plus)



## Oxygen Holder

An attachment for placing an oxygen bottle in C size.

Part no.: 90707

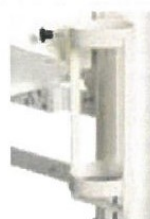

## Self help pole

The self-help pole helps patients to get up in the bed and change positions.

Part no.: SHP600 (FloorLine LTC)

Part no.: 90700 (FloorLine i and FloorLine-i Plus)



## Side rails

1/2 side rail

Side rail extends 64mm above bed platform in lowest position.

H: 345 mm

B: 820 mm

Part no.: 72791




## Side rails

1/4 side rail

H: 345 mm

B: 410 mm

Part no.: 72790

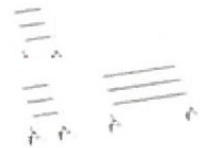

## Side rails

Split side rail (1/2 + 1/4 length side rail)

Side rail extends 64 mm above bed platform in lowest position.

H: 345 mm

B: 820 + 410 mm

Part no.: 72792



## Side rails

3/4 side rail

Side rail extends 64 mm above bed platform in lowest position.

H: 345 mm

B: 1390 mm

Part no.: 72793



## Bumper

The bumper protects the head of the bed and are mounted on the lower support bracket between the wheels.

Part no.: 90705 (FloorLine LTC)

Part no.: 90706 (FloorLine-i and FloorLine-i Plus)



## Swing-away trapeze

The free-standing swing-away trapeze allows patient to reposition themselves. No tools needed for assembling.

Part no.: 896

JØHL HUMAN CARE- Firskovvej 6- 2800 Kgs. Lyngby- telefon 3920 2007-

[info@johlhumancare.dk](mailto:info@johlhumancare.dk)- [www.johlhumancare.dk](http://www.johlhumancare.dk)

**JØHL**



**JØHL**



Jøhl Human Care – Firskovvej 6 – 2800 Kgs. Lyngby – Telefon 3920 2007

[info@johlhumancare.dk](mailto:info@johlhumancare.dk) – [www.johlhumancare.dk](http://www.johlhumancare.dk)

