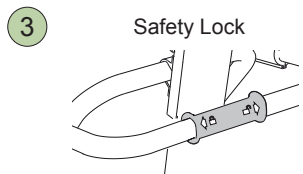
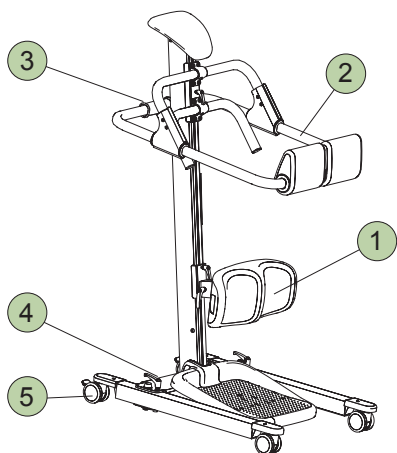
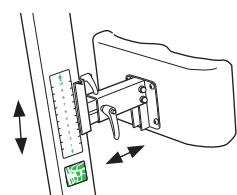
RollOn

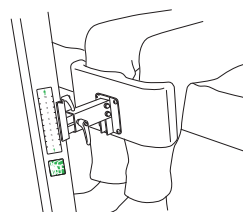
Quick Reference Guide

1 Lower-leg Support


Adjustment
The height and depth of the lower-leg support are adjustable.


Try-out
Adjust the height of the lower-leg support so that it provides comfortable counter pressure just beneath the patient's kneecaps. If needed, the lower-leg support can also be vertically adjusted.

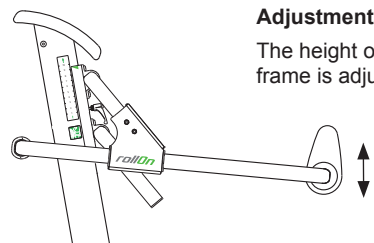2 Seat Frame


Adjustment
The height of the seat frame is adjustable.

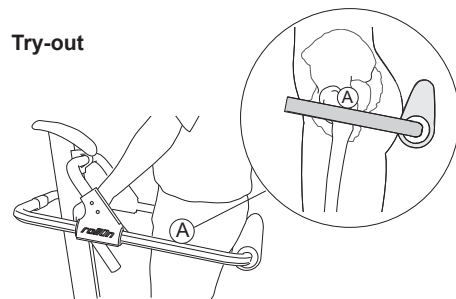Try-out


To prevent the patient from sitting on the rear supports and to prevent him/her from sliding down and out of the support, the frame should be set just under the patient's hip joints (A).

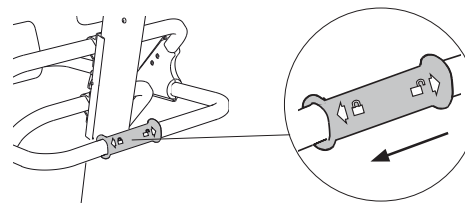3 Safety Lock


⚠ The safety lock should be in the locked position during transfer.

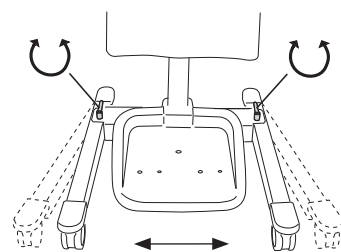4 Base

5 Locking the Wheels

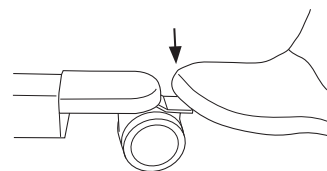⚠ Lock the wheels during raising.

Areas of Use:

- to/from bed and chair/wheelchair
- to/from chair/wheelchair and toilet
- repositioning in a chair/wheelchair.

The RollOn bridges the gap between conventional walkers and powered sit-to-stand lifts.

Problems during Use

If the patient does not achieve a sufficiently upright position, we recommend a lift from Liko's product range; the first choice being the sit-to-stand lift Sabina.

Ergonomics

- Do not lift manually. Let the patient and RollOn do the job.
- Take it easy. Work calmly and methodically
- Don't strain your back. Work in a walking-stance with your back straight
- Don't twist your back. Move your feet
- Reduce the lever effect. Work close to the patient and the support.

Cleaning

When necessary, clean RollOn with a moist cloth.
 For more information read the Instruction Guide.

Other Quick Reference Guides

Liko's Quick Reference Guides are available for downloading at www.liko.com.

© Copyright Liko AB

A Hill-Rom Company

www.liko.com

Manufacturer:

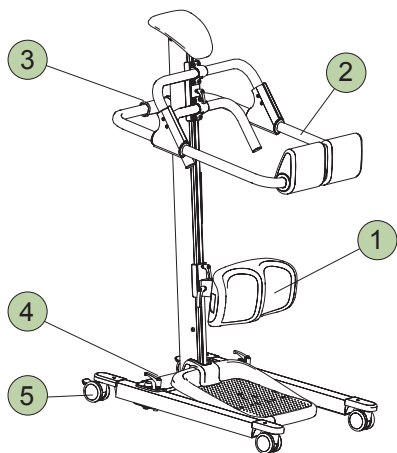
Liko AB
 Nedre vägen 100
 SE-975 92 Luleå
 Sweden
info@liko.se

IMPORTANT!

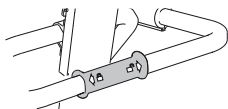
This Quick Reference Guide do not replace RollOn's Instruction guide, which can be downloaded from www.liko.com.

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