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Congratulations on your new hip protector, the WonderHip™. WonderHip™ wears comfortably and is near invisible under your garment. WonderHip™ provides safety and considerably reduces risk of hip fractures. Due to the comfortable shorts and the excellent shape, WonderHip™ can be worn all day and even through the night.

HERE WE GIVE SOME ADVICE ON HOW YOU BENEFIT THE MOST FROM YOUR WONDERHIP™.

- The cushions must be placed inside the pockets, one on either side of the shorts. The curved side of the cushion is out away from the body, cf the fig. 1.
- The shorts may be used instead of ordinary under wear, or outside it – your choice. On the inner upper lining the word **WONDERHIP** is woven into the fabric. This is the front side of the shorts.
- WASHING INSTRUCTIONS:** Remove the cushions before washing. The shorts are best washed at 60°C with ordinary house detergent. They can be spin-dried and tumbled dry. The cushions can be hand-washed and spin-dried or, better, air-dried. However, this might require 24 hours. **NB! Cushions must not be treated with textile softeners! Neither may they be cleansed with alcohol or other solvents!** Just use ordinary household detergents and rinse well with water.

- STORAGE:** The cushions to be kept out of direct sun shine. Use the box for storage.

We hope you will be satisfied by your new WonderHip™. Feel free to contact us on any issue regarding WonderHip™. At www.wonderhip.com you can find other products that can make life more comfortable.

PRODUCER: Vital Base AS, Myravegen 9, NO-6060 Hareid.
Tel. +47 70 09 59 50, mail: firmapost@vitalbase.no

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