

Automatisk Penispumpe til rejsningsproblemer



Medintim

Placer hjælperingene
og til sidst keglen.

Medintim

Brug keglen til nemmere
at glide en penisring
på.

A shirtless man is shown from the chest down to the waist, holding a pair of scissors and cutting a blue ring. The background is a solid orange color. The man's torso is muscular, and he is wearing dark-colored shorts. The scissors are held in both hands, and the blue ring is being cut in the middle. A blue dot is placed on the ring, with a line pointing to the text box on the right.

Medintim

Alternativt kan du bruge hænderne til at trække penisringen nedover hjælperingen.

Medintim

Pumpen er nu klar.



A person is shown from the side, using a medical pump on their arm. The pump is a blue device with a clear tube. The person's hand is on the pump, and the tube is connected to their arm. The background is a solid blue color.

Medintim


Brug af pumpen på
kroppen.

A close-up photograph of a person's torso and arm. The person is holding a blue and white medical device, possibly a catheter or probe, against their skin. Another person's hand is visible, applying a white lubricant from a small tube to the device. The background is a warm, orange-toned wall.

Medintim

Påfør
glidecreme.

Medintim



Sæt penis i cylinderen
og pres pumpen tæt
ind imod kroppen.
Begynd at pumpe.

Medintim



Der laves vakuum i cylinderen,
som skaber en rejsning.

Medintim

Glid penisringen
af nedover penis.

6LIV
med
ELSE O

Medintim

Penisringen holder
rejsningen (max. 30 min.)



Medintim

Efter brug:
Fjern penisringen.
Brug evt. glidecreme.

Medintim

Tilstand efter brug.

Medintim

www.medintim.de

CE 0483

FDA registered device
Made in Germany



KESSEL Medintim GmbH
Kelsterbacher Str. 28
64546 Mörfelden-Walldorf
Germany
+ 49 (0) 6105/203 720