
•SmartVision 2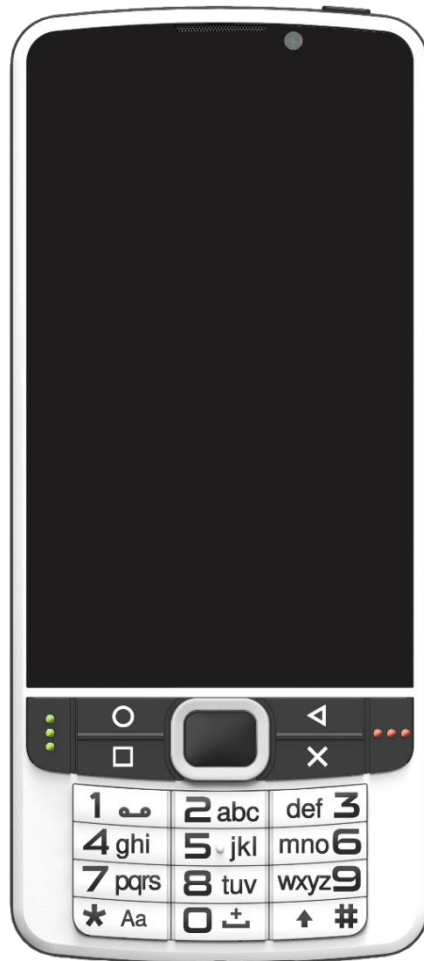

Quick starting guide
Guide de démarrage rapide
Schnellstartanleitung
Guida rapida
Guía rápida
Korte handleiding
Hurtig Start Guide
Sprievodca rýchlym štartom
Turc?
Hebreux?

www.kapsys.com

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3. WARRANTY – LIMITATION OF LIABILITY GUARANTOR (ENGELSK)

The guarantor is Kapsys, manufacturer of the device, with registered office at 694, avenue du Docteur Maurice Donat, Mougins Sophia Antipolis (06250), France.

LIMITED WARRANTY

KAPSYS warrants to the purchaser that the product complies with the description thereof (and available on www.kapsys.com website) in accordance with the provisions of Articles L.211-1 and seq. of the French Consumer Code.

KAPSYS warrants to the initial purchaser that from the date of purchase and as long as it is used under normal conditions and according to its intended purpose, the product is free from any defects in workmanship and materials.

This product is guaranteed for a period of two (2) years and the battery is guaranteed for a period of six (6) months from the date of purchase.

During this commercial warranty period, the product will be either repaired or replaced free of charge by Kapsys or its approved service providers (excluding transportation cost).

This warranty applies to KAPSYS telephone in France where the warranty is registered. Please contact your vendor for any queries about warranty service. Users who apply for coverage will be requested to provide a proof of purchase of their product (address and date of purchase).

In accordance with article L 211.16 of the French Consumer Code, as long as the product is unavailable for use or cannot be used for more than seven (7) days, the warranty period of a product repaired under warranty is automatically extended by the duration of repair.

KAPSYS warranty does not cover:

- Accessories supplied with the product
- Normal wear and tear of the product and reduction in the battery charge holding capacity

- Damage resulting from improper use, exposure to humidity or liquid, or close proximity to a heat source
- Cracked or broken devices, or devices with visible signs of shocks
- Failure to respect the precautions for use, accidents, negligence, misuse or use inconsistent with the instructions provided with the device, or commercial use of the product
- Short-circuits of the battery or use of the battery in another device than your device
- Use of accessories or connectors not recommended by KAPSYS
- Damage as a result of attempt repair by anyone not authorized by Kapsys

The limited warranty included herewith and complying with the applicable legislation excludes any other implied warranty and/or duty incumbent upon the seller, in particular but not limited to any obligation in terms of satisfactory quality, reliability or availability of the accuracy or completeness of responses, results and information supplied by the product or of data accessibility. You are recommended to make one or several backups of your data stored in your product. KAPSYS cannot be held responsible for loss of data resulting from the product malfunction, repair or replacement.

4. LIMITATION OF LIABILITY (ENGELSK)

To the maximum extent permitted by applicable law, in no event shall KAPSYS be liable to you, any user, or third party for any indirect, special, consequential, incidental or punitive damages of any kind, arising in contract, tort, or otherwise, including, but not limited to, injury, loss of revenue, loss of goodwill, loss of business opportunity, loss of data, and/or loss of profits, regardless of the foreseeability thereof or whether KAPSYS has been advised of the possibility of such damages. And in no event shall the total liability of KAPSYS exceed the amount received from you, regardless of the legal theory under which the cause of action is brought. The foregoing does not affect any statutory rights which may not be disclaimed.

Network servers and or cellular system services are provided by third parties via communication systems, networks and media on which KAPSYS does not have any control and therefore cannot ensure availability and operation efficiency. We recommend that you contact directly the relevant providers for any support related to their services.

5. ABOUT SPECIFIC ENERGY ABSORPTION RATE

Your SmartVision2 by KAPSYS is a radio transmitter/receiver. It was designed and manufactured to comply with the radiofrequency (RF) exposure thresholds recommended by the European Union.

This device meets the EU requirements (1999/519/EC) on the limitation of exposure of the general public to electromagnetic fields by way of health protection. The unit of measurement for European council's recommended limit for mobile telephones is the "Specific Absorption Rate" (SAR). This SAR limit is 2.0 W/kg averaged over 10 g of body tissue. This device meets the requirements of the International Commission on Non-Ionizing Radiation Protection (ICNIRP) and European Standard EN 62209-1 / EN 62209-2 for mobile radio devices.

For body-worm operation, this phone has been tested use with accessories that contain no metal and that position the handset a minimum of 1.5 cm from the body. Use of other accessories may not ensure compliance with RF exposure. If you do not use a body-worm accessory and are not holding the phone at the ear, position the handset a minimum of 1.5 cm from your body when phone is switched on.

During use, the actual SAR level is usually much lower than the maximum value. In general, the closer you are to the base station, the lower the transmission output of your mobile telephone. To reduce exposure to RF energy, use a hands-free accessory or other similar option to keep the device away from your head and your body.

The highest SAR value of this device is listed below:

- Head: XXXXX W/kg (averaged over 10g of body tissue)
- Body : XXXXX W/kg (averaged over 10g of body tissue)

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RoHS COMPLIANCE:

This product is in compliance with Directive 2011/65/EU of European Parliament and of the Council of 8 June 2011, on the restriction of the use of certain hazardous substances (RoHS) in electrical and electronic equipment and its amendments.

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KAPSYS ord, brands og logoer er registrerede varemærker fra KAPSYS SAS.
ABBYY navne, mærker og logoer er registrerede varemærker fra ABBYY Software Ltd.
Bluetooth navne, brands og logoer er registrerede varemærker fra Bluetooth SIG Inc.
Nuance er et registrerede varemærker fra Nuance Corporation Inc.