

# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

## Quick guide

(DK) **Light Turner** - til Light Cover  
(UK) **Light Turner** - for Light Cover

The Original One-Layer System

24\_20220919 DK/UK



MC 002-2700



MC 002-2701

Vare nr./Item no.	HMI nr./no.	Navn/Name - Størrelse/Size	UDI-DI
○ MC 002-2700	114964	Light Turner - L	5744001972267
○ MC 002-2701	114965	Light Turner - M	5744001972274



### Eftersyn:

Produktet skal kontrolleres jævnlig og gerne inden hver brug. Kontroller, at der ikke er skader på stof, stropper, sømme mm. Har produktet høj friktion eller på anden vis er i stykker, skal produktet kasseres.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torne fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



(DK) Forordning MDR 2017/745 for medicinsk udstyr klasse 1  
(UK) Regulation MDR 2017/745 for medical devices class 1



(DK) Efterlad aldrig bruger på produktet på sengekanten.  
(UK) Never leave the user on the product when lying on the edge of the bed.




(DK) Efterlad aldrig produktet liggende på gulvet.  
(UK) Never leave the product on the floor.



(DK) Efterlad aldrig en bruger liggende på produktet, mens dette er koblet på en lift.  
(UK) Never leave the user lying on the product whilst connected to a hoist.



(DK) Sørg altid for, at brugeren ikke hænger fast i stropperne.  
(UK) Always make sure that the user is not caught in the straps.

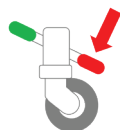

(DK) Sørg for, at alle relevante bremser er låst, inden forflytning foretages.  
(UK) Make sure that all relevant brakes are activated before transferring the user.



(DK) Må ikke bruges som liftsejl – kan med fordel kombineres med lift til fx. vendinger i seng.  
(UK) Do not use as a sling – can be combined with a hoist for repositioning in bed.

ISO 12 31 03 21

The Original Nordic Human Handling Concept





(DK) For mere information, brugervejledning, videoer mm., se  
(UK) For more information, User Manual, videos etc. visit  
[www.master-care.dk](http://www.master-care.dk)

Developed & Manufactured by:





# Light Turner


## The **Original** One-Layer System

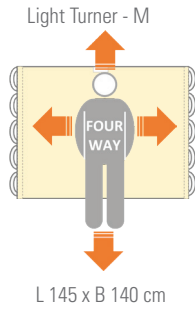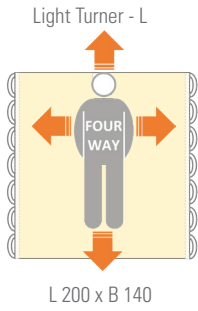


(DK) Kombinationsmuligheder / (UK) Combinations

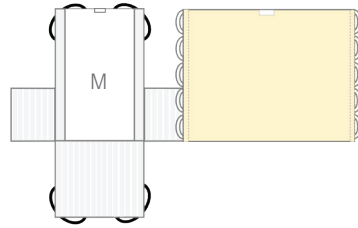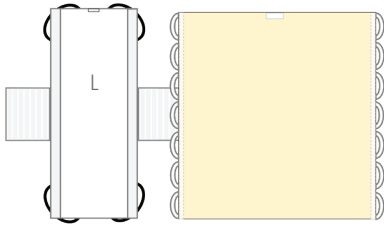

(DK) Anvendelsesmuligheder / (UK) Applications

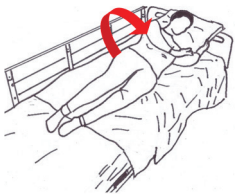


1.

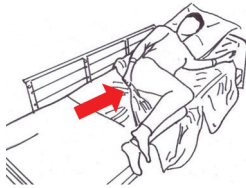


2.



3.

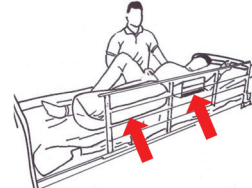


4.

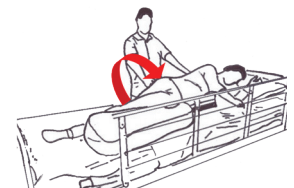

5.



6.

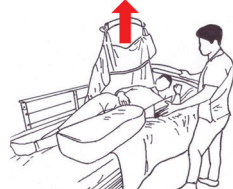


7.




8.




9.

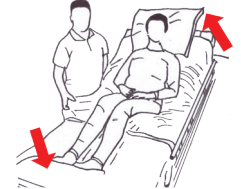



10.

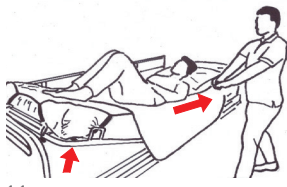

11.

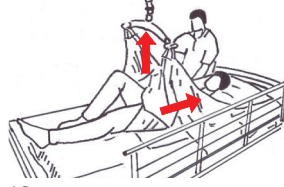

12.

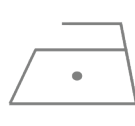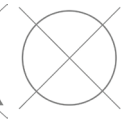

85°C

(DK) Skånsom vask og tørretumbling forlænger holdbarheden væsentligt.  
(UK) Gentle washing and dry tumbling significantly prolongs the durability.

Max. 40°C

(DK) Anvend aldrig skyllemiddel, da dette forringer glideegenskaberne væsentligt.  
(UK) Never use fabric softener as this will reduce the sliding properties significantly.

Max. 40°C

(DK) Skånsom strygning forbedrer glideevnen.  
(UK) Gentle ironing improves sliding abilities.



Find us on:



master care A/S, Sofienlystvej 3, DK 8340 Malling, Danmark  
Tel. +45 86 93 85 85, E-mail: info@master-care.dk

(DK) Vi tager forbehold for evt. trykfejl og mangler. (UK) Reservations are made for printing errors.