

Quickguide Yola rollator

Unfolding and folding the rollator

- Firstly, unfold front wheels and secure them with locking knobs in the mounting openings of the frame in the part of wheels mounting. Then, unfold rear wheels and secure them with locking knobs at the top of the rear legs.
- To unfold the rollator, press the seat in the middle so that all the elements fit together and you hear “click”.
- Fix the shopping bag in the front part of the rollator.
- Before first use, check the tightening of mounting screws.
- Remove plastic casing from wheels.
- To fold the rollator, pull the strap located in the middle of the seat upwards.

Note! Be careful while folding or dismantling the rollator due to the risk of smashing fingers in hinges.

Brakes

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the brake with locking mechanism by pushing down the brake handle.

To release the brake, pull the brake handle upwards.

Height adjustment of handles

- To adjust the height of the handles, press metal pin which is located on the outer side of the frame, select appropriate height and lock the position of the tubes in relation to each other.

Note! The handles must be set on the same level.

Adjustment of brakes

Adjustment of lower brake

- To adjust the brake and increase the tension, unscrew the adjustment screw with the use of appropriate tools and tighten the locking screw by holding the adjustment screw.