

Quick guide Classic Tim kørestol

UNFOLDING AND ASSEMBLY

Step 1. Unfolding the wheelchair

Place the wheelchair on the ground. Unfold the wheelchair by using the hand grips which are at the back of the backrest. Then, press the front and rear edge of the seat so that all the elements fit together.

Step 2. Assembly of footrests

Assemble the right and the left footrest in the front part of the frame by placing them on two pins and turn them inwards until they are locked.

Step 3. Adjustment of footplates

Adjust the right and the left footplate with the use of pins and locking clips which is placed on the footrest. In case of elevating legrest, adjust the angle of inclination to the ground using the lever. Proper space should be left between the footplates and the ground as footplates set too low may cause difficulties in overcoming such obstructions as curb. Adjust the footplates so that the angle between trunk and thighs is 90° (optimum sitting position is provided to the user, then).

Step 4. Assembly of protective strap

Put the protective strap on the footrests as it prevents user's legs from sliding down under the wheelchair from footplates.

Note! Do not use the wheelchair without protective strap.

On completion of the above activities, the wheelchair is ready to use.