

Quick guide EVERYDAY TIM kørestol

Step 1. Unfolding the wheelchair

Place the wheelchair on the ground. Unfold the wheelchair using the hand grips which are at the back of the backrest. Then, press the front and rear edge of the seat so that all the elements fit together.

Step 2. Assembly of footrests

Assemble the right and the left footrest. Put the assembly element into the opening in the front part of the frame and turn the footrest inwards until the footrest lever is locked.

Step 3. Adjustment of footplates

Adjust the right and the left footplate by tightening the screw in the suitable mounting hole. Proper space should be left between the footplates and the ground as footplates set too low may cause difficulties in overcoming such obstructions as curb. Adjust the footplates so that the angle between trunk and thighs is 90° (optimum sitting position is provided to the user, then).

Step 4. Assembly of protective strap

Put the protective strap on the footrest as it prevents user's legs from sliding down under the wheelchair from footplates.

Note! Do not use the wheelchair without protective strap.

Step 5. Adjustment of anti-tip wheels

In order to adjust anti-tip wheels, which are located at the back of the wheelchair, pull the lock, move the wheels upwards or downwards, and select appropriate height.

Step 6. Checking the assembly of wheels

Lift the wheelchair slightly and spin the right wheel and then the left one by pulling the handrims and check if the wheels spin properly.

On completion of the above activities, the wheelchair is ready to use.