

Quickguide Badestol med ryg- og armlæn

ASSEMBLY

Step 1. Assemble the backrest with the backrest tubes using four mounting screws (wing screws).

Step 2. Assemble the backrest tubes into the mounting openings located under the seat until the locking pins click.

Step 3. Place the assembled chair on the legs.

Step 4. Assemble the side handgrips into the mounting openings located on the outer side of the seat.

HEIGHT ADJUSTMENT

- To adjust the height of the chair, press the metal pin, select appropriate height of the leg, and then, lock the position of the legs relative to each other with the locking pin. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level in each leg.