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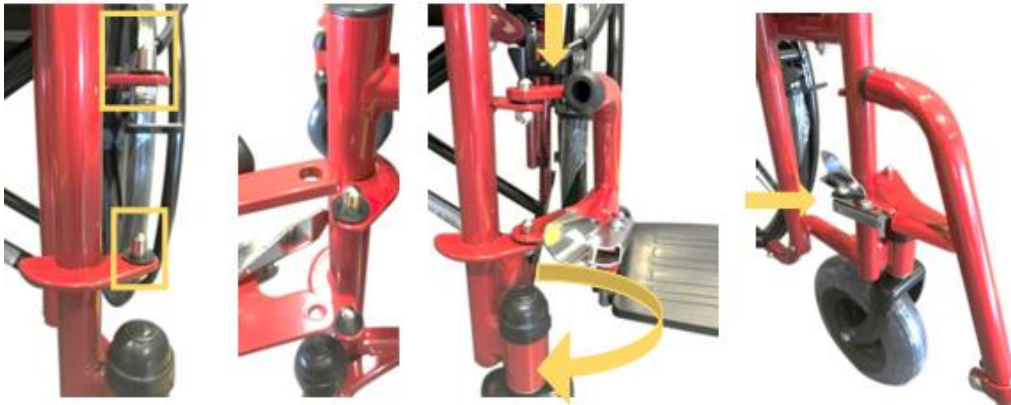
The wheelchair that you have in your hands may differ from the one in the following photos. The assembly process is similar.

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To open: Get the handles and pull outwards, then push the bars on the side of the seat, downwards until the wheelchair is completely open.

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To close: Follow the reverse procedure.

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Assembly footrest by installing the hinge plates that are on the part onto the hinge pins on the wheelchair frame. Push the footrest towards inside of the wheelchair until it locks into place. Repeat the same procedure for the other foot.

Remove the footrest by pushing the release lever, rotate outwards, lift the footrest off the hinge pins.

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*Use the tool, that are with the product package, to **adjust the length** of the legrest. Loosen the screw and tighten to the desired length to secure it.*

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*Use the included **footrest belt** to make sure that the patient's feet are properly positioned.*

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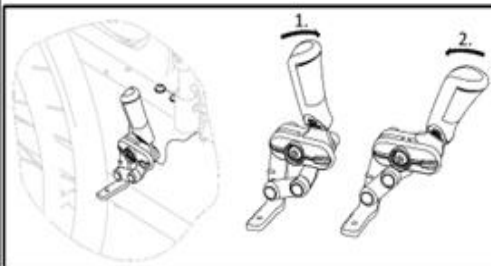
*The **brakes** are used to immobilize the wheelchair when it is stationary to prevent it from rolling away.*

To engage the brake, push the brake lever forwards with the palm. (1)

To disengage the brake, pull the brake lever backwards with the low fingers (2)

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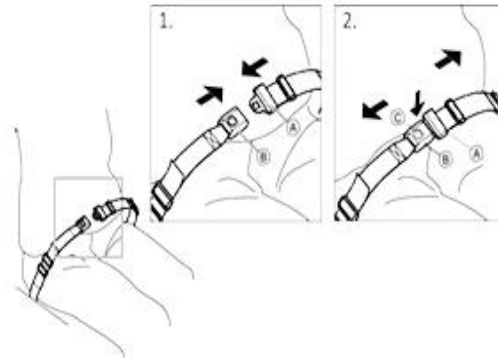
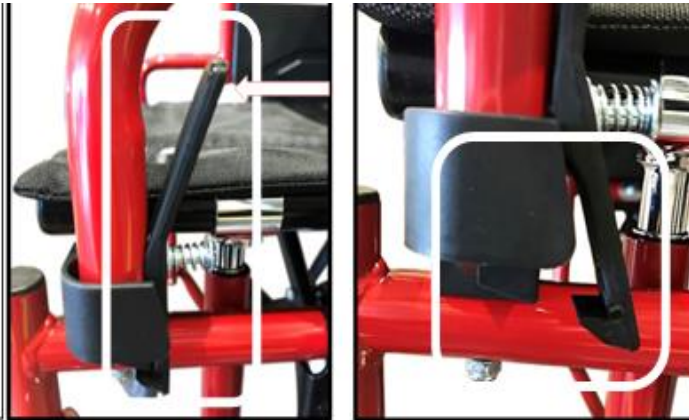
Never engage the parking brakes while you are moving.

Do not engage the parking brakes to slow down the wheelchair.

Do not lean on the parking brakes for support or transfer.

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To raise the folding armrest, press the safety lever and pull upwards.

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It prevents the user from sliding downward in the wheelchair or from falling out of the wheelchair.

A loose belt can allow the user to slip down and create a risk of strangulation.

Ensure that you are sitting fully back in the seat and that the pelvis is as upright and symmetrical as possible.

To close, push the catch A into the buckle clasp B.

To open, push the Press button C and pull the catch A out of the buckle clasp B.