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The wheelchair that you have in your hands may differ from the one in the following photos. The assembly process is similar. First, we check the contents of the package.

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To open: Get the handles and pull outwards, then push the bars on the side of the seat, downwards until the wheelchair is completely open.

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To close: Follow the reverse procedure.

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Assembly the Footrest by installing the hinge plates that are on the part onto the hinge pins on the wheelchair frame. Push the footrest towards inside of the wheelchair until it locks into place. Repeat the same procedure for the other footrest.

Remove the Footrest by pushing the release lever, rotate outwards, lift the footrest off the hinge pins.

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Use the included footrest belt to make sure that the patient's feet are properly positioned.

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Optional with the purchase of the elevating footrests 0811358 & 0811359.

For the ref: 0811607 the Elevating Footrests are standard.

You will not need the footrest belt. Remove the standard footrests and install the elevating footrests with the same way as the standard one.

To move the footrest upwards, pull the main part and leave it at the desired height. To move the footrest downwards, push the lever and lower it, holding it from the lower part. This function, for security reasons, is recommended to be done only from the attendant and not by the user himself.

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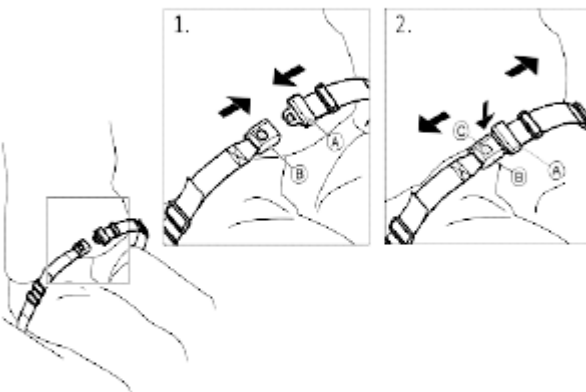
It prevents the user from sliding downward in the wheelchair or from falling out of the wheelchair.

A loose belt can allow the user to slip down and create a risk of strangulation.

Ensure that you are sitting fully back in the seat and that the pelvis is as upright and symmetrical as possible.

To close, push the catch A into the buckle clasp B.

To open, push the Press button C and pull the catch A out of the buckle clasp B.

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Anti-tilt system (Upon Request- It is not included in the packaging)

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In the case where the wheelchair includes an overturning system, remove the cap from the pipes at the bottom of the wheelchair.

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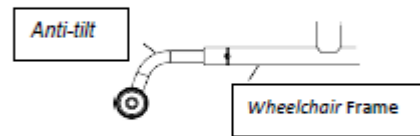
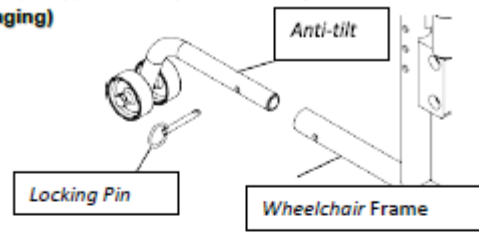
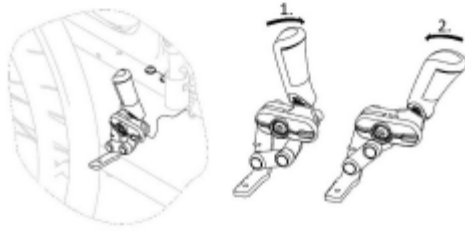
Install the lock pin to the intersecting holes. By inverting the process, we can remove the rollover avoidance system.

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The brakes are used to immobilize the wheelchair when it is stationary to prevent it from rolling away.

To engage the brake, push the brake lever forwards with the palm. (1)

To disengage the brake, pull the brake lever backwards. (2)

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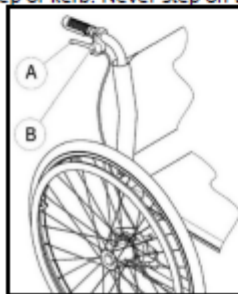
Never engage the parking brakes while you are moving.

Do not engage the parking brakes to slow down the wheelchair.

Do not lean on the parking brakes for support or transfer.

Keep your fingers away from movable parts when using the brake, always keep your hand on the brake lever.

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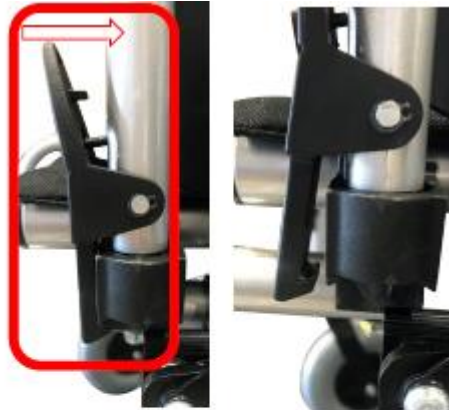
For the Wheelchairs 0811306, 0811301, 0811300, 0811610, 0811611 are standard. For the wheelchairs 0811301B, 0811300B & 0811609, are optional with the extra ref. code 0811398.

With attendant's brakes, an assistant can brake the wheelchair while it is moving. The attendant's brake can also be used as a parking brake.

To brake, pull the brake lever A on the push handle. To park, pull the brake lever firmly until the safety lever B engages.

To release, pull the brake lever and release the safety lever underneath the brake lever

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For the Wheelchairs 0811306, 0811301, 0811300, 0811610, 0811611 & 0811607

How to remove the Quick Release rear wheels:

The Wheelchair has a quick-release system on the rear wheels so they can be removed easy and can reduce also the storage volume in the car and home.

To disassemble the wheel, press the black button in the center of the wheel, to unlock, and at the same time pull the wheel outwards until it comes out completely from its base.

Repeat the same process on the other wheel.

To assemble follow the reverse procedure.