# MADE IN EUROPE

This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

## Quick guide

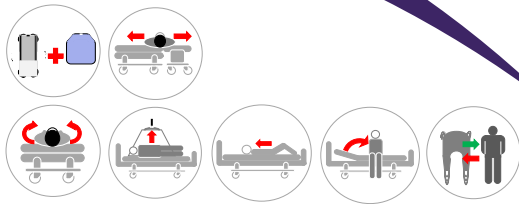
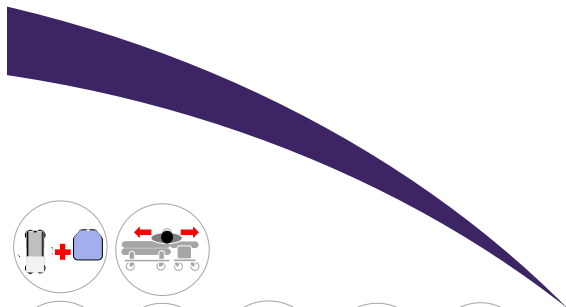
(DK) **Master Turner Royal** - til madrascovers

(UK) **Master Turner Royal** - for mattress covers

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### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.

(DK) Forordning MDR 2017/745 for medicinsk udstyr klasse 1  
(UK) Regulation MDR 2017/745 for medical devices class 1

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(UK) Never leave the user on the product when lying on the edge of the bed.

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(UK) Never leave the product on the floor.

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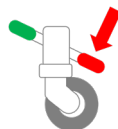
(UK) Never leave the user lying on the product whilst connected to a hoist.

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(UK) Always make sure that the user is not caught in the straps.

(DK) Sørg for, at alle relevante bremsere er låst, inden forflytning foretages.

(UK) Make sure that all relevant brakes are activated before transferring the user.

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(UK) Do not use as a sling – can be combined with a hoist for repositioning in bed.

ISO 12 31 03 21

The Original Nordic Human Handling Concept

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
(UK) For more information, User Manual, videos etc. visit

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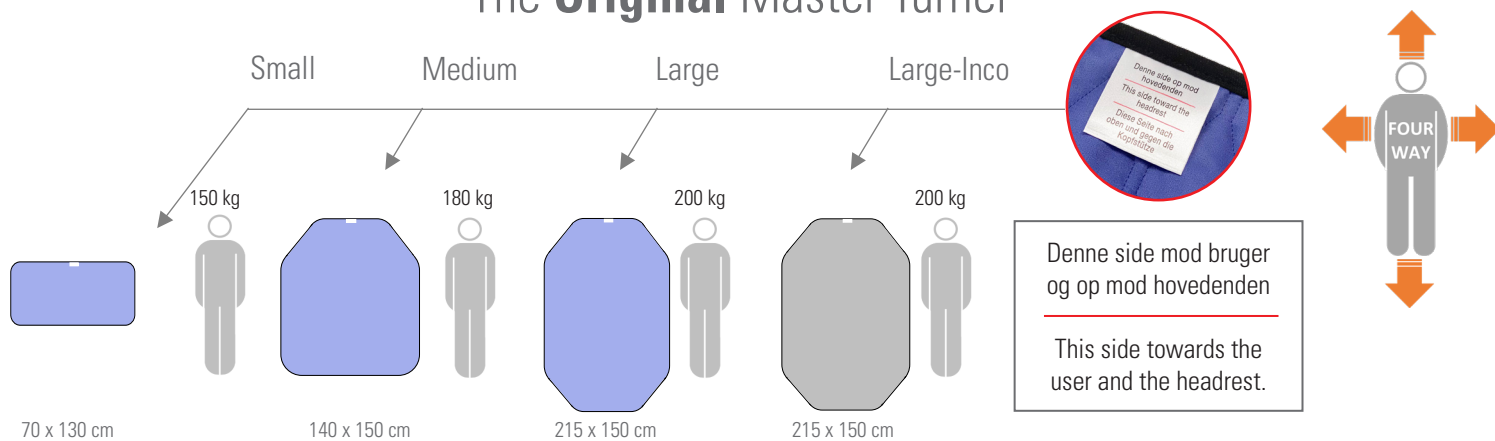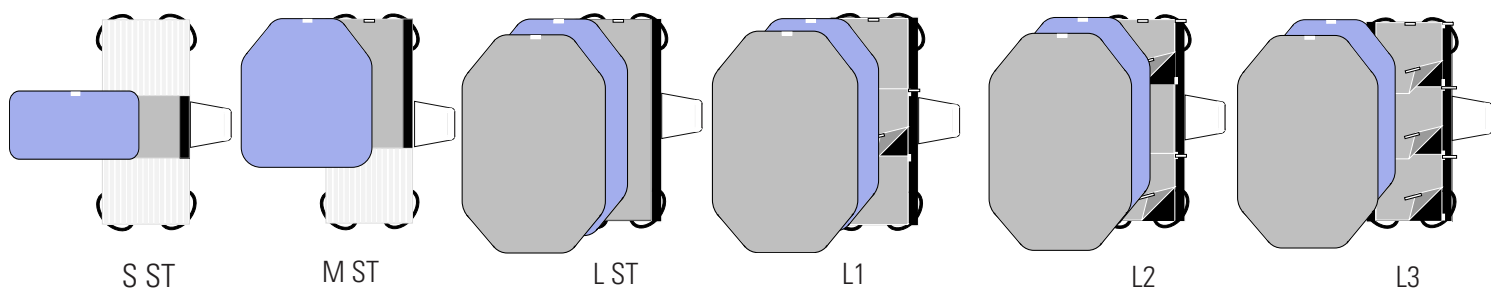
(UK) Gentle washing and dry tumbling significantly prolongs the durability.

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(UK) Never use fabric softener as this will reduce the sliding properties significantly.

Find us on:

master care A/S, Sofienlystvej 3, DK 8340 Malling, Danmark  
Tel. +45 86 93 85 85, E-mail: info@master-care.dk