

PARTS LIST

# Parker



# General information

---

Note. All article numbers starting with **85** indicates that more than one detail is included in the article number.  
Us-version from serial number **1111000366**. **Blue text indicates changes compared to previous manual.**

## How to order

Before ordering spare parts study the exploded view closely and make sure that you note all involved spare parts.

Please note that gaskets and o-rings normally are included in kits and can not be ordered separately.


When taking contact with Arjo Parts sales always have the following information at hand:

- The correct name of the product. See type label.
- The serial number of the product. See type label.


**Serial Number From November 2016**

**REF** PRODUCT CODE  
**SN** SERIAL NUMBER

Consecutive number

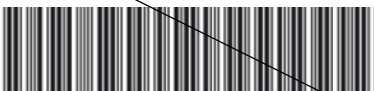


(01)GTIN No.



(11)YYMMDD(21)Serial No.


Site (VEPL= Veplas)



YYYY-MM-DD 8618944\_2


**Serial Number From May 2014**

**REF** XXXXXXXXXXXXXXXX




**SN** XXXXXXXXXXXXXXXX

Consecutive number



Site (VEPL= Veplas)



YYYY-MM 8618944 A

**Serial Number**

The serial number consists of the following parts:

**0806000001**

Year    Month    Consecutive Number

Manufacturing country and Manufacturing Works can no longer be readout.

# *Contents*


---

<b>Overview .....</b>	<b>4</b>
<b>A Base and Yoke .....</b>	<b>6</b>
<b>B Tub and Drain .....</b>	<b>8</b>
<b>C Tilt Actuator and Tub Attachments .....</b>	<b>10</b>
<b>D Panels .....</b>	<b>12</b>
<b>E Door .....</b>	<b>14</b>
<b>F Mixer .....</b>	<b>16</b>
<b>G Shower&amp; Disinfection US .....</b>	<b>18</b>
<b>H Shower&amp; Disinfection EU/GB .....</b>	<b>20</b>
<b>I Control Box .....</b>	<b>22</b>
<b>J Air-Spa .....</b>	<b>24</b>
<b>K Shower Pole .....</b>	<b>26</b>
<b>L Sound and Vision (S&amp;V) .....</b>	<b>28</b>
<b>Preventive Maintenance Parts/Recommended Spares .....</b>	<b>30</b>
<b>Service Articles and Special Tools .....</b>	<b>32</b>

## *Overview*

<b>Pos</b>	<b>Description</b>	<b>See page</b>
<b>A</b>	Base and Yoke	<b>6</b>
<b>B</b>	Tub and Drain	<b>8</b>
<b>C</b>	Tilt Actuator and Tub Attachments	<b>10</b>
<b>D</b>	Panels	<b>12</b>
<b>E</b>	Door	<b>14</b>
<b>F</b>	Mixer	<b>16</b>
<b>G</b>	Shower& Disinfection US	<b>18</b>
<b>H</b>	Shower& Disinfection EU/GB	<b>20</b>
<b>I</b>	Control Box	<b>22</b>
<b>J</b>	Air-Spa	<b>24</b>
<b>K</b>	Shower Pole	<b>26</b>
<b>L</b>	Sound& Vision (S&V)	<b>28</b>

# Overview

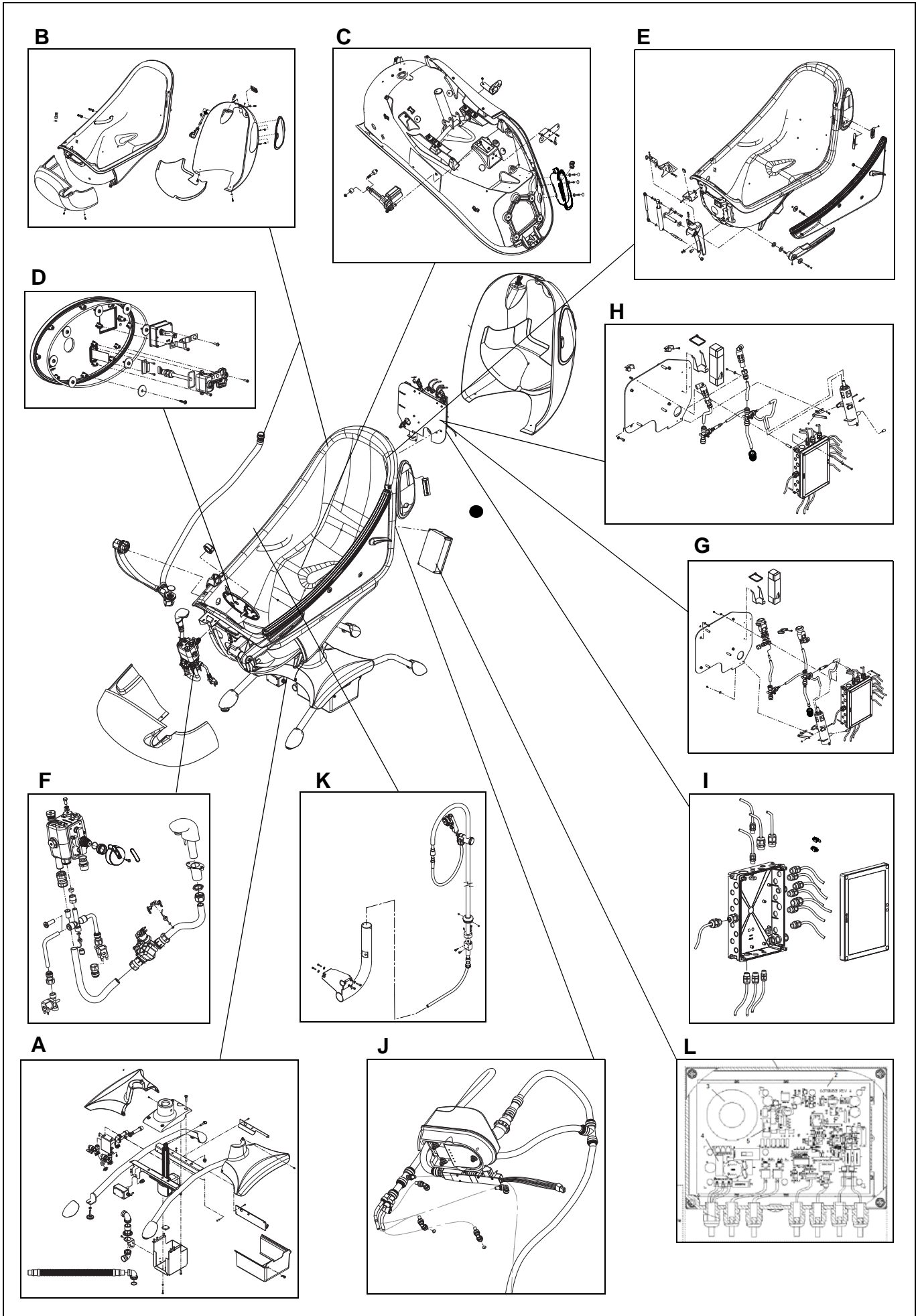


## A

## Base and Yoke

Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8662396-036	1	Base			
2	8361511-036	1	Suspension Box			
3	8457052	1	Spacer			
4	8462042	1	Chassis casting			
5	6305177	4	Foot			
6	PF0806	1	Lock			With disinfection
7	PH0255	1	Lift Actuator matrix			
8	6305121	1	Shut-off valve HW, complete			
9	6305122	1	Shut-off valve CW, complete			
10	8662438-036	1	Plate			Without disinfection
11	8361265-036	1	Bracket valves			New version 2014
12	<b>8662684</b>	2	Connection			Incl nut and ring
13	8260750-036	1	Box disinfection			With disinfection
14	8360747-036	2	Rail			With disinfection
15	<b>8563213</b>	1	<b>Floor drain attachment kit</b>			Incl. screws, washers and nuts
16	6151258B	6	Screw FZB MF6S ISO 10642 M5x2-8.8			
17	8456971	2	Yoke bearing housing			
18	PF1333	1	Yoke detent pin			
19	6130612	4	Screw A4 MC6S ISO 4762 M6x12-80			
20	6130616	5/8	Screw A4 MC6S ISO 4762 M6x16-80			
21	6200003	12	Washer A4 BRB ISO 7089 6.4x12x1.6			200HV
22	6210043	8	Lock nut A4-DIN 985 M6-80			
23	6210035	4	Nut A4 ML6M ISO 4035 M10-040			
24	PF0142	6	Screw FZB MC6S ISO4762 M8x25-8.8			
25	6151050	4	Screw FZB MF6S ISO10642 M10x50-10.9			
26	6210246	4	Locking nut FZB M6MF ISO 7043 M10-8			Flange nut
27	8663057-036	1	Bracket water trap			
28	6262469	1	O-ring			
29	8663063	1	Plastic nut			
30	8663212	3	Elbow G1 1/2"			
31	8662944	1	Pipe connection			
32	6319539	2	Hose clamp 10mm			
33	8461324	2	Adaptor			
34	6907788	2	Filter			
35	6190337	3	Screw A4 MRT-TT ISO 14583 M6 x 12			
36	6190511	4	Screw A4 MRT-TT ISO 14583 M6 x 25			
37	6250891	2	Lubrication nipple			
38	6200053	1	Toothed washer A4 AZ DIN 6798 M6			
39	<b>S8562145</b>	1	<b>Field wiring box, complete</b>			
40	8664471	1	Key			
41	8663061	2	Connection pipe 40			
42	8461601	1	Hose 690mm			
43	8563020-618	1	Connection hose GB, 618mm			
	8563020-1200		Connection hose GB, 1200mm			
	8563020-1700		Connection hose GB, 1700mm			
	6270394		Connection hose, 618mm			
	6270395		Connection hose 1200mm			
6270396	Connection hose, 1700mm					
6270397	Connection hose, 2400mm					
44	<b>S8561125-035</b>	1	<b>Assembly feet cover</b>			<b>All covers incl. straps</b>
45	<b>S8560730-035</b>	1	<b>Assembly base cover</b>			<b>Incl. screw and nut</b>
46	8662130	1	Fiber gasket			
47	<b>8664476</b>	1	<b>Yoke complete</b>			
48	8663074	1	Connection pipe 38			
49	141087	1	Skintop			
50	8662943	1	Connection			
51	8663065	1	Hose 2100mm			
52	S8664479-xx	1	Large base cover			Colour code see B1
53	<b>8564474</b>	1	<b>Seismic anchor 4 pcs</b>			<b>Incl. nuts and washers</b>

A

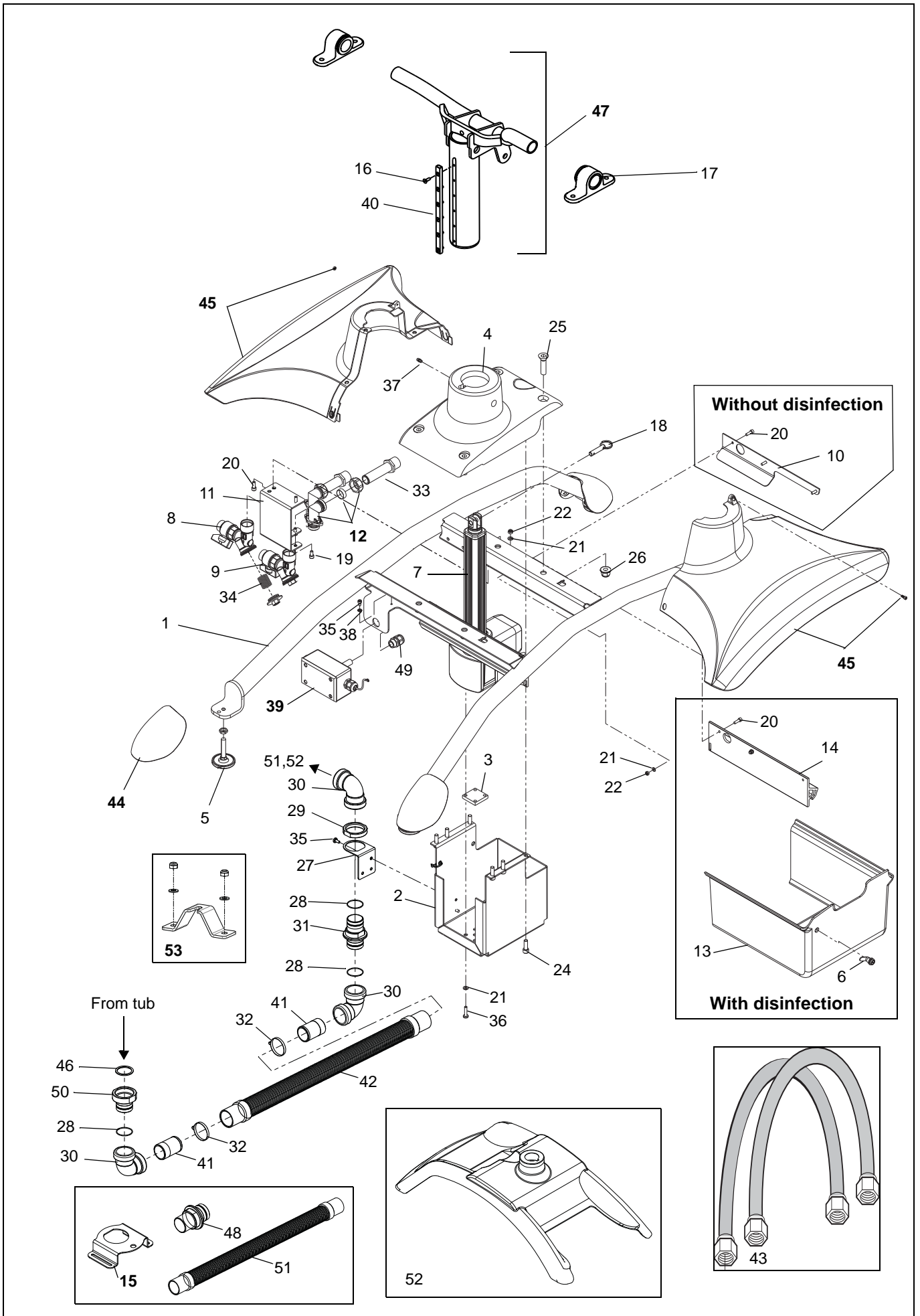


## B

## Tub and Drain

Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8662401-xx 8662402-xx	1	Back skirt Back skirt with door cut			-04=White -016=Pastel blue -061=Cream -065= Ivory -071=Pastel green New version november 2013 to fit option shower flush
2	8461474	1	Inner plate			Incl tape. Only w disinfection
3	8461441-036	1	Door disinfection, complete			Disinfection, UK S&V
4	8337059-03	1	Bending cover			
5	8460610-035	1	Handcontrol holder			
6	6200001	8	Washer A4 BRB ISO 7089 4.3x9x0.8			
7	8444769	2	Washer, plastic			Disinfection, UK S&V
8	6314161	2	Blind rivet M4 UPO45 A1			
9	8358126	1	Shower hose, grey			UK version see K
10	<b>S6303096</b>	<b>1</b>	<b>Shower handle, grey</b>			UK version see K
11	6190601	4	Screw, FZB TX PT K40x12			
12	6190602	2	Screw, FZB TX PT K40x16			Disinfection, UK S&V
13	6190607	2	Screw FZB M6S ISO 4017 M8x25- 8.8			Driloc
14	6200044	2	Washer A4 BRB ISO 7089 8.4x24x1.6			200HV
15	8461432-035	1	Shower holder w. seal			Not UK
16	S6319521	2	Hose clamp			
17	8662403-xx	1	Front skirt			Colours see pos 1
18	PC0680	1	Plug			UK versions only
19	6190307	5	Screw A4 K6S ISO 7380 M6x25-80			
20	6200054	5	Washer A4-RBS ISO 7093 6.4x18x1.6			200HV
21	<b>6907875</b>	<b>1</b>	<b>Drain plug complete</b>			
22	6907926	1	Gasket			
23	S8454960	1	Knob			
24	8453083	1	Gasket positive drain			
25	400.00.00.16 400.00.00.09		Neck pillow, blue Neck pillow, grey			
26	400.00.00.02 400.00.00.15		Back pillows grey Back pillows blue			
27	8257376-033		Leg support, grey			
28	S6360016	1	Bottom connection			
29	S6319522	2	Hose clamp			
30	8261224 8361595	1	Drain complete WW Drain complete GB			
31	8440528	3	Nipple			
32	6200005	4	Washer A4 BRB ISO 7089 8.4x16x1.6			200HV
33	6210185	5	Nut A4 ML6M ISO 4035 M8-035			
34	8442442	1	Nut temp sensor			
35	8547031-013 S8360308-034	1	Trim kit Blue Trim kit Grey			
36	6314159	1	Emergency stop			
37	8361943	1	Rotation stop			
38	8359329	1	Grab handle			
39	<b>8543697</b>	<b>1</b>	<b>Strainer kit, grey</b>			2 strainers handle, 2 strainers hose, o-ring, screw, plug
40	<b>8543698</b>	<b>1</b>	<b>Repair kit, grey</b>			2 pins, sleeve, spring, 3 o-rings, 2 complete valves with o-rings
41	AH1005427	2	Screw FZB TX PT K40x20			Not UK




**B**

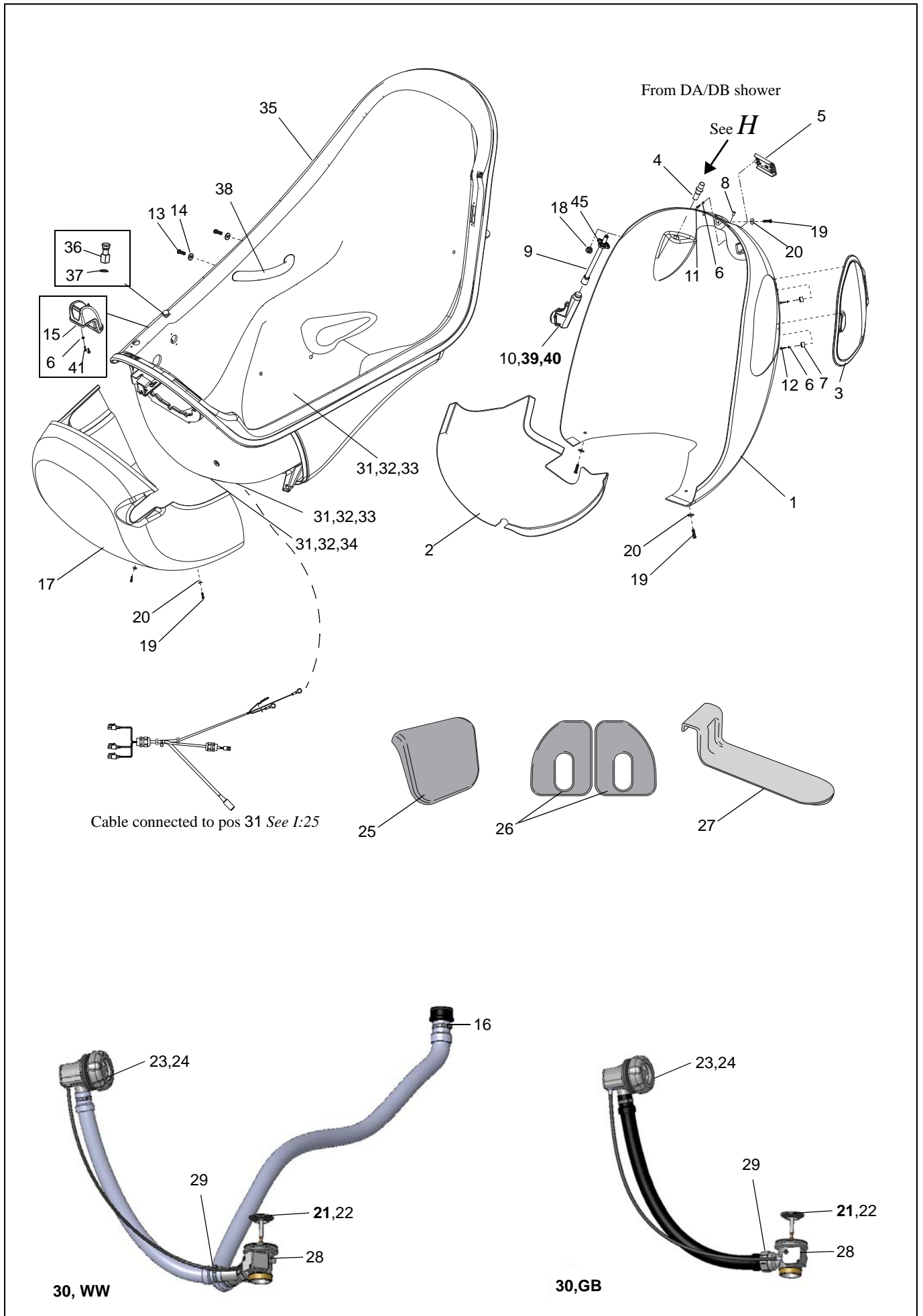

<b>C</b> <i>Tilt Actuator and Tub Attachments</i>						
<b>Pos</b>	<b>Art. No.</b>	<b>Qty</b>	<b>Description</b>	<b>From ser. No.</b>	<b>Up to ser. No.</b>	<b>Remarks</b>
1	PH0254	1	Tilt actuator matrix			
2	6210046	2	Locking nut A4 DIN 985 M12-80			
3	6303133	2	Spacer			
4	6101290	2	Screw A4 M6S ISO 4014 M12 x 90-80			
5	8361976	1	Bracket airspa			
6	6210044	2	Locking nut A4 DIN 985 M8-80			
7	8461520	1	Bracket handset			
8	6200053	1	Toothed washer A4 AZ DIN 6798 M6			
9	6210043	1	Lock nut A4-DIN 985 M6-80			
10	8459309	1	Potential equalizer male			
11	6370284	1	Potential equalizer female			
12	6190307	6	Screw A4 K6S ISO 7380 M6x25-80			
13	6311258	3	"List"			
14	8461577	1	PE Cable			To mixer
15	8361433	1	DIN8 cable			
16	8360661-034	1	Disinfection interior			
17	6311367	3	Cover, plastic			
18	6320246	3	Washer			
19	6210185	2	Nut, A4 ML6M ISO 4035 M8 - 035			
20	8662442-034	1	Gland			




C

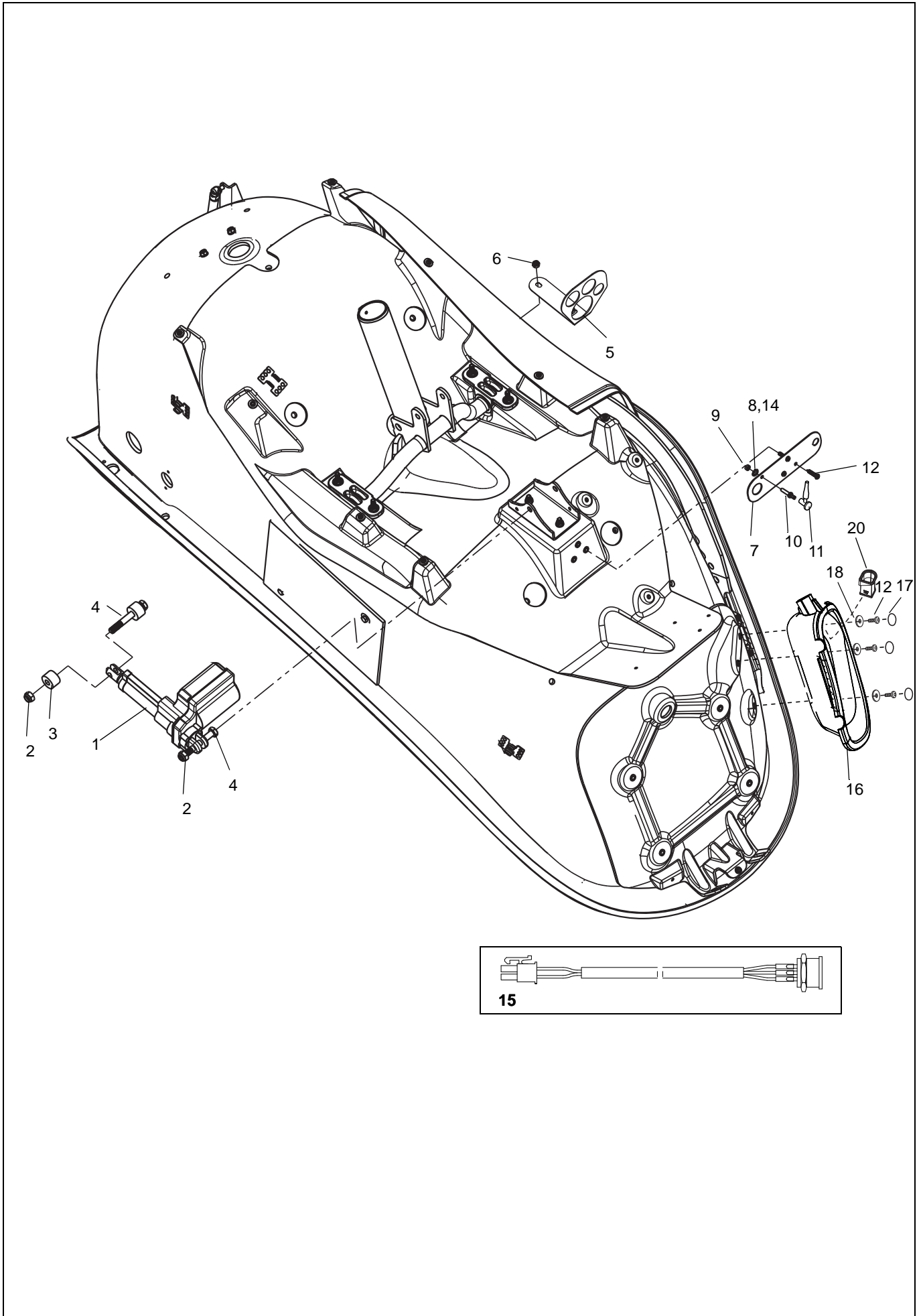

**D***Panels*

<b>Pos</b>	<b>Art. No.</b>	<b>Qty</b>	<b>Description</b>	<b>From ser. No.</b>	<b>Up to ser. No.</b>	<b>Remarks</b>
1	8361028-036	1	Panel Disinfection & airspa			Not GB
2	8361029-036	1	Panel Air-spa			
3	8362379-036	1	Panel Standard			
4	8362378-036	1	Panel Disinfection			Not GB
5	8451745	1	Spring			
6	8662573	1	Seal			
7	6190560	2	Screw FZB TX PT K25X6			
8	8259038-034	1	Closing			
9	6190559	3	Screw FZB TX PT K30X10			
10	8461032	1	Panel seal			
11	6370624	1	Link board			Will be replaced by 8664719
12	8359746	1	Panel Cable			
13	6200176	8	Washer A2 SRKB ISO 4759 5,3x25x1,5			
14	6190602	10	Screw FZB TX PT K40x16			
15	8560688	1	Thermometer			
16	8560045	1	Hand Control Air-spa			
17	8560050	1	Hand Control Standard			

D

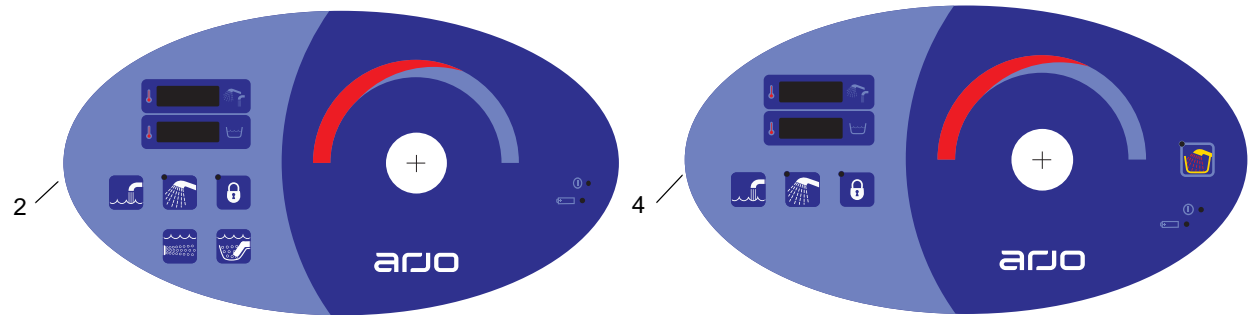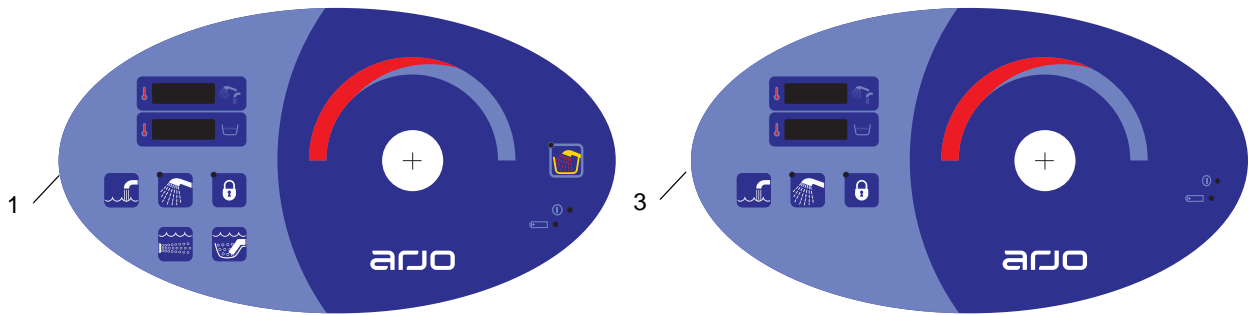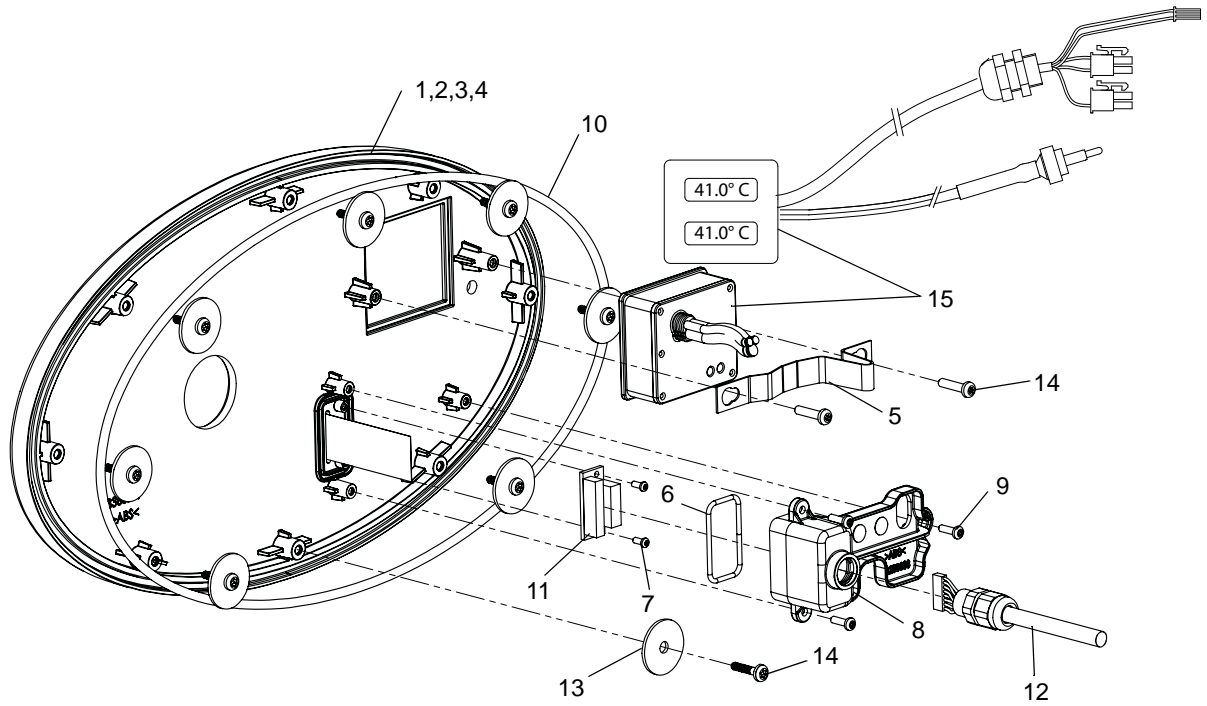

16





17

## E

## Door

Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8662357	1	Hinge bracket			
2	PF0288	1	Screw FZB MC6S ISO 4762 M5x16-12.9			
3	PD0090	1	Door hinge key			
4	8662359	1	Lower connection screw			
5	6210014	1	Nut A4- M6M ISO 4032 M8 – 80			
6	6100825	1	Screw A4-M6S ISO 4017 M8x25-80			
7	8662343	1	Hinge spindle			
8	8662434	1	Gas strut			
9	6200160	4	Washer FZB SRKB ISO 4759 11x35x3			
10	6210045	3	Locking nut A4 DIN 985 M10-80			
11	PF0143	2	Locking nut FZB M6MF ISO 7043 M5-8			Flange
12	8662440	3	Spacer ring			
13	PD0332	3	Flange washer			
14	PC0461	3	Cap			
15	6131030	1	Screw, A4 MC6S ISO 4762 M10x30-80			
16	8661118-036	1	Door hinge			
17	PF1323	2	Screw A4 MC6S ISO 4762 M10x60-80			
18	<b>8361572-xx</b>	<b>1</b>	<b>Door complete</b>			-xx colours see page B:1
19	8664012	1	Lock plate			New version 13090000055
20	8359706	1	Lock block			
21	8662432	1	Compression spring			
22	8662710	1	Rod			
23	8662414	1	Flat bar nut			
24	6100610	3	Screw A4 M6S ISO 4017 M6x10-80			
25	6200003	4	Washer A4 BRB ISO 7089 6.4x12x1.6			200HV
26	8662413	1	Bracket			
27	8662423-xx	1	Door cover			-xx colours see page B:1
28	6320246	2	Washer			
29	6311418	2	Cover			
30	6190426	2	Screw A4 K6S ISO 7380 M6x16-80			
31	6319542	1	Circlip 13mm			
32	6319543	1	Circlip 16mm			
33	8662358	1	Upper connection screw			
34	8662711	1	Tube			
35	PC0680	1	Knock out seal			
36	8662416	1	Door lock			New version July 2012
37	8357230-03	1	Door trim			
38	8662580-036	1	Door lever			
39	8359329	1	Grab handle			
40	6210043	3	Lock nut A4-DIN 985 M6-80			
41	6210241	2	Nut FZB ML6M ISO 4035 M16 - 05			
42	8662433	1	Washer			
43	6316280	2	Cover			
44	8456839	2	Flange washer			
45	6130860	2	Screw A4 MC6S ISO 4762 M8x60-80			
46	S8662690-03	1	Door seal			
47	PF0108	2	Screw A2 FTS ISO 14586 ST4.2x32-80			New version July 2012
48	6131016	1	Screw A4 MC6S ISO 4762 M10x16-80			
49	6200009	1	Washer A4 BRB ISO 7089 13 x 24 x 2.5			200HV
50	6306168	2	Bearing IGUS			
51	6307218	1	Protection cover			
52	8618811	1	Label Parker			
53	8662191	1	Mixer bracket			


**E**

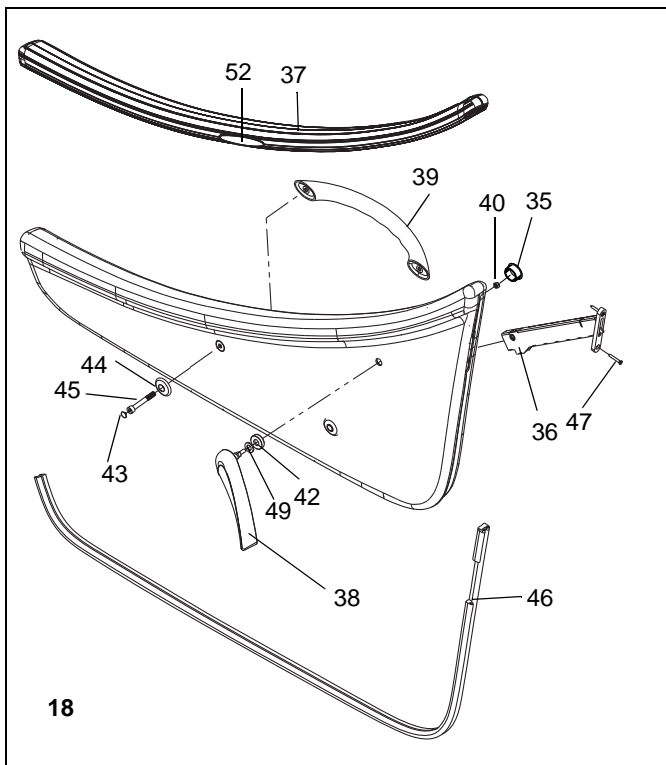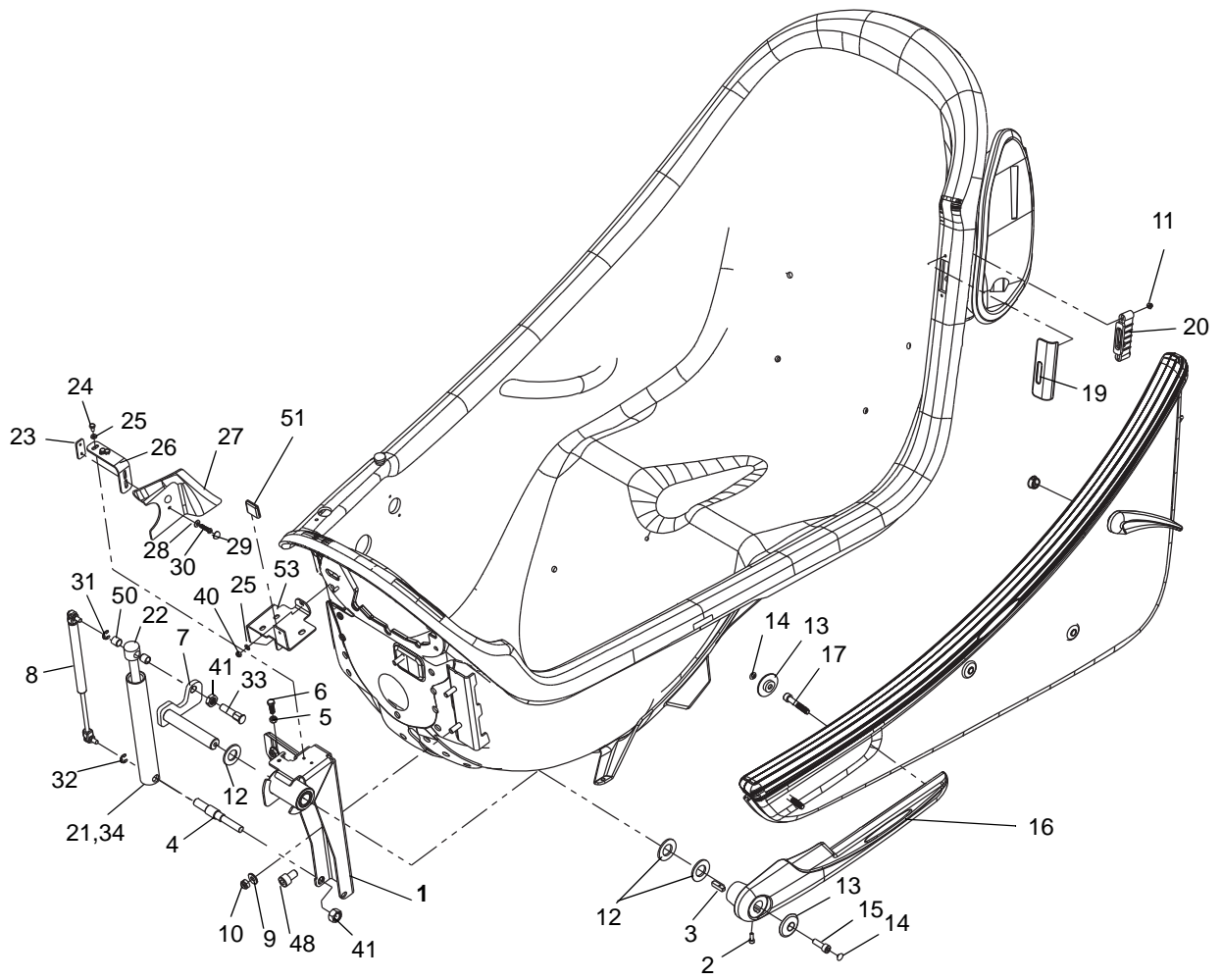

<b>F</b>						
<i>Mixer</i>						
Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8547094	1	<b>Mixer complete</b>			
2	S8359123	1	Spout complete			
3	8665043	1	Solenoid Valve RPE 3/4" 24VDC			Including couplings
4	8361676	1	PEX-pipe			
5	8361675	1	PEX-pipe			
6	8458215	1	T-pipe			
7	8361322	2	PEX-pipe			
8	6210155	1	Nut 20			
9	8546992	1	<b>Kit tempsensor attachment</b>			
10	8360717	1	Spout spacer			
11	6310120	2	Plug-in coupling			
12	PP0841	2	Solenoid Valve RPE 1/2" 24VDC			
13	8462053	2	Temp sensor			
14	8461317	2	Connection FMM			
15	6250315	1	Straight coupling			
16	6319494	4	Sleeve			
17	6260100	1	O-ring			
18	6250110	1	Compression ring			
19	8458268	1	Conex nipple			
20	6270385	2	<b>Hose softpex steel DN10</b>			Including fibre washers
21	6316139	1	Fibre washer 1/2"			
22	8662467	2	<b>Hose softpex steel DN15</b>			Including fibre washers
23	6250040	1	Fibre washer 3/4"			
24	8457463	1	Sliced nut			
25	6200070	1	Toothed washer AZ M8 A4			
26	6100816	3	Screw A4 M6S ISO 4017 M8x16-80			
27	6311411	5	Lid			
28	6316288	1	Plug			W/o disinfection or UK only
29	6310136	1	Plug-in coupling 15 mm			UK only
30	8546655	1	O-rings kit for mixer			
31	6907715	4	Nut			
32	8360433	1	Cabling solenoid valves			See also I:5
33	6260018	1	O-ring			
34	6319334	1	Stop ring			
35	8547048-034	1	<b>Control knob</b>			Incl. cover and screw
36	8461219-036	1	Cover			
37	8547097	1	<b>Repair kit mixer</b>			3 cartridges with springs and 3 non-return valves, all o-rings
38	6907717	1	Adjustment plate			
39	6200005	3	Washer A4 BRB ISO 7089 8.4x16x1.6			200HV
40	6250264	1	Compression coupling 3/4"x22			




F

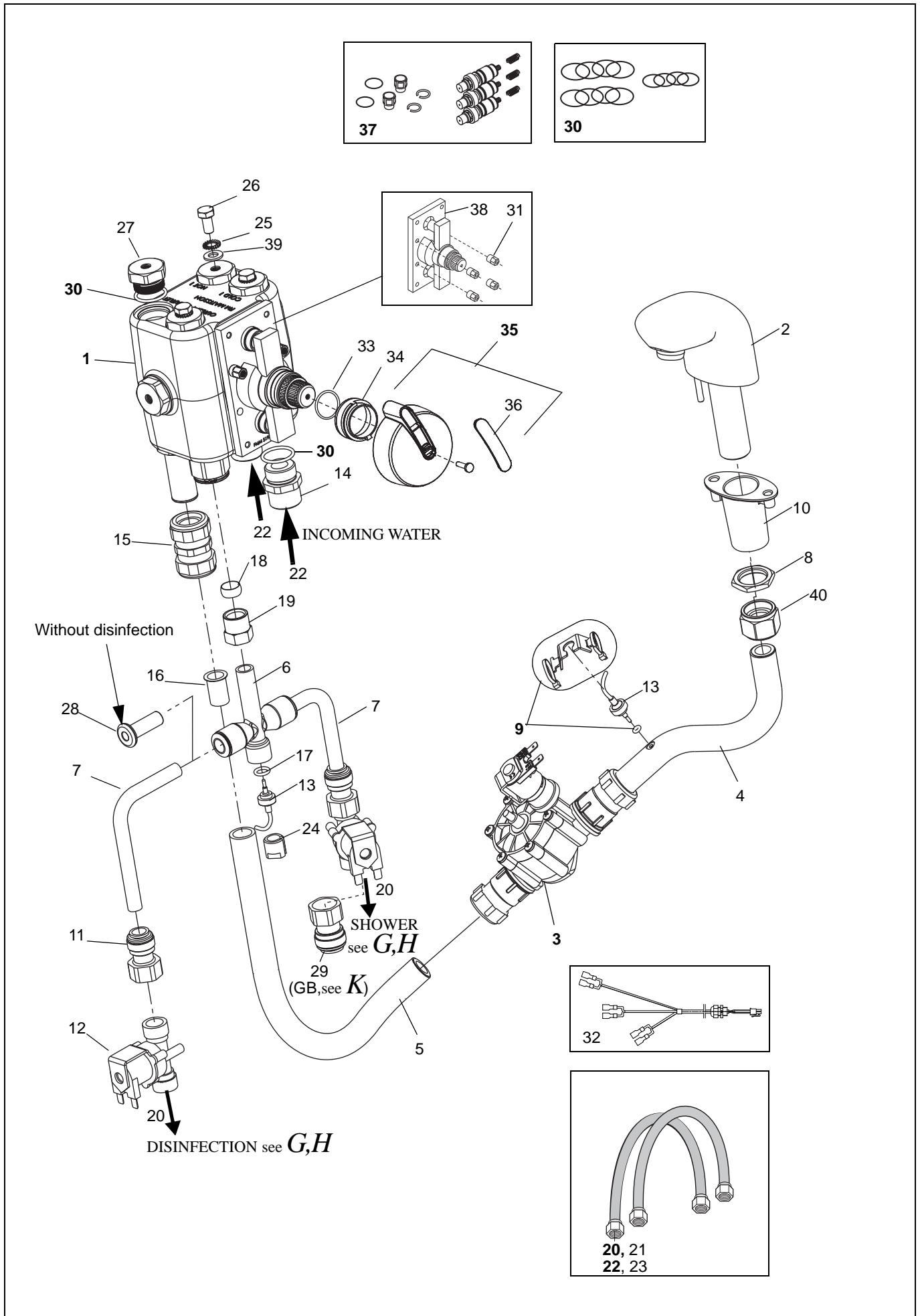

## G Shower & Disinfection US (option)

Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8261264	1	Bracket			
2	6310120	1	Stud			
3	8462010	1	PEX-pipe, disinfection US			
4	8661588	1	Plastic hose, LLDPE 12 mm blue			
5	6270176	1m	Hose			a=370mm b=220mm
6	8662572	2	Leak connector			Incl. flow restrictor 1,0 l/m
7	8150670	1	Air-gap drain			
8	6303146	4	Spacer			
9	6210041	4	Locking Nut A4 DIN 985 M4-80			
10	6326015	2	Vacuumbreaker DA			
11	8461270	2	Clip			
12	8461508	1	PEX-pipe, shower top US			
13	6907932	1	Repair kit Vacuumbreaker DA			
14	6316244	1	Plug			
15	6200054	4	Washer A4-RBS ISO 7093 6.4x18x1.6			200HV
16	6130625	4	Screw A4 MC6S ISO 4762 M6x25-80			
17	6130330	2	Screw A4 MC6S ISO 4762 M3x30-70			
18	6130416	2	Screw A4 MC6S ISO 4762 M4x16-70			
19	6130445	4	Screw A4 MC6S ISO 4762 M4x45-70			
20	6210040	2	Locking Nut A4 DIN 985 M3-80			
21	8359776	1	Battery 24V incl. cable			
22	8667040	1	Battery bracket			
23	6319493	2	Sleeve			W. disinfection two
24	8459472	2	Nipple Ø 12			
25	6260108	4	O-ring			
26	6250312	2	Compression ring			
27	8458182	2	Conex nut			
28	6250888	2	Elbow			
29	6250889	2	Adaptor 3/4"-1/2"			
30	8458181	4	Nipple			
31	6316330	2	Sleeve			
32	8461182	1	PEX pipe			
33	8543494	1	Injector			
34	6312197	1	Flow restrictor, disinfection			Grey 4 l/min
35	6200001	8	Washer A4 BRB ISO 7089 4.3x9x0.8			200HV
36	6270116	1m	Hose			
37	6319513	3	Hose clamp			
38	8343434	1	Flowmeter US			
39	8661802-03	1	Bottle connection			
40	6303148	1	Disinfection hose			
41	<b>S6303097</b>	<b>1</b>	<b>Disinfection handle, yellow; complete</b>			
42	6316139	4	Washer, fibre			
43	<b>8543696</b>	<b>1</b>	<b>Strainer kit, yellow handle</b>			Strainer handle, o-ring, screw, plug
44	<b>8545142</b>	<b>1</b>	<b>Repair kit, yellow handle</b>			Pins, sleeve, spring, 3 o-rings, 2 complete valves with o-rings
45	<b>6907870</b>	<b>1</b>	<b>Swivel, complete yellow handle</b>			Incl plastic washer
46	6260107	1	O-ring			
47	8662374	1	Attachment			
48	6250902	1	Straight coupling PM0412E			Introduced 1307000062
49	6301211	2	Cable tie			
50	6130420	1/2	Screw A4 MC6S ISO 4762 M4x20-70			W. disinfection two

G

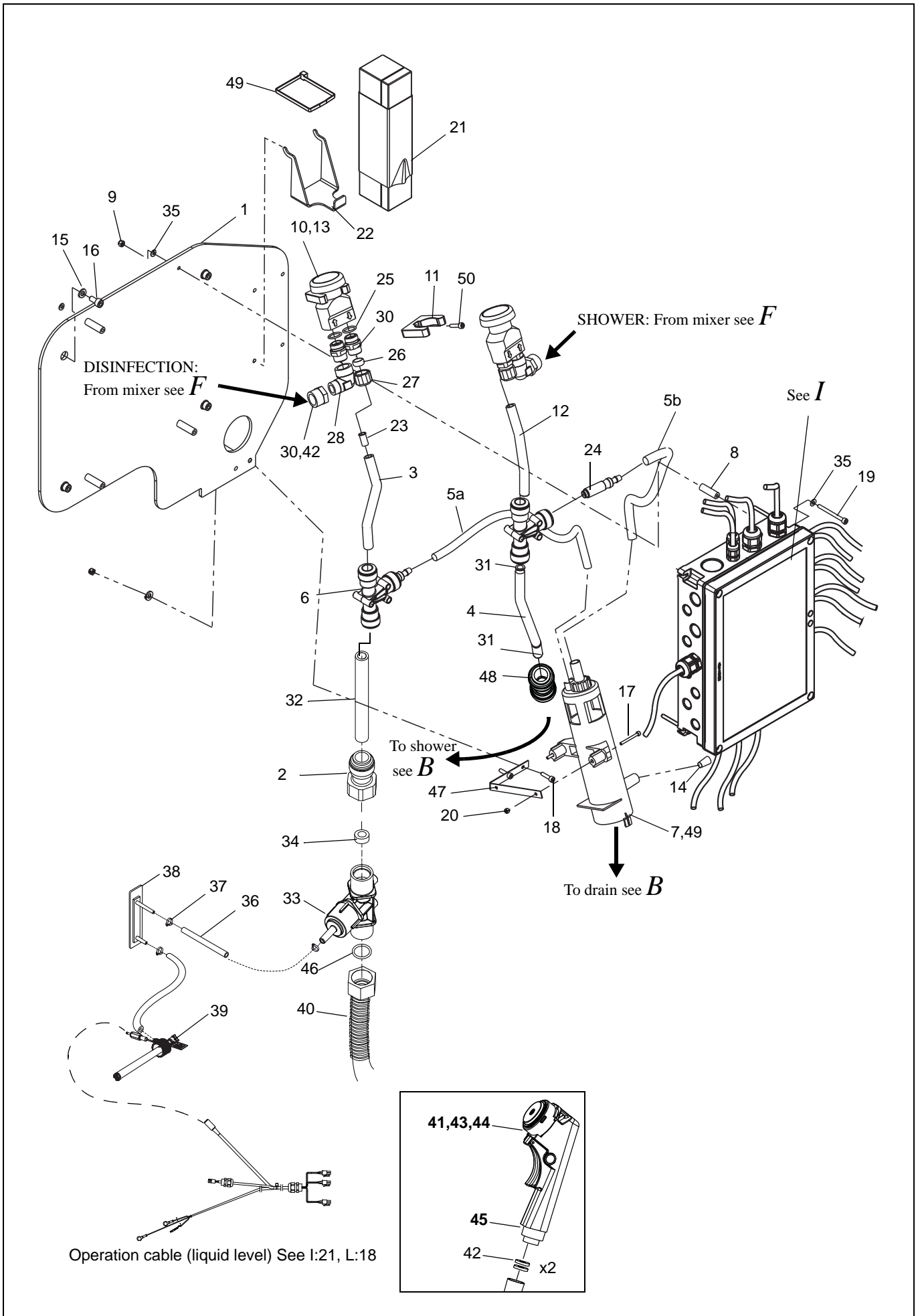


## H Shower & Disinfection EU/GB (option)

Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	8261264	1	Bracket			
2	6310120	1/2	Stud			Not GB, With disinfection two
3	8462011	1	PEX-pipe, disinfection			Not GB
4	8661588	1	Plastic hose, LLDPE 12 mm blue			Not GB
5	6270176	1m	Hose			a=370mm Not GB ant No disinfection. b=220mm, Not GB
6	8662572	1/2	Leak connector			Not GB, W. disinfection two, Incl. flow restrictor 1,0 l/m
7	8150670	1	Air-gap drain			Not GB
8	6303146	4	Spacer			
9	6210041	4	Locking Nut A4 DIN 985 M4-80			
10	6318133	1/2	Vacuum breaker DB			Not GB, W. disinfection two
11	PF0807	1/2	Clip			Not GB, W. disinfection two
12	8461319	1	PEX-pipe, shower top			Not GB
13	6260107	1	O-ring			
14	6316244	1	Plug			Not GB
15	6200054	4	Washer A4-RBS ISO 7093 6.4x18x1.6			200HV
16	6130625	4	Screw A4 MC6S ISO 4762 M6x25-80			
17	6130330	2	Screw A4 MC6S ISO 4762 M3x30-70			Not GB
18	6130416	2	Screw A4 MC6S ISO 4762 M4x16-70			Not GB
19	6130445	4	Screw A4 MC6S ISO 4762 M4x45-70			
20	6210040	2	Locking Nut A4 DIN 985 M3-80			Not GB
21	8359776	1	Battery 24V incl. cable			
22	8667040	1	Battery bracket			
23	8662734	1	Attachment			
24	8459472	1/2	Nipple Ø 12			Not GB, W. disinfection two
25	6316330	2	Sleeve			
26	8461182	1	PEX pipe			
27	8543494	1	Injector			
28	6312197	1	Flow restrictor, disinfection			Grey 4l/min
29	6303148	1	Disinfection hose			
30	6200001	8	Washer A4 BRB ISO 7089 4.3x9x0.8			200HV
31	6270116	1m	Hose			
32	6319513	3	Hose clamp			
33	8337053	1	Flowmeter EU			
34	8661802-03	1	Bottle connection			
35	PRB2000	1	Disinfection bottle, 3x3l			
36	<b>S6303097</b>	<b>1</b>	<b>Disinfection handle, yellow; complete</b>			
37	6316139	4	Washer, fibre			
38	<b>8543696</b>	<b>1</b>	<b>Strainer kit, yellow handle</b>			Strainer handle, o-ring, screw, plug
39	<b>8545142</b>	<b>1</b>	<b>Repair kit, yellow handle</b>			Pins, sleeve, spring, 3 o-rings, 2 complete valves with o-rings
40	<b>6907870</b>	<b>1</b>	<b>Swivel, complete yellow handle</b>			Incl plastic washer
41	6301211	2	Cable tie			
42	6250902	1	Straight coupling PM0412E			Introduced 1307000062
43	6130420	1/2	Screw A4 MC6S ISO 4762 M4x20-70			W. disinfection two

H




EU version illustrated. GB version only includes bracket (1) with battery (21) and control box.

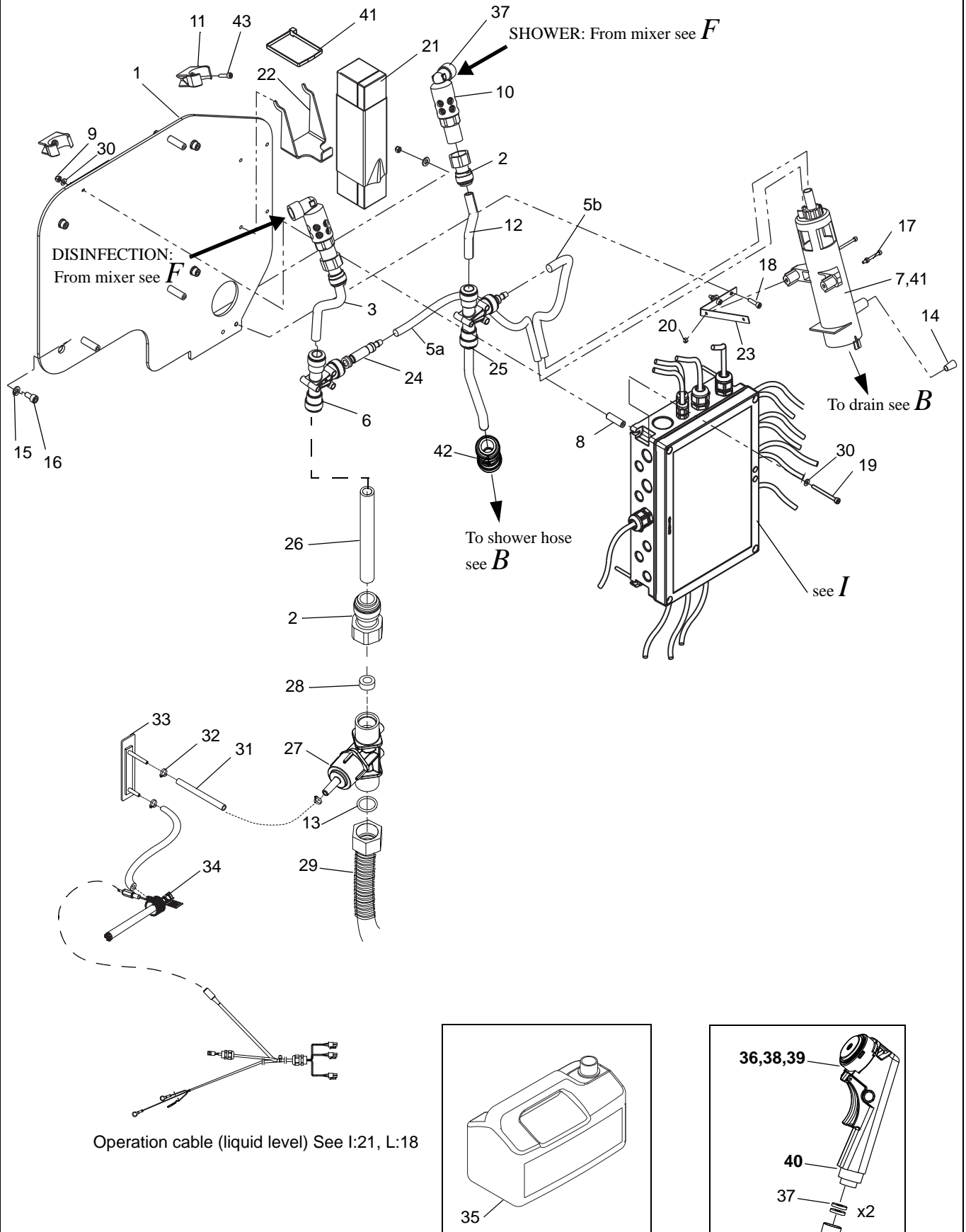

*I**Control Box*

Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	<b>S8560885</b>	1	<b>Control Box complete with cabling</b>			Incl. pos 6-10, 21 and kit 120V
2	8461537	1	PE cable			600 mm, to PE
3	8361135	1	Mains cable			J6
4	8360436	2	Actuator cable			J7,J14
5	8360433	1	Cable dis, fill, shower valves			J17 See also F:32
6	8361424	1	Hand control cable			J15
7	6303116	1	Screw DSS M4070x25			PE
8	6210222	3	Nut FZB ML6M ISO 4035 M4-05			PE
9	6190539	1	Screw FZB M6S ISO 4014 M4x12 - 8.8			PE
10	6200163	1	Washer FZB - AZ DIN 6798 M4			PE
11	6100650	1	Screw M6S 6x50 A4			
12	6210043	1	Lock nut A4-DIN 985 M6-80			
13	8459981 8459980	1	Strap 120V Strap 230V			
14	6200054	2	Washer A4-RBS ISO 7093 6.4x18x1.6			200HV
15	8664055	1	PCB			In production 1311001037
16	6320237	1	Transformer			
17	8360435	1	Emergency stop cable			J10
18	8361026	1	Air spa actuator cable			J12
19	See F:13	2	Temp sensor			(8462053) J27, J28
20	See D:16	1	Thermometer			(8560688), J22,J24,J26
21	8260432	1	Operation cable: level, S&V mute,and disinfection level			J1, J20,J21. See also L:18
22	See D:13	1	Cable panel			(8359746), J16
23	<b>8547101</b>	1	<b>Jumper</b>			<b>10 pcs.</b> JP2,3,4
24	6379112	1	Fuse T1AL 5x20 (230V)			230V: 1, 120V: 2
	6379093	1	Fuse T2AL 5x20 (120V)			
	6379095	1	Fuse T6,3AL 5x20 (230V)			
	6379014	1	Fuse T8AL 5x20			
	6379153	1/2	Fuse T10AL 5x20			
25	See G,H:21	1	Battery cable			(8359776) J9
26	See J.27	1	Air Spa mains cable			(8662619) J2,J11
27	6370224	2	Skintop PG11			
28	6370223	1	Skintop PG9			
29	6210242	3	Nut PG7			
30	6210243	8	Nut PG9			
31	6210244	5	Nut PG11			
32	6370741	1	Sealing plug PG9			
33	6370737	2	Sealing plug PG11			
34	See L:13	1	Power cable S&V			
35	8261611	1	Lid			
36	6130516	4	Screw A4 MC6S ISO 4762 M5x16-80			
37	8661895	1	Sealing			

I

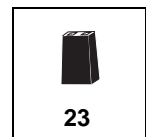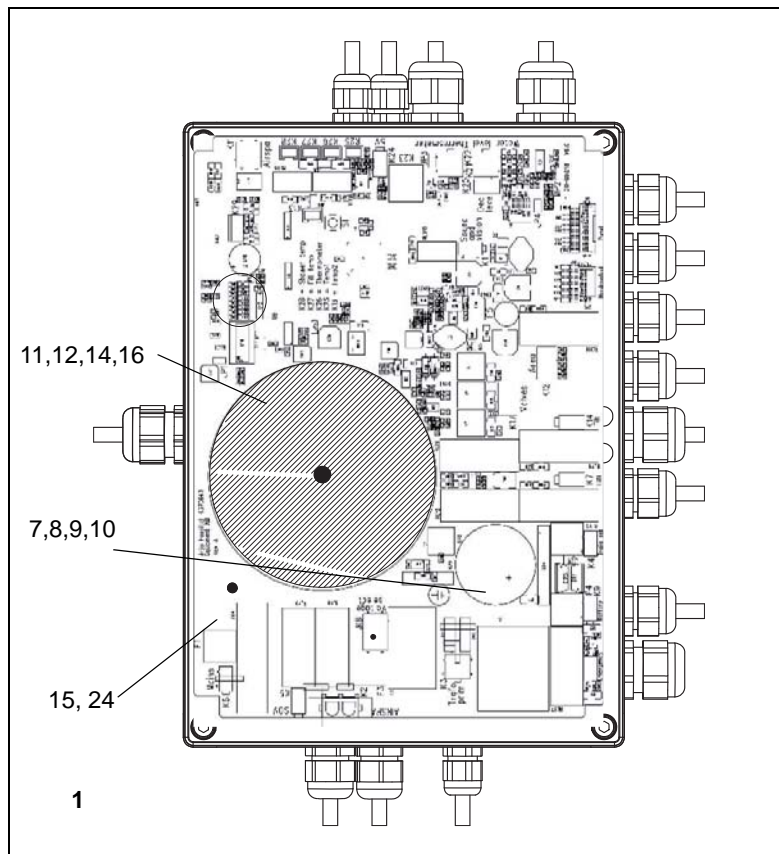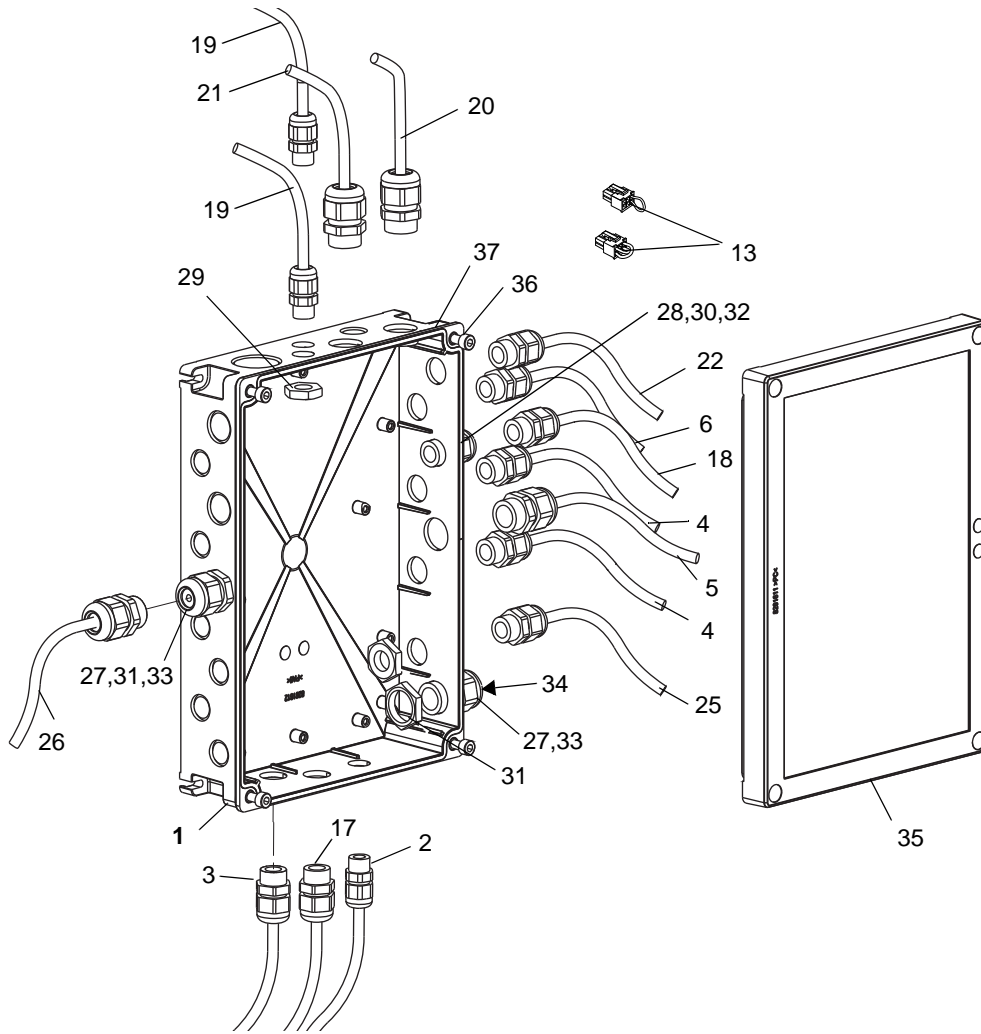


**J****Air-Spa (Option)**

Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	PP0861	10	Elbow, equal 10mm			
2	PP0977	9	Locking Nut			
3	PC0952	9	O-ring 12x2mm			
4	PP0976	9	Air Jet			
5	PP0523	9	Air Jet diaphragm			
6	PP0978	9	Air Jet cap, chrome			
7	6270363	1m	Hose 10mm, white			a=50 b=60 c=130 d=165 e=410 f=470 g=630 h=1910
8	8461799	2	Reduction washer			
9	6250833	1	Elbow 10-10			
10	PP0860	1	Tee 10mm			
11	PP0858	2	Non return valve			
12	6250890	1	Divider 10mm			
13	<b>8546872</b>	<b>1</b>	<b>Ferrule</b>			Incl. washers and screws
14	8361499	1m	Hose 22 mm			a=80 a=500 b=1100 c=1350
15	6250892	1	Tee 22mm			
16	PP0856	2	Manifold assembly 22mm-4x10mm			
17	6210043	1	Lock nut A4-DIN 985 M6-80			
18	6200054	3	Washer A4-RBS ISO 7093 6.4x18x1.6			200HV
19	6130625	3	Screw A4 MC6S ISO 4762 M6x25-80			
20	<b>S8562649-230</b> <b>S8562649-120</b>	<b>1</b>	<b>Blower 230V</b> <b>Blower 120V</b>			Complete.
21	<b>PP0862</b>	<b>1</b>	<b>Bayonet</b>			Including o-ring
22	6328033	1	Adaptor			
23	PY7012	1	Stem adaptor			
24	6250845	1	Equal straight connector			
25	6305156	1	Air Filter			
26	<b>S8561266</b>	<b>1</b>	<b>Air Spa switch, complete</b>			Incl. cabling
27	8662619	1	Air spa mains cable			
28	6200074	2	Toothed washer A4 AZ DIN 6798 M5			
29	6130512	2	Screw A4 MC6S ISO 4762 M5x12-80			




J

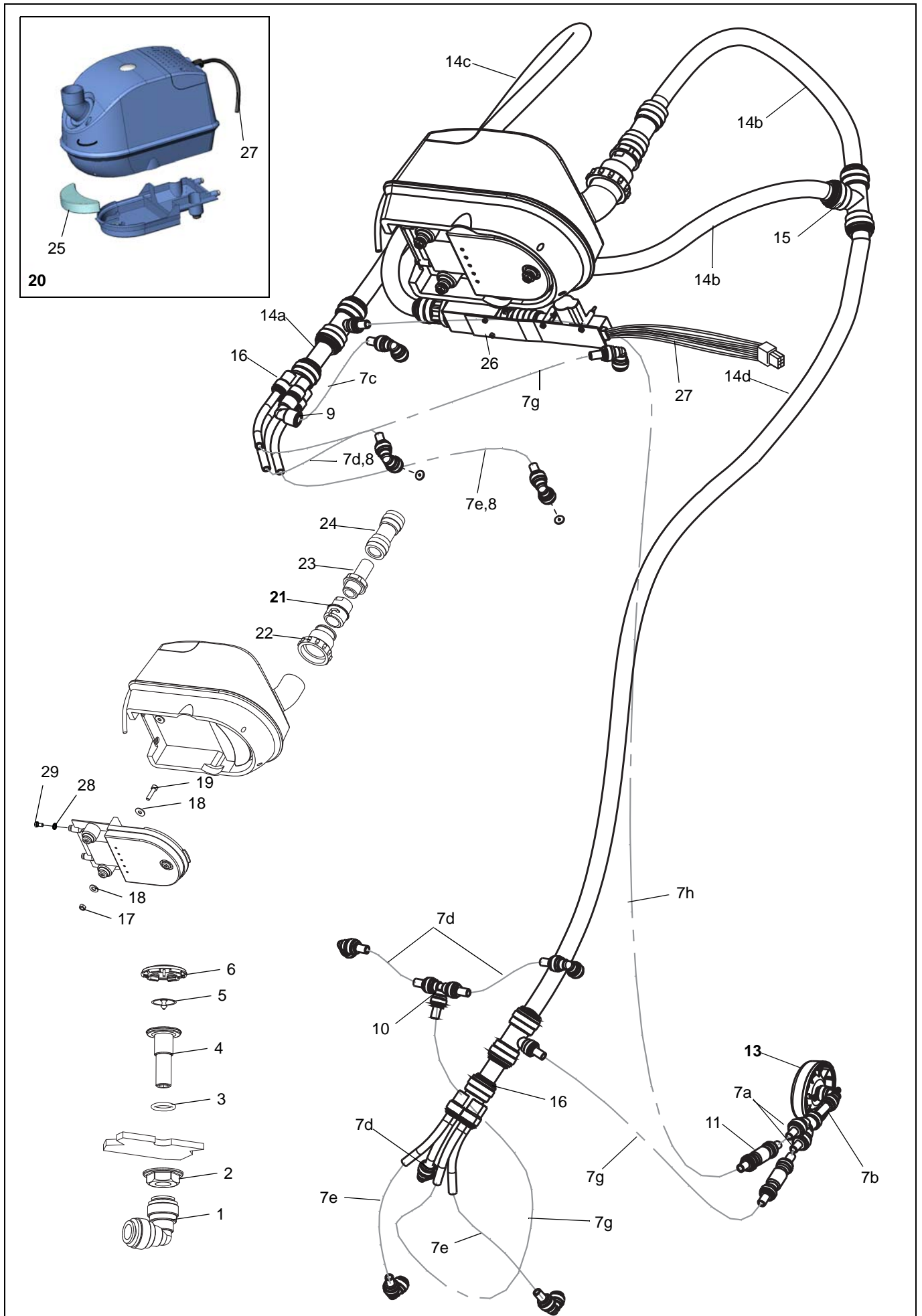

# *K Shower Pole (GB-version)*


Pos	Art. No.	Qty	Description	From ser. No.	Up to ser. No.	Remarks
1	PD0691	1	Shower mast			
2	8662395-036	1	Shower stand			
3	6306009	2	Rubber seal			
4	<b>S6303096</b>	<b>1</b>	<b>Shower handle, grey</b>			Kits See B:39,B:40
5	8454729	1	Adapter bush			
6	8337059-03	1	Bending cover			
7	6303147	1	Shower hose			
8	6303130	1	Shower handle holder			
9	6200003	4	Washer A4 BRB ISO 7089 6.4x12x1.6			200HV
10	6130630	1	Screw A4 MC6S ISO 4762 M6x30-80			
11	PC0488	3	Cord grip bush			
12	PC0487	1	Thread adapter			
13	6130612	2	Screw A4 MC6S ISO 4762 M6x12-80			
14	PC0705	1	Mounting block			
15	PF0194	2	Set screw A4 MSK6SS ISO 4029 M6x12-21H			KÄO
16	PC0706	1	Turn stop			
17	6190566	2	Set screw A4 MSK6SS ISO 4029 M6 x 6-21H			
18	PC0703	1	Upper bush			
19	6130625	4	Screw A4 MC6S ISO 4762 M6x25-80			

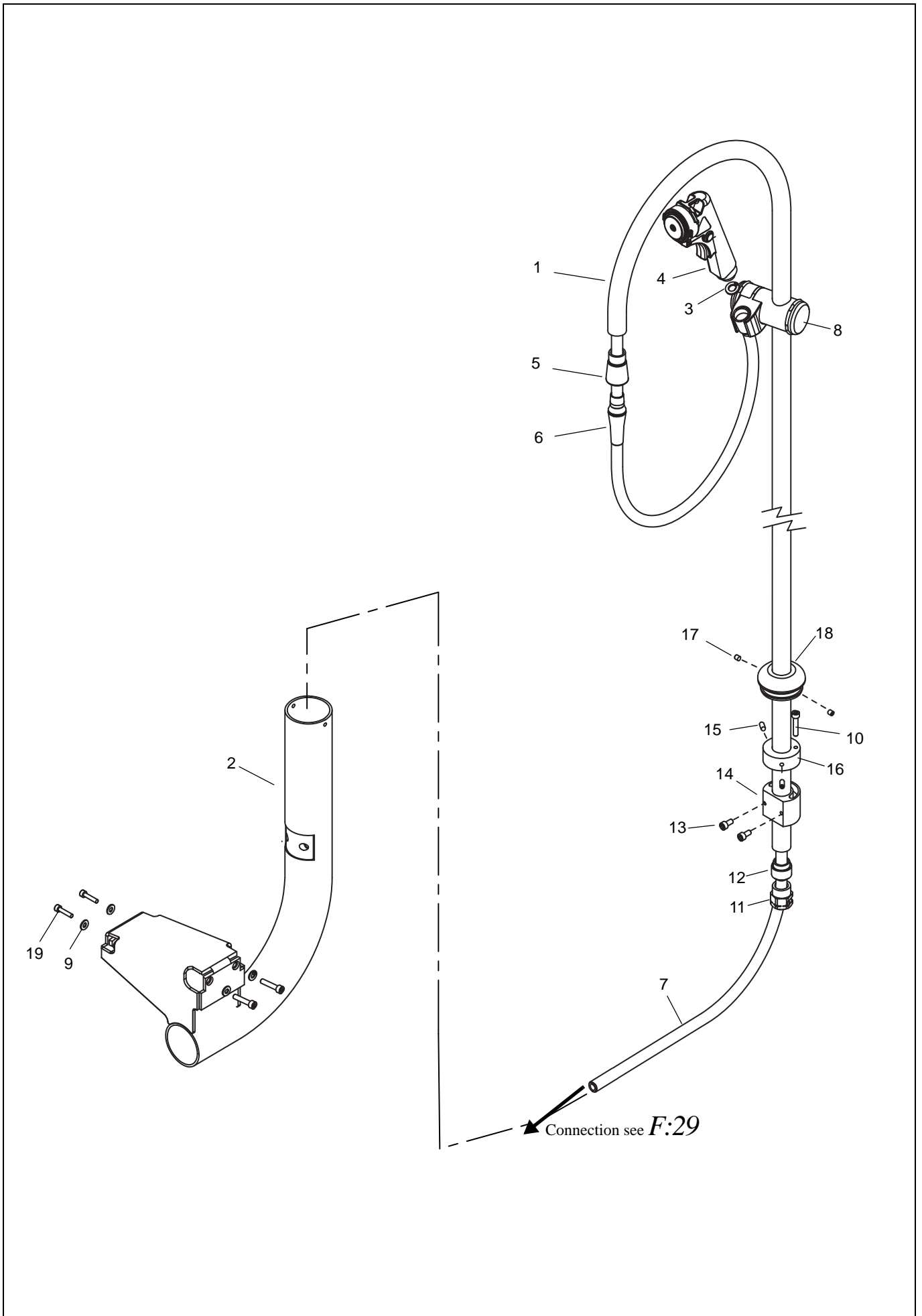**K**

**L****Sound & Vision**

<b>Pos</b>	<b>Art. No.</b>	<b>Qty</b>	<b>Description</b>	<b>From ser. No.</b>	<b>Up to ser. No.</b>	<b>Remarks</b>
<b>1</b>	<b>S8561139</b>	1	<b>Control Box, complete</b>			Incl pos 2-6, 13 and labels
<b>2</b>	8663757	1	PCB			Introduced Oct. 2013
<b>3</b>	6320235	1	Transformer			
<b>4</b>	6379038	1	Fuse T500mAL 5x20mm			
<b>5</b>	6379093	1	Fuse T2AL 5x20mm			
<b>6</b>	8360697	1	USB Cable			
<b>7</b>	6210256	1	Nut			Replaces 8460894
<b>8</b>	6200171	1	USB cover			
<b>9</b>	6316314	1	Seal			
<b>10</b>	8360872	1	IR receiver cable			
<b>11</b>	8461122	1	IR receiver clip			On PCB Control Box
<b>12</b>	6316313	1	Plug			
<b>13</b>	8360506	1	Power cable			
<b>14</b>	8360890	1	Remote control			Incl. holder
<b>15</b>	8663637	1	Remote control holder			Introduced Dec. 2012
<b>16</b>	8261138	1	LED and cabling			
<b>17</b>	6371062	2	Speaker incl cabling			
<b>18</b>	8260432	1	Operation cable (S&V mute)			See also I:21
<b>19</b>	8461314	1	USB memory			Incl. Nine Classic Works
<b>20</b>	8418658	1	Warning sign, circular			
<b>21</b>	6130445	4	Screw A4 MC6S ISO 4762 M4x45-70			
<b>22</b>	6303146	4	Spacer Skiffy 005 4400 000 02			
<b>23</b>	8418253	1	Warning sign, triangular			
<b>24</b>	8662603-034	1	Disinfection cover			With S&V but no disinfection
<b>25</b>	8662597	1	Disinfection plug			With S&V but no disinfection

L

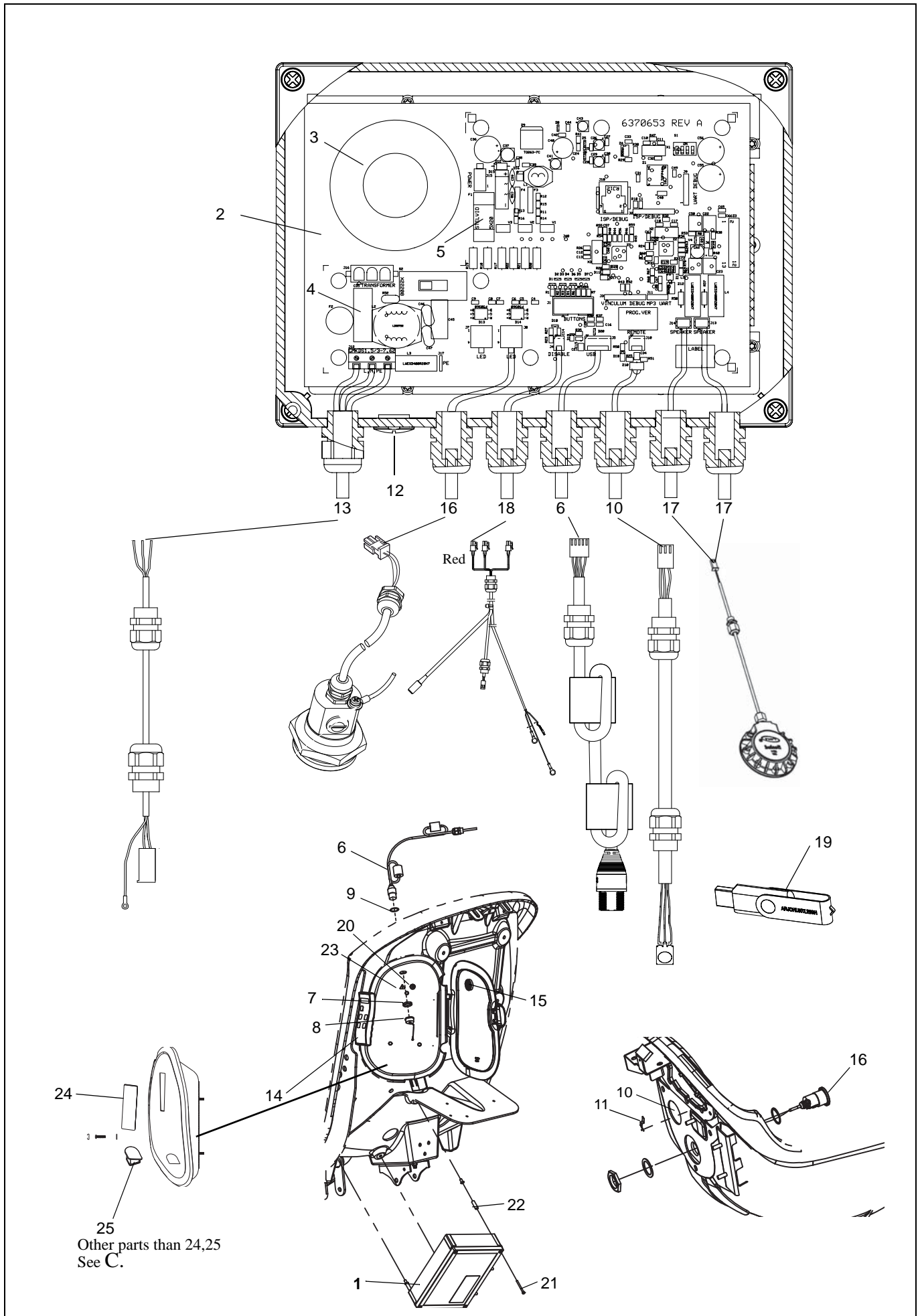

25  
Other parts than 24,25  
See C.

# Parker

## Preventive Maintenance Parts/Recommended Spares

---

Preventive Maintenance Parts	Part number	Number of baths		Page
		<b>1</b>	<b>5</b>	
Air Filter (for air-spa)	6305156	1	5	<b>J:25</b>
Air Jet Caps, chrome (for air-spa)	PP0978	8	40	<b>J:6</b>
Air Jet Diaphragms (for air-spa)	PP0523	8	40	<b>J:5</b>
Battery pack	8359776	1	5	<b>G/H:21 I:25</b>
Connection hose, alternative lengths WRAS, GB or WW	See page <b>A:43</b>	2	10	<b>A:43</b>

Recommended spares	Part number	Number of baths		Page
		1	5	
Actuator, HI/LO	PH0255	1	2	A:7
Actuator, tilt	PH0254	1	1	C:1
<b>Disinfection handle, yellow</b>	<b>S6303097</b>	1	2	<b>G:41, H:36</b>
Drain plug	6907875	1	1	B:21
Flowmeter US	8343434	1	2	G:38
Flowmeter EU	8337053	1	2	H:33
Fuse 250V/T1AL 5x20 (230V)	6379112	1	2	I:24
Fuse 250V/T2AL 5x20 (120V)	6379093	1	2	I:24
Fuse 250V/T6,3AL 5x20 (230V)	6379095	1	2	I:24
Fuse 250V/T8AL 5x20	6379014	1	2	I:24
Fuse 250V/T10AL 5x20	6379153	1	2	I:24
Hand control 4-button	8560050	1	1	D:18
Hand control 6-button	8560045	1	1	D:17
Hose clamp (flowmeter)	6319513	2	6	G:37 H:32
<b>Repair kit mixer</b>	<b>8547097</b>	3	3	<b>F:37</b>
O-rings kit mixer	8546655	1	1	F:30
<b>Shower handle, grey</b>	<b>S6303096</b>	1	2	<b>B:10, K:4</b>
Shower holder	8461432-035	1	1	B:15
Shower hose (not GB)	8358126	1	1	B:9
Shower hose (GB)	6303147	1	1	K:7

# Parker

## Service Articles and Special Tools

---

Service Articles	Type	Quantity	Part Number
Berulub Sihaf 2	Lubrication	2kg	6907944
Electrolube HTC	Heat transfer paste	60g	6319270
Essve 193	Sanitary Silicone	310ml, cartridge	6319106
Loctite 243	Locking	10ml	6950607
Loctite 5776	Locking	50ml	6311460
Loctite 5140	Sealant	85g	6311423
OKS 410	Lubrication	1000g	6907950
Permabond 2010	Adhesive	20g	6311413
Shell Cassida RLS2	Lubrication	400ml cartridge	6907792
Shell Gadus S3 A1300C	Lubrication	400g cartridge	6907945
Plexus MA300	Glue	50ml	6311313
Adhesive, Axson A230, 400 ml	Glue (repair)	400ml	6907960

Special Tools	Part Number
Door Hinge Jig Kit	PT0106
Alemite gun	Not supplied by Arjo
Glue gun	6311313-GUN
Adhesive gun (repair)	6907961
Tool Conex 303-22x3/4"	8665047



At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.



ArjoHuntleigh AB  
Hans Michelsensgatan 10  
211 20 Malmö, Sweden  
[www.arjo.com](http://www.arjo.com)

**arjo**