

## Serviceanvisning Badestol med ryg- og armlæn

### Maintenance

- Clean the shower chair regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.
- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Systematically check:
  - tightening of the wing screws
  - the condition of the locking pins
  - the condition of the tubes (if cracks)It is recommended to check the rubber tips regularly. In case of worn-out, replace the tips with the new ones.

### 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.