

Serviceanvisning Basic badebænk med håndtag, ryglæn og Hygiejneudskæring

Maintenance

- Clean the shower chair regularly using only water, sponge and mild detergents for plastic materials such as soap. Dry thoroughly.
- **Do not use** for cleaning: strong detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence – loss of warranty rights).
- Systematically check:
 - tightening of wing screws
 - condition of locking pins
 - condition of tubes (if cracks)

It is recommended to check rubber tips regularly. In case of worn-out, replace the tips with the new ones.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.