

## 6. Maintenance

### Reviews

At least once every three months make a visual inspection of all the connecting components (latch, hinge) of the product including:

- the condition of the locking pins
- the condition of the castors and brakes
- stability and the look of the frame

### Cleaning

Carry out regular maintenance of the technical condition of the chair:

- Clean the commode chair regularly using only water and mild detergents. Dry thoroughly.
- **Do not use** for cleaning: strong polishing agents and detergents, abrasive agents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Empty and disinfect the toilet bucket and the toilet seat after each use. Use toilet cleaning agents according to producer's instructions.