



Serviceanvisning Basic letvægtskørestol, varenummer: 0811300B

MAINTENANCE:

Periodic review of the procedures below will help to maintain the life and effectiveness of your wheelchair.

WEEKLY

1. Check the wheel pressure by following the instructions on the wheels. If there are pneumatic wheels on the wheelchair.
2. Check that the central bolt of the transverse arm is tight and in good condition.
3. Clean the fabric with soapy water or sufficient products.
4. Clean the metal parts of the chair with a dry cloth or with special products for chrome surfaces or painted surfaces.
5. Check the wheel bolts and adjust if necessary.
6. Check for damage to front and rear wheels.

QUARTERLY

1. Check and adjust if necessary: wheel forks, the wheel axels and the hand-rim of the rear wheels.
2. Grease or oil all components that are subject to friction or impending wear.
3. Make sure that all nuts and bolts are tight, but not too tight.
4. General revision of the chair's status and repair to whatever is necessary.

ANNUALLY

1. We recommend a full review once a year to a qualified service provider.
2. For the safety of the patient, we recommend that the wheelchair will be checked by the manufacturer every 2 years. In case of repair, use only authentic spare parts.

CLEANING:

In addition to maintenance and inspection, both the frame and the upholstery can be thoroughly cleaned.

- To maintain its appearance, the fabric must be cleaned regularly.
- Dirt can be removed with a sponge and a mild detergent with warm water, sweeping in the direction of texture.
- Rinse with clean water before letting it dry.
- **DO NOT leave the wheelchair wet.**
- Do not place the fabric in a tumble dryer. If the dirt is deep, the surface should be cleaned with a soft brush.
- The frame should be cleaned regularly with a mild detergent.
- The wheelchairs with leather seats must be cleaned with dry cloth.
- Clean the wheels with warm water and mild soap. Pay attention in drying the wheels. it is dangerous use the wheelchair with wet wheels.
- If you need to disinfect the product, use a common, mild disinfectant.
- Protect the item from scratches, cuts and punctures.

