

- ⚠ Keep this device out of reach of children under 3 years.
- ⚠ Be aware that alarms and notifications can be missed if batteries run out, connectors become disconnected, or if you are out of the system's radio range.
- ⚠ Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
- ⚠ Protect cables from any potential source of damage.
- ⚠ Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
- ⚠ This device is designed for indoor use only. Do not expose the device to moisture.
- ⚠ Use only power adapters and battery types that are specified in this user manual.
- ⚠ Do not make any changes or modifications to this device.
- ⚠ Protect the device from shocks during storage and transport.
- ⚠ Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!
- ⚠ Please note that the deliberate or accidental switching off the volume or power switch on the Pager can result in a missed alarm.
- ⚠ The product is intended to be used as a care alerting system but should not be used as the only security in life-threatening situations. This product does not replace the need for personal supervision of caretakers.
- ⚠ The product has no power switch. To turn off the product, the charger and power supply must be disconnected from the mains power and all batteries need to be removed. Make sure that the charger and power supply are easily accessible.

Information on product safety

- ⓘ Failure to follow these instructions could result in damage to the device and void the warranty.
- ⓘ Do not use the device in areas where electronic equipment is prohibited.
- ⓘ Do not expose batteries to fire or to direct sunlight.
- ⓘ Do not drop your device. Dropping onto a hard surface can damage it. If dropped, always test it before using it.

- ⓘ The device may only be repaired by an authorized service center.
- ⓘ If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
- ⓘ If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit bellman.com for contact info.
- ⓘ This device should not be used on aircrafts unless specifically permitted by flight personnel.
- ⓘ Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C.
- ⓘ Disconnect the power supply from the mains during thunderstorms or if the product is not in use for a long period of time.

Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit bellman.com for contact information.

- Radio Equipment Directive (RED)
- Medical Device Regulation (MDR)
- EC General Product Safety Directive
- Electromagnetic Compatibility Directive (EMC)
- Restriction of Hazardous Substances Directive (RoHS)
- REACH Regulation
- Waste Electrical & Electronic Equipment (WEEE)
- EC Battery Directive

ISO Certification of legal manufacturer

Certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.
 SS-EN ISO 9001 Certification Number: CN19/42071
 SS-EN ISO 13485 Certification Number: CN19/42070

Certification Body

SGS United Kingdom Ltd
 Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK