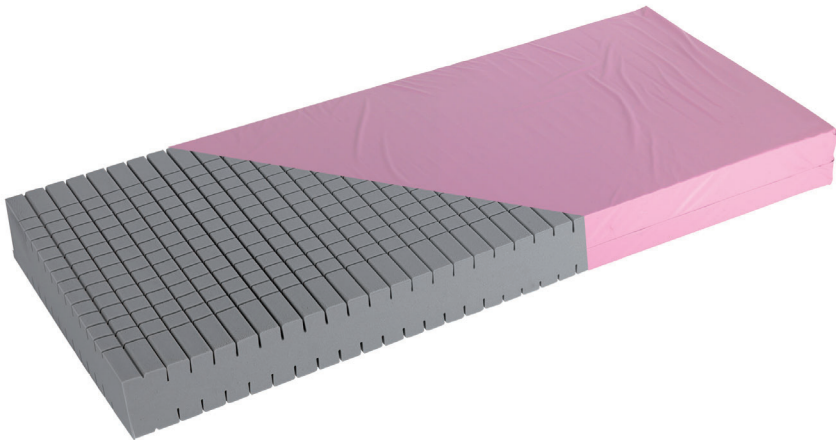
INSTRUCTIONS FOR USE

Pentaflex Elegance

Mattress replacement

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WARNING

To avoid injury, always read this Instructions For Use and accompanied documents before using the product.

Mandatory to read the Instructions For Use.

Design Policy and Copyright

® and ™ are trademarks belonging to the Arjo group of companies.

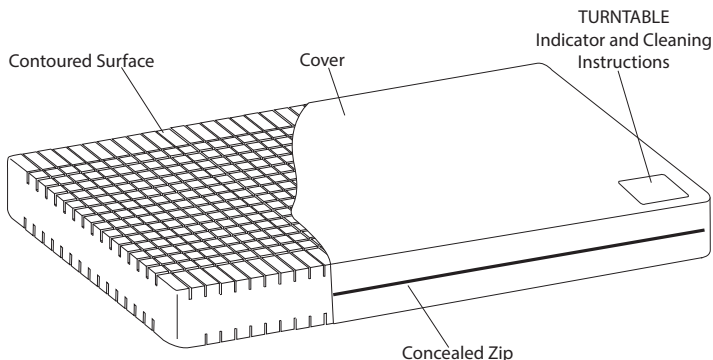
© Arjo 2019.

As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of Arjo.

Product Descriptions

Mattress replacement system

The Pentaflex® Elegance mattress replacement design comprises of a reversible 4-way turn, high resilience polyurethane core with 5 anatomical pressure redistributing zones per side, protected by a 2-way stretch, vapour permeable, water resistant cover.

A *Pentaflex* Elegance foam squab (part no. SQB2) is available for use with in-built bed extensions, providing an extra 15 cm for the taller patient.

WARNING

Mattress handles are for mattress transport only.

Clinical applications

Indications for Use

Pentaflex Elegance is indicated for users up to 140 kg (308 lbs / 22 stones). Suitable for a wide range of patients including those with superficial ulcers (in association with an individualised plan of care).

Note: These are guidelines only and should not replace clinical experience or judgement. If there is no improvement in the patient's condition, specialist advice should be sought.

WARNING

The cover of this product is vapour permeable but not air permeable and may present a suffocation risk. It is the responsibility of the care giver to ensure that the user can use this product safely.

If a serious incident occurs in relation to this medical device, affecting the user, or the patient then the user or patient should report the serious incident to the medical device manufacturer or the distributor. In the European Union, the user should also report the serious incident to the Competent Authority in the member state where they are located.

Instructions for Use

Mattress replacement system:

1. Remove all packaging.
2. Lay the *Pentaflex* Elegance mattress replacement onto the bed frame, ensuring that there are no sharp edges which may rip the cover.

CAUTION

Do not use the mattress without a cover. It provides a protective barrier against biological contamination of the foam.

3. Ensure the correct month is showing on the Turntable™ indicator at the foot of the cover. If the incorrect month is showing, rotate the mattress to show correct month at the foot end. Months are indicated as follows:
1/12 = January
2/12 = February, etc.
4. Use the *Turntable* indicator as a guide for the monthly rotation and turning of the mattress. This helps to even out wear of the mattress and to extend the life of the product.

Decontamination

To Clean the Cover

Wipe all exposed surfaces with a cloth which has been moistened in simple detergent and water. Ensure to remove any organic debris that may be adherent to the surfaces. Dry thoroughly.

To Disinfect the Cover

For disinfection we recommend a chlorine-releasing agent such as NaOCl or NaDCC diluted to 1,000 ppm (range may vary between 250 & 10,000 ppm depending on contamination status and local policy).

Wipe with a cloth moistened with clean water and dry thoroughly.

Alternatively, use a cloth moistened with alcohol (70%) all allow to air dry.

Routine checks

The foam core and cover should be inspected regularly for any signs of damage, soiling or staining.

Technical Data

Typical Bed size	-	-	Acute Single Size	-	-	Community Standard Size	Community Double Size
Suitable for use on Arjo Bedframe	20050C Child Anniversary™	Contura™ 420 and 430	Contura 350, 360, 460, 480, 560 and Minuet® 2	Minuet 1	Contura 880 and 980	-	-
Size (cm) L x W x D x Hx	170 x 86 x 15 x H2	182 x 78 x 15 x H2	198 x 86 x 15 x H2	200 x 90 x 15 x H2	202 x 88 x 15 x H2	190 x 92 x 15 x H2	190 x 138 x 15 x H2
PENTAFLEX ELEGANCE Product Code	PE170/086H2	PE182/078H2	PE198/086H2	PE200/090H2	PE202/088H2	PE190/092H2	PE190/138H2
Material	High resilience polyurethane foam						
Cover Material	Polyurethane coated polyester fabric with enhanced durability						

Pentaflex Elegance mattresses are available in other custom made sizes, e.g. PEYYY/ZZZHx, where YYY specifies the length (between 140 and 216 cm), and ZZZ specifies the width (between 66 and 140 cm), Hx specifies the number of handles (available number of handles 2, 4, 5, 6 or 7). All sizes must be specified in increments of 2 cm.

When combining beds and mattresses outside the table given above, it is the customer's responsibility to ensure compatibility of mattress and bed frame. Please refer to your bed manufacturers recommended mattress sizes. For further details contact Arjo.

Fire retardancy:

System meets the requirements of BS EN 597-1: 1995, BS EN 597-2: 1995, and BS 6807: 2006, 5.

FIRE WARNING

Cigarettes, lighters, naked flames and exposed heating elements e.g. in electric fires (and other electrical appliances) are sources of fire hazard and could ignite bed linen, blankets and clothing. It is essential that these fire sources are kept away from the mattress.

NB: It is highly dangerous to smoke in bed.

End of Life Disposal

The following advice is recommended:

- Fabric material used on the mattresses or any other textiles, polymers or plastic materials etc. should be sorted as combustible waste.
- Mattresses at the end of life should be disposed of as waste according to the national or local requirements which may be landfill or combustion.

Note: Follow all local regulations and, if applicable, healthcare facility standard operating procedures.

Warranty and Service

Standard terms and conditions apply to all sales. A copy is available upon request. These contain full details of warranty terms and do not limit the statutory rights of the consumer.

For service, maintenance, and any questions regarding this, or any other product, please contact your local distributor.

Symbols

Recommended wash temperature:
15 min at 60°C
Maximum wash temperature: 15 min at 95°C

Do not iron

Do not use phenol-based cleaning solutions

Hospital name

Tumble dry at 60°C
Maximum drying temperature 80°C

Wipe all surfaces with cleaning solution, then wipe with a cloth moistened with water and dry thoroughly.

Date of first use

Refer to user instructions

Routine disinfection: 1,000 ppm.

Fire retardancy

Safe Working Load (SWL)
140 kg / 309 lb

CE marking indicating conformity with European Community harmonised legislation

Indicates the product is a Medical Device according to EU Medical Device Regulation 2017/745

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HONG KONG
Arjo Hong Kong Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7600
Fax: +852 2960 1711

ITALIA
Arjo Italia S.p.A.
Via Giacomo Peroni 400-402
IT-00131 ROMA
Tel: +39 (0) 6 87426211
Fax: +39 (0) 6 87426222
E-mail: Italy.promo@arjo.com

MIDDLE EAST
Arjo Middle East FZ-LLC
Office 908, 9th Floor,
HQ Building, North Tower,
Dubai Science Park,
Al Barsha South
P.O Box 11488, Dubai,
United Arab Emirates
Direct +971 487 48053
Fax +971 487 48072
Email: Info.ME@arjo.com

NEDERLAND
Arjo BV
Biezenwei 21
4004 MB TIEL
Postbus 6116
4000 HC TIEL
Tel: +31 (0) 344 64 08 00
Fax: +31 (0) 344 64 08 85
E-mail: info.nl@arjo.com

NEW ZEALAND
Arjo LV
34 Vestey Drive
Mount Wellington
NZ-AUCKLAND 1060
Tel: +64 (0) 9 573 5344
Free Call: 0800 000 151
Fax: +64 (0) 9 573 5384
E-mail: nz.info@Arjo.com

NORGE
Arjo Norway AS
Olaf Helsetts vei 5
N-0694 OSLO
Tel: +47 22 08 00 50
Faks: +47 22 08 00 51
E-mail: no.kundeservice@arjo.com

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Address page - REV 24: 04/2019

At Arjo, we are committed to improving the everyday lives of people affected by reduced mobility and age-related health challenges. With products and solutions that ensure ergonomic patient handling, personal hygiene, disinfection, diagnostics, and the effective prevention of pressure ulcers and venous thromboembolism, we help professionals across care environments to continually raise the standard of safe and dignified care. Everything we do, we do with people in mind.

ArjoHuntleigh AB
Hans Michelsensgatan 10
211 20 Malmö, Sweden
www.arjo.com

arjo

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Arjo Inc.