- GB** ARIA - Pressure relieving mattress - User manual
- D** ARIA - Wecheldruckmatratze - Gebrauchsanleitung
- I** Materasso a bolle - mod. ARIA - Manuale d'uso
- DK** ARIA - Vekseltryksmadras - Brugervejledning

Item nr.
Art. Nr. **270016**
Codice
Varenr.

CE

MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
www.mobilex.dk

GB ARIA - Pressure relieving mattress

Intended use of the device

The Mobilex product is a medical device which is intended to be used only as an air support surface system for the human body. It is suitable for pressure ulcer prevention and treatment. It has been specifically designed for prevention of bedsores and pressure ulcers and offers an affordable solution to 24-hour pressure area care. It can maximally be loaded with 130 kg. **Any other use is prohibited.**

Contraindications

Patient conditions for which the application of pressure relieving therapy on an alternation system is contraindicated are as follows: cervical or skeletal traction and unstable spinal cord injuries.

Safety requirements

General

- ✓ Make sure that this user manual is read by all persons using the device. Manufacturer does not take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- ✓ Only use the product in flawless condition.
- ✓ If defects or errors are detected you must immediately contact the dealer.
- ✓ Follow the information on the product label.
- ✓ Only use the product for the described purpose.
- ✓ Avoid making constructive changes on the device, unless you have the manufacturer's written acceptance for such modifications.
- ✓ The device must only be used on beds equipped with regular mattresses. It is forbidden to put the antidecubitus mattress directly on the bed frame.
- ✓ The device is dedicated to be used indoors only.
- ✓ A physical/occupational therapist should always assist in first adjustment of the device for optimum support and correct functionality.

Electrical

To avoid incidents and to reduce the risk of burns, electrocution, fire, or injury to persons while using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

- ✓ Always unplug this product immediately after use.
- ✓ Do not use while bathing or near water sources.
- ✓ Do not place or store product where it can fall or be pulled into a tub or sink.
- ✓ Do not place in or drop into water or other liquids.
- ✓ Do not reach for a product that has fallen into water. Unplug immediately. The device is not intended to be self-propelled while seated.
- ✓ Close supervision is necessary when this product is used by, on, or near children or disabled people.
- ✓ Use this product only for its intended use as described in this user manual.
- ✓ Do not use attachments not recommended by the manufacturer.
- ✓ Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Return the product to a service center for examination and repair.
- ✓ Keep the cord away from heated surfaces.
- ✓ Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
- ✓ Never drop or insert any object into any opening or hose.
- ✓ Do not use outdoors or operate where aerosol (spray) products are being used.
- ✓ Connect this product to a properly grounded outlet only. See Grounding instructions below.

Electrical connection

Before any connection is made, make certain that this product is connected safely to a conductor via the two-wire power cord.


Product description

The device comes with all the necessary features for efficient decubitus prevention. It consists of few various elements. No tools are required for assembly. Devices are supplied in carton boxes, detached in a few elements for easier transportation.

The content is:

- ✓ 1 Pump
- ✓ 1 Mattress pad with inflation tubes
- ✓ 1 User manual

Power cord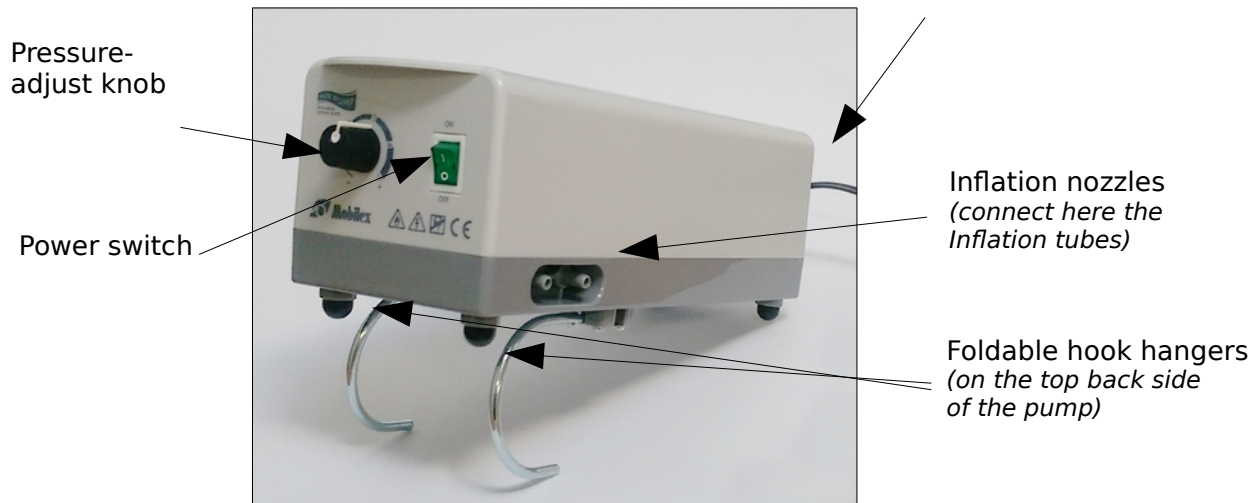

P05 compressor (pump)

Assembly description

Looking on the picture above the device should be assembled in a quite instinctive way.

We recommend you to follow below steps:

- In the first place take all elements out of the box and check with the list from above if nothing is missing.
- If you'd discover at that stage that some parts are missing or damaged, do not continue assembly process. Contact your supplier of the product right away.
- Lay the mattress pad on a foam base or other regular bed mattress. Do not put it directly on the bed frame. Please refer to the Specifications section for details.
- The inflation tubes should be towards the foot end so that they can be connected to the inflation nozzles on the pump.
- Cover it with a cotton sheet to avoid direct skin contact and reduce friction.
- Hang the pump over the frame or board at the foot end of the bed using the foldable hook hangers. Make sure the pump is secured.
- Connect the ends of the inflation tubes to the mattress pad nozzles and to the pump's inflating nozzles. Make sure the tubes are properly attached and that they are fitted tight on the nozzles.
- Make sure the inflation tubes are not kinked or tucked under the pad.
- Before inserting the plug into the outlet, make sure the voltage is compatible.
- Plug the power cord into an electrical outlet with AC power. Replace or repair if damaged.
- Make sure this product is well grounded.
- Make sure to disconnect the pump by unplugging the power cord when it is not in use.

Function description

- Turn on the power by pressing the power switch at the right side of the pump. The indicator of the power switch will come on. The pump starts to pump air into the mattress pad.
- For time-saving in the first inflation, turn the pressure knob to the full Firm/Hard position. It requires approximately 10 minutes for a bubble pad to be fully inflated.
- When mattress pad is fully inflated turn the pressure-adjust knob to set a comfortable pressure level from soft to firm according to patient's weight and comfort.
- It is recommended that the pressure-adjust knob being turned to Firm when the bubble pad is firstly inflated. Users can then easily adjust the pressure level of the bubble pad to a desired firmness by themselves or according to the suggestion from a health care professional.
- If the pressure is consistently low, check for any leakage occurring in inflation tubes. Contact your local dealer for replacement of any damaged tubes.
- The deflating/inflating cycle runs automatically, when pump is turned on.
- The air pressure can be adjusted with the control knob between settings 1 to 5. The pump pressure is correctly set, when the patient's body does not rest on the underlying mattress.

Cleaning & maintenance

It is important to follow these procedures for cleaning and decontamination of the pump and bubble pad before using the system on patients. The cleaning is required at least once a week to maintain personal hygiene.

Pump

- DO NOT immerse or soak the pump.
- Check for external damage and move the pump to a cleaning area.
- Place the pump on a work surface and spray or wipe the outside of the case with quaternary ammonium solution.
- DO NOT spray any cleaning solution directly on the surface of the pump.

- DO NOT use a Hypocarbonate or Phenolic based cleaning solution as this may cause damage to the case. Allow the solution to incubate for 10 minutes or accordingly as stated by the cleaning product used.
- Spray cloth with cleaning solution and clean faceplate. DO NOT allow excess cleaning solution on faceplate or control panel (if solution gets inside, damage will occur).
- Allow surface to thoroughly dry after cleaning.
- After the pump is thoroughly cleaned and dried, proceed to plug in the pump and test to see if it runs normally.

Bubble pad

- Any obvious blood spots should be wet thoroughly with 1:9 Hypochlorite solution
- (1 part bleach to 9 parts water) and allow drying for at least 10 minutes. Then use a clean, damp cloth.
- Brush or wipe down all surfaces with soap and water before applying any liquid.
- Allow it to thoroughly air dry. Once the inside is dry, turn it back: wipe down the outside of the bag with disinfectant.
- Dry the bubble pad on a sunless area after cleaning.

Maintenance

- Check the power cord and plug to see if there are abrasions or excessive wear.
- Plug in the pump and check the airflow from the hose connection port. The airflow should alternate between ports every half-cycle time.
- For storage lay the bubble pad flatly. Fold or roll it carefully. Unplug the pump and store with proper identification tag. Store device in dry places with non freezing temperatures.

Recommended service life: Normally 10 years depending on use.


Warning

 **Please note that if the product is bent or broken, the product must not be recycled but must be replaced.**

Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to above mentioned advice.

Disposal of the product

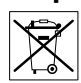 The product cannot be disposed of with household waste but must be brought to the local recycling centre. Follow the national requirement to dispose the pump. It should be delivered to the deposit station for re-use or recycling.

Most common problems and solutions

Problems	Solutions
<i>The pump doesn't work</i>	<ol style="list-style-type: none"> 1. Check if the plug is inserted firmly into the outlet. 2. Turn on the power switch again. 3. If the power indicator is ON and the pump still doesn't work, contact your local dealer immediately. 4. If the power indicator is OFF, there may be a faulty outlet. Try to connect the power cord to another outlet. If the power indicator is still OFF, contact a qualified electrician for main power check.
<i>Incomplete inflation (Low pressure)</i>	<ol style="list-style-type: none"> 1. For a quick check, adjust the pressure to Firm. 2. Check to see if the inflation tubes connected to the pump are twisted or disconnected, or any leakage occurring. Always keep the tubes straight. Change the tubes if there is any leakage.

Product label

Mobilex
a better life in motion

MOBILEX A/S
 Grønlandsvej 5
 DK - 8660 Skanderborg
 Tel: +45 87 93 22 20
 www.mobilex.dk

Mobilex product name
 EX. For outdoor and indoor use

Item nr. xxxxxx

Max. 150 kg

Max. width = 63 cm

Max. load in basket = 5 kg

Produced 12.11.2020

Serial nr. 001

5 740001 403084

Example of the product label (not original)

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Eksempel på typeskilt

Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

Technical data

Pump P05	Specification
Dimensions	27,5 x 10 x 10 cm
Weight	1.1 kg
Pressure Range	70 ~ 130 mm Hg
Cycle Time	6 min

Mattress Pad	Description	Dimensions	Material	Min./Max. Load
B01	Bubble pad w/positioning extension	200x90x6 cm	PVC 0.3 mm	20 kg /130 kg

System	Specification
Classification	Class I
Applied Part	Mattress Pad
Input Rating	230V 50Hz, < 10W, 1A
Fuse Rating	250V, F1AL
Air Output	6-7 L/min
Environment Requirements	<ul style="list-style-type: none"> Temperature: Operation 0°C to 40°C Humidity: Operation 30%~75% non-condensing
Safety Standards	CE, Protection Against Class I Electric Shocks

Declaration of Conformity (CE)

(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

hereby declares that the

(erklært hermit, dass das Produkt / erklærer hermed, at / dichiara che il)

Mobilex ARIA Pressure relieving mattress

Pump type: P05 & Mattress type 801 PVC,

Item No. 270016

(Artikel Nr., Varenr., Codice)

is designed and produced in accordance with the technical documentation and the basic requirements of the:

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In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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